


QUICK START GUIDE - A-450/A-625 Ceiling Lift

Manual: 753139 [Version 2.0]


1 Charge the Lift

i Before initial use, the lift unit must be charged for 5 hours.

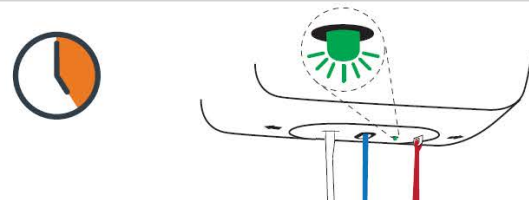


2 Connect the Hand Control

i Make sure the white plastic tab on the Emergency Stop strap is pushed fully in place.
Contact your local Handicare dealer for Hand Control connection / disconnection servicing.

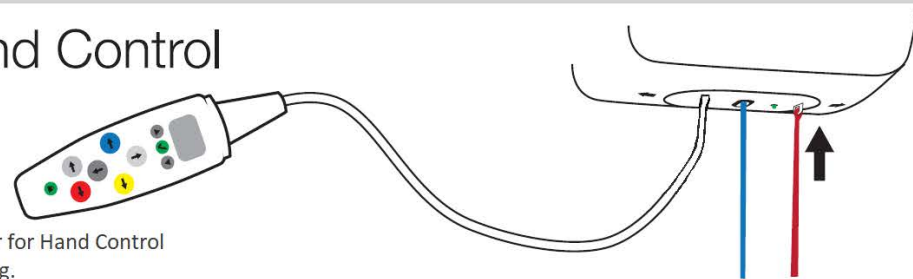

3 Turn On the Lift

Press any button on the hand control to turn the lift on.

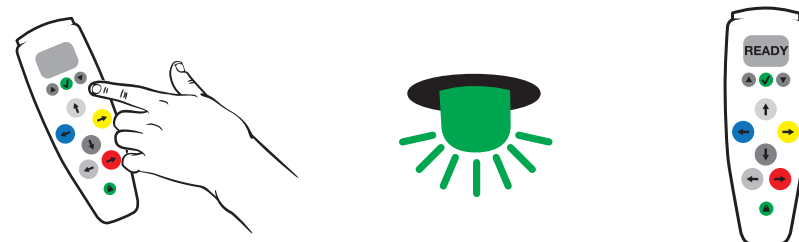

4 Turn Off the Lift

The lift automatically shuts off after approximately 5 minutes of non-use.



i In an emergency, the lift can also be shut off by pulling the red Emergency Down strap.


5 Hand Control Operation

The A-450/A-625 lift hand control:



- Controls lift motion
- Presents active lift status display
- Provides usage data and diagnostics.

Available models:

Manual Traverse, Power Traverse, and Power Traverse X/Y.

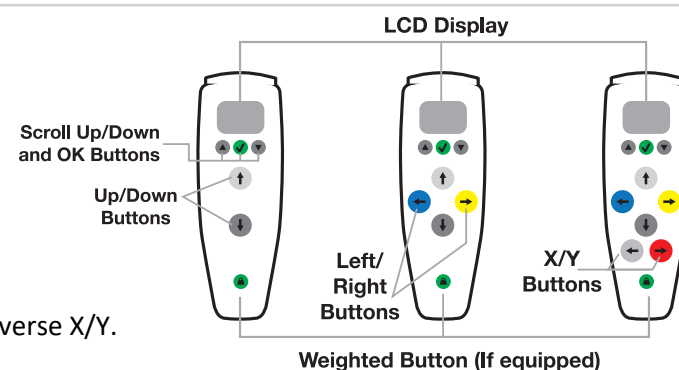


6 Transfer an Individual

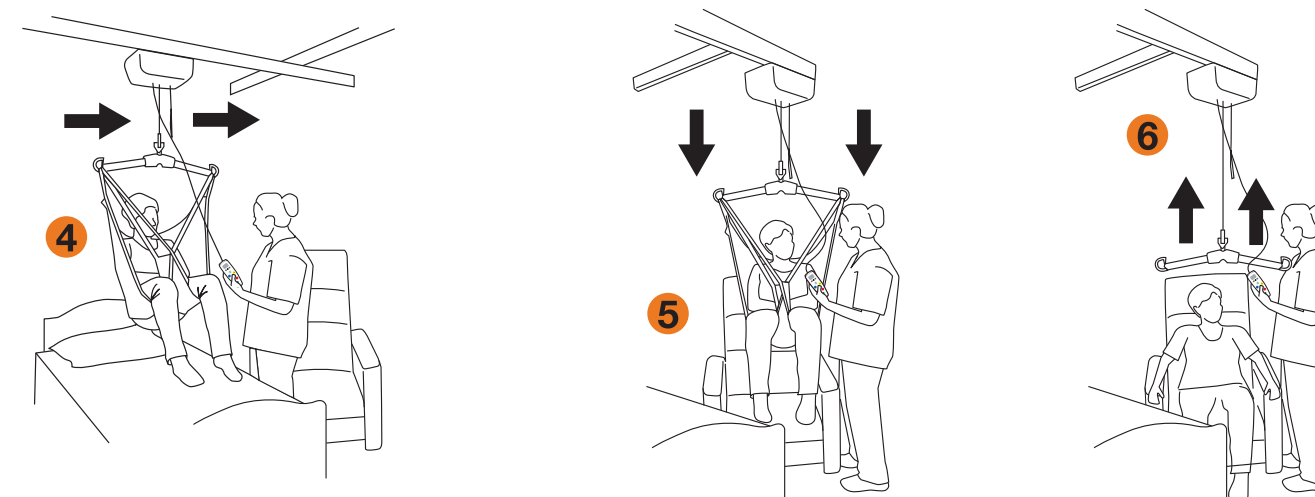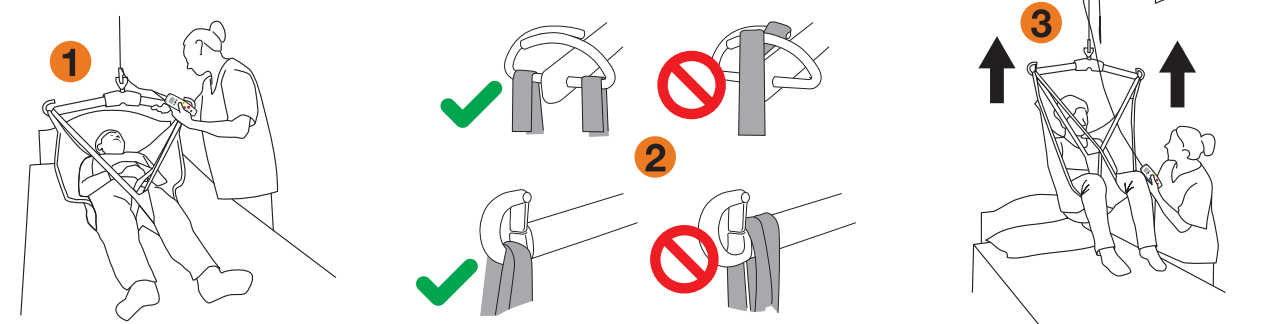

! Never drag the lift along the track by pulling with the hand control.
! Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.

7 Emergency Stop

The lift unit has an Emergency Shut-off feature so that in an emergency the operator can completely disable power to the lift.

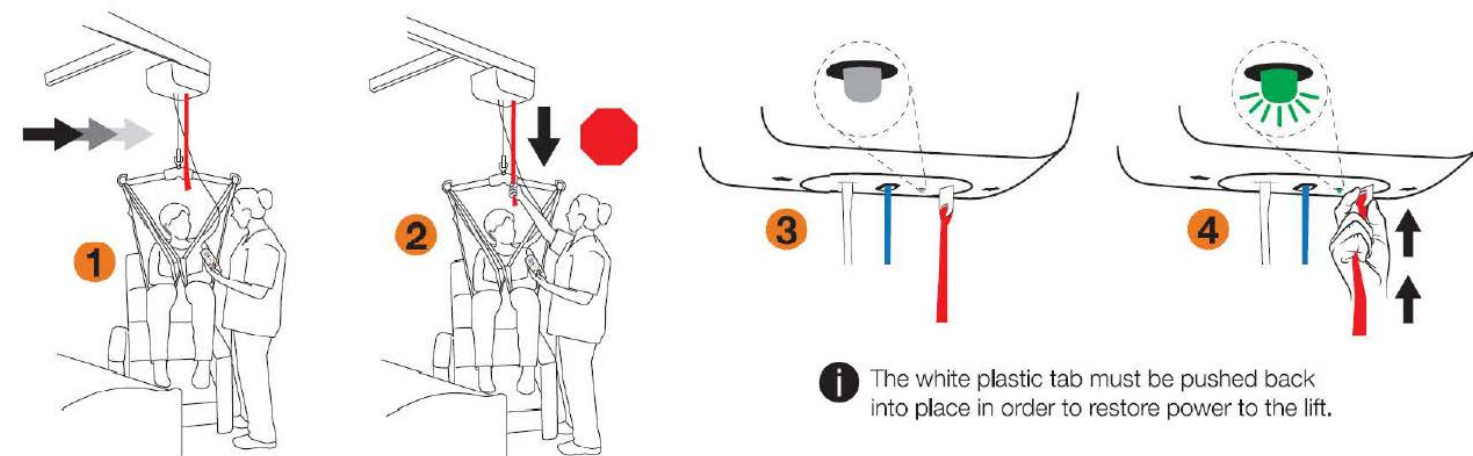

i The white plastic tab must be pushed back into place in order to restore power to the lift.

A-450 and A-625 Ceiling Lifts

The A-450/A-625 is a lifting aid used by health care professionals and caregivers in homes, institutions, and long term care facilities to lift, position and transfer clients or a disabled family member. Functions such as lifting up or down or moving left or right are controlled by pressing buttons on the digital hand control attached to the lift.

! Before using the A-450/A-625 lift system, check the lift, track, and sling for any unusual wear or damage. Refer to the owner's manual for each piece of additional equipment to determine what should be checked. If anything looks unusual, contact your local Handicare dealer before use. Failure to check the lift could result in damage to the lift or serious injury to the operator or the individual being lifted.

! Failure to comply with warnings in this Quick Start Guide may result in injury to the operator or the individual being lifted or transferred, as well as damage to the lift and related components. If you have any questions after reviewing this guide, contact your local authorized Handicare dealer.

! Prior to attempting these steps, review the instructions for all parts and accessories; Contact your local authorized Handicare dealer if you have any questions or concerns.

HANDICARE USA, INC.
10888 Metro Court
St. Louis, MO 63043 USA
Tel: 1-866-891-6502
www.handicare.com

HANDICARE CANADA
81 Romina Drive
Concord, ON L4K 4Z9 Canada
Tel: 1-905-850-0093
www.handicare.ca