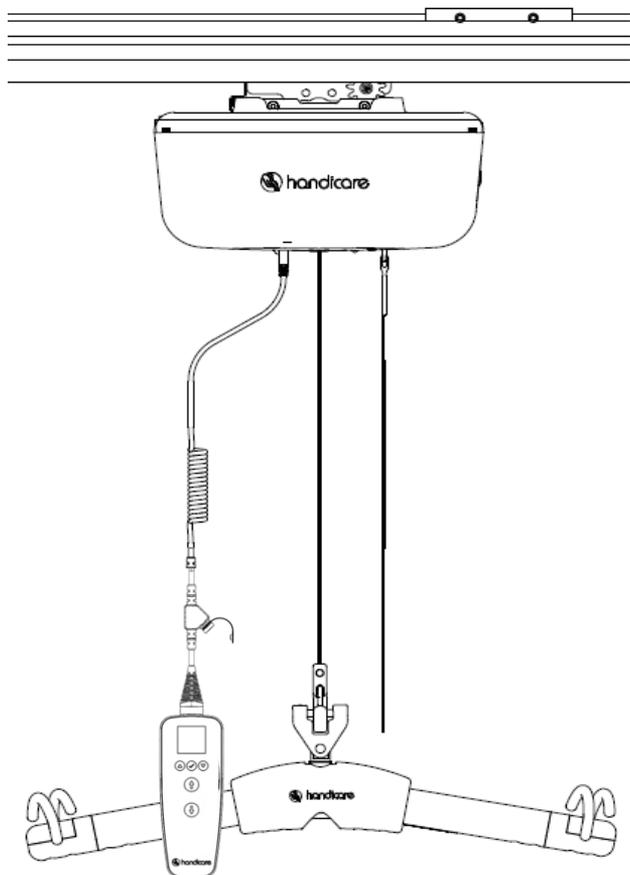
A-450/A-625

Handicare

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---

# Contents

Cautions.....	3
Use and Care .....	4
Overview of A-450/A-625 Lift System .....	4
Turn On the Lift .....	7
Turn Off the Lift.....	7
Hand Control Operation.....	8
Raise or Lower the Carry Bar.....	8
Move the Lift on the Track.....	9
Transfer an Individual .....	9
Hand Control Information and Diagnostics .....	13
Emergency Stop .....	13
Emergency Manual Lowering .....	14
Clean and Disinfect the Lift System.....	15
Optional Features.....	15
Contraindications .....	22
Troubleshooting.....	23
General Inspection and Preventive Maintenance .....	25
Recurrent Performance and Safety Testing.....	26
A-450/A-625 Lift Specifications .....	28
Hand Control Icons.....	30
IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines.....	34
Warranty .....	38
About Handicare .....	38
A-Series .....	38

## Cautions

- The A-450/A-625 **must be installed** only by persons authorized by Handicare. The installation guide is available to and for use by authorized dealers only.
- Under no circumstance should the A-450/A-625 track, lift, sling, or the entire system be put in control of a person who has not been properly trained in the use and care of this equipment. Failure to adhere to this warning may result in serious injury to the operator, and/or the individual being lifted/transferred.
- The A-450/A-625 lift, and associated track and slings are not toys. Do not use it for unsafe practices. Do not allow children to play with the lift or any of its components.
- The manufacturer's warranty is void if persons unauthorized by Handicare perform work on the A-450/A-625 lift system.
- There are no user serviceable parts inside the cover. Do not remove cover screws, or open the lift unit, as this may result in serious injury and will VOID THE WARRANTY. Contact your local Handicare dealer or your local Handicare business development manager for service.
- In facilities where more than one operator will be responsible for using the A-450/A-625 lift and associated track and slings it is imperative that all such individuals be trained in its proper use. A training program should be established by the facility to acquaint new operators with this equipment. Contact your local Handicare dealer or your local Handicare business development manager for training assistance.
- Never expose the A-450/A-625 lift or hand control directly into water. Warranty does not cover any misuse or abuse of the lift system.
- To maintain optimum function, the A-450/A-625 should be inspected and maintained on a regular basis. See the section titled "General Inspection and Maintenance" See page 22.
- Any accessories used with the A-450/A-625 including but not limited to track and slings, should be checked to ensure that they are in good working order. Check for signs of wear or fraying prior to each use. Report any unusual wear, or damage immediately to your local Handicare dealer or your local Handicare business development manager.
- The A-450/A-625 lift and associated accessories, track, and slings are intended only for lifting, repositioning and transferring of a person. Handicare will not be responsible for any damage caused by the misuse, neglect or purposeful destruction of the lift, or its associated components. Do not attempt to modify/ alter the A-450/A-625 lifts.
- Do not in any circumstance exceed the maximum allowable load of this lift. Refer to the "Specifications" section of this manual, or the labels on the lift.
- The installation of the lift, track, accessories, and sling are certified to a maximum load. Do not exceed the maximum rated load of any of the components.
- There is a risk of explosion if the lift is used in the presence of flammable anesthetics.
- Ensure that a clear space is maintained around the lift and track. Move all curtain material and other obstacles out of the way before performing a transfer.
- The charger must be located outside the patient vicinity at all times. The patient vicinity is the space with surfaces likely contacted by the patient or an attendant who can touch the patient.
- A-450/A-625 lifts should be decommissioned/disposed of after recommended 10 year service life in accordance with regional component specific disposal recommendations.

# Use and Care

## Introduction

To ensure safe operation of your A-450/A-625 lift, before using this equipment, carefully read this entire manual, especially the section on "Cautions". The A-450 and A-625 are designed to be used in conjunction with Handicare/Prism Medical lift track, accessories, and slings; Refer to the user guides supplied with these components before using them and refer to them while reviewing this manual.

Be sure you understand the contents of this manual completely before using this equipment.

**Failure to comply with warnings in this manual or proper lifting instructions may result in injury to the operator or the individual being lifted or transferred, as well as damage to the lift and related components.**

If you have any questions after reviewing this manual, Contact your local Handicare dealer or your local Handicare business development manager.

Store this manual with the documents included with the lift system, sling, and all related components. Contents of this manual are subject to change without prior written notice.

## Overview of A-450/A-625 Lift System

The A-450/A-625 is a lifting aid used by health care professionals and caregivers in homes, institutions, and long-term care facilities to lift, position and transfer clients or a disabled family member.

The A-450/A-625 is one part of a ceiling lift system that lifts an individual directly from above, making it possible to transfer and reposition mobility impaired individuals safely, with dignity, and in comfort while minimizing both strain and risk to the caregiver. The ceiling lift also does not take up valuable floor space as most traditional methods do.

The A-450/A-625 lift is one of the three major parts that make up a ceiling lift system:

- **Lift** - Motorized unit that raises and lowers an individual directly from above
- **Track** - Custom tailored and securely mounted to a ceiling structure, wall or floor provides the path for the lift to traverse
- **Sling** - Specially designed fabric accessory that attaches to the lift's carry bar to hold and support an individual for positioning or transfer

Refer to any user guides supplied with the accessories and reference them while reviewing this lift guide.

## A-450 and A-625 Ceiling Lifts

The A-450/A-625 is designed to raise and transfer an individual along a track from one surface to another surface. The lift remains on the lift track when in use or charging.

The A-450/A-625 ceiling lift can be either a manual or powered model. With the manual version, the caregiver effortlessly moves the individual horizontally along the track. The powered version has a secondary motor that moves the individually horizontally.

Functions such as lifting up or down or moving left or right are controlled by pressing buttons on a digital hand control attached to the lift.

## Lift System

The following parts are included with your new A-450/A-625 lift system:

- A-450/A-625 lift (with Integrated Scale and Wi-Fi or without Integrated Scale and Wi-Fi)
- Digital Hand control
- Lift Charger to be mounted on the wall or ceiling at the end of the track
- Owner's Manual

Additional parts may include:

- **Slings** - If a sling has been supplied with the lift, refer to the instructions included with the sling.
- **Accessories** - If additional accessories such as a turntable or transition gate system have been supplied with the lift, refer to the instructions included with those items.

Figure 1 illustrates sample floor plans for an installed lift system.


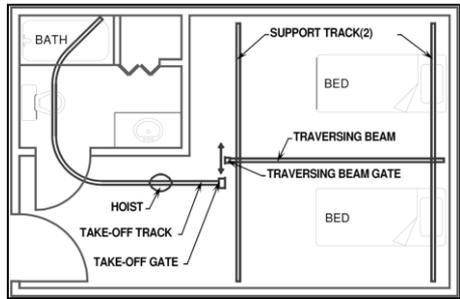
Figure 1: Installed Lift System Sample Plans

## Charge the Lift

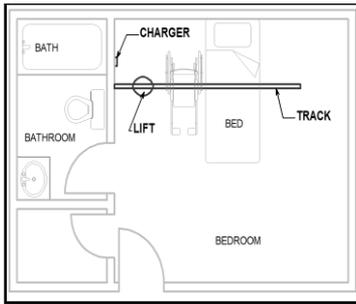Before initial use, the lift unit must be charged for 5 hours.

Charge the batteries regularly. Handicare recommends the lift be left on charge when not in operation, and at the end of each day, which will maximize the life cycle of the batteries.

The lift may remain connected to the charger indefinitely since the charger has a built-in regulator to eliminate the danger of overcharging.

The ceiling lift charges in one of three ways, depending on its configuration:

- **End-Point Charging** – The rail features a charging point at one end, and the lift must be moved to that point in order to charge. Please refer to A-450/ A-625 Lift Specifications for appropriate charger options.
- **Constant Charging (Optional)** – Charging strips run along the inside of the rail and the lift charges automatically, at any place on the rail, whenever it is low. When the Constant Charger is charging, the lift indicator lights solid orange to indicate that the batteries are charging; once charging is completed, it lights green. The Constant Charger can be attached to all A-450/A-625 lifts except those equipped with the Return to Charge feature (page 21). The batteries can remain connected to the charger indefinitely, since the charger has a built-in regulator to eliminate danger from overcharging. Please refer to A-450/ A-625 Lift Specifications for appropriate charger options.
- **Hand Control Charging** – Using an optional barrel charger, the lift can charge through the hand control cable. While charging, the lift indicator lights orange; it lights green once the lift is completely charged. The lift can be charged while turned on or off by the Emergency Down Strap/Switch. See below for the procedure for charging the lift using the hand control.

Figure 2: Charger with Barrel Connector

To charge A-450/A-625 lifts through the hand control:

1. Ensure your lift hand control has a charger input connector and you have an A-450/A-625 charger with barrel connector as shown in Figure 2.
2. Pull out the protective cap from the hand control connector.
3. Align the charger's barrel connector with the opening on the hand control's charger connection point.
4. Insert the charger's barrel connector into the hand control. Once the charger is connected properly, the lift indicator lights orange and turns green when the lift is completely charged. The lift can be charged whether the lift is turned on or off.

**The lift will not raise or lower when the hand control charger is connected; Emergency lowering still functions.**

**Do not position the lift where it is difficult to disconnect the charger in the event of an emergency. The unit can be isolated electrically from "Mains" by unplugging the charger.**

## Low Battery

If the batteries of the lift are low (25-30%) and require charging, the indicator located on the underside of the lift lights orange, a beep audible alarm sounds when button is pressed, and the hand control displays the message **Low Battery**. If a transfer is in progress, complete the transfer (the Emergency Down and X-Y Traversing also continue to operate) and then move lift to the end of the track where the charger is located.

- The lift may be used after one hour of charging from low battery; however, Handicare recommends overnight charging or minimum of 5 hours.

If the batteries are fully discharged and require charging, the indicator on the lift control panel lights red a beep audible alarm sounds when button is pressed, and the hand control displays the message **Very low battery**.

**When the battery is discharged the UP function is disabled. The Down, Emergency Down, and X-Y Traversing, continue to operate.**

**Power supply is considered a part of the A-625/A-450 system. Use only the charger that was supplied with the lift. Use of any other charger will void all warranties and may cause damage to the lift.**

**Do not position the lift where it is difficult to disconnect the charger in an emergency.**

**The unit can be isolated electrically from "Mains" by unplugging the charger.**

## Plug Types

Plug types vary by geographical location. Refer to local electrical guidelines on the type of plug or voltage which can be used (connection to the charger is an IEC 60320-C7 plug).

## Attach the Hand Control to the Lift

- Should you have any concerns or questions contact your local Handicare dealer or your local Handicare business development manager.
- Should the hand control disconnect from the underside of the lift, contact your local Handicare dealer or your local Handicare business development manager.
- Do not pull the lift along the track by the hand control cord.
- Be aware of any surrounding objects which might become entangled with the hand control cord.
- Keep the hand control cord away from any patient who might wrap the cord around themselves.

**Misuse of the hand control cord can lead to strangulation**

## Test the Hand Control

Perform a brief test to ensure proper connectivity:

1. Turn on the lift as shown on page 7.
  2. Press any button on the hand control. The hand control screen lights up.
  3. Raise and lower the carry bar using the Up button  and Down button .
    - If the carry bar moves up and down, the hand control is properly connected.
    - If the carry bar does not move correctly, double-check
      - That the lift is on
      - That the hand control connector is full connected
      - That the hand control does not display **Slack Detected**
- If there are still problems with the lift, contact your local Handicare authorized dealer for service.

## Change the Hand Control Hook

If you want to use a C450/C625 carry bar (360760) with your A-450/A-625 lift, Handicare recommends you install an A-series hand control hook (635669) on the hand control for a more secure fit.

## Turn On the Lift

**Before using the A-450/A-625 lift system, check the lift, track, and sling for any unusual wear or damage. Refer to the owner's manual for each piece of additional equipment to determine what should be checked.**

**If anything looks unusual, contact your local Handicare dealer or your local Handicare business development manager before use. Failure to check the lift could result in damage to the lift or serious injury to the operator or the individual being lifted.**

1. Press any button on the hand control to turn the lift on. The indicator on the underside of the lift lights green and the display screen on hand control displays **[Ready]**.
  - If the lift batteries require charging, the indicator on the lift's underside lights orange, a beep audible alarm sounds when the button is pressed, and the hand control displays the message **[Low Battery]**.
  - If the lift batteries are discharged to 25-30% of capacity, the hand control displays the message **[Low Battery]** and the lift continues to function until the battery status reaches 10-15%.
  - If the lift batteries are discharged to 10-15% of capacity, the hand control displays the message **[Very low battery]**, the indicator on the lift's underside lights red, and a beep audible alarm sounds when the button is pressed. The lift will not raise, but will lower and the emergency lowering will still function.

## Turn Off the Lift

The lift automatically shuts off after approximately 2 minutes of non-use.

**In an emergency, the lift can also be shut off by pulling the Emergency Down strap. See "Emergency Stop" (page 13) for details on Emergency Stop and restarting the lift after an Emergency Stop.**

# Hand Control Operation

The A-450/A-625 lift hand control

- Controls lift motion
- Presents active lift status display
- Provides usage data and diagnostics.
- Figure 3 shows the hand controls for Manual Traverse, Power Traverse, and Power Traverse XY lift systems.

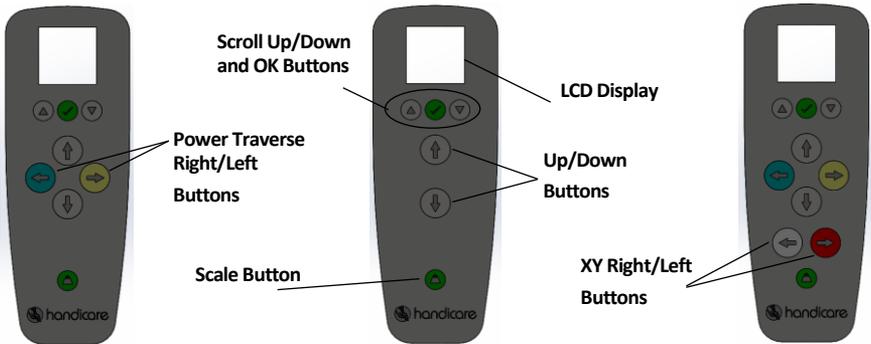
Figure 3: A-450/A-625 Hand Controls for Power Traverse, Manual Traverse, and Power Traverse XY Lift Systems (Left to Right)

## Hand Control Display Functionality

The A-450/A-625 lift hand control works in three modes:

- **Control Mode** to display command icons and active lift status display
- **Weighing Mode** to display user weight and scale control functions (for lifts equipped with a load cell)
- **Diagnostic/Information Mode** to display lift usage and diagnostic information

See “Hand Control” on page 30 for details on the display information and what it means.

## Raise or Lower the Carry Bar

1. Hold the hand control in one hand.
2. Hold the carry bar in the other hand so that it will not accidentally sway knock into an individual or nearby object.
  - To raise the carry bar, press the Up button .
  - To lower the carry bar, press the Down button .

## Auto Raise Function

All A-series lifts are equipped with Auto Raise Function (ARF) to automatically raise the lift’s carry bar to a set height, allowing the user to quickly and conveniently move the carry bar away from the patient after use. Auto Raise has four carry bar height settings

- **Raise 6 inch** – Raises carry bar to 6 inches from the top limit
- **Raise 12 inch** – Raises carry bar to 12 inches from the top limit
- **Raise 18 inch** – Raises carry bar to 18 inches from the top limit
- **Raise 24 inch** – Raises carry bar to 24 inches from the top limit

Contact your Service Technician to change these settings.

To activate Auto Raise,

1. Simultaneously press and hold the Up button  and Down button  for 3–5 seconds. The unit beeps and the carry bar raises automatically to the programmed height.
  - If you inadvertently enable Auto Raise while a patient is in the lift, the lift senses excess load and stops automatically.
    - The overload threshold is approximately 10 lbs (4.5 kg).
  - If during Auto Raise the carry bar accidentally gets tangled with obstacles such as furniture or drapery, the lift automatically detects the change in the load, and if more than 10 lbs (4.5 kg), the lift stops automatically.
2. To interrupt ARF manually,
  - Press any button on the hand control.  
or
  - Grab the carry bar and hold it firmly.

**A-450/625 lifts are shipped with the Auto Raise function off; contact your Handicare authorized dealer to set up the Auto Raise Feature. Always ensure the carry bar is attached to the lift before activating Auto Raise. The weight of the carry bar is required to ensure correct function.**

## Move the Lift on the Track

The lift is normally parked at the charging station end of the track when not in use. Depending on the type of lift (manual traversing or power traversing), you can move the lift along the track to a position directly above the person to be lifted in one of two ways:

### Manual Traversing Lift

1. Lower the carry bar to a comfortable height so you can hold it easily.
2. Grab hold of the carry bar and move the lift along the track by gently pushing the carry bar or the individual in the sling.

### Power Traversing Lift

1. Use the blue and yellow directional hand control buttons ( and ) to move the lift.
  - The blue and yellow buttons correspond to the blue and yellow directional arrows on the underside of the lift; simply press the button matching the one on the lift for the direction you want it to move.

**Never drag the lift along the track by pulling with the hand control.**

**Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.**

## Transfer an Individual

The following steps are intended to generally illustrate the procedure of using the lift to transfer an individual from one location to another. Contact your local Handicare dealer or your local Handicare business development manager if you have any questions or concerns.

**Configurations vary by installation. Prior to attempting these steps, review the instructions for the sling and additional accessories you are using with the lift.**

To transfer an individual using the lift,

1. Complete the “Each Use” inspection on page 25 of the General Inspection and Preventive Maintenance instructions.
2. Move the lift close to the individual to be transferred.
  - Follow the procedures for up and down and moving along the track as described in “Raise or Lower the Carry Bar” on page 8 and “Move the Lift on the Track” on page 9.

**Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift.**

3. Prepare the individual being transferred with the appropriate sling.

- Refer to the instructions supplied with the sling.

**Prior to lifting, make sure that the sling is correctly fitted and adjusted around of the individual for maximum comfort and safety.**

4. Position the lift directly over the individual.

5. Lower the carry bar.

**Always check to ensure that the lift is correctly positioned directly above the person to be lifted. The lift straps may fray over time, if repeatedly positioned improperly.**

**Ensure that the carry bar has no cuts, dents, or sharp edges that may come in contact with the straps of the sling and cause damage to them. Report any concerns to your Handicare dealer or your local Handicare business development manager.**

6. Attach the sling loops to the carry bar.

- The straps on each side of the sling are generally attached to the corresponding side of the carry bar.

7. Double-check to ensure:

- The straps are properly attached to the carry bar.
- The individual being lifted is properly positioned in the sling prior to lifting.

**Prior to lifting an individual, ensure the hand control cord is free of the carry bar.**

**Prior to lifting an individual, make sure that the straps of the sling are securely placed on the hooks of the carry bar and that the straps will not come off. (Figure 4).**

**Figure 4: Sling Straps Properly and Improperly Placed**

8. On the hand control, press the Up button . The lift raises the individual.

**Make sure the individual is lifted to an appropriate height for a safe transfer that avoids any obstructions that may cause injury to the individual in the sling or damage to the lift.**

9. Move the individual along the track to the desired location.
  - *If using a manual traversing lift,*
    1. Lower the carry bar to a comfortable height so you can hold it easily.
    2. Grab hold of the carry bar and move the lift along the track by gently pushing the carry bar or the individual in the sling.
  - *If using a power traversing lift,*
    1. Use the blue and yellow directional hand control buttons (⬅ or ➡) to move the lift.
    2. The blue and yellow buttons correspond to the blue and yellow directional arrows on the underside of the lift; simply press the button matching the one on the lift for the direction you want it to move.
10. When the individual is at the desired location, press the Down button (⬇) to lower the individual to the correct height.
 

 **Before you remove the sling loops from the carry bar, check that the transferred individual is securely supported in the appropriate position.**
11. Press the Down button (⬇) to lower the carry bar so the sling straps can be easily removed from the carry bar.
 

 **Make sure the carry bar does not contact the individual in the sling.**
12. Remove the sling straps from the carry bar.
13. Press the Up button (⬆) to raise the carry bar to a safe height.
14. Use the blue and yellow directional buttons (⬅ or ➡) to move the lift out of the way, if equipped (i.e., away from the immediate area).
15. Remove the sling from the individual.
  - Store the sling in a safe place.
16. Press the Up button (⬆) to raise the carry bar to height where it is out of the way while not in use.
17. Charge the lift as described in “Charge the Lift” on page 5.

**WARNING: STRANGULATION HAZARD**

**On completion of Lift Cycle, ensure that the carry bar, strap, and hand control are raised up away from reach of children, toddlers, or anyone not intended to use the product.**


**Psychological, mental, and emotional state assessment is key to safe use of a ceiling lift; ensure that use and storage of the ceiling lift and accessories are safe for the patient’s mental condition.**

**While raising a ceiling lift system with or without patient, ensure that the patient, caregiver, other individuals, furniture, and other items are not at risk of becoming entangled with the moving, hand control cord, lift strap and carry bar.**

## Traversing Beam Transfer

If you are using a Handicare X-Y system with a traversing beam, be sure you understand the traversing beam and how it works in transferring an individual. As shown in Figure 5, the X-Y system features two parallel support tracks with one traversing beam mounted across them.

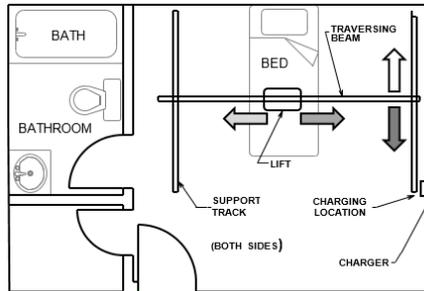**Figure 5: Handicare X-Y System Featuring Traversing Beam**

The lift can be moved along the traversing beam, and the traversing beam itself can be moved along the two parallel support tracks. Because of this, an X-Y system provides greater movement and positioning ability for an individual since the coverage area is everywhere between the support tracks as opposed to just the area beneath a section of track, as in a single-track installation.

- **If using a manual traversing beam**, move the beam along the support tracks manually to transport the beam, lift, and individual together.
  1. Lower the carry bar to a comfortable height so you can grab it easily.
  2. Grab hold of the carry bar and move the lift and traversing beam along the track by gently pushing the carry bar or the individual in the sling.
- **If using a power traversing beam**, move the beam along the support tracks by pressing either the red button or gray button on the hand control to transport the beam, lift, and individual together.
  1. Use the gray and red directional hand control buttons (⬅️ and ➡️) to move the traversing beam.
    - The gray and red buttons correspond to the gray and red directional arrows on the underside of the power traverse X-Y system lift; simply press the button matching the one on the lift for the direction you want it to move.
  2. Use the blue and yellow directional hand control buttons (⬆️ and ⬇️) to move the lift along the traversing beam.
    - The blue and yellow buttons correspond to the blue and yellow directional arrows on the underside of the traversing lift; simply press the button matching the one on the lift for the direction you want it to move.

**When moving the traversing beam, avoid any obstructions that may cause injury to the individual in the sling or damage to the lift or track.**

# Hand Control Information and Diagnostics

In addition to controlling the lift, the A-Series hand control is an interface that provides usage and diagnostics data from the lift.

To view data and setting through the hand control:

1. Press the  key of the hand control. The Menu screen displays.

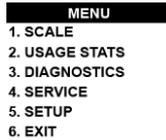
Figure 6: The Hand Control Menu Screen

2. Use the   keys to scroll through the menu options.
  - **The Scale menu** is for using and reviewing the lift's scale capabilities.
  - **The Usage Stats menu** is for reviewing usage information for the lift.
  - **The Diagnostics menu** is for service and technical support of the lift.
3. On the menu you want to review, press the  key. The selected menu displays.
  - If you select the wrong menu, at the bottom of the display:
    - Select **EXIT/GO BACK** and press the  key to return to the previous screen
    - or
    - Select **EXIT ALL** and press the  key to return to the main screen.

## Emergency Stop

The lift unit also has an Emergency Shut-off feature so that in an emergency the operator can completely cut the power to the lift.

To perform an Emergency Stop:

1. Pull down once on the red emergency lowering cord on the underside of the lift unit.
  - The lift immediately stops, beeps once, and all power to the lift shuts off
    - *To lower the patient after an emergency stop*, pull and hold down the red emergency lowering cord
  - All lift functions are disabled
  - The On indicator light and display turn off
  - The white plastic tab at the top of the red emergency lowering cord pops out

To turn the lift on after an Emergency Stop:

1. Have a Handicare authorized service dealer inspect your lift to determine the cause for needing to use the Emergency stop.
2. Use a finger to press the white plastic tab at the top of the red emergency lowering cord back into place.

**The white plastic tab *must* be pushed back into place in order to restore power to the lift.**

# Emergency Manual Lowering

Use the Emergency Manual Lowering system only if the lowering procedures described in “Raise or Lower the Carry Bar” (page 8) do not work. If you have any questions, contact your local authorized Handicare dealer or your local Handicare business development manager.

A proper safety ladder or stool may be required in order to remove the cover of the device. Use extreme caution if this is required. If you have any questions, contact your local Handicare dealer or your local Handicare business development manager.

**DO NOT** attempt to use the lift’s functions while using manual lowering.

To perform Emergency Manual Lowering:

1. Locate the 4 mm Allen key stored in the top cover.
2. Locate the small circular clip on the side of the lift (Figure 7).

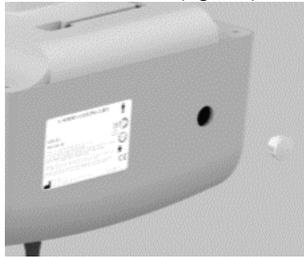
Figure 7: Circular Clip

3. Remove the clip to reveal the Allen key slot.
4. Insert the Allen key into the slot as shown in Figure 8.

Figure 8: Allen Key in Slot

5. Turn the Allen key clockwise to lower the lift strap.

**DO NOT USE THE LIFT AFTER THE MANUAL LOWERING MECHANISM HAS BEEN USED.** The lift must be reset by a qualified lift technician after manual lowering. Contact your local Handicare dealer or your local Handicare business development manager.

# Clean and Disinfect the Lift System

## Clean the Lift

To clean and disinfect the exterior of the lift:

1. Dampen a cloth with isopropyl alcohol.
2. Wipe down the entire exterior of the lift and carry bar.

**Use only isopropyl alcohol to clean and disinfect the exterior of the Lift system. No other chemicals or liquids should be used to clean and disinfect this lift!**

**Ensure that no liquids get inside the lift unit. This lift unit is not watertight or drip proof; Failure to protect the lift from liquids may result in damage to the lift and result in personal injury.**

## Clean the Anti-Microbial Strap

The A-450/A-625 is equipped with a strap featuring specialized antimicrobial coating at the fiber level. The anti-microbial coating provides excellent hygiene and material protection, including:

- Preventing microbial development
- Providing excellent protection against Methicillin-resistant Staphylococcus aureus (MRSA)
- Preventing mold stains and algae infestation
- Providing rot-proof performance

Disinfect the strap using an accelerated hydrogen peroxide disinfecting product.

## Clean the Lift Accessories

Follow the cleaning and disinfecting instructions included with the accessory.

## Optional Features

### Integrated Load Scale

#### Weigh an Individual

The A-450/A-625 can be purchased with an integrated load cell to conveniently weigh an individual while lifting or transferring.

To weigh an individual using the lift:

1. Attach a carry bar and sling to the lift strap.
2. Press any button on the hand control to turn it on. The Ready screen displays.
3. On the hand control, press the Weigh button.
4. Next, press the  key to open the scale menu
5. Use the   buttons to scroll down and select the Tare option
6. The Tare/Zero screen (Figure 9) displays on the hand control.

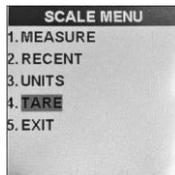
Figure 9: Tare/Zero Screen

7. Select **TARE/ZERO**.
8. Wait 2-3 seconds until the hand control screen displays 0.0 lb (0.0 kg).
9. Use the lift to lift the individual.
10. Once the individual is fully lifted, press the **Weigh** button. The hand control displays the individual's weight.

**To change the weighing unit of measure, see “Hand Control Information and Diagnostics” on page 13.**

The integrated scale in the A-series lift is **NOT** approved to determine mass in medical practice, including weighing patients in connection with health monitoring. Because of its highly sensitive sensors for measuring weight, avoid banging or crashing the lift into the track end stops.

## Quick Release Hook

A Quick Release Hook allows a carry bar to be quickly attached to and removed from a lift.

### Attach the QRS Carry Bar to the Quick Release Hook

1. Hold down the red latch and align the Quick Release Hook slot with the carry bar's Clevis pin.

Figure 10: Clevis Pin of Carry Bar through the Red Latch of the QRS Hook

2. Hook the Quick Release Hook onto the carry bar's Clevis pin.

Figure 11: Carry Bar Attached to Quick Release Hook

3. Make sure the latch is closed safely once the Clevis pin is sitting in the Quick Release Hook.

**If the latch does not close safely, DO NOT USE THE QUICK RELEASE HOOK and contact your local Handicare dealer or your local Handicare business development manager.**

### Detach the QRS Carry Bar from the Quick Release Hook

To detach the QRS (Quick Release System) Carry Bar from the Quick Release Hook:

1. Hold the hook and carry bar.
2. Use your finger to push the latch down and open it.

Figure 12: Open the Latch

3. Hold the latch down and tilt the hook about 90°.

**Figure 13: Tilt the Hook**

4. Remove the carry bar from the quick release hook.

**Figure 14: Remove the Hook**

### **Install a Quick Release Hook**

See the Carry Bar Connector Installation instructions included with the carry bar.

## Wi-Fi

The A-450/A-625 can be purchased with an optional Wi-Fi diagnostic and usage module.

### Connect to the Lift via Wi-Fi

To connect Wi-Fi-enabled lift to a Wi-Fi enabled PC, Laptop, Tablet, or Cellphone via Wi-Fi:

1. Turn on the lift.
  - The lift must be On; Wi-Fi doesn't work if the lift is on Standby.
2. Press any button on the hand control to turn on the lift.
3. Power on the device you are connecting to the Lift system.
  - If using a laptop, open the network connecting wizard.
  - If using a smartphone or tablet, open the Wi-Fi menu.
4. Select the connection for the lift **handicare\_XXXXXXXXXXXXXXXX**, where **XXXXXXXXXXXXXXXX** is the **fifteen digit serial number** of the lift as per the product label. (Figure 15).

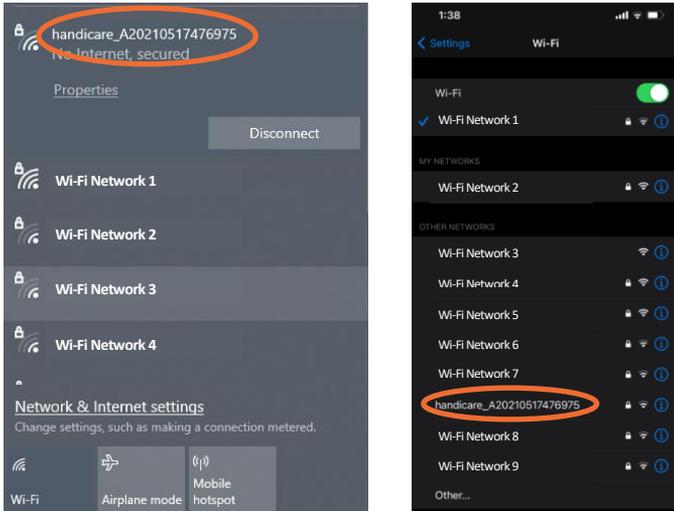
Figure 15: WiFi Networks Window

5. Enter **0987612345** as the WiFi network *password*.
6. Depending on the operating system of the connecting device, a **No Internet Access** message may display. If one displays select **Yes**.
7. Open the Internet Browser of your device (Microsoft Edge, Google Chrome, Mozilla Firefox, Safari).
8. In the navigation bar, enter **192.168.4.1**

9. Press [Enter]. The Handicare Wi-Fi main menu screen displays (

Figure 16).

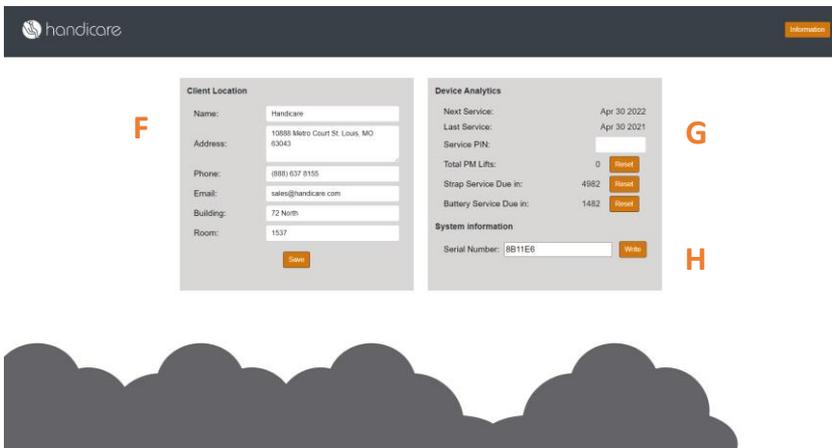
Figure 16: Wi-Fi Main Menu Screen

10. All relevant information is shown on the main menu screen.

- A. **System Information** – Displays unique lift information:
  - i. **Model** – Name of the Handicare lift.
  - ii. **Serial Number** – The unique identifier assigned by Handicare to each A-Series lift.
  - iii. **Date of Manufacturing (DOM)** – The date the lift was manufactured
- B. **Client Location** – Displays customer information which can be edited:
  - i. **Name** – The customer’s company/site/location
  - ii. **Address** – The customers’s address
  - iii. **Phone** – The desired customer’s phone number
  - iv. **Email** – The desired customer email address
  - v. **Building** – The customer building/area
  - vi. **Room** – The customer room
- C. **Device Status** – Displays live A-Series status and information while connected:
  - i. **System State** – The current status of the lift
  - ii. **Charger Status** – Charging status (fully charged, charging, not charging)

- iii. **Battery Voltage** – The live voltage of the batteries and state of charge (low battery, very low battery, dead battery)
  - iv. **Motor Current** – The live current draw on the motor
  - v. **Scale Weight** – The live reading of the scale (if equipped with load cell option)
  - vi. **Case Temperature** – The live reading of the lift's cover temperature
  - vii. **Tilt Angle (Side/Length)** – The live reading of the angle of tilt the lift is experiencing
- D. **Device Analytics** – Displays lift statistics and usage parameters:
- i. **Next Service** – The next yearly Preventative Maintenance (PM) date
  - ii. **Last Service** – The most recently completed Preventative Maintenance (PM) date
  - iii. **Total PM Lifts** – The Preventative Maintenance lift counter
  - iv. **Strap Service Due in** – An **estimated** number of lift cycles remaining until strap service is due (Please refer to General Inspection and Preventative Maintenance on page 23. for safe patient handling maintenance)
  - v. **Battery Service Due in** – An **estimated** number of lift cycles remaining until battery service is due (Please refer to General Inspection and Preventative Maintenance on page 23. for safe patient handling maintenance)
  - vi. **Total Lifts** – The Total number of lifts the A-Series has completed since being installed
- E. **Cycle History** – Shows the **20** most recent lift cycles alongside the lift type, scale reading, and timestamp.
- i. **Lift Type** – Defines the type of lift performed (normal, heavy, overloaded)
  - ii. **Weight** – The scale reading during a lift (if equipped with load cell option)
  - iii. **Date & Time** – The date and time that corresponds with when the lift was performed
11. To edit customer/client information Select **Configuration**. The Configuration screen (Figure 17: Configuration Screen) displays, showing:

**Figure 17: Configuration Screen**

- F. **Client Location** – Edit customer information. Press Save to update customer information and return to Main Screen.
  - G. **Device Analytics** – For Maintenance or Service, please refer to the product technical manual. Resetting maintenance alarms and updating service dates requires a SERVICE PIN
  - H. **System Information** – For Handicare Manufacturing use ONLY. Requires a MANUFACTURING PIN
12. To Export data from the lift Select **Export** as show in Figure 18: Export Button

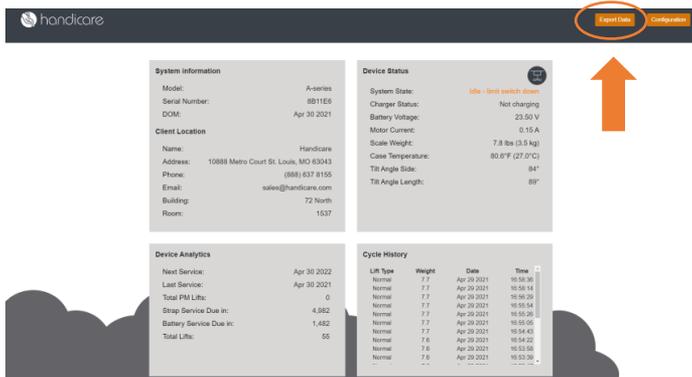
Figure 18: Export Button

## Return to Charge

If your power traversing lift has a return-to-charge feature (RTC), press and hold the blue and yellow colored directional hand control buttons simultaneously for 3-5 seconds until the lift beeps. The lift automatically retracts the carry bar and drives along the track until it docks at the charger.

The weight sensitive adaptive return-to-charge feature offers added protection during Return to Charge movement. Significant features of the Load Sensitive RTC include:

- If the user/operator inadvertently enables Return to Charge while a patient is in the lift, the Return to Charge function stops automatically by sensing excess load on the lift.
- The overload threshold is approximately 10 lbs (4.5 kg) when equipped with a load cell. Without a load cell, the lift has a 2 Amp overcurrent threshold.
- During Return to Charge, if the lift's carry bar accidentally gets tangled with obstacles such as furniture or drapery, the lift automatically detects the change in the load.
- *If the load varies more than 15-30 lb (6.8-13.6 kg), lift stops automatically.*

The Return to Charge feature can be interrupted manually either by pressing any button on the hand control or by grasping the carry bar firmly.

The Return to Charge feature does not work with power traversing lifts outfitted with the Constant Charge (Omni) option.

- Return to Charge features several settings that can be customized:

- **RTC Max. Time** - The maximum time allowed for the lift to travel to and dock at the charger before aborting; may be set in 60 second increments from 60 to 240 seconds
  - **RTC Drop Time** - The time the lift devotes to lowering the carry bar upon docking at the charger; may be set in 3 sec increments from 9 to 24 seconds
  - **RTC Speed** - The relative speed the lift travels at to reach the charger; may be set to 2, 4, or 8
- Contact your Service Technician to change these settings.

**Make sure the carry bar is attached to the lift before activating Return to Charge. The added weight of the carry bar is required to ensure correct working of limit switches, and failure to have carry bar attached can result in a limit switch error (see "Troubleshooting" on page 23).**

**Avoid any obstructions that may cause injury to the individual in the sling or damage to the lift or track.**

# Contraindications

**There are no known “contraindications” associated with the use of A-450/A-625 lifts and their accessories, provided they are used according to manufacturer’s recommendations and guidelines.**

# Troubleshooting

If you encounter problems using the A-450/A-625, review the Troubleshooting chart.

## Troubleshooting Chart

Use the Fault column to locate the fault you've encountered, then complete the recommended solution in the adjacent column on the row.

If you cannot find the fault or the recommended solution does not correct the problem, immediately contact your local Handicare authorized dealer for service.

Fault	Recommended Solution
The hand control of the lift has become disengaged.	Refer to "Attach the Hand Control to the Lift" on page 6. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The hand control LCD does not Turn On even after pressing hand control buttons.	Ensure that hand control is properly connected (see Attach the Hand Control to the Lift") Ensure the white plastic tab located at the end of the red emergency lowering cord is fully pressed into the lift unit. Ensure the lift is charged; refer "Charge the Lift" on page 5 If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The hand control LCD turns on after pressing hand control buttons but lift does not move.	Ensure the hand control charger is not connected to the lift—the lift will not move when hand control charger is connected. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The carry bar/accessories attached to the lift does not operate up or down even when the hand control is properly connected and hand control LCD is on.	The indicator light on the control panel located on the underside of the lift should be green. Press the On button or Up/Down arrow buttons or any colored button on the hand control. This should activate the lift and the indicator light turn green. If the lift still does not function, the batteries may be low and require charging. Refer to "Charge the Lift" on page 5 and charge the lift for at least one hour, then try to raise or lower the carry bar. If the emergency lowering has been used, the Up and Down functions will not operate. DO NOT use the lift. Contact your local authorized dealer immediately so that the lift can be checked to ensure proper continued operation.
The green light on the underside of the lift is on and hand control LCD is on but the lift does not operate in the DOWN direction.	There is a built-in slack tape detector in the lift. This may be sensitive. Apply weight to the carry bar while pressing the Down button. If this corrects the problem temporarily but not permanently, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The red indicator light on the underside lift turns red or a loud alarm sound is heard when an individual is raised.	The batteries are low and require charging. Refer to "Charge the Lift" on page 5. Charge the lift for at least one hour and then try to raise or lower the carry bar. If this does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.

Fault	Recommended Solution
One side of the lift strap is starting to fray after continued use.	Discontinue using immediately and replace strap. Check to be sure that the lift is always directly above the individual being lifted, especially with power traversing lifts. Refer to "Transfer an Individual" on page 9 for correct lift positioning. If fraying still continues, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
The lift does not pass through a track component such as a turntable or transition gate.	Refer to the Owner's Manual for the specific piece of equipment in question. If the recommended solution does not correct the problem, contact your local authorized dealer immediately to check the lift and ensure proper continued operation.
No Power	Ensure the Emergency Lowering tab has not come out. If it has, simply press the tab back into the lift and check battery for charge.
Lift starts beeping while pulling Emergency Down Strap/Switch with strap not coming down.	The A-series lift is equipped with dual sensors for strap tension and position, the Emergency Down Strap/Switch system only works with load on the strap and Emergency Down Strap/Switch motion stop immediately after the lift strap is loose.
The user weight readings fluctuate too much while weighing a patient on a load cell equipped lift.	Ensure the user is stable and there is no swinging movement while weighing a patient. If problem persists, contact your local authorized dealer immediately so that the lift can be checked to ensure proper continued operation.
The user weight readings are not accurate while weighing a patient on a load cell equipped lift.	Ensure the user is stable and there is no swinging movement while weighing a patient. Also ensure the scale is properly tared/zeroed as described in "Weigh an Individual" on page 15. If the problem persists, contact your local authorized dealer immediately so that the lift can be checked to ensure proper continued operation.
The lift is recommending preventive maintenance.	Complete the "Semi-Annually" maintenance procedure on page 25 prior to resetting the lift counter. Instructions to reset the lift counter are detailed in the A-Series Technical Manual. Counter can only be reset by an authorized Handicare dealer.

# General Inspection and Preventive Maintenance

Do not perform any inspection or maintenance while the lift is in use with a patient.

## Each Use

*(To be completed by User)*

Prior to each use the A-450/A-625 lift and associated track, accessories, and slings must be visually inspected. Refer to the accessory and sling user guides for specific details regarding their inspection.

Should any of these items fail the inspection do not use the lift. Contact your local authorized dealer for service.

Visually check for the following:

- ✓ The lift strap shows NO signs of fraying or breaking along its entire length.
- ✓ The stitching on the lift strap where it connects to the carry bar shows no signs of fraying or breaking.
- ✓ If the lift strap wear indicators are grey, replace the lift strap regardless of number of cycles.
- ✓ The sling that will be used show no signs of unusual wear and tear. The straps of the sling that connect to the carry bar of the lift show no signs of fraying or breaking. Refer to specific sling instructions.
- ✓ The hand control cable is not kinked, twisted, knotted, disconnected, or damaged.
- ✓ All the functions on the hand control work correctly (e.g. Up/Down/Left/Right, etc.).
- ✓ There are no cuts, dents or sharp edges on the carry bar that may damage the sling straps.
- ✓ The lift has no unusual sounds when the carry bar is moved up or down or the lift is moved left or right.
- ✓ Ensure that end stops are installed at each end of the track.

## Monthly

*(To be completed by User)*

Monthly, the A-450/A-625 lift and associated track, accessories, and slings, must be inspected.

Should any of these items fail the inspection do not use the lift. Contact your local authorized dealer for service.

- ✓ Complete the visual inspection as noted in the “Each Use” section above.
- ✓ *With no one in the sling nor attached to the lift* check that the lift moves freely along the entire length of the track.

## Semi-Annually

*(To be completed by User)*

In high frequency transfer areas (more than 1500 lifts per year) or situations where heavier than normal clients are regularly lifted, maintenance should also be completed every 6 months. It is recommended that this service history be documented.

Should any of these items fail the inspection, do not use the lift. Contact your local authorized dealer for service.

Check for the following:

- ✓ Check emergency stopping and emergency lowering functions using the red emergency cord.
- ✓ Inspect carry bar for damage; verify insert and hooks. Ensure there are NO cuts, dents, or sharp edges that may damage the strap of the unit or sling.
- ✓ Lower the lift strap all the way out of the unit; verify there are no signs of fraying or breaking along its entire length.
- ✓ *(C-Series Carry Bars only)* Remove the plug from the carry bar and inspect the integrity of the strap around the pin.
- ✓ The stitching on the lift strap where it connects to the carry bar shows NO signs of fraying or breaking.

- ✓ Verify all functions on the hand control work correctly (Up, Down, Traversing, Return to Charge, etc.).
- ✓ The lift makes no unusual sounds when the carry bar is moved up or down or the lift is moved left or right.
- ✓ Press and hold the Up button until the carry bar is at the top. The motor should stop automatically.
- ✓ Using the hand control, let the strap out all the way to the ground. The motor should stop before the strap begins to wind backwards.
- ✓ While using either the up or down buttons, angle the strap more than 15 degrees. The motor should stop automatically.
- ✓ Engage the unit with charger and check that the unit is charging properly. The indicator on the lift should light orange.
- ✓ Check that the lift's LED light lights green when on, dark when off, and orange when charging
- ✓ Check for cracks or wear in the case.

**If the audible/visual preventive maintenance alert sounds, complete the Preventive Maintenance Form before resetting the lift counter. Instructions to reset the lift counter are detailed in the A-Series Technical Manual. Counter can only be reset by an authorized Handicare dealer.**

## Annually

*(To be completed by a lift technician)*

**This section to be only completed by a qualified service technician as authorized by Handicare.**

Annually, the A-450/A-625 system must be inspected. Be sure to documents this service history as directed by the A-series Tech Manual.

- ✓ Complete the inspection as noted in the “Monthly” section above.

## Recurrent Performance and Safety Testing

The following table highlights A-450/A-625's Essential Performance Parameters, with their subsequent testing method and frequency. If your A-450/A-625 does not meet the following criteria, contact your local Handicare authorized dealer for service.

Performance Criteria	Recommended Check	Frequency
The A-450/A-625 should be capable of lifting 450 lb / 204.5 kg (for A-450) or 625 lb / 284 kg (for A-625)	Load test at full load (625 lb / 284 kg for A-625 and 450 lb / 204.5 kg for A-450).	Annual
The A-450/A-625 will operate as intended throughout its service life, provided all recommended maintenance and usage recommendations are followed.	Load test at full load (625 lb / 284 kg for A-625 and 450 lb / 204.5 kg for A-450) Pre-lifting inspection by user caregiver. Preventive maintenance by trained tech.	Annual. Every use Semi-annually or annually, per usage frequency. See “ General Inspection and Preventive Maintenance” on page 25.

Performance Criteria	Recommended Check	Frequency
<p>The A-450/A-625 should be capable of working in conjunction and compatibility with Handicare/Prism Track, trollies and accessories (Carry bars and Slings).</p>	<p>Load test at full load (625 lb / 284 kg for A-625 and 450 lb / 204.5 kg for A-450).</p> <p>Pre-lifting inspection by user caregiver.</p> <p>Preventive maintenance by trained tech.</p>	<p>Annual</p> <p>Every use</p> <p>Semi-annually or annually, per usage frequency</p> <p>See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>
<p>The A-450/A-625 should have “Free Fall” protection system.</p>	<p>Preventive maintenance by trained tech.</p>	<p>Semi-annually or annually, per usage frequency. See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>
<p>The A-450/A-625 should have upper and lower limits.</p>	<p>Preventive maintenance by trained tech.</p>	<p>Semi-annually or annually, per usage frequency. See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>
<p>The A-450/A-625 strap should have a minimum safety factor of 6 in terms of break load.</p>	<p>Load test at full load (625 lb / 284 kg for A-625 and 450 lb / 204.5 kg for A-450).</p> <p>Pre-lifting inspection by user caregiver.</p> <p>Preventive maintenance by trained tech.</p>	<p>Annual</p> <p>Every use</p> <p>Semi-annually or annually, per usage frequency</p> <p>See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>
<p>The A-450/A-625 should have visual “Error” status display.</p>	<p>Preventive maintenance by trained tech.</p>	<p>Semi-annually or annually, per usage frequency.</p> <p>See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>
<p>The A-450/A-625 hand control locking mechanism used as connections should be “Self-Locking”.</p>	<p>Preventive maintenance by trained tech.</p>	<p>Semi-annually or annually, per usage frequency.</p> <p>See “</p> <p>General Inspection and Preventive Maintenance” on page 25.</p>

## Reference

### A-450/A-625 Lift Specifications

Lift Motor:	24 VDC
Traverse Motor:	24 VDC (Optional at time of Purchase)
XY Traverse Motor:	24 VDC (Optional at time of Purchase)
Charger:	Standard Model: Soneil PS2403SRM26 (End-Point Charger/ Hand Control Charger) Alternative Model: Soneil 2403SRM 20 (End-Point Charger only)
	Standard Model Input: 100-240 VAC 0.8 Amps, 50-60 Hz Alternative Model: 100-240 VAC 1.5 Amps, 50-60 Hz
	Standard Model Output: 24 VDC, 1.33 Amps Alternative Model Output: 24 VDC, 1 Amps
	Note: Only use the Soneil PS2403SRM26 and Soneil 2403SRM 20 charger with the A-450/A-625 Lift. Other chargers are not compatible to the A-450/A-625.
Batteries:	24 VDC (2 x 12 VDC) 5.0 AH, Sealed Lead Acid
Lift Case:	Flame Retardant ABS
Hand control:	Digital with IPx6 and IPx7 rating IPX6 = Protected against water jets IPX7 = Protected against temporary immersion (1m, 30 minutes)
Main Unit IP Rating	IP22 IP22 = Protected against access to hazardous part by finger, protected against vertically falling water drops up to 15 degrees
Lifting Range:	Up to 96" (2438mm)
Lift Weight:	21—23.5 lb (9.5-10.65 kg)
Maximum Safe Working Load:	450 lb / 204.5 kg (A-425) or 625 lb / 284 kg (A-625). Note: The maximum safe working load (SWL) of the installed lift is determined by referring to the product label located on the bottom of the lift.
Duty Cycle:	1 Min On—9 Mins Off
Max Sound Level:	Raising Max load 50 dB, Lowering Max load 50 dB on a lift in good working order.
Load Cell:	Relative Accuracy: +/- 0.1% of measured weight Absolute Accuracy: +/- 0.2% of measured weight
System Integrated Load Cell:	Capacity: 0—SWL Absolute Accuracy: +/- 0.6% of measured weight Display Resolution: 0.1 lb (0.05 kg)
Service Life:	22,500 cycles or 10 years, whichever comes earlier. As a precautionary measure, the lifting strap should be replaced every 5000 cycles, 3 years, or when the wear indicators of the lift strap are grey, whichever comes earlier. Higher usage lifts may necessitate more frequent replacement of the lifting strap. See “  General Inspection and Preventive Maintenance” on page 25.
Usage	Temperature: +5° to +40° C (41 to 104 °F)

Environmental Conditions:	Relative Humidity: 15 to 90% RH																																			
	Atmospheric Pressure: 70 to 106 hPa																																			
Storage Environmental Conditions	Temperature: -40° to +70° C (-40 to 158 °F)																																			
	Relative Humidity: 10 to 100% RH																																			
	Atmospheric Pressure: 500 to 1060 hPa																																			
Battery Charging Cycle Rated Performance:	161 lifts at 200 lb (91 kg), 20 secs "On"/10 secs "Off" duty cycle, lower lifting range with lift 6" (152 mm) from ground																																			
	90 lifts at 440 lb (200 kg), 20 secs "On"/10 secs "OFF" duty cycle, lower lifting range with lift 6" (152 mm) from ground																																			
	50-60 lifts at 450 lb (204 kg), 1 min "On"/9 min "OFF" duty cycle, each lift being 24" (610 mm) at the middle of the lifting range (from 54" (1370 mm) strap out to 30" (762 mm) strap out) per full battery)																																			
	30-40 lifts at 625 lb (284 kg), 1 min "On"/9 min "OFF" duty cycle, each lift being 24" (610 mm) at the middle of the lifting range (from 54" (1370 mm) strap out to 30" (762 mm) strap out) per full battery)																																			
	Note: The lift has a break in period; breaking in of the lift will need to be done before these numbers will be achieved. The breaking in period varies from lift to lift and depends on the frequency of use and the types of load being applied. Higher loads and a greater frequency of use break in the lift faster.																																			
Return to Charge (RTC) Speed:	Varies Based on Setting: #1 = 0.123 m/s (4.86 in/s) #2 = 0.137 m/s (5.4 in/s) #3 = 0.167 m/s (6.59 in/s) #4 = 0.192 m/s (7.57 in/s) #5 = 0.242 m/a (9.53"/s)																																			
Traverse Speed:	Varies Based on Setting: #1 = 0.123 m/s (4.86 in/s) #2 = 0.137 m/s (5.4 in/s) #3 = 0.165 m/s (6.51 in/s) #4 = 0.189 m/s (7.46 in/s) #5 = 0.244 m/a (9.59"/s)																																			
Lifting Speed:	<table border="1"> <thead> <tr> <th colspan="5">Lifting Speed, m/s (in/s)</th> </tr> <tr> <th>Setting</th> <th>Up with load</th> <th>Down with load</th> <th>Up without load</th> <th>Down without load</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>0.027 (1.06)</td> <td>0.045 (1.77)</td> <td>0.05 (1.97)</td> <td>0.052 (2.05)</td> </tr> <tr> <td>4</td> <td>0.0243 (0.957)</td> <td>0.0405 (1.59)</td> <td>0.045 (1.77)</td> <td>0.0468 (1.84)</td> </tr> <tr> <td>3</td> <td>0.0216 (0.850)</td> <td>0.036 (1.42)</td> <td>0.04 (1.57)</td> <td>0.0416 (1.64)</td> </tr> <tr> <td>2</td> <td>0.0189 (0.744)</td> <td>0.0315 (1.24)</td> <td>0.035 (1.38)</td> <td>0.0364 (1.43)</td> </tr> <tr> <td>1</td> <td>0.0162 (0.638)</td> <td>0.027 (1.06)</td> <td>0.03 (1.18)</td> <td>0.0312 (1.22)</td> </tr> </tbody> </table>	Lifting Speed, m/s (in/s)					Setting	Up with load	Down with load	Up without load	Down without load	5	0.027 (1.06)	0.045 (1.77)	0.05 (1.97)	0.052 (2.05)	4	0.0243 (0.957)	0.0405 (1.59)	0.045 (1.77)	0.0468 (1.84)	3	0.0216 (0.850)	0.036 (1.42)	0.04 (1.57)	0.0416 (1.64)	2	0.0189 (0.744)	0.0315 (1.24)	0.035 (1.38)	0.0364 (1.43)	1	0.0162 (0.638)	0.027 (1.06)	0.03 (1.18)	0.0312 (1.22)
Lifting Speed, m/s (in/s)																																				
Setting	Up with load	Down with load	Up without load	Down without load																																
5	0.027 (1.06)	0.045 (1.77)	0.05 (1.97)	0.052 (2.05)																																
4	0.0243 (0.957)	0.0405 (1.59)	0.045 (1.77)	0.0468 (1.84)																																
3	0.0216 (0.850)	0.036 (1.42)	0.04 (1.57)	0.0416 (1.64)																																
2	0.0189 (0.744)	0.0315 (1.24)	0.035 (1.38)	0.0364 (1.43)																																
1	0.0162 (0.638)	0.027 (1.06)	0.03 (1.18)	0.0312 (1.22)																																

# Hand Control Screen Displays

Display Image	Details	Comments
	Boot Up Icon	Lift is turning On by pressing any button of Hand Control.
	Up motion Icon	Lift is going up. Real-time battery percentage also shown.
	Down motion Icon	Lift is going down. Real-time battery percentage also shown.
	Slack/Loose strap Icon	Strap is slack/loose. Insufficient weight pulling down on the strap. Real-time battery percentage also shown.
	Lower Limit Icon	Lower limit reached. Strap is fully unrolled. Real-time battery percentage also shown.
	Upper Limit Icon	Upper limit reached. Strap can go no higher. <i>Real-time battery percentage also shown.</i>
	Battery Very Low Icon	Battery is critically low. <b>Unit will lower but not raise.</b> <i>Real-time battery percentage also shown.</i>
	Battery Low Icon	Battery is low. <b>Unit will still function.</b> <i>Real-time battery percentage also shown.</i>
	Overload Icon	Lift is overloaded / Too much weight on strap. <i>Real-time battery percentage also shown.</i>
	Service Due Icon	Lift has accumulated at least 1500 post PM cycles.
	Service Limit Exceeded	Lift has surpassed recommended cycles without servicing

## A-450/A-625 Lift Display Symbols

Symbol	Definition	Symbol	Definition
	Class II equipment		Type b applied part
	Pertinent Product Information.		Refer to instructions manual/booklet
	Certified by TUV		Off Power
	On Power		On/Off
	Body Weight		EMERGENCY DOWN
	Status LED's for battery level and system error indication		Authorized Representative in European Community
	Manufacturer as defined in EU directive 93/42/EEC		Refer to Instructions, Manual, Booklet
	Date of Manufacturing		Model/Catalogue Number
	WEEE wheeled bin		Product Serial Number
	General Warning Sign		European Conformity

# LED, Hand Control LCD, and Audio Symbols

Lift Status	Lift Status LED Color	Audio Signals	Hand Control LCD	Available Functionality		
				Up	Down	E-Down
Lift fully charged	Solid Green 	None		None		
Lift ready for operation	Solid Green 	None				
Charging hand control	Solid Orange 	None		None		
Low Battery	Solid Orange 	Beep on button press				
Very Low Battery	Solid Red 	Beep on button press		None		
Over Load	Blinking Orange 	Beep on button press		None		
System Fault	Blinking Orange 	None		None	None	
Lift has over 1,000 cycles	Solid Orange 	None				
Lift has over 1,500 cycles	Blinking Red 	Beep on button press				

## Carry Bars

Several carry bars are available for the A-450/A-625 lift.

All of the following carry bars have a 625 lb / 284 kg SWL (Safe Working Load) and QRS (Quick Release System) with the bull horn connection as shown below.

**Note:** The C450/C625 carry bar (360760) is compatible with the A-450/A-625 lift but requires a different control hook (635669).

Part Number	Length between Slings
360931	525mm/20.7in

All of the following carry bars have a 625 lb / 284 kg SWL with bull horn connection and attaches directly to the strap as shown below.

Part Number	Length between Slings
360935	525mm/20.7in

All of the following carry bars have a 625 lb / 284 kg SWL and QRS with spring latch connection as shown below.

Part Number	Length between Slings
360982	461mm/18.2in
360981	550mm/21.6in
360980	690mm/27.2in

All of the following carry bars have a 625 lb / 284 kg SWL with spring latch connection and attaches directly to the strap as shown below .

Part Number	Length between Slings
360986	461mm/18.2in
360985	550mm/21.6in
360984	690mm/27.2in

# IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

*The table below elaborates the recommended electromagnetic working environment.*

**Table C.2 – Example Immunity Test Level Compliance**

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV Contact ±15 kV Air	±8 kV Contact ±15 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV line to line & ±0.5 kV, ±1 kV, ±2 kV line to ground	±0.5 kV, ±1 kV line to line & ±0.5 kV, ±1 kV, ±2 kV line to ground	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines  IEC 61000-4-11	0% $U_T$ (100% dip in $U_T$ ) for 0,5 cycle  0% $U_T$ (100% dip in $U_T$ ) for 1 cycles  70% $U_T$ (30% dip in $U_T$ ) for 25/30 cycles  0% $U_T$ (100% dip in $U_T$ ) for 5 sec	0% $U_T$ (100% dip in $U_T$ ) for 0,5 cycle  0% $U_T$ (100% dip in $U_T$ ) for 1 cycles  70% $U_T$ (30% dip in $U_T$ ) for 25/30 cycles  0% $U_T$ (100% dip in $U_T$ ) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the <b>equipment</b> requires continued operation during power mains interruptions, it is recommended that the <b>equipment</b> be powered from an uninterruptible power supply or a battery
Power Frequency Magnetic Field (50/60 Hz) IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE $U_T$ is the a.c. mains voltage prior to application of the test level.			

# IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

**The table below elaborates the recommended electromagnetic working environment.**

**Table C.3 – Example Immunity Test Level Compliance**

<p>The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.</p>			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
<p>Conducted RF IEC 61000-4-6</p>	<p>3 Vrms 150 kHz to 80 MHz</p>	<p>3 Vrms 150 kHz to 80 MHz</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the <b>equipment</b> including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> <p><math>d = 1.2\sqrt{P}</math>  <math>d = 1.2\sqrt{P}</math>  <math>d = 2.3\sqrt{P}</math></p> <p>80 MHz to 800 MHz              800 MHz to 2.7 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey should be less than the compliance level in each frequency range.</p>
<p>Radiated RF IEC 61000-4-3</p>	<p>6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,7 GHz</p> <p>RF communication equipment inside 80 MHz to 6 GHz</p>	<p>6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,7 GHz</p> <p>RF communication equipment inside 80 MHz to 6 GHz</p>	
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p>			
<p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re orienting or relocating the equipment.</p>			

# IEC 60601-1-2:2014—(Ed 4.0) EMC/EMI Compliance Guidelines (Cont'd)

*The table below elaborates the recommended separation between the equipment from RF communication devices.*

**Table C.4 – Example Recommended Separation Distances**

<b>Recommended separation distances between portable and mobile RF communications equipment and the equipment in question</b>			
The equipment is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the equipment can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the equipment as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter  W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz  $d = 1.2\sqrt{P}$	80 MHz to 800 MHz  $d = 1.2\sqrt{P}$	800 MHz to 2.7 GHz  $d = 2.3\sqrt{P}$
<b>0.01</b>	<b>0.12</b>	<b>0.12</b>	<b>0.24</b>
<b>0.1</b>	<b>0.38</b>	<b>0.38</b>	<b>0.73</b>
<b>1</b>	<b>1.2</b>	<b>1.2</b>	<b>2.3</b>
<b>10</b>	<b>3.8</b>	<b>3.8</b>	<b>7.3</b>
<b>100</b>	<b>12</b>	<b>12</b>	<b>23</b>
For transmitters rated at a maximum output power not listed above, the recommended separation distance $d$ in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

# Warranty

This Warranty does not affect or in any way limit your Statutory Rights.

Handicare guarantees all equipment, which includes Ceiling Lifts, Floor Lifts, Slings, Service Parts and Track and accessories, supplied as new, against failure within the period of 1 year from date of installation or 18 months from date of manufacturing, whichever is shorter, by virtue of defects in material or workmanship.

Handicare guarantees all refurbished equipment supplied against failure within a period of three months from date of installation or six months from date of purchase whichever is shorter.

This guarantee does not apply to failure attributable to normal wear and tear, damage by natural forces, user neglect or misuse or to deliberate destruction, or to batteries more than 90 days after original purchase.

This guarantee shall be void if the equipment is not serviced by Handicare or its authorized service agents in accordance with the manufacturer's recommendations or if any unauthorized person carries out works on the equipment.

The liability of Handicare under the terms of this guarantee shall be limited to the replacement of defective parts and in no event shall Handicare incur liability for any consequential or unforeseeable losses.

## About Handicare

Handicare offers solutions and support to increase the independence of the physically challenged, mobility impaired or elderly people and to enable them to live an active life—on their terms—and to facilitate for their care providers and family. The offering encompasses a comprehensive range of curved and straight stairlifts, transfer, lifting and repositioning aids, vehicle adaptations and medical equipment. Handicare is a global company with sales in more than 20 countries and is a market leader in this field. The head office is in Stockholm, Sweden and manufacturing is located at six sites distributed across North America, Asia and Europe.

## A-Series

### The A-Series Advantage

Built on the proven technology of the Handicare C-Series, the new A-450 and A-625 ceiling lifts raise patient handling to the next level. Increase patient protection with antimicrobial materials, simplify patient transfers with its tilting design and easily access real-time data and diagnostics. The A-Series ceiling lifts give you the extra advantage in safe patient handling.

### **Always make sure you have the correct version of the manual**

The most recent version of all manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

For questions about the product and its use

Contact your local Handicare dealer or your local Handicare business development manager.

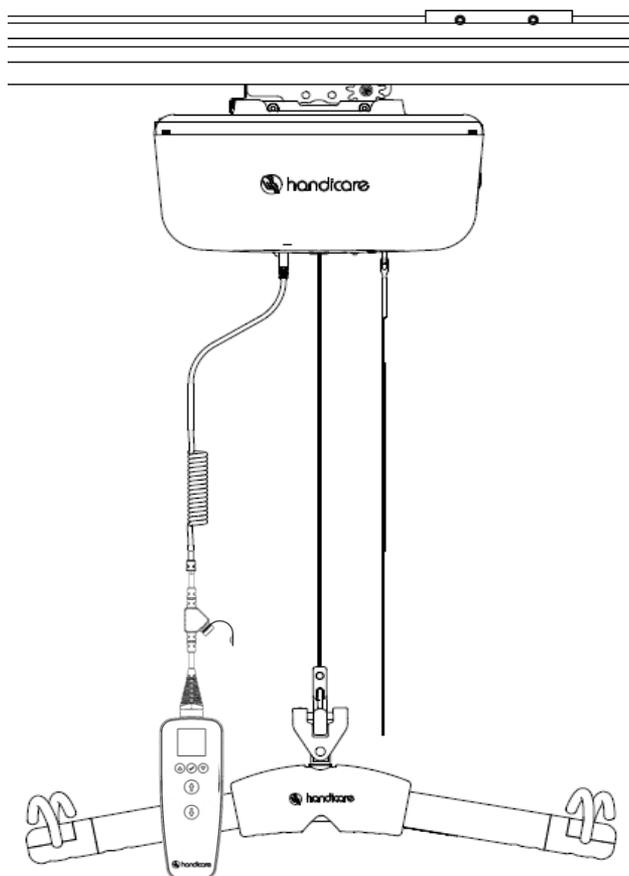
This Page Intentionally Left Blank

**Handicare USA, Inc.**  
10888 Metro Court  
St. Louis, MO 63043 USA  
Toll Free: 1-866-891-6502  
Fax: 314-692-7858  
[www.handicare.com](http://www.handicare.com)

**Handicare Canada**  
81 Romina Drive  
Concord, ON L4K 4Z9 Canada  
Toll Free: 1-877-304-5438  
Fax : 1-416-260-0256  
[www.handicare.com](http://www.handicare.com)

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The screenshot shows the configuration interface for a Handicare device. It features a dark header with the Handicare logo and an 'Information' button. The main content area is split into two panels. The left panel, titled 'Client Location', contains input fields for Name, Address, Phone, Email, Building, and Room, with a 'Save' button at the bottom. The right panel, titled 'Device Analytics', displays service dates (Next Service, Last Service), a Service PIN field, Total PM Lifts, Strap Service Due In, and Battery Service Due In, each with a 'Reset' button. Below this is a 'System Information' section with a Serial Number field and a 'Write' button. Large letters F, G, and H are overlaid on the form fields to indicate specific areas of interest.

Figure 16: Écran de configuration

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