


P703 PRIME LONG TERM CARE BED

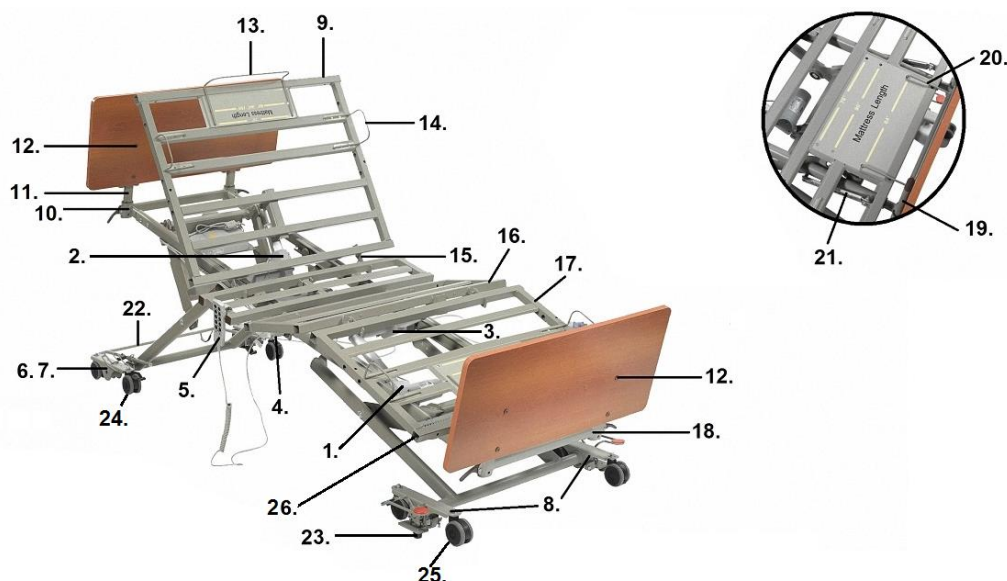

1. Hi/Lo Motor- SP01-TA23-1060-001
Hi/Lo & Foot Motor Extension Cord- SP01-P703-80E
 2. Head Motor- SP01-TA31-1060-001
 3. Foot Motor- SP01-TA31-1060-002
 4. Motor Control Box- SP01-TC21-1060-001
"Y" Cable- SP01-P703-70E
Power Cord- SP01-P703-60-02E
 5. Hand Pendant (8 button)- SP02-TH21060003
 6. Head Deck Truck Assembly Left- SP01-P703-04-03E
 7. Head Deck Truck Assembly Right- SP01-P703-04-02E
 8. Foot Deck Truck Assembly- SP01-P703-20E
 9. Head Deck w/Hardware- SP01-P703-10001E
 10. Head Board Adjustment Bracket- SP01-P703-10008E
 11. Head Board Mounting Bracket- SP01-PCB901-10003E
 12. Head/Foot Board Mounting Bolt (ea)- SP01-UF16108061E
 13. Head End Mattress Retainer (ea)- SP01-P703-02-03E
 14. Width Mattress Retainer (ea)- SP01-P703-02-02E
 15. Head/Thigh Deck Hardware- SP01-P703-10002E
 16. Thigh Deck w/Hardware- SP01-P703-10003E
 17. Foot Deck w/Hardware- SP01-P703-10005E
Foot Deck Hardware- SP01-P703-10004E
 18. Foot Board Adj. Length Mtg. Bracket w/Hardware- SP01-P703-10009E
 19. Foot Board Mounting Bracket w/Hardware (pr)- SP01-PCB901-10019E
 20. Foot End Mattress Retainer- SP01-P703-03-03E
 21. Foot Deck Ratcheting Assembly- SP01-PCB300-01-57E
 22. Wall Guard- SP01-P703-30E
 23. Brake Stop- SP01-PCB900-24BS
 24. Locking Caster- SP01-TL500-37E
 25. Non-Locking Caster- SP01-TL500-29E
 26. End Cap- PC20303001E-CO
- Not Shown
- Head Width Extension Left- SP01-P703WE-10002E
Head Width Extension Right- SP01-P703WE-10001E
Thigh Width Extension Left- SP01-P703WE-10004E
Thigh Width Extension Right- SP01-P703WE-10003E
Foot Width Extension Left- SP01-P703WE-10006E
Foot Width Extension Right- SP01-P703WE-10005E