Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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Padded Commode/Shower Chair with Wheels

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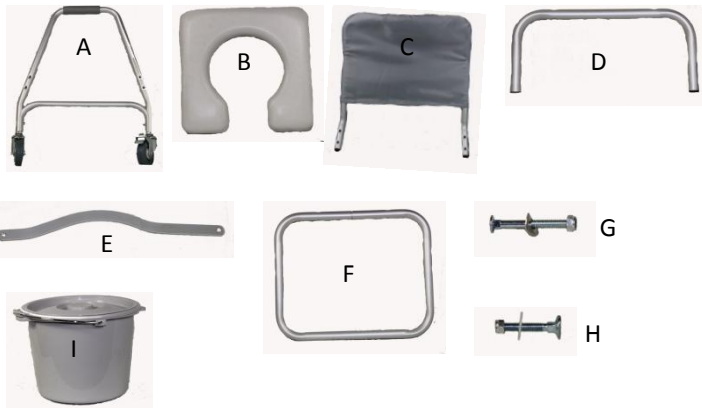
3

ATTACH BACKREST TO ARM SUPPORTS - FIXER LE DOSSIER AUX APPUIS-BRAS

Using 4 large screw combinations attach the backrest to the arm supports.
 Note: Back angles slightly outward
 Note: Locking casters (like in bubble above) are at the back of the commode
 Note: The tubing has holes in it to insert the screw, the head of the screw fits into the square cut-out making the screw flush to the tubing.

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Using 4 small screw combinations attach the pail brackets
 Attach the seat by snapping the clips over the seat support tubing
 Note: The pail will drop in between the pail support brackets

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