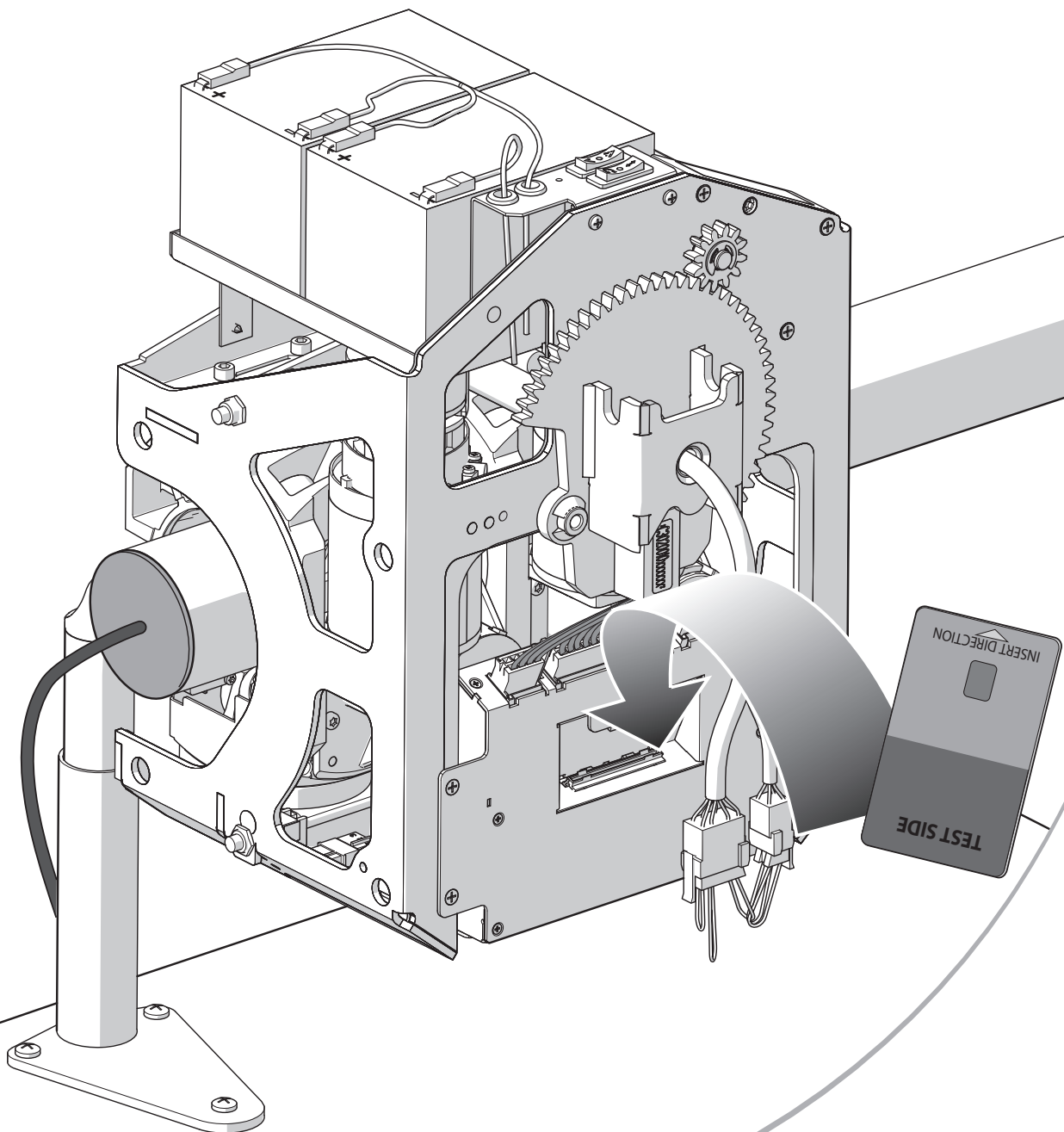
FREECURVE | INSTALLATION MANUAL ***MONORAIL*** | MANUAL D'INSTALLATION

Classic
Elegance
Alliance

EN *All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual may also be changed without prior warning. This manual is intended for the standard models of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications that differ from those of the standard model of the lift supplied to you.*

This manual has been compiled with the best intentions and every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof.

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UK

The Dutch manual is the original language

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
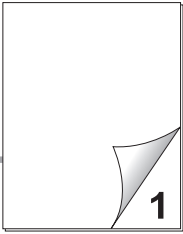
1

2

Thank you for choosing a Handicare stairlift.
This manual is for the Classic and Elegance/
Alliance seat types, and includes installation instruc-
tions for the standard features and optional upgrades.

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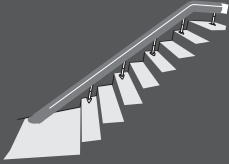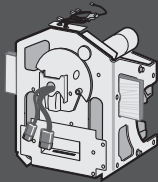
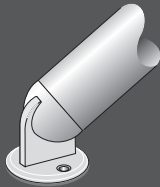
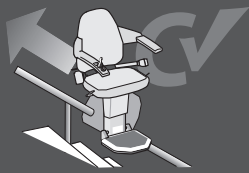
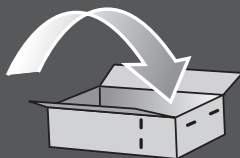
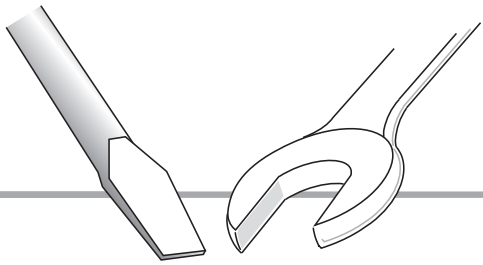
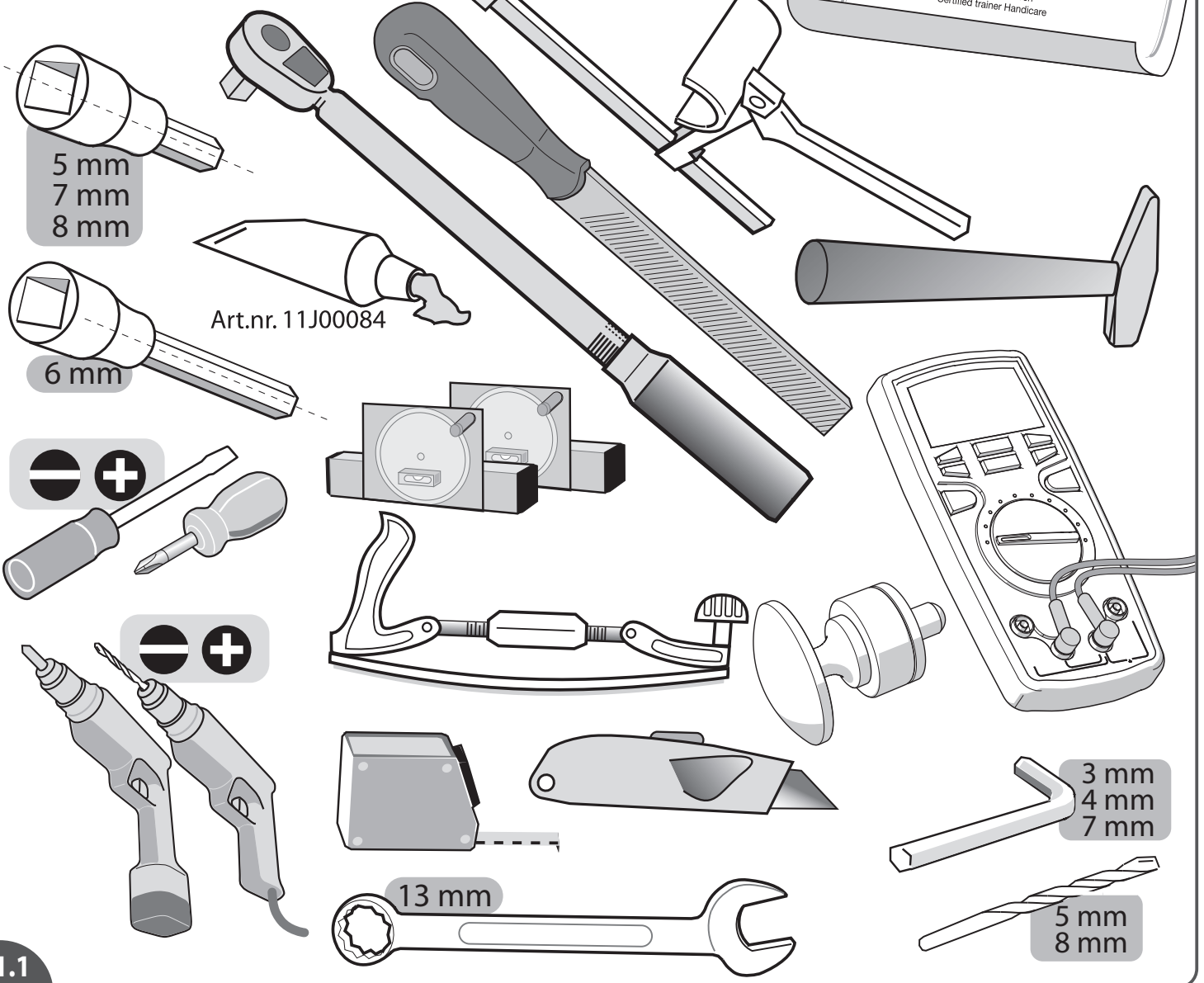
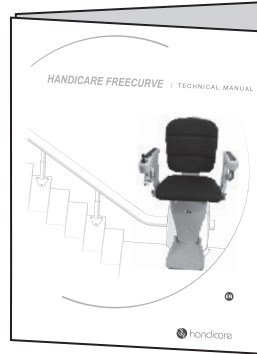
Nm **Hand tight**
 Not loose, but not really tight either.
 Somewhere around 1 or 2 Nm.

1.1

Only a **certified** technician should install this lift (general required knowledge can be found in the Freecurve Technical Manual). The latest instructions of the lift and instructions for options are available for download at www.handicarepartners.com. The installations of the options are not all described in this manual. These instructions are provided with the corresponding option kit. Ensure good and sufficient tools; the shown tools are minimum required to install the stairlift. In case of installation a new part on a existing lift always replace the part by a part with the same article number.

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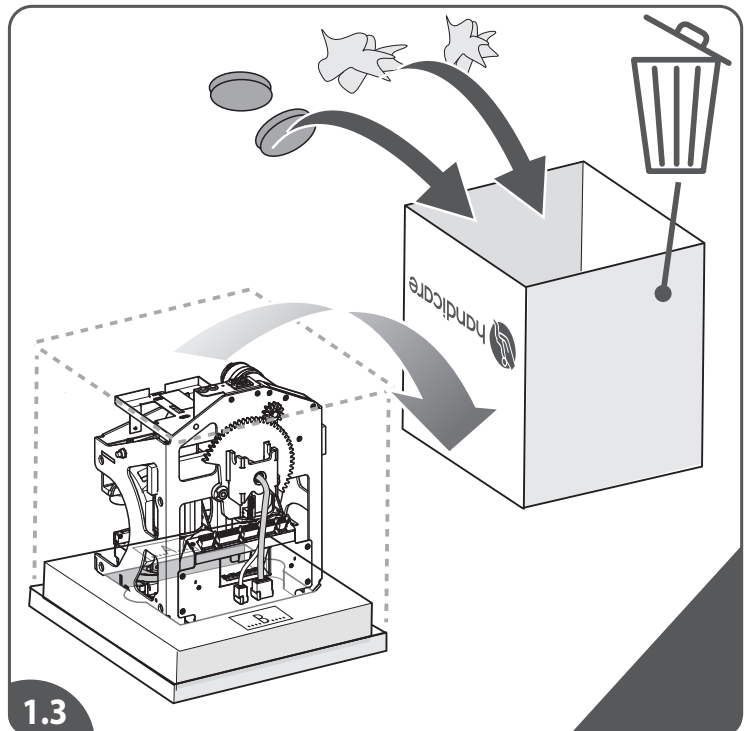
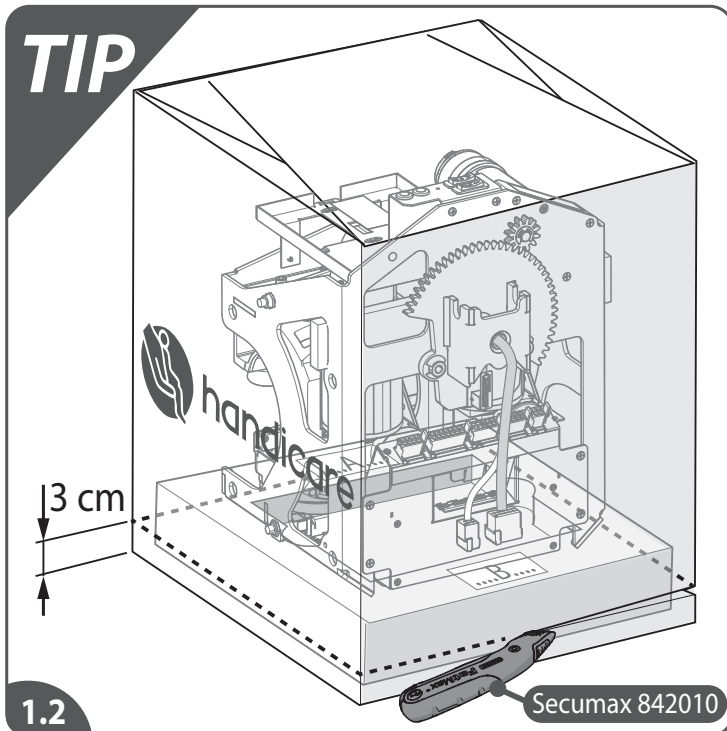
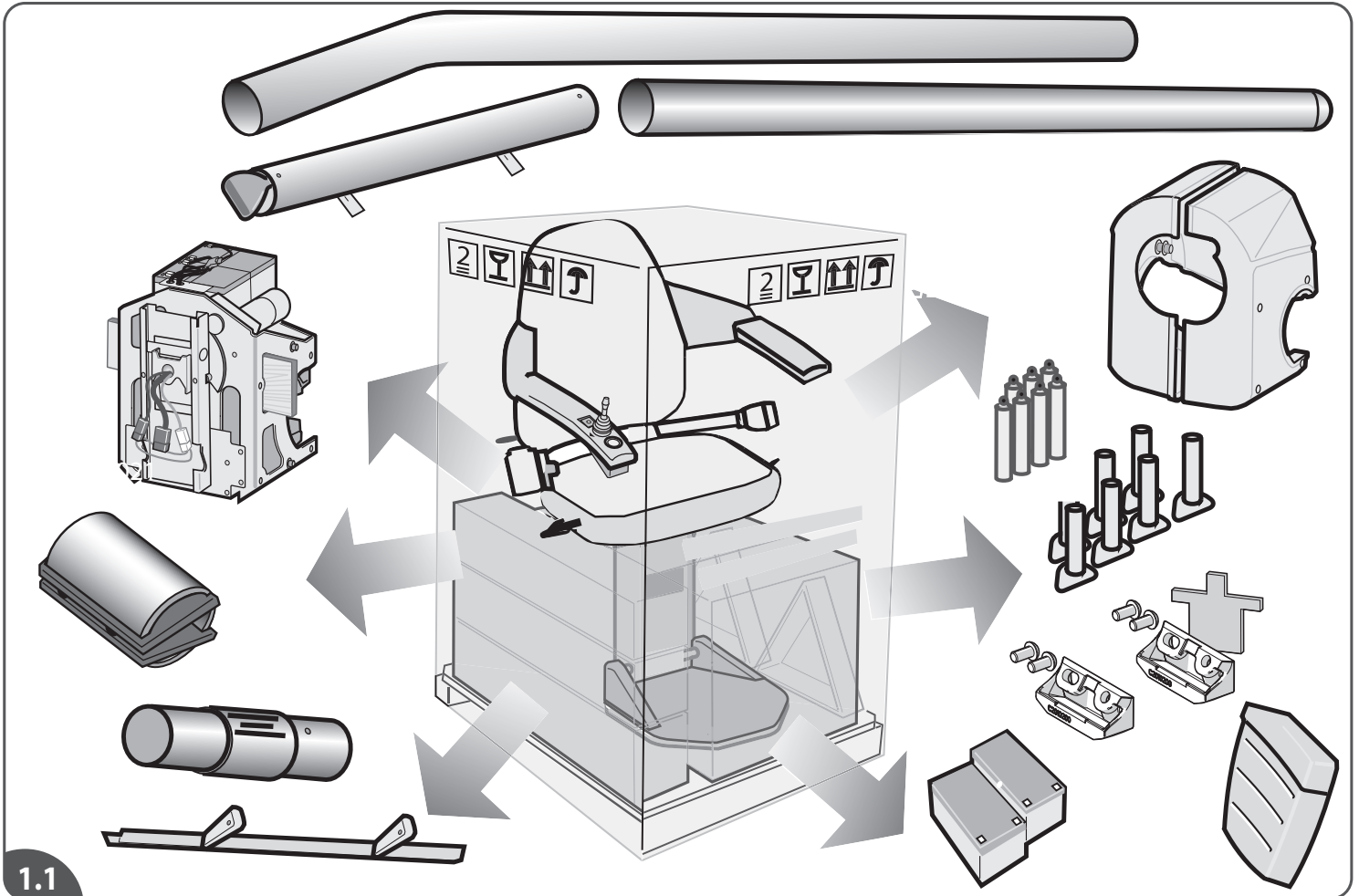
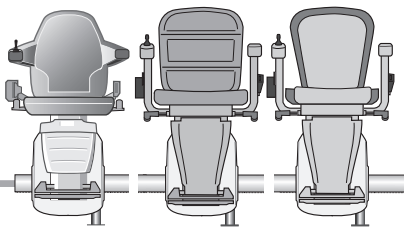
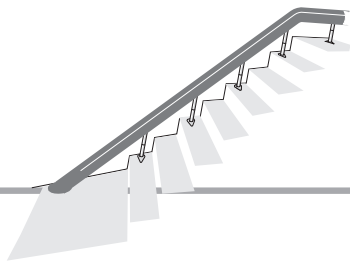
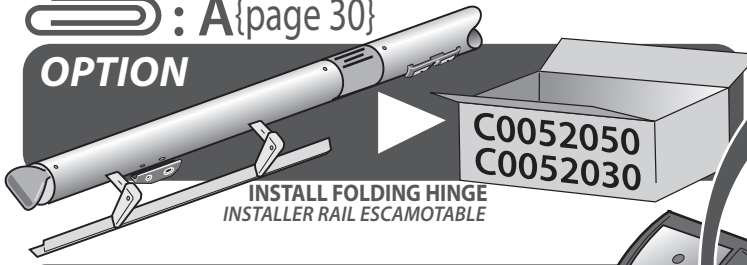
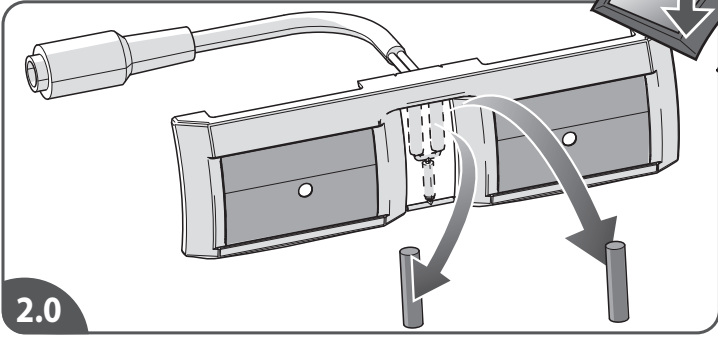
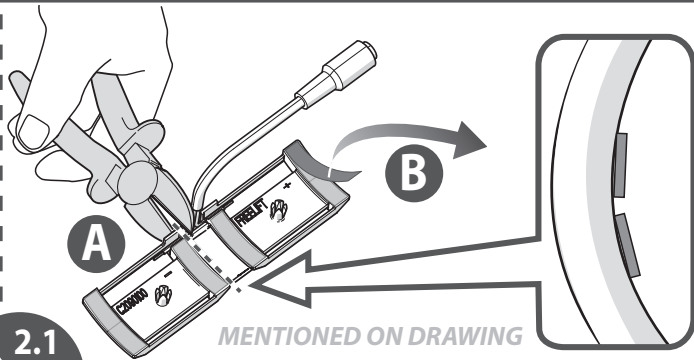
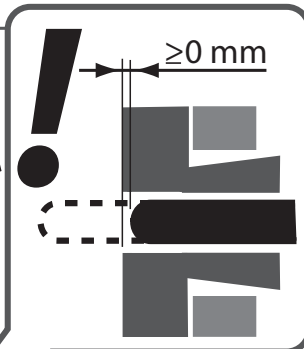
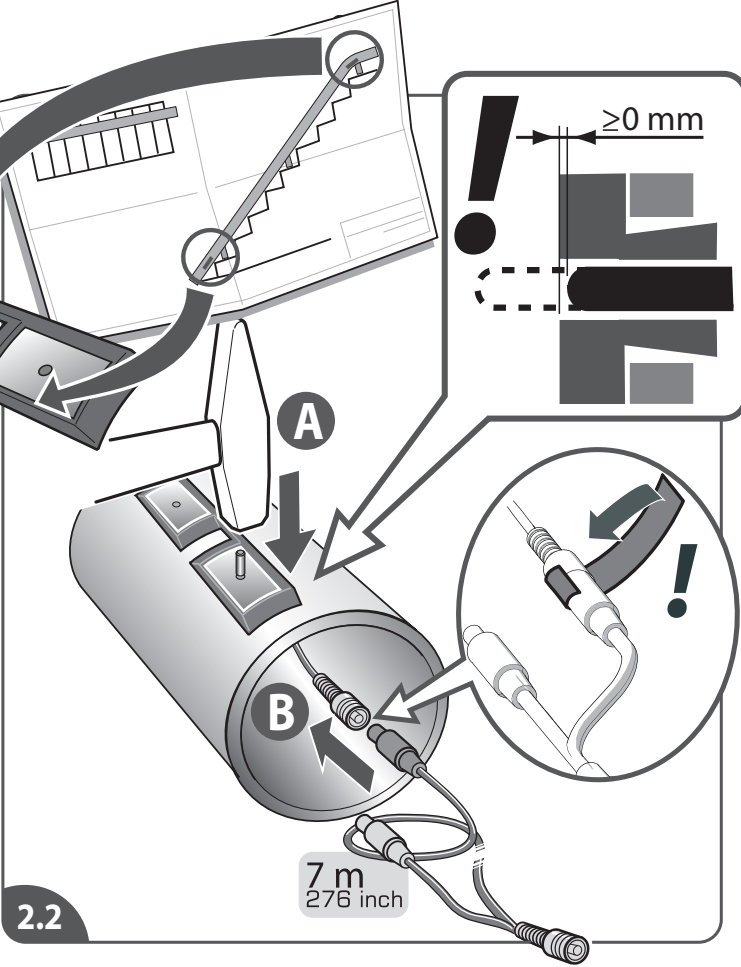
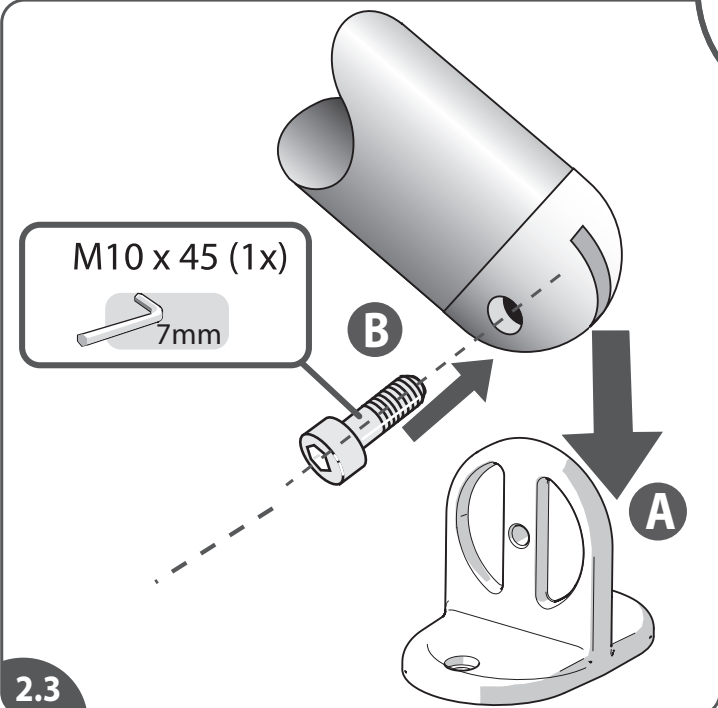
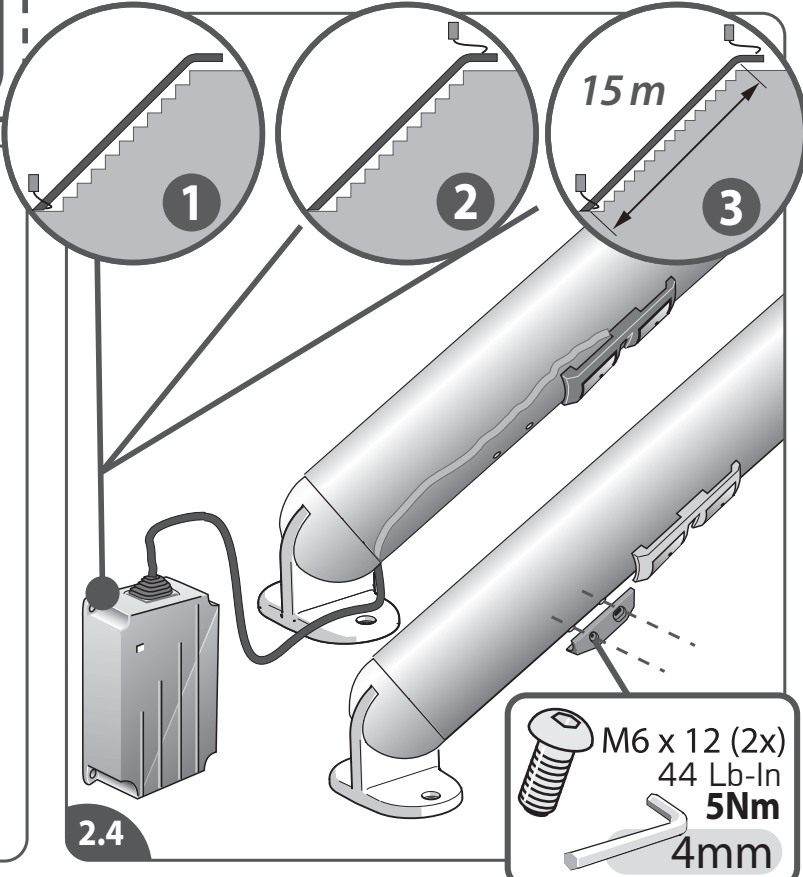
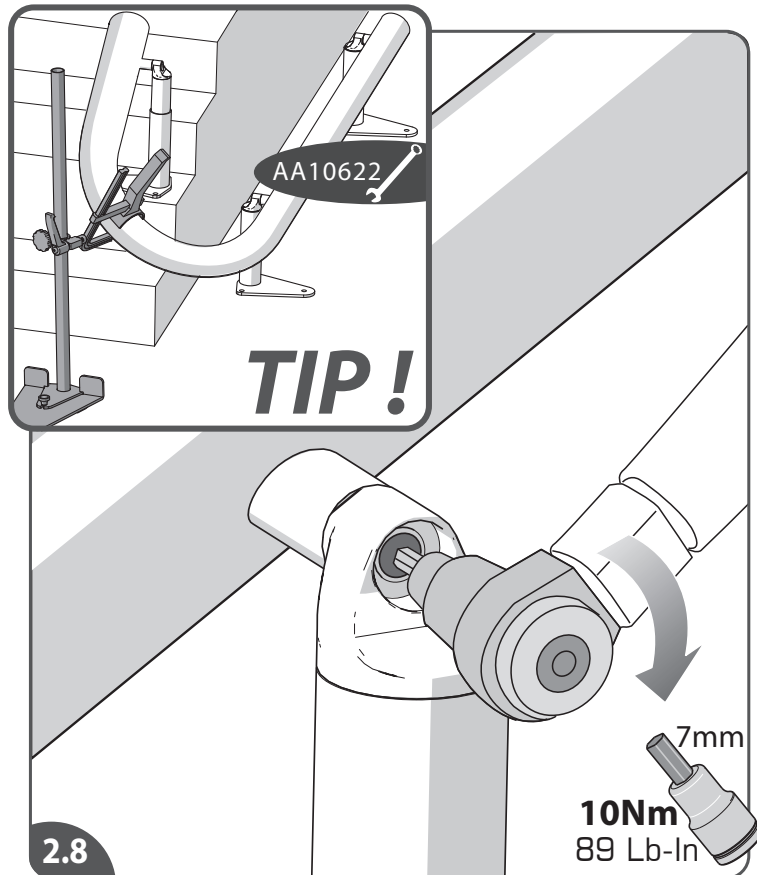
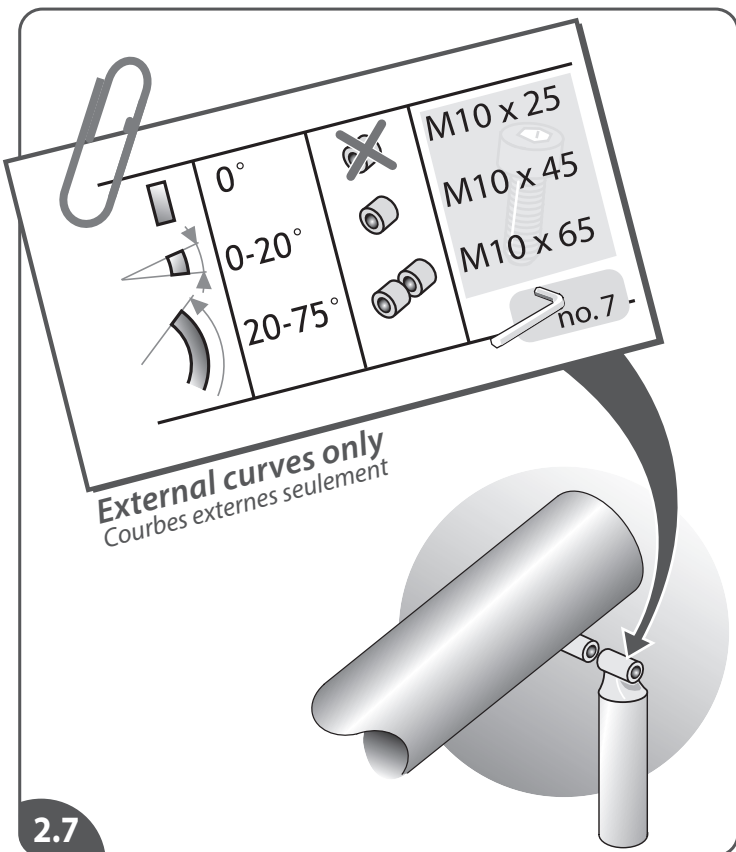
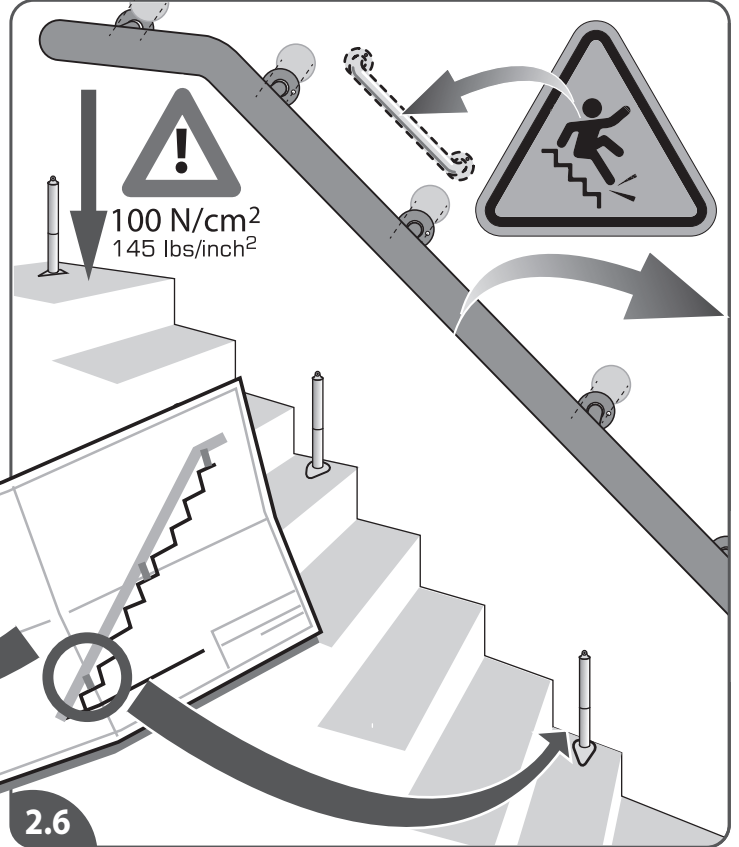
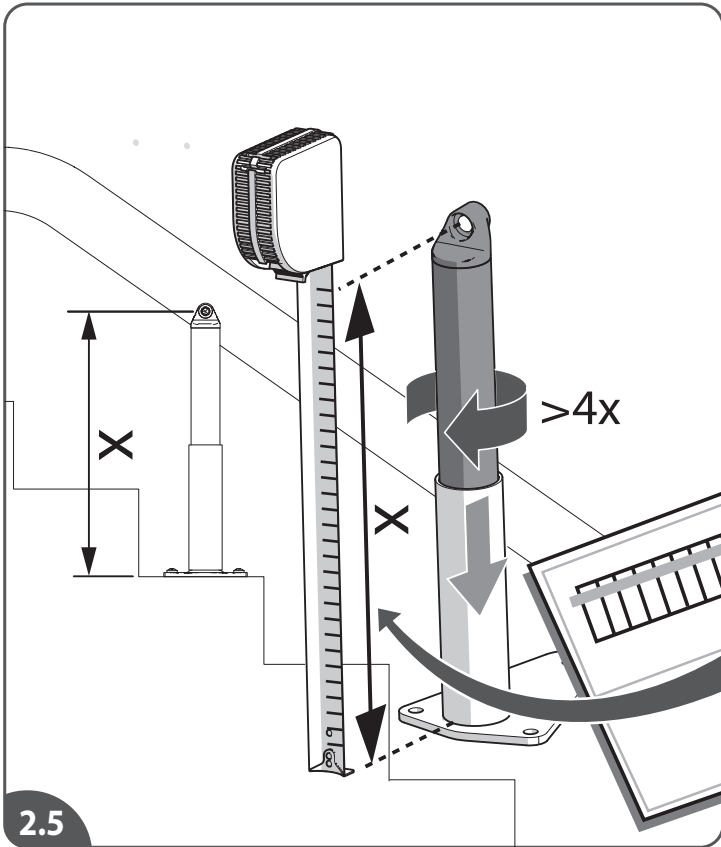
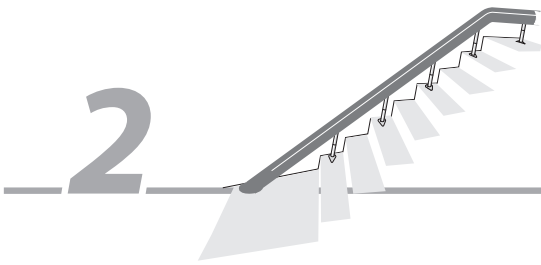
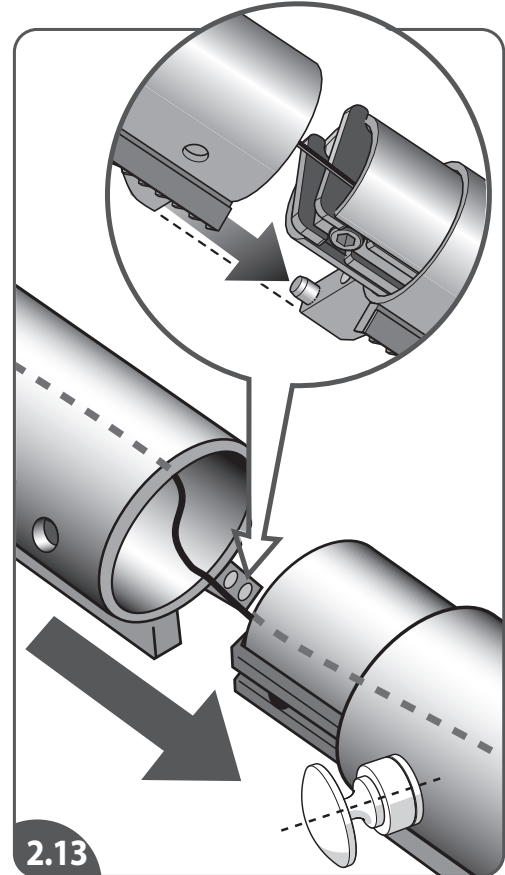
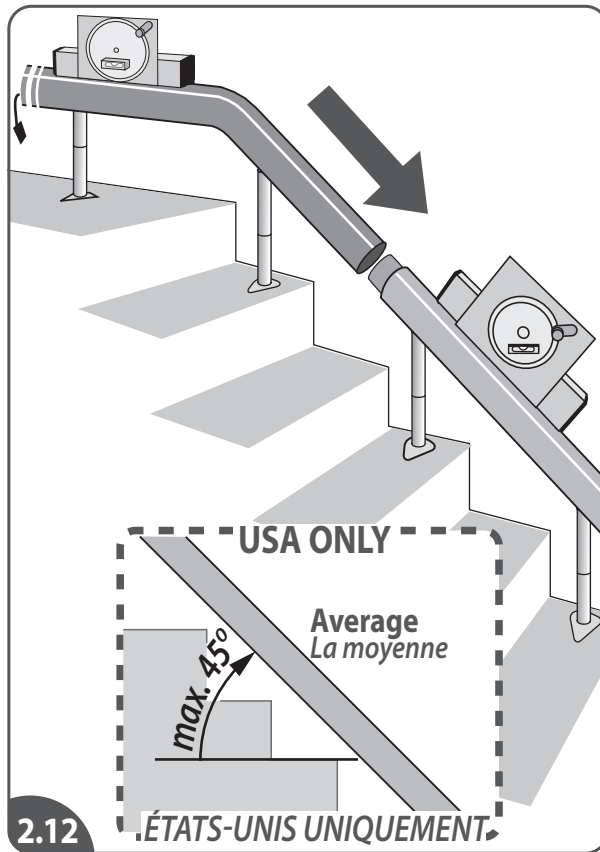
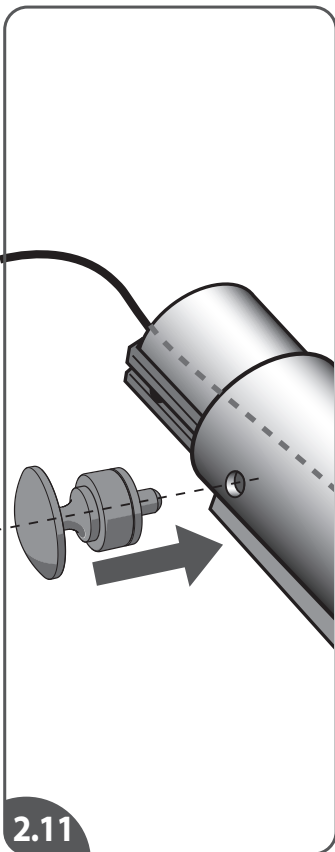
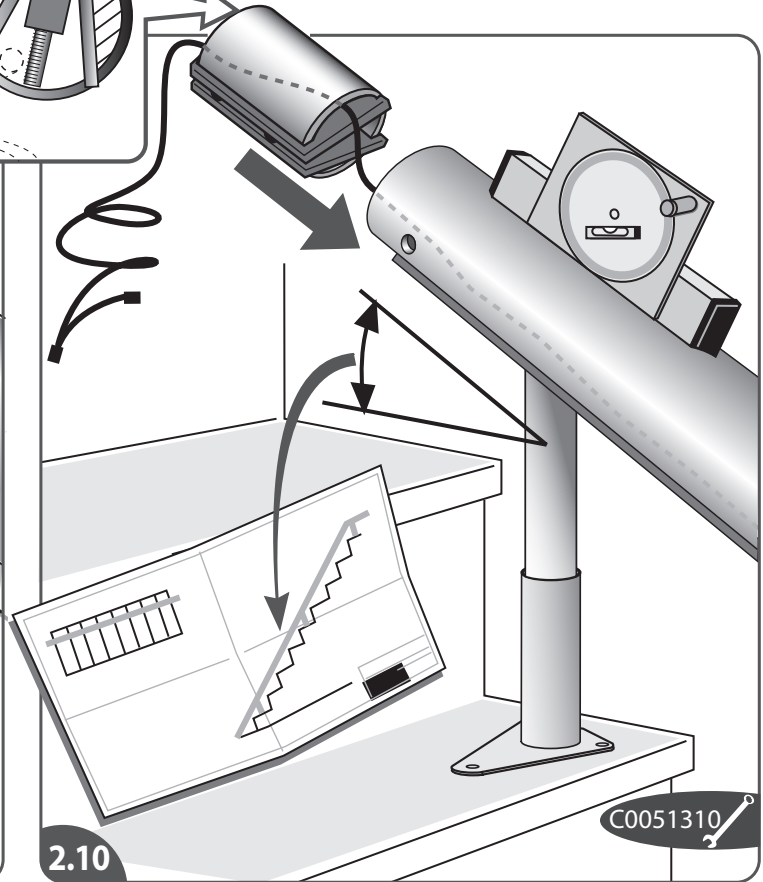
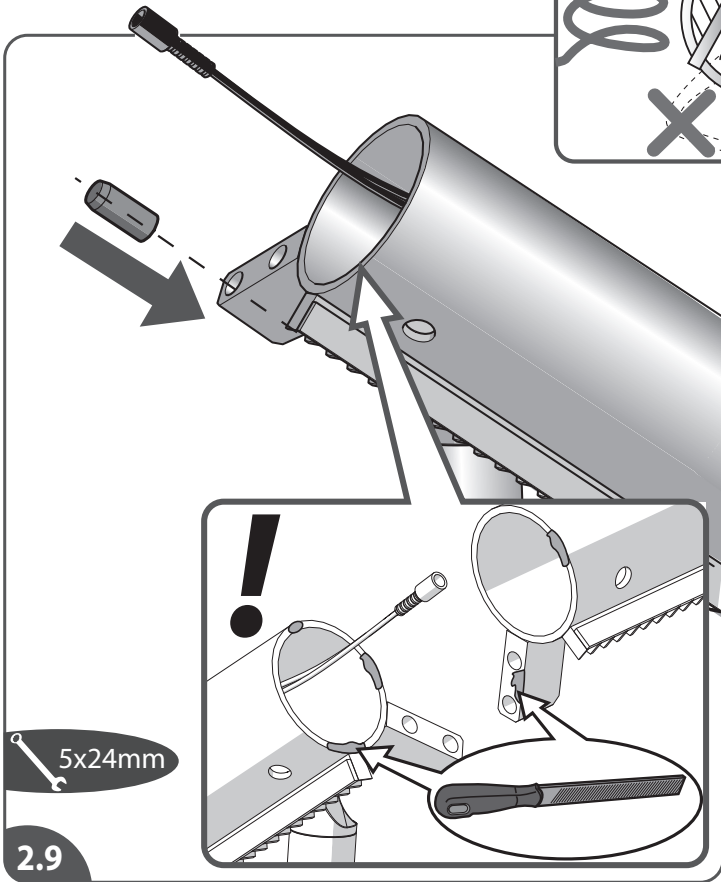
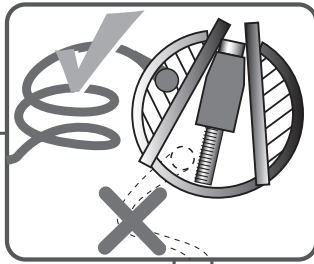
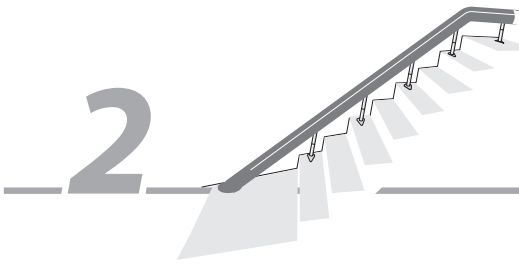
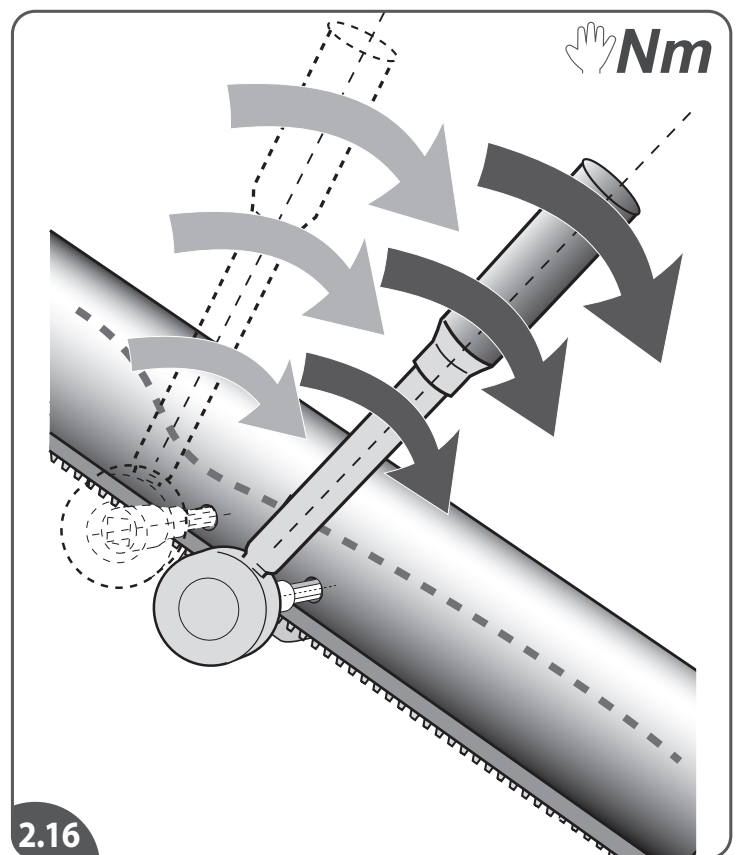
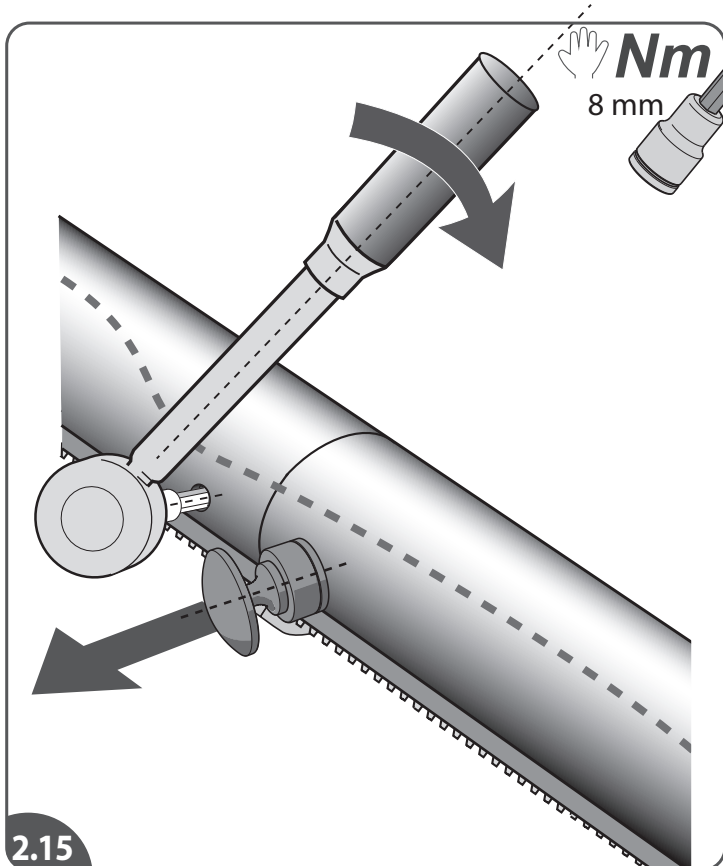
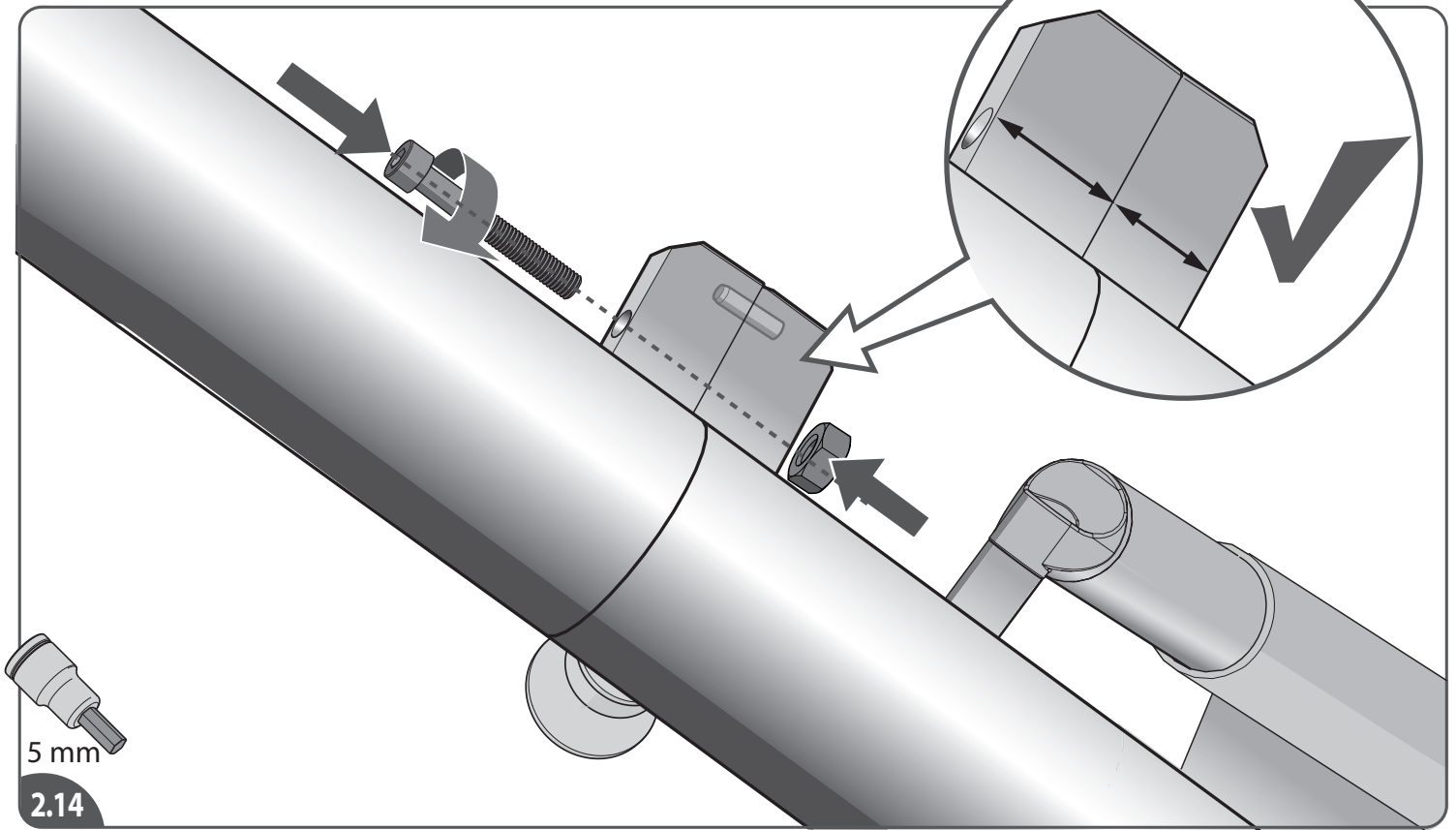
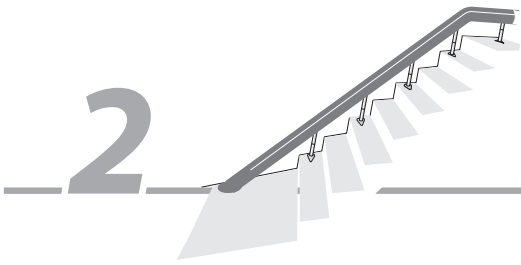
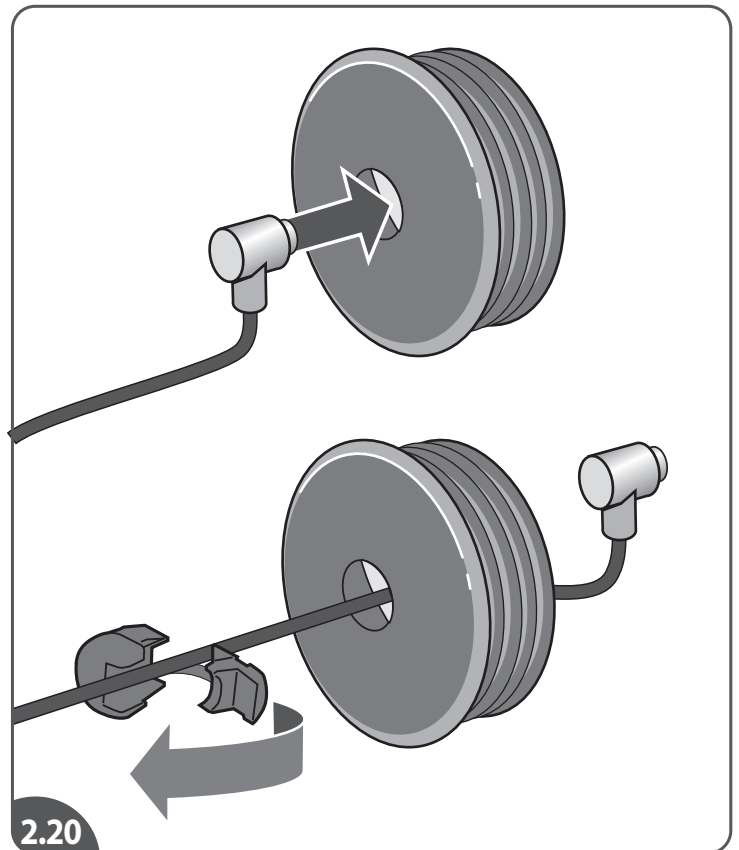
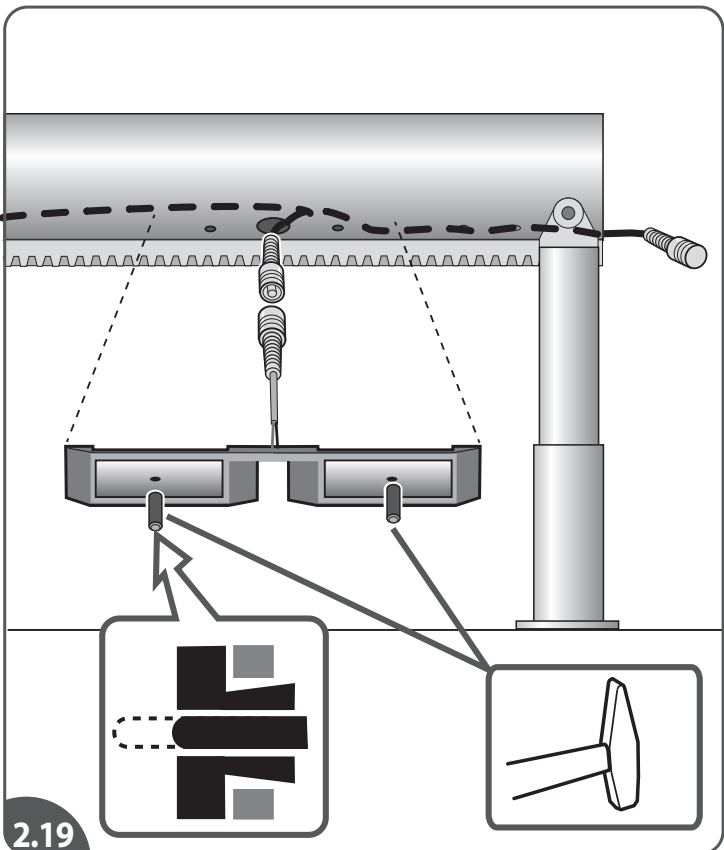
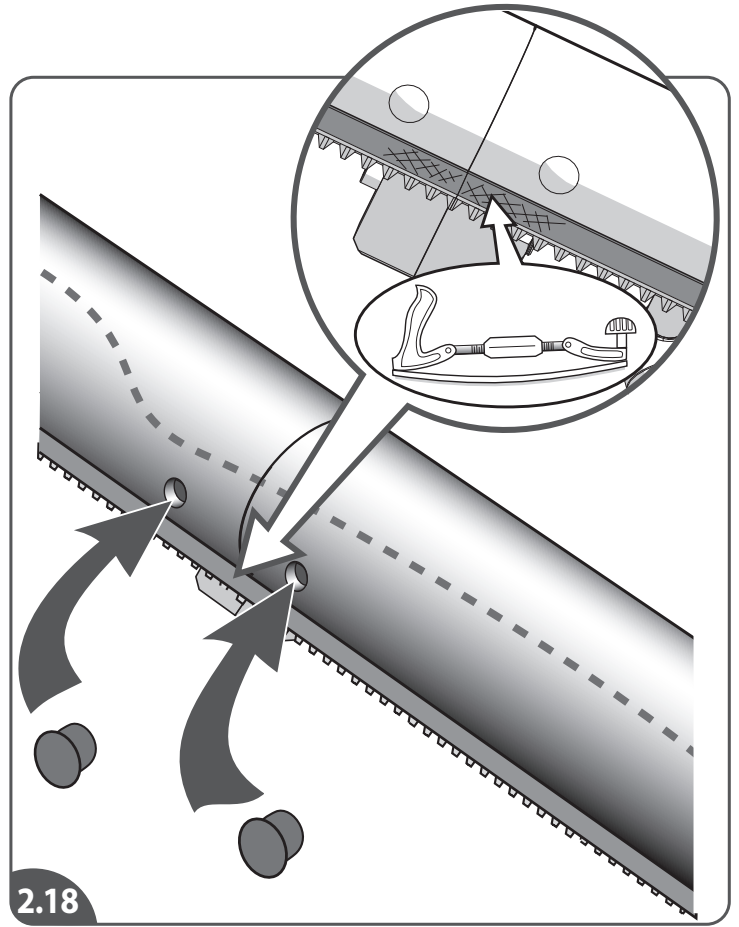
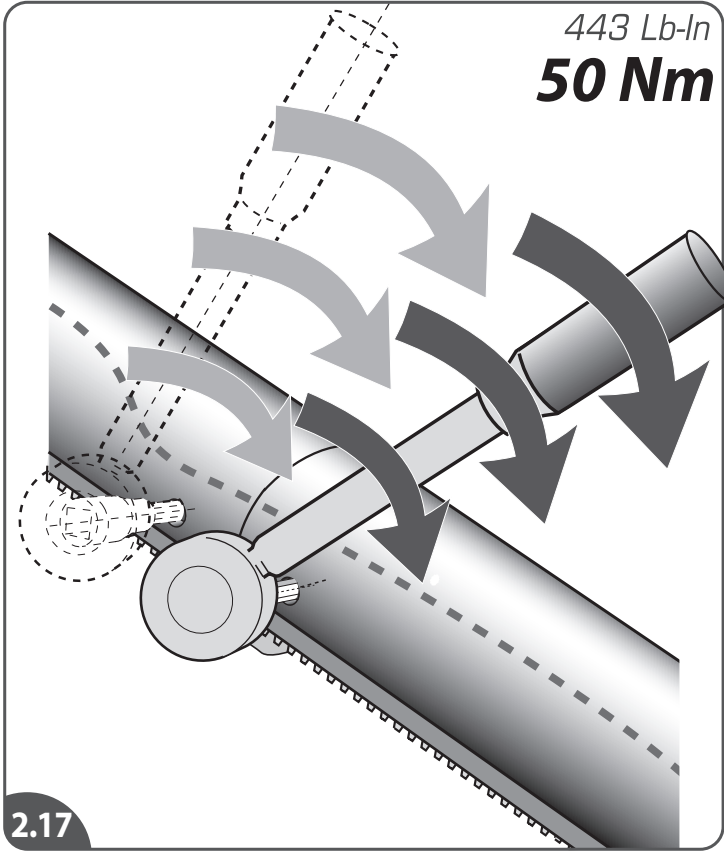
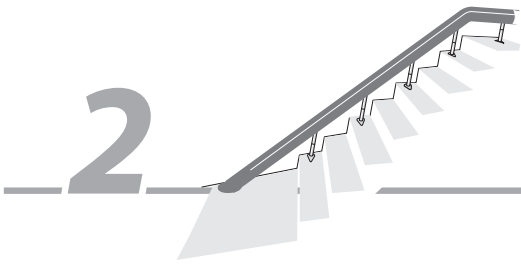
2.4


2

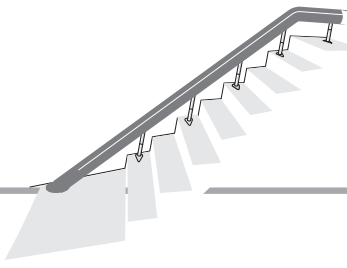USA ONLY
The installation of electrical equipment and wiring shall conform to the requirements of NEC and ANSI/NFPA70.

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

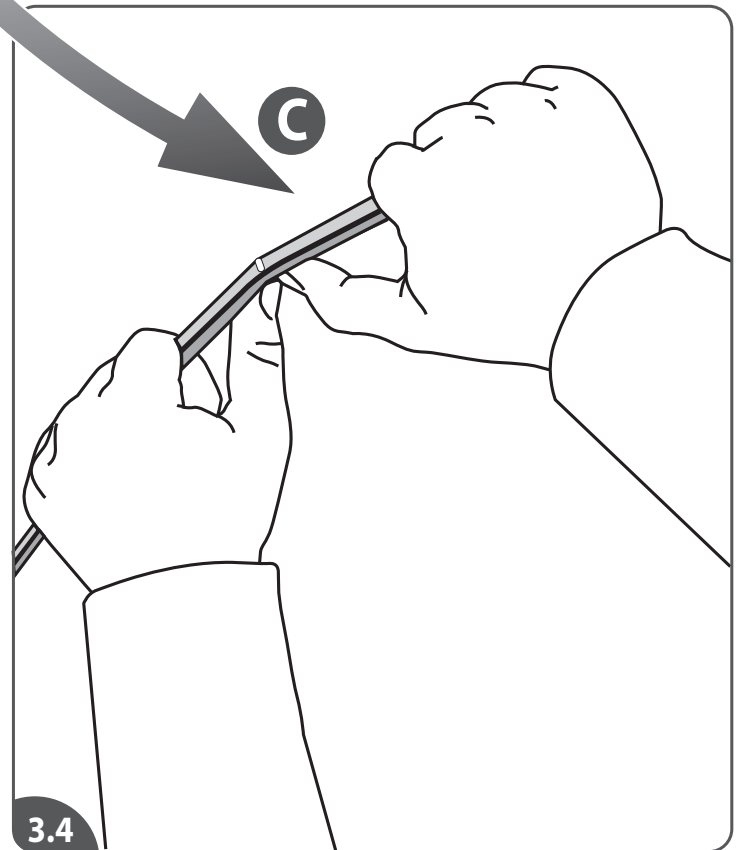
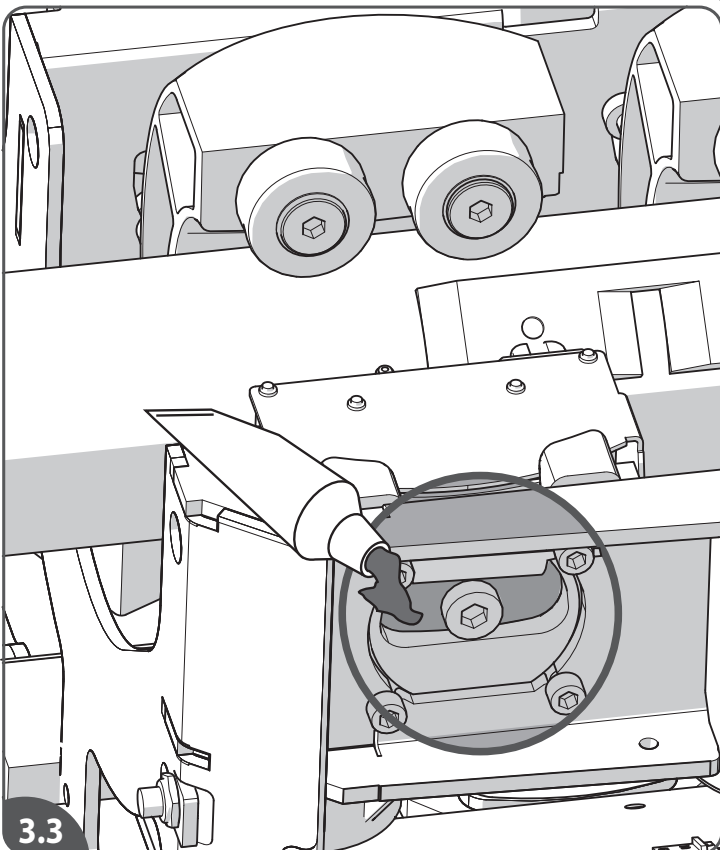
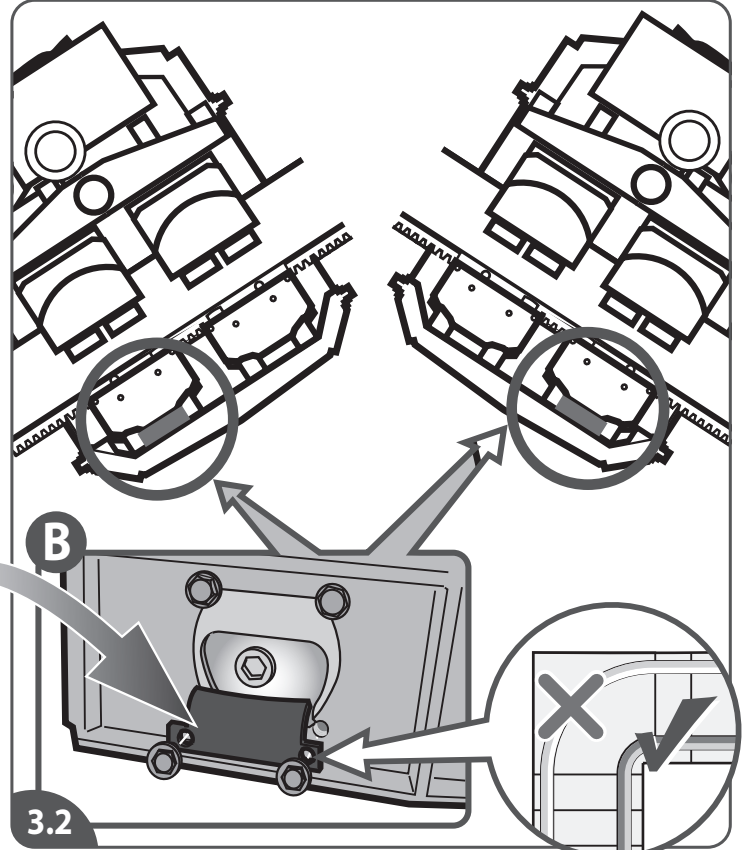
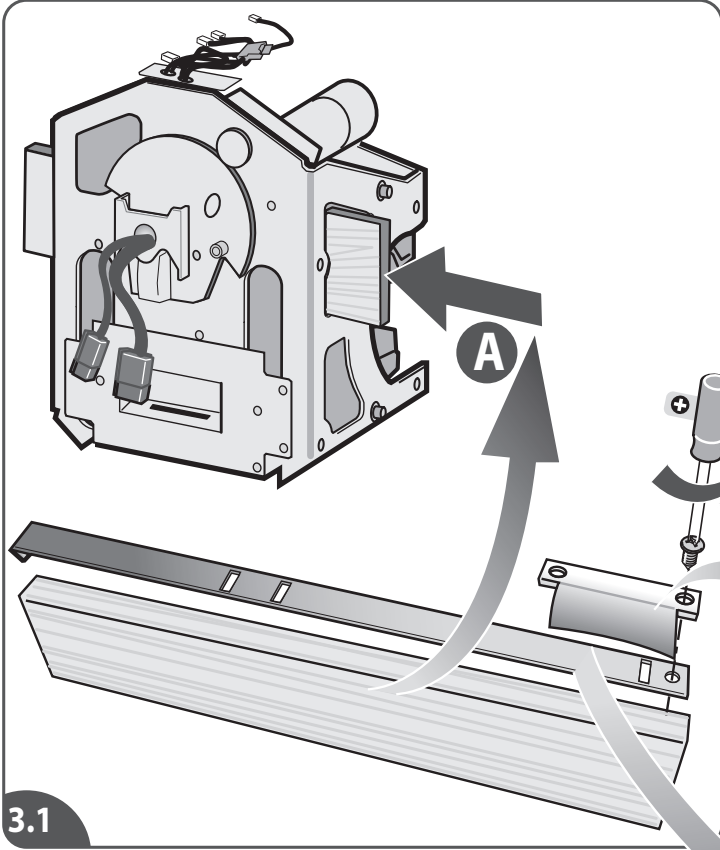
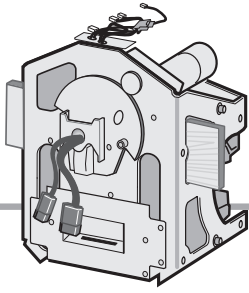
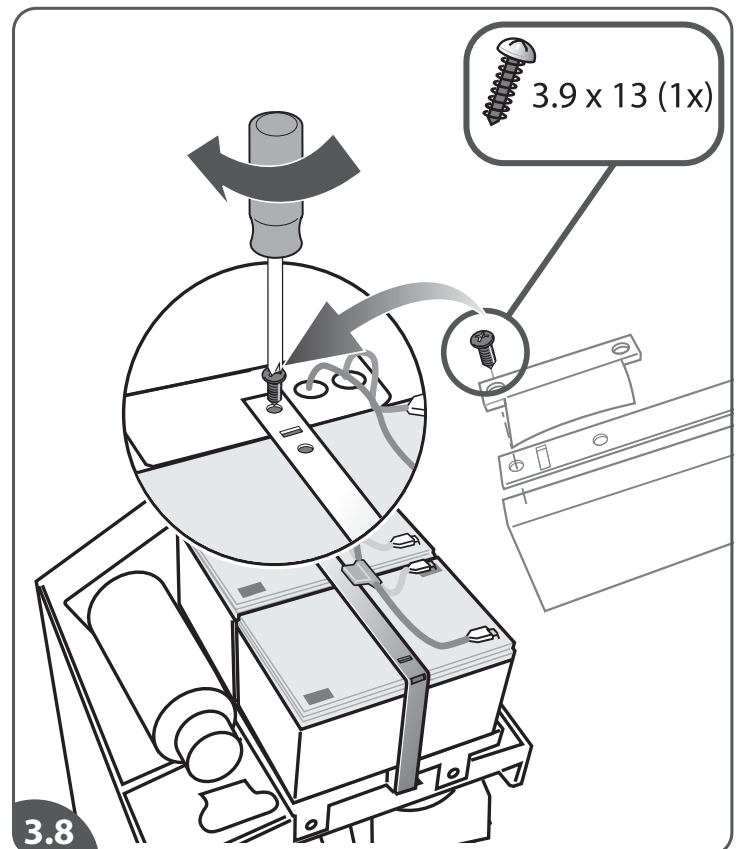
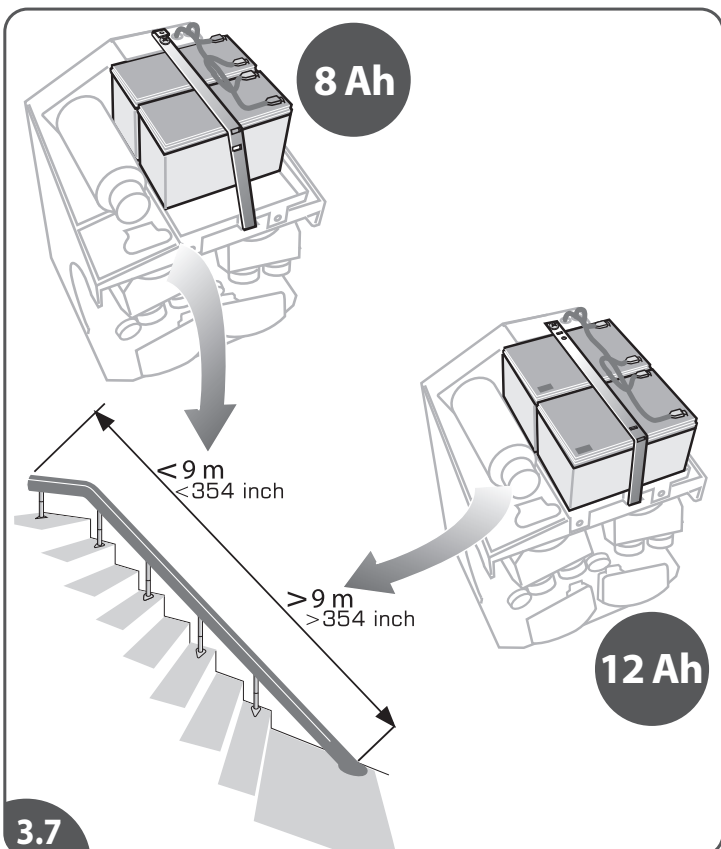
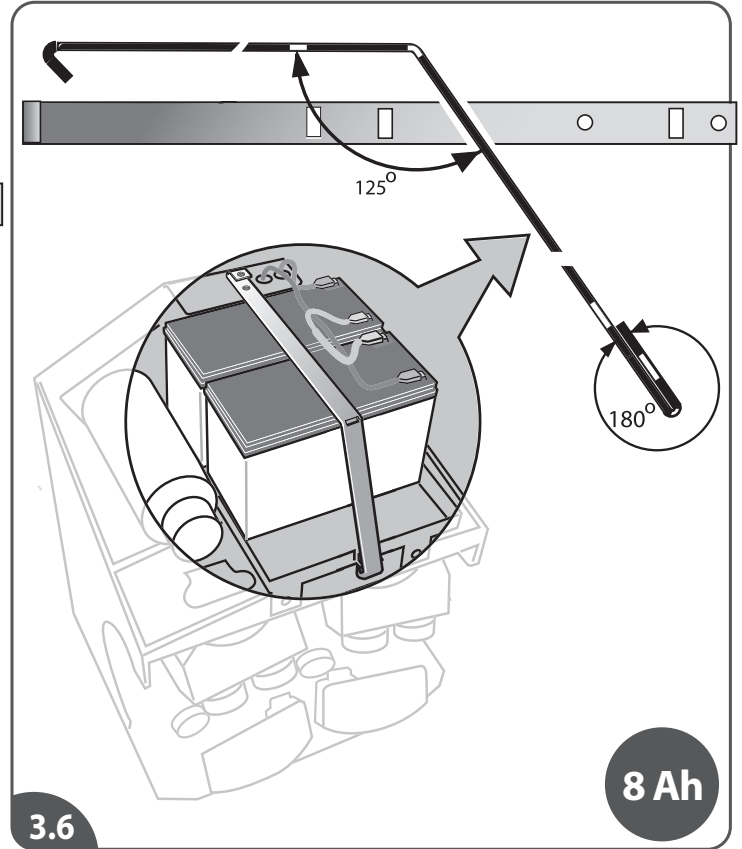
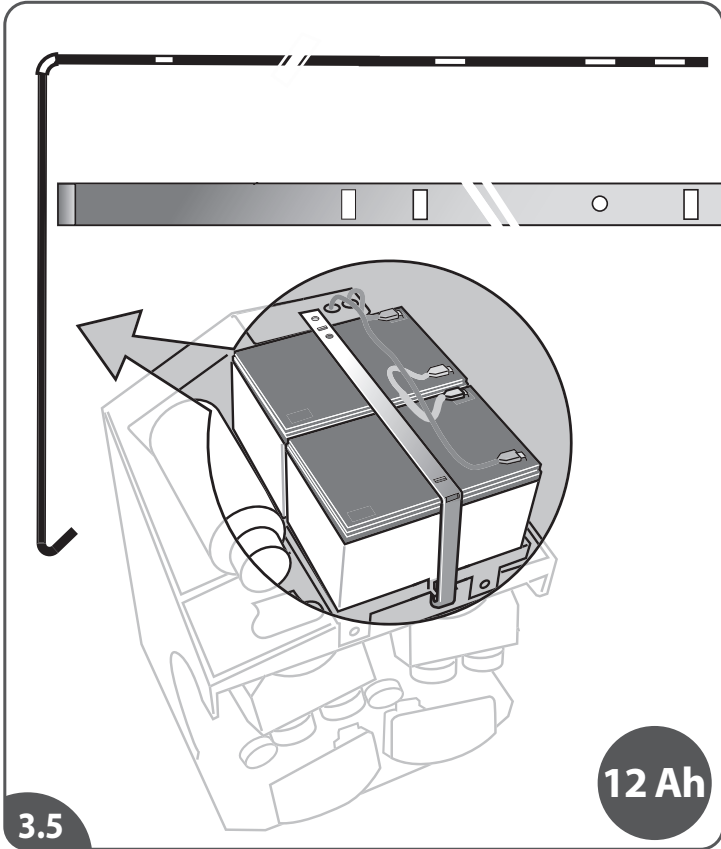
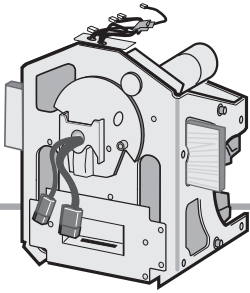
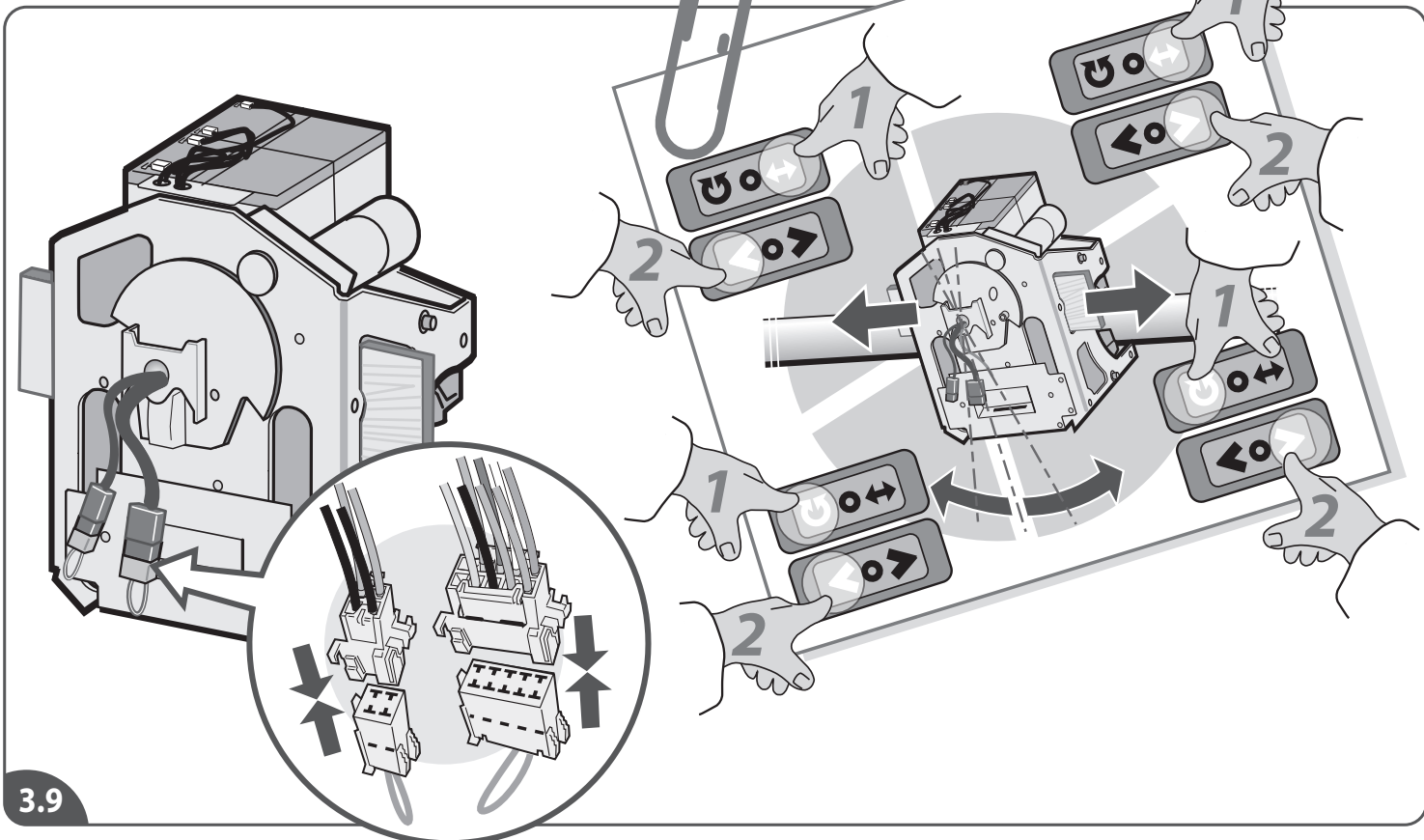
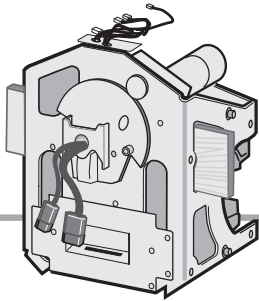
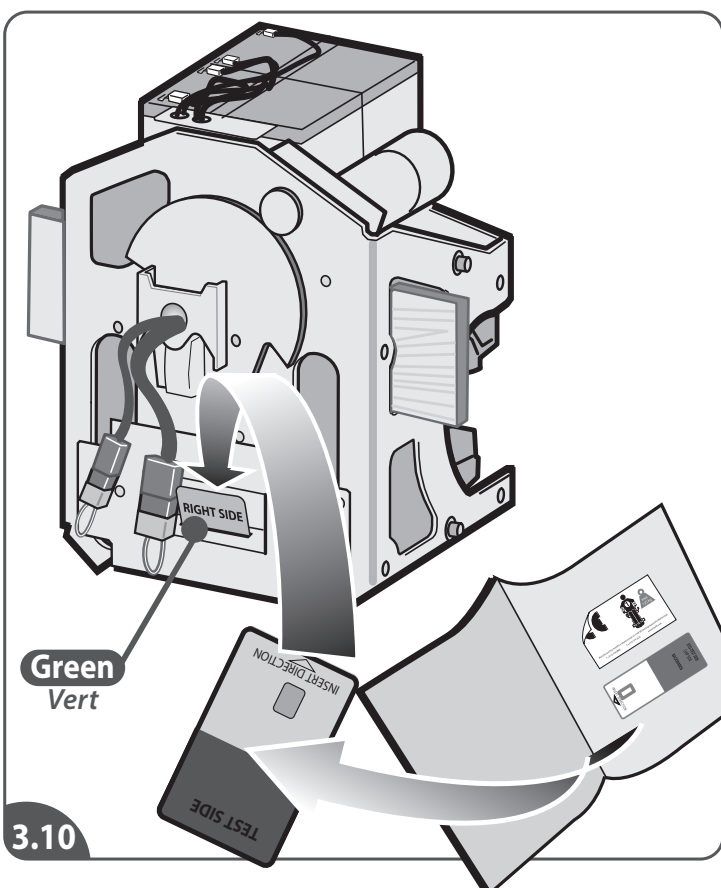
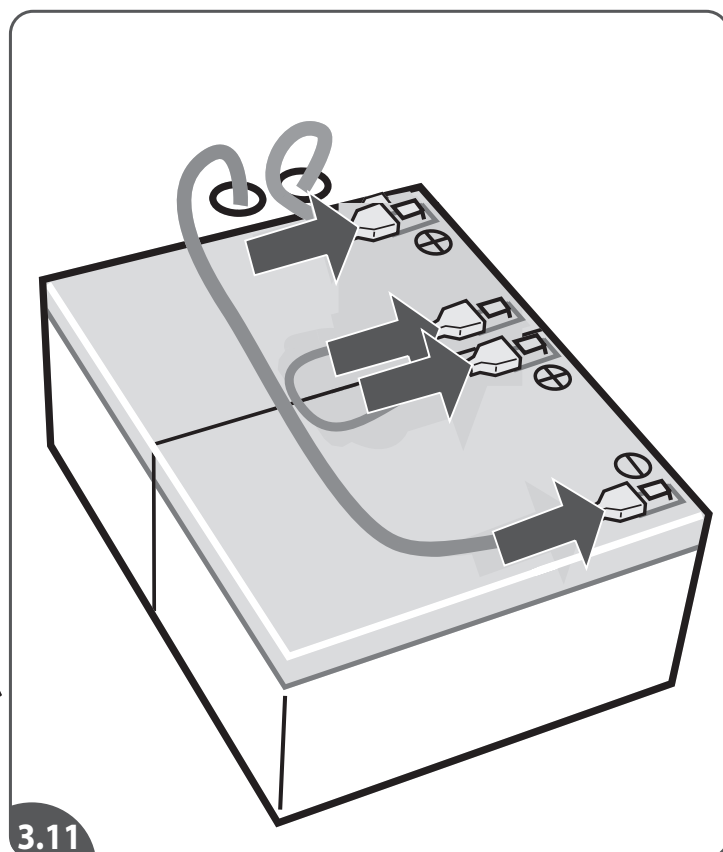
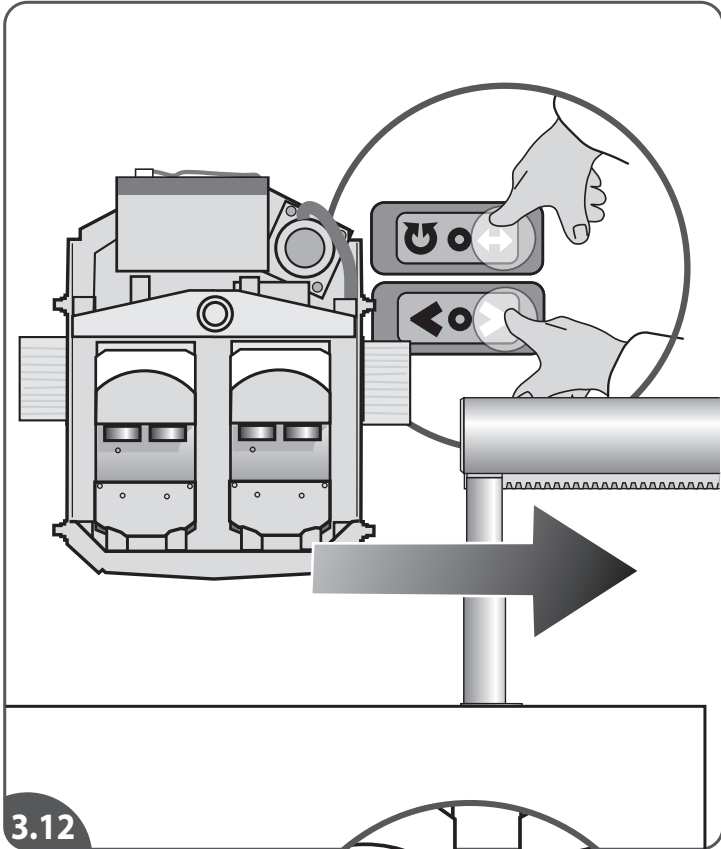
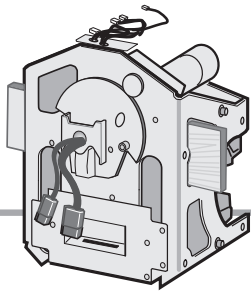
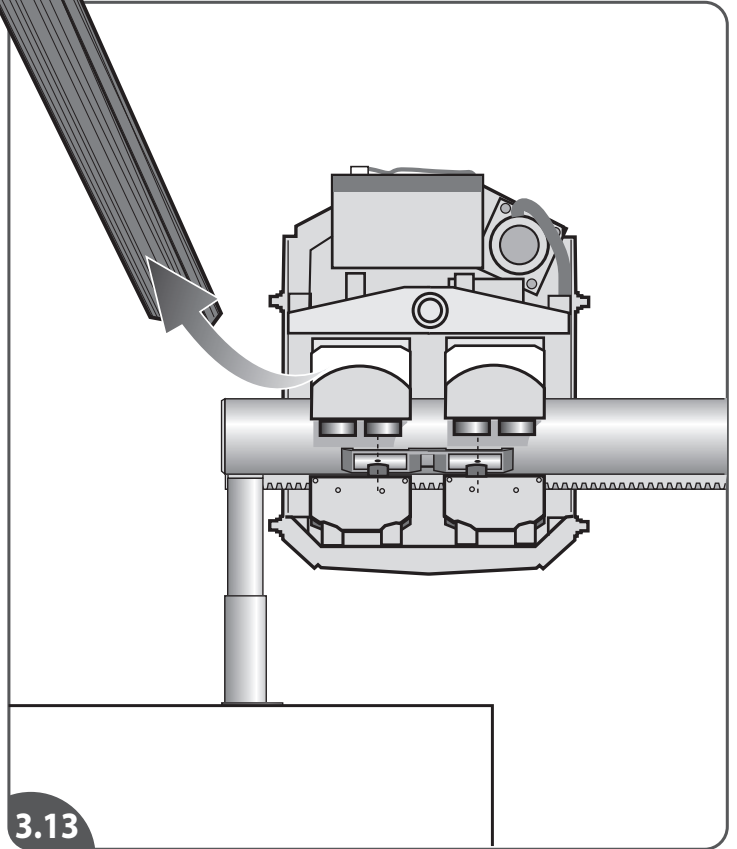
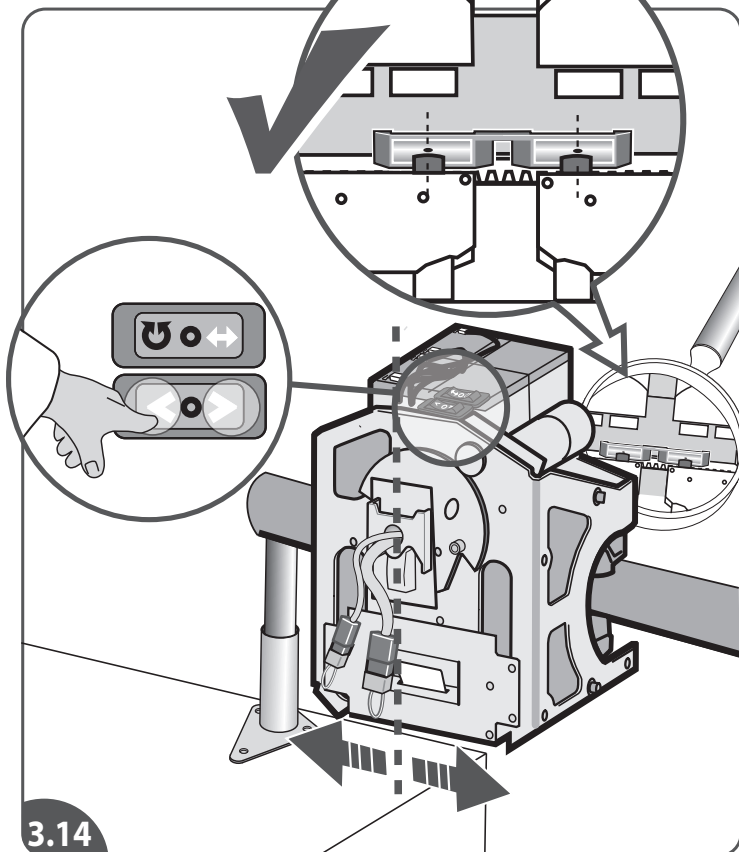
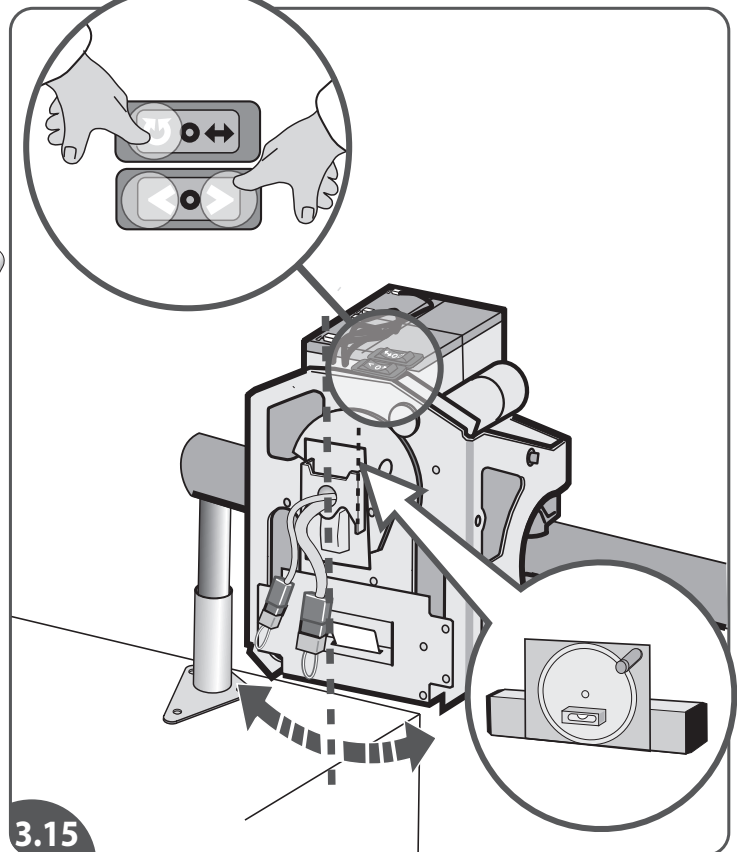
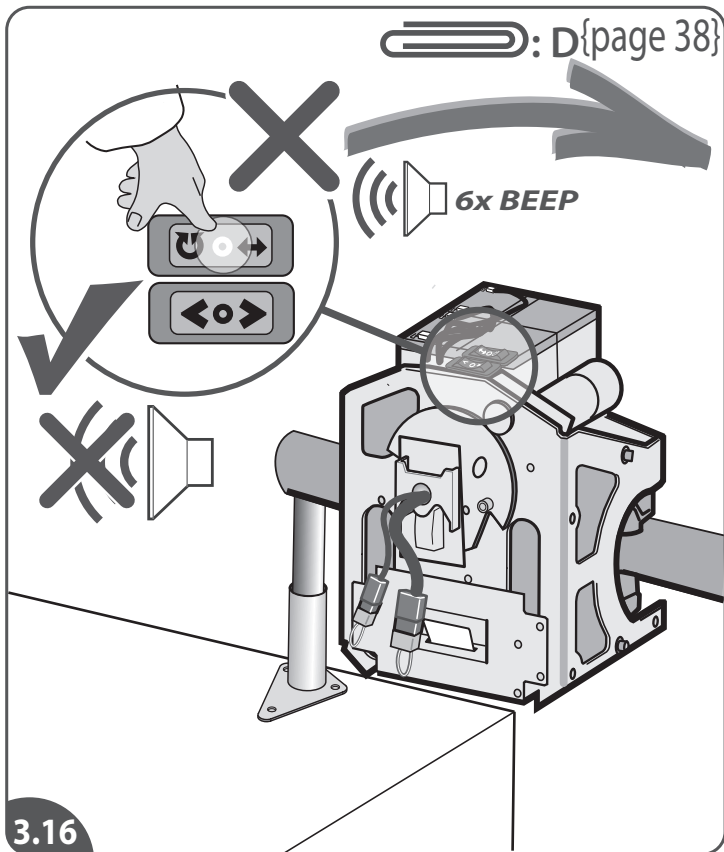
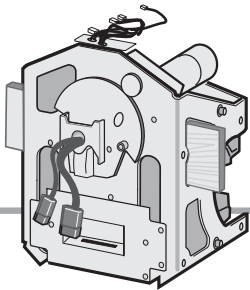
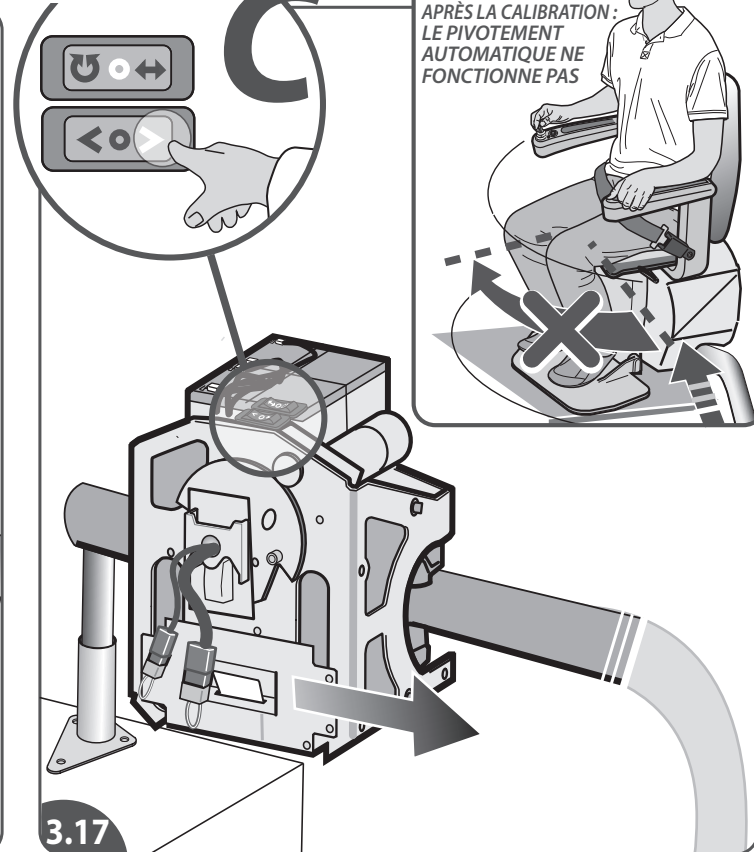
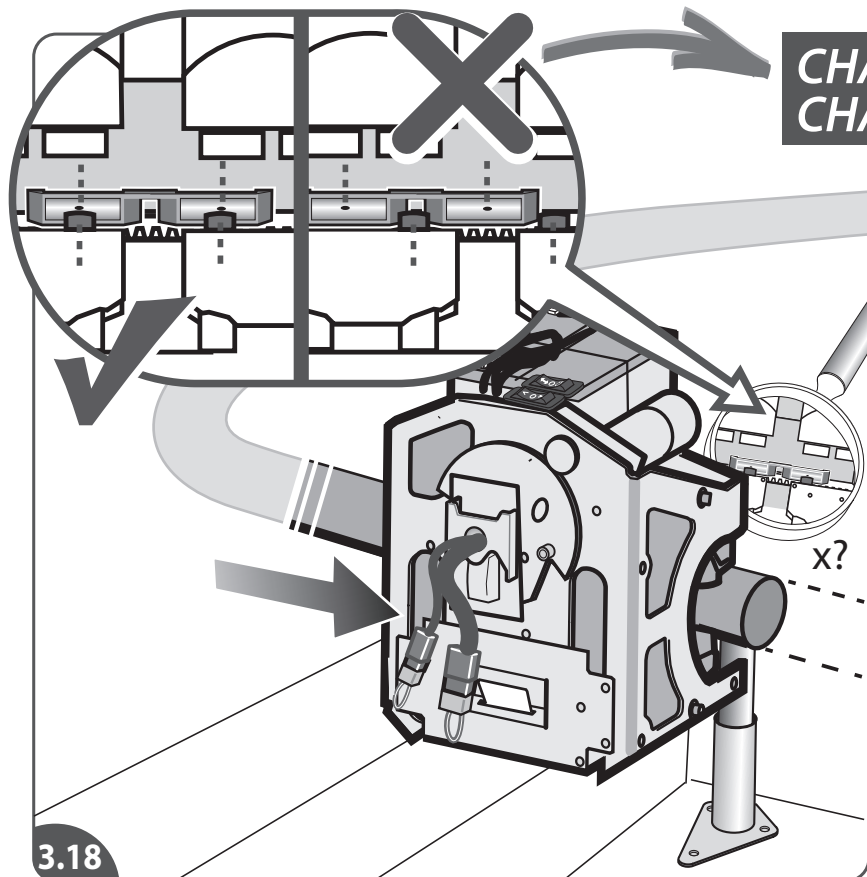
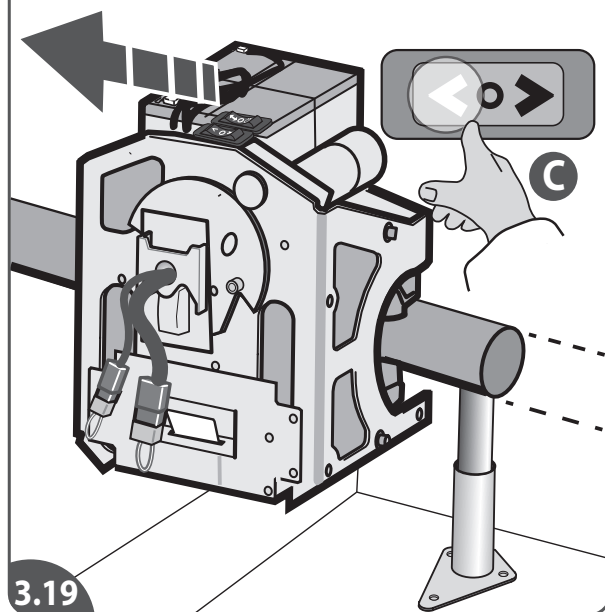
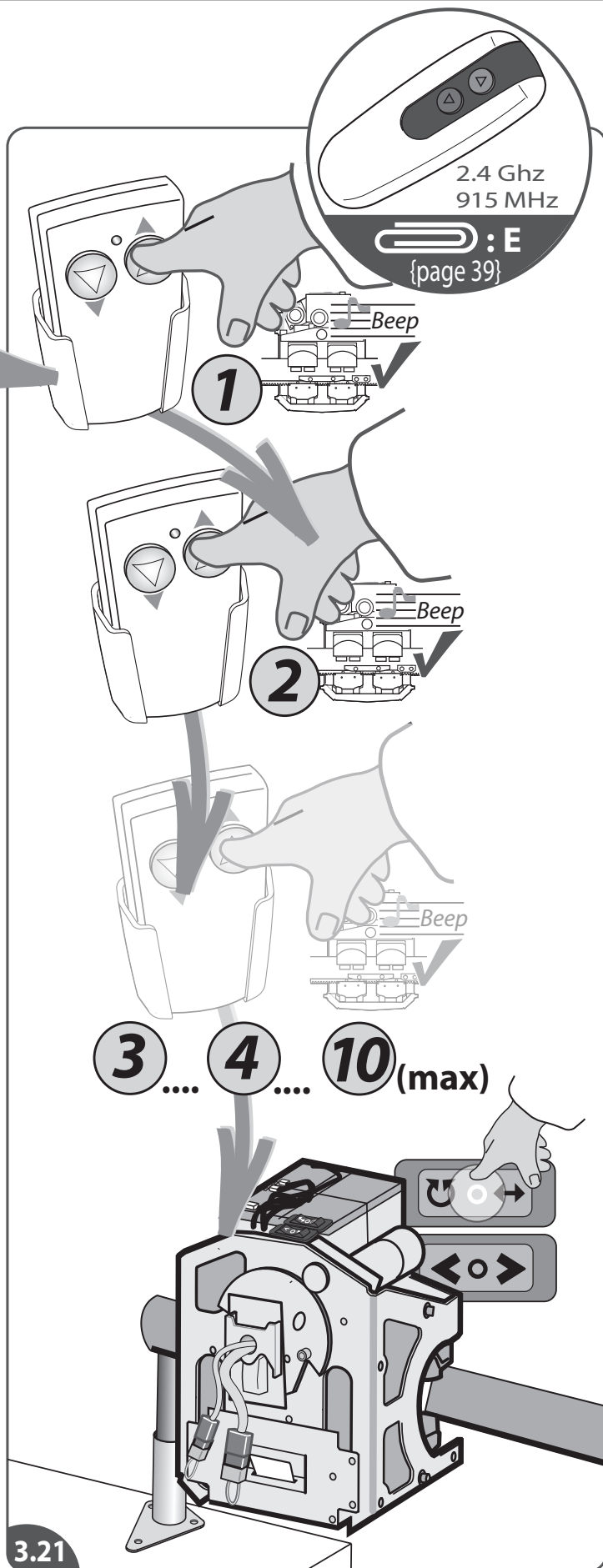
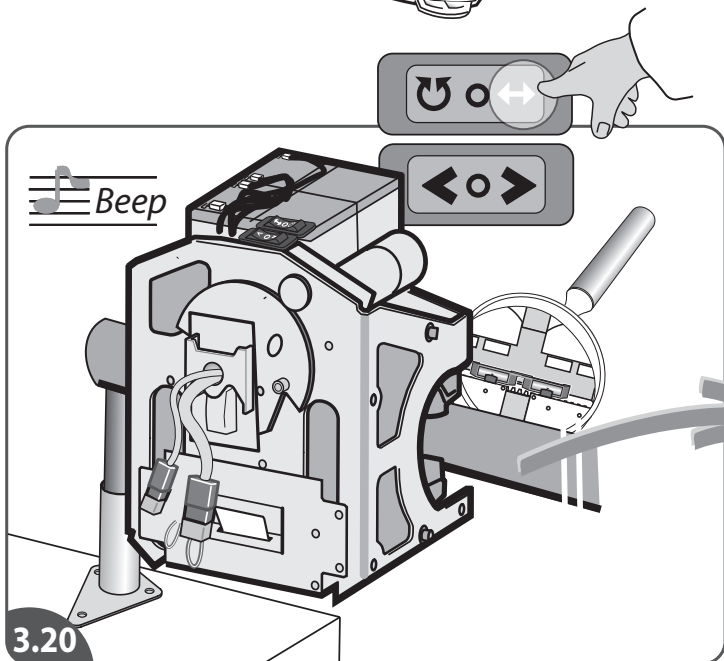
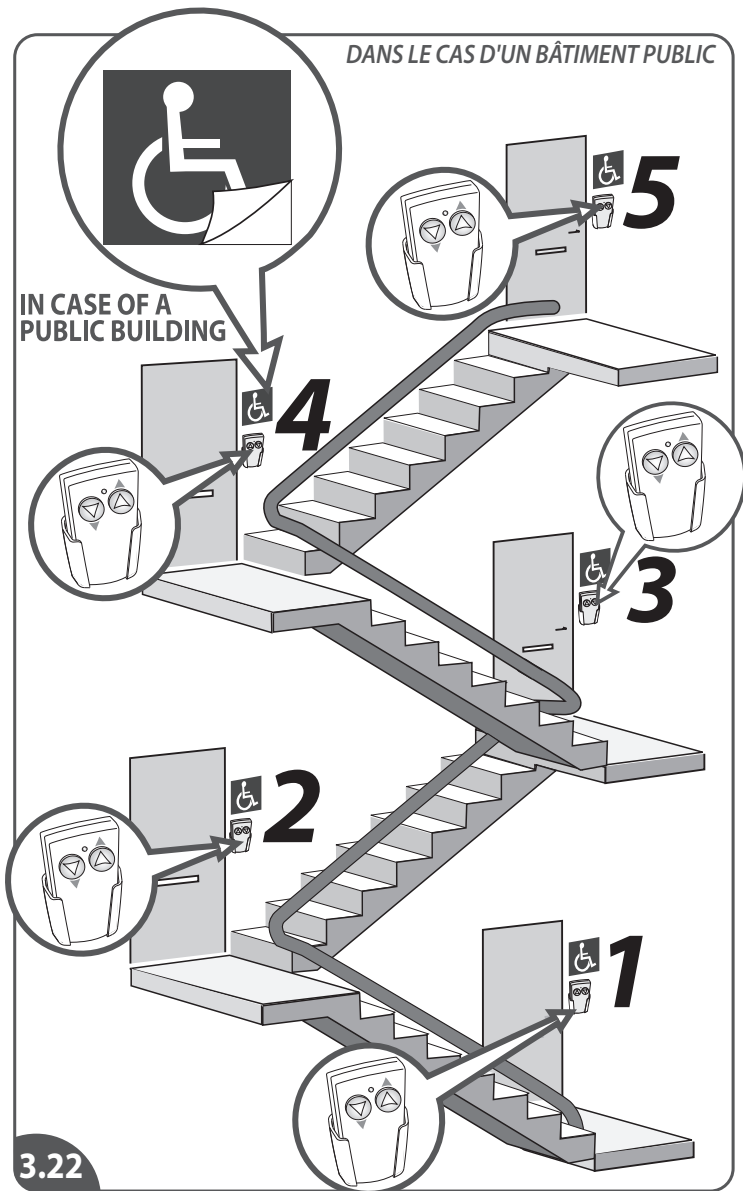
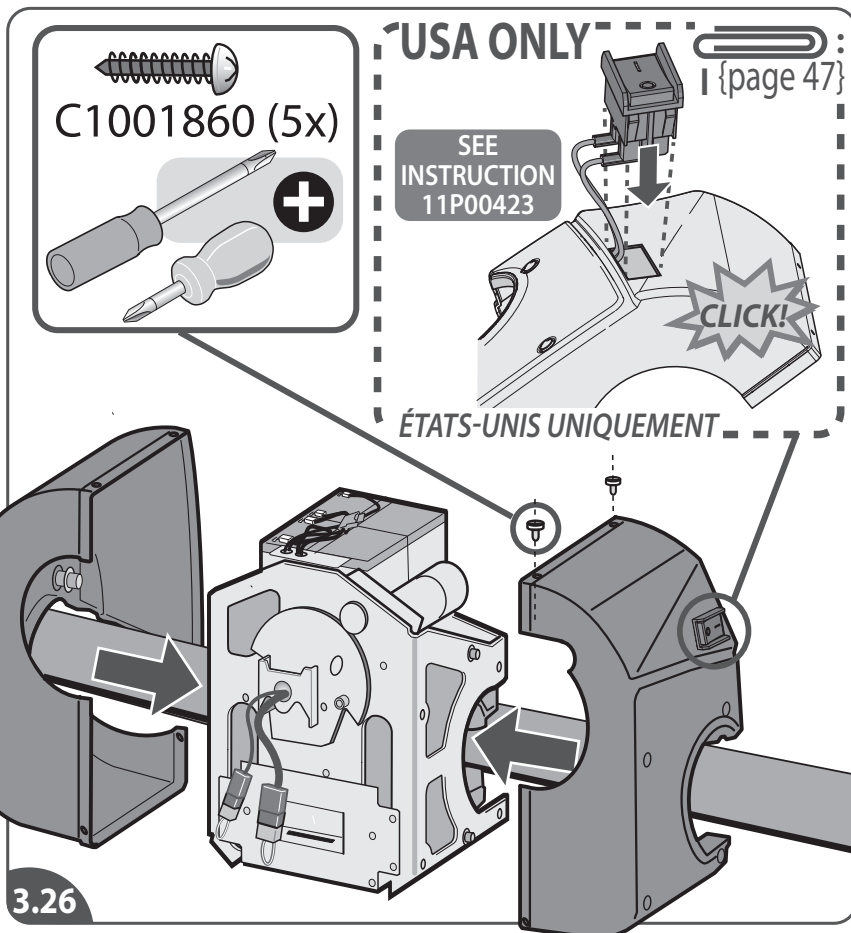
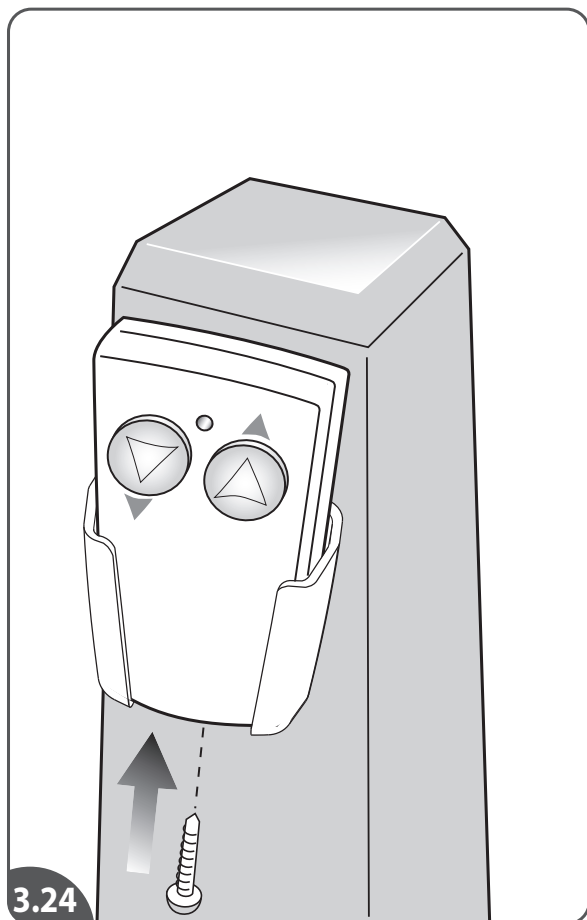
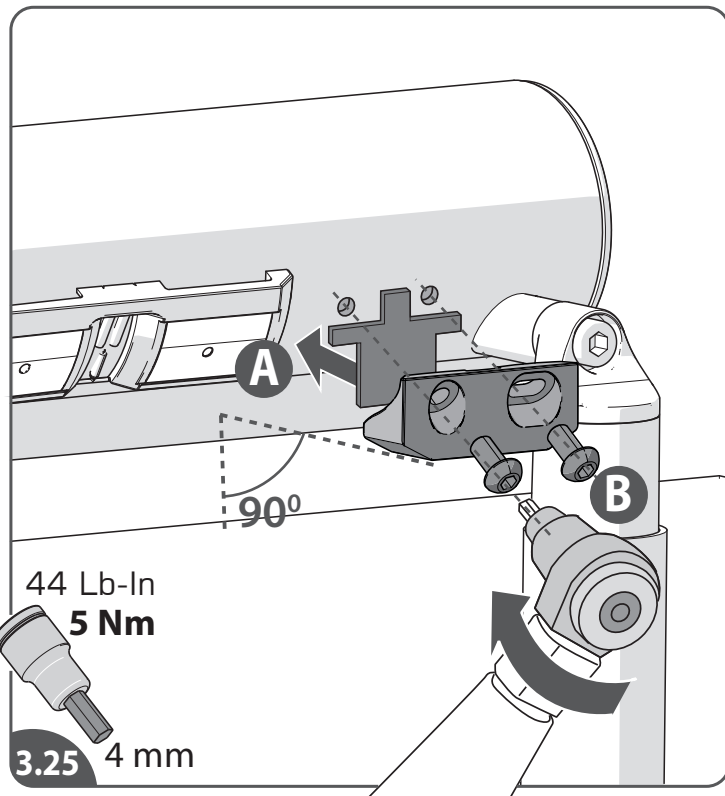
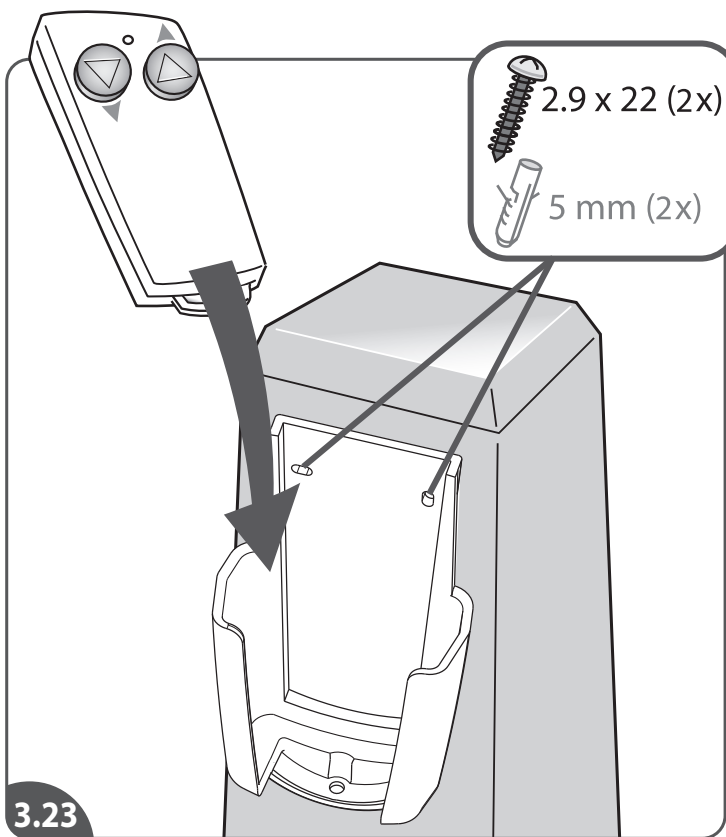
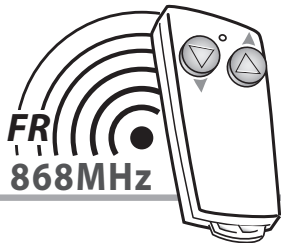
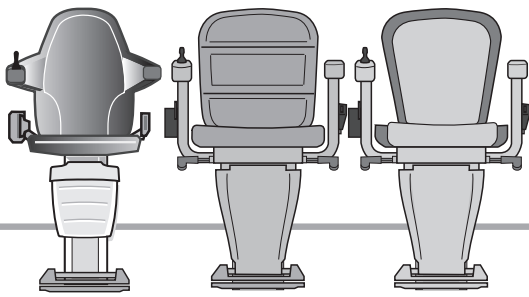
: **B**{page 34}

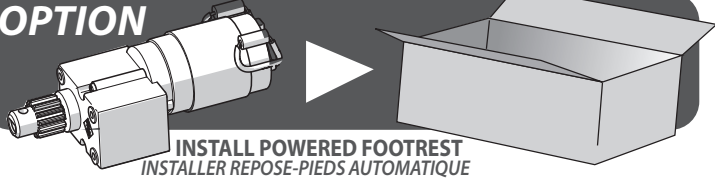
OPTION

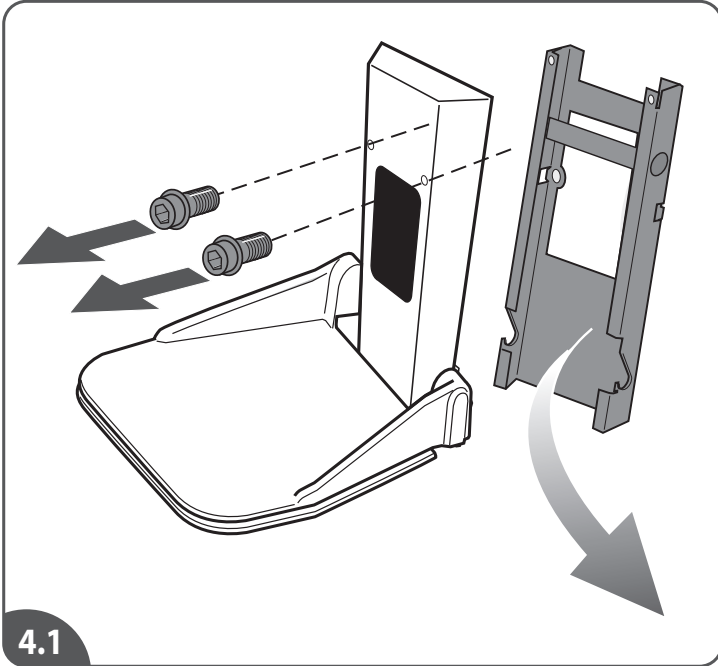INSTALL POWERED FOOTREST
INSTALLER REPOSE-PIEDS AUTOMATIQUE

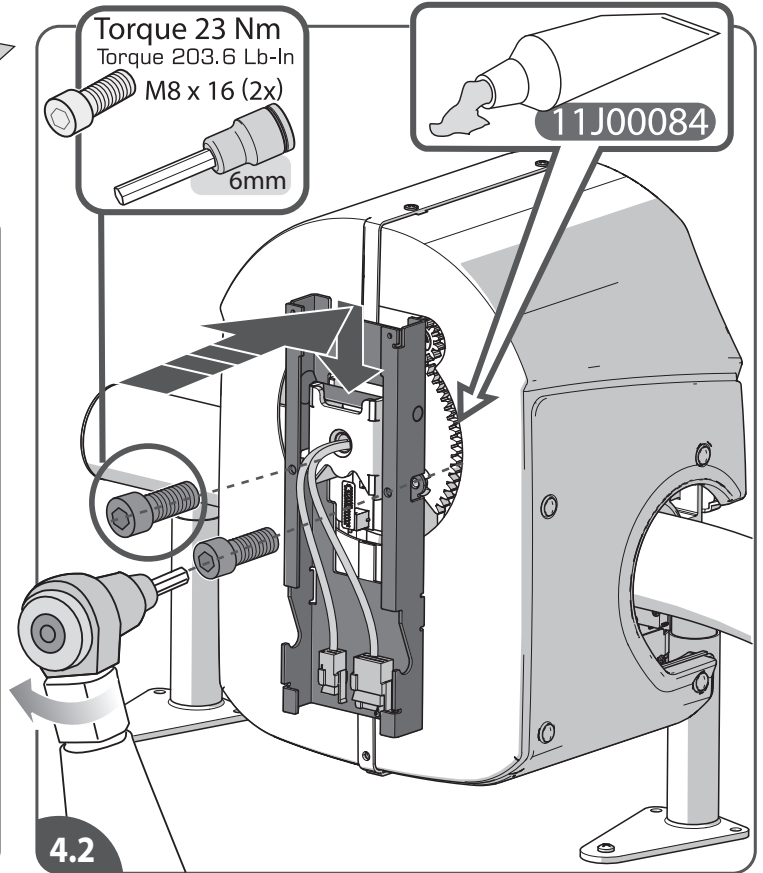4.1


4.2


CONFIGURATION FOOTREST DIRECTION

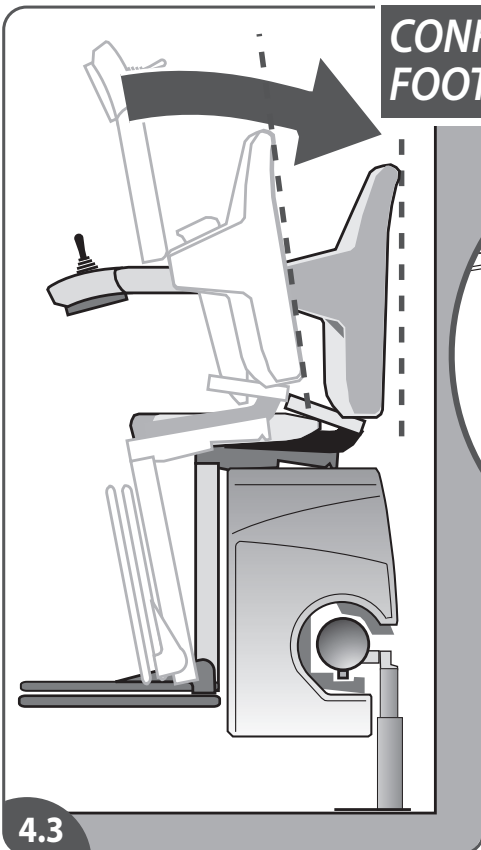Torque 5,5 Nm
Torque 48.7 Lb-In
M5 x 10 (2x)
4mm





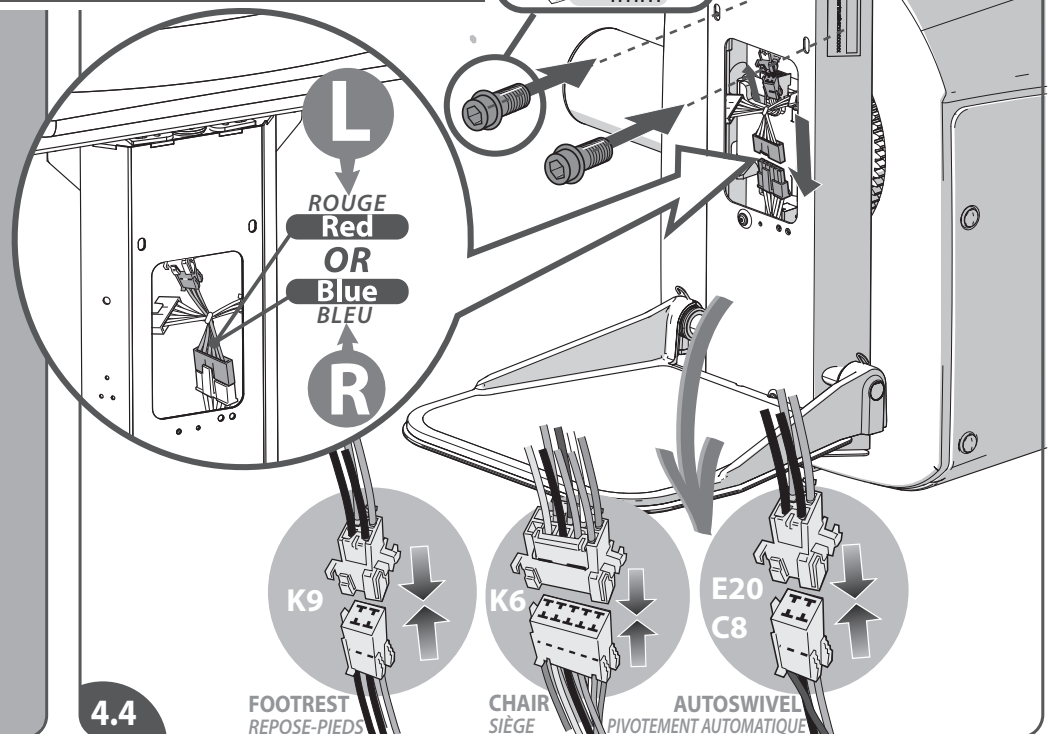
4.3


4.4

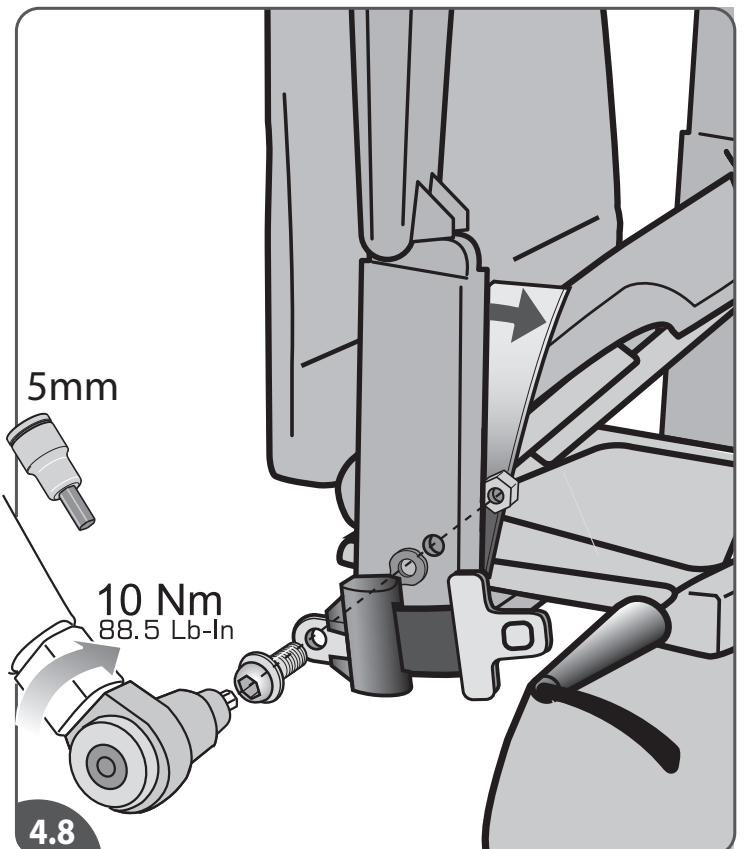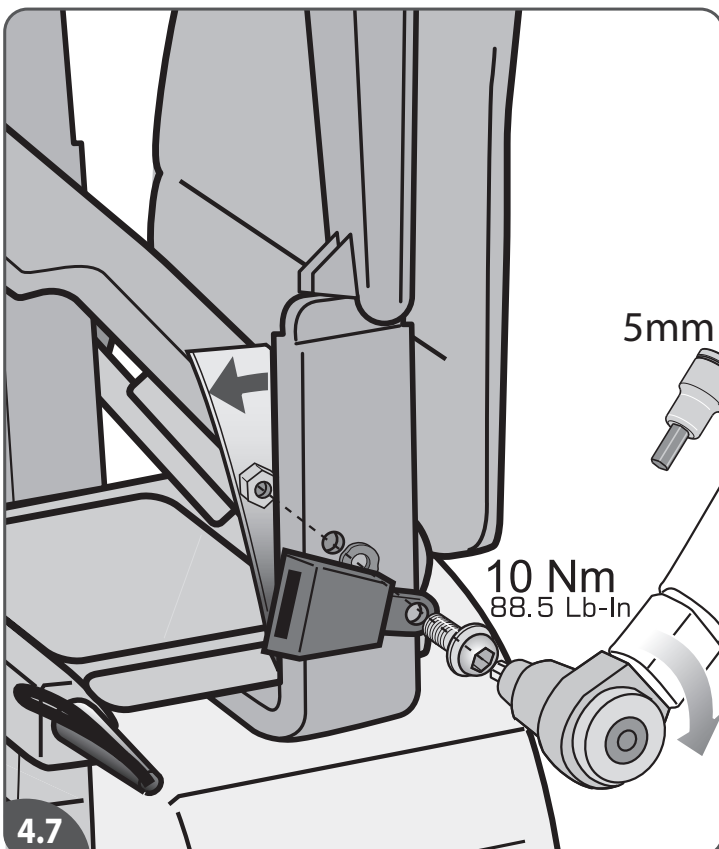
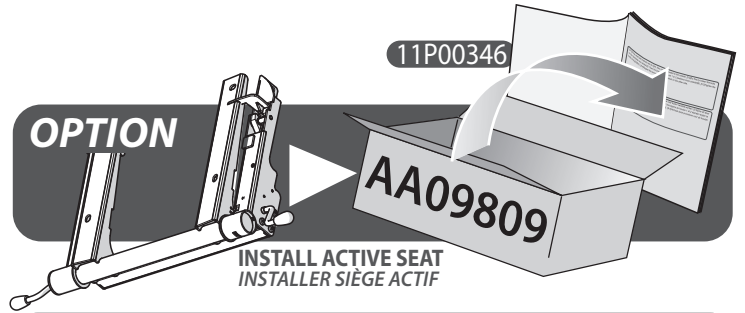
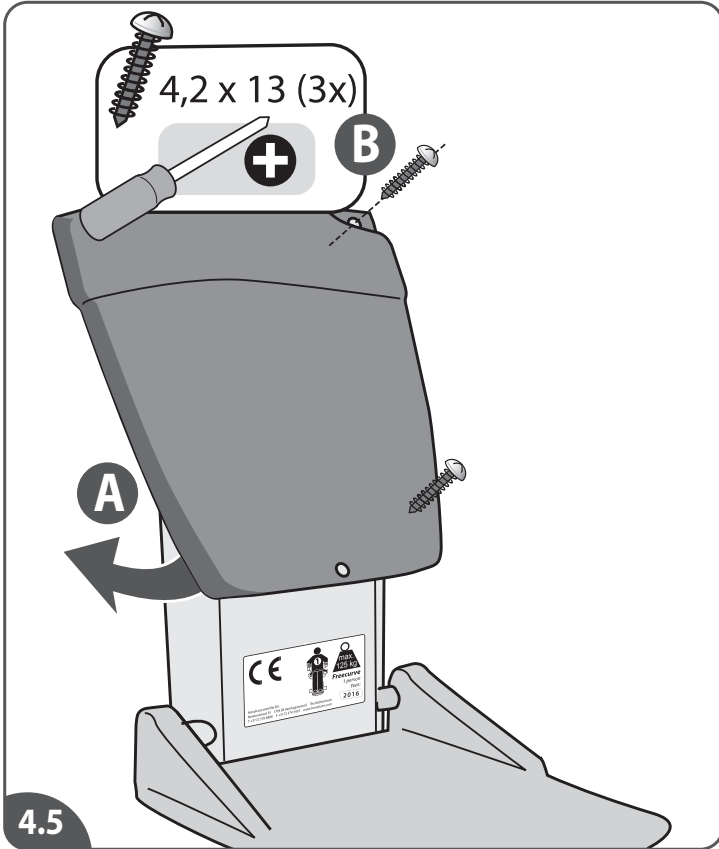
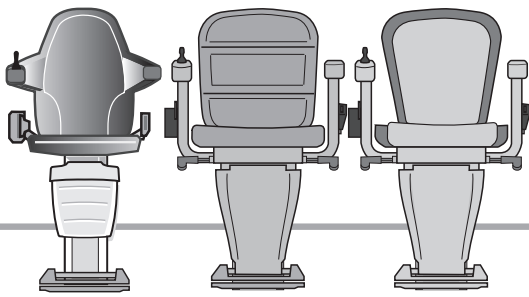
4


4





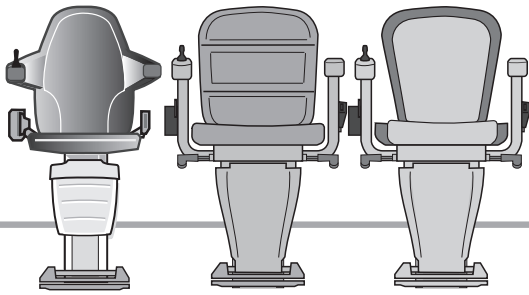
MANUAL

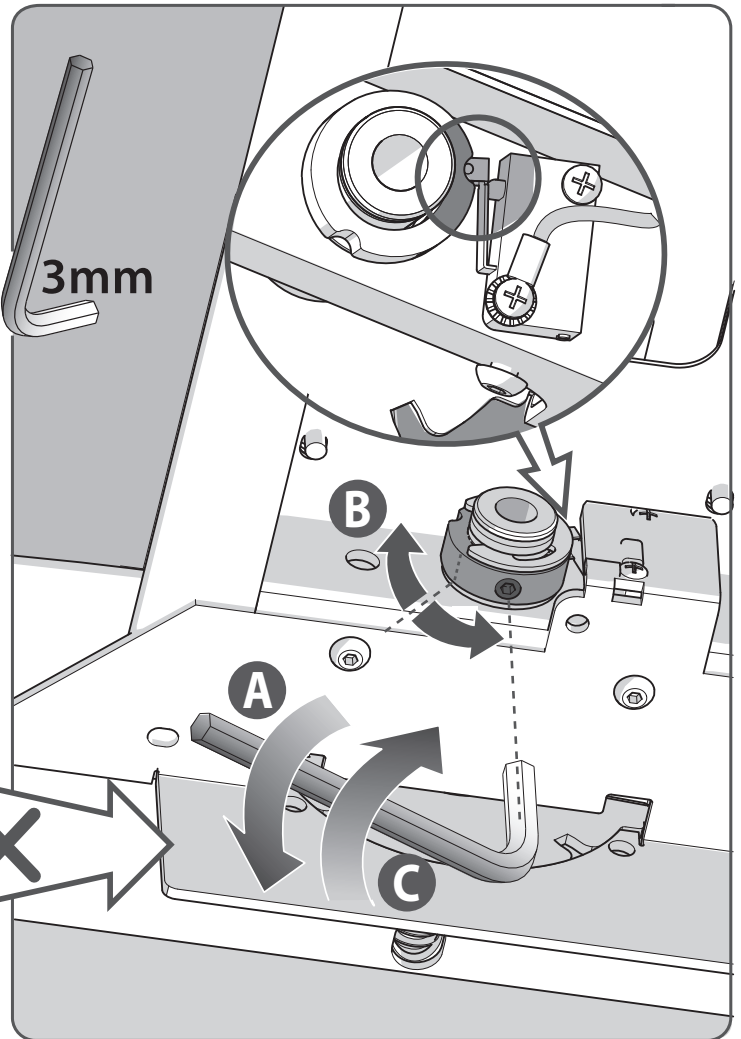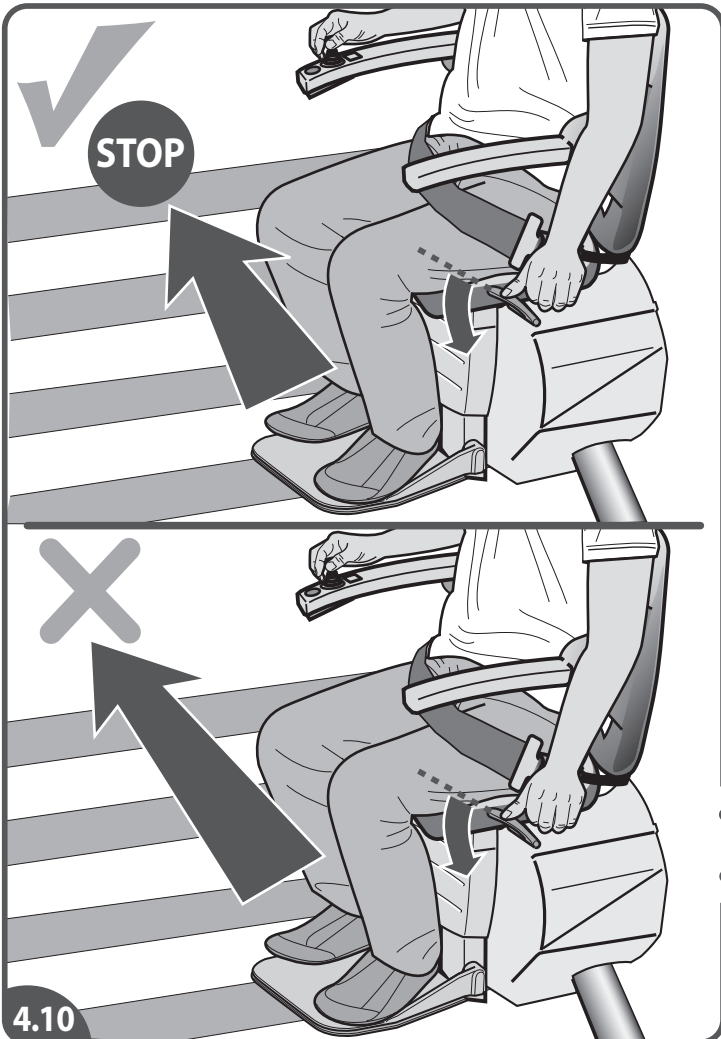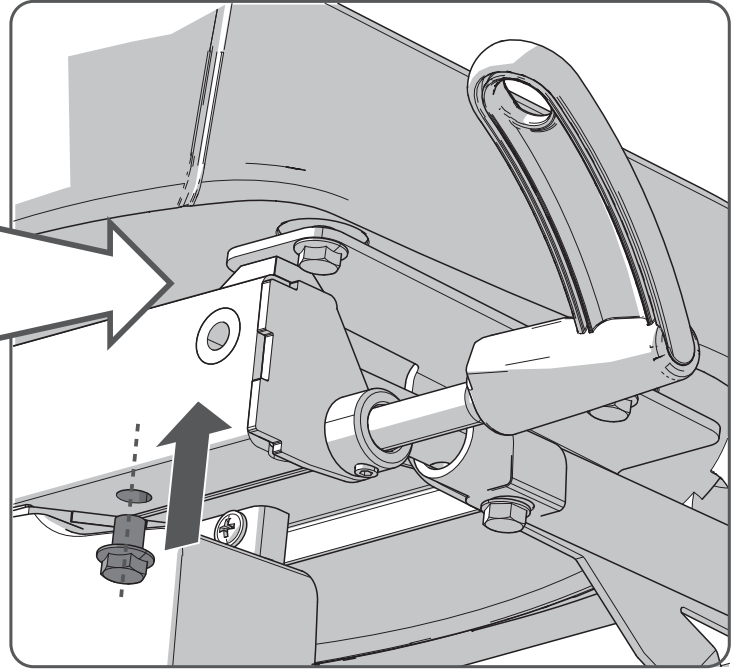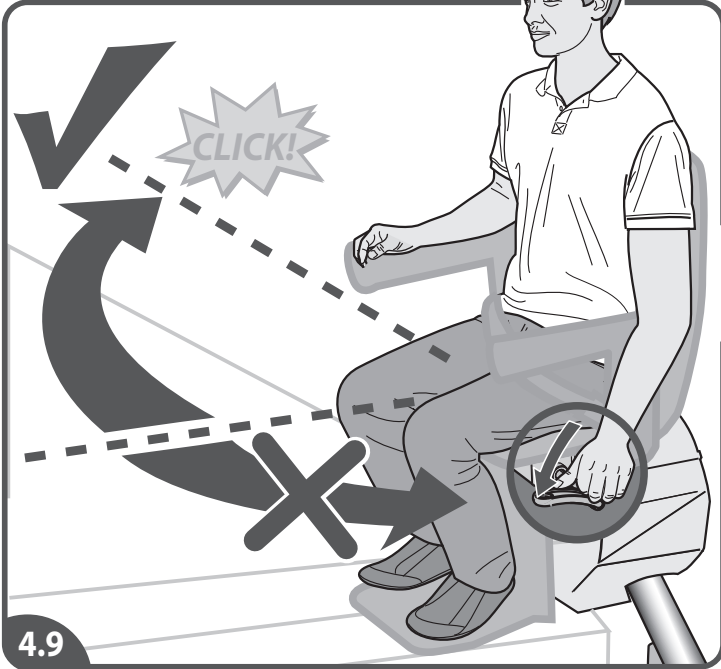
4

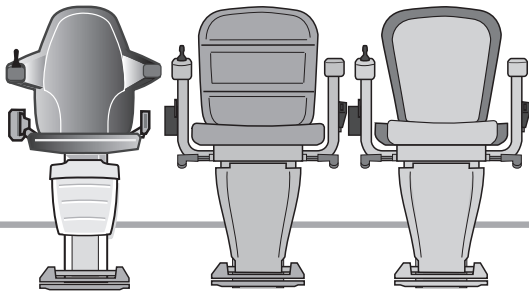POWERED

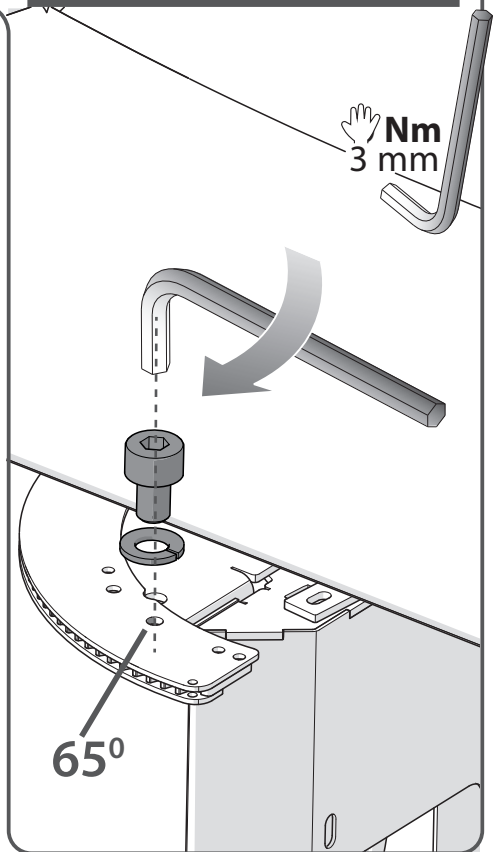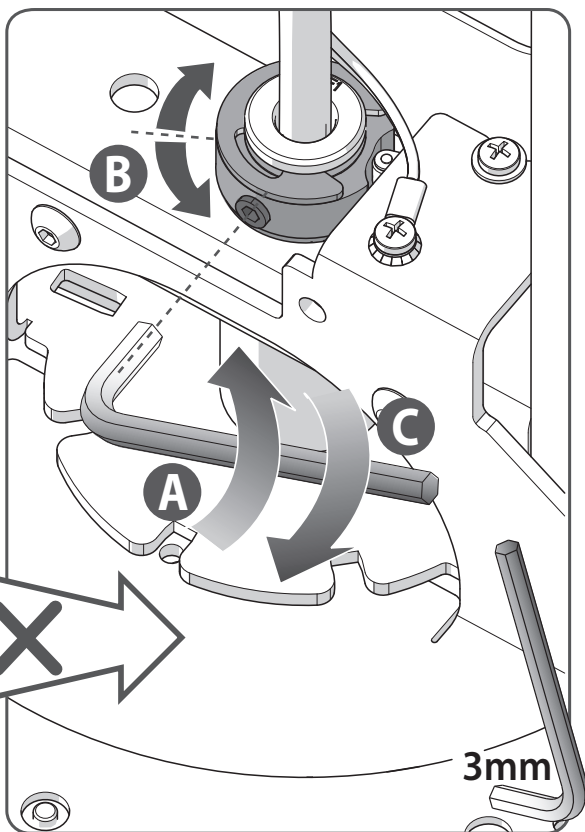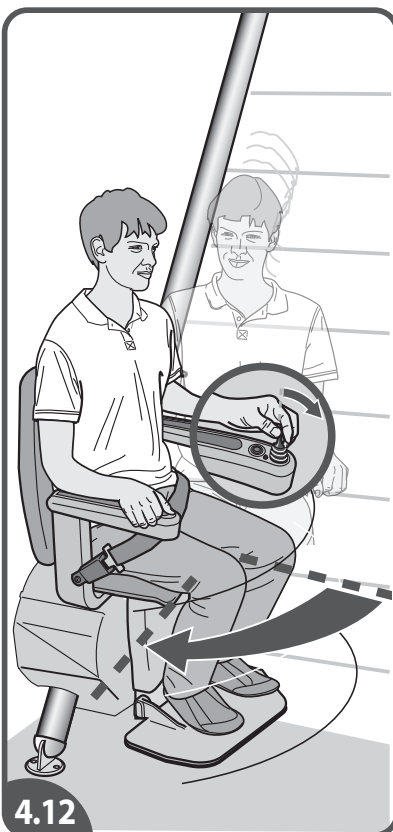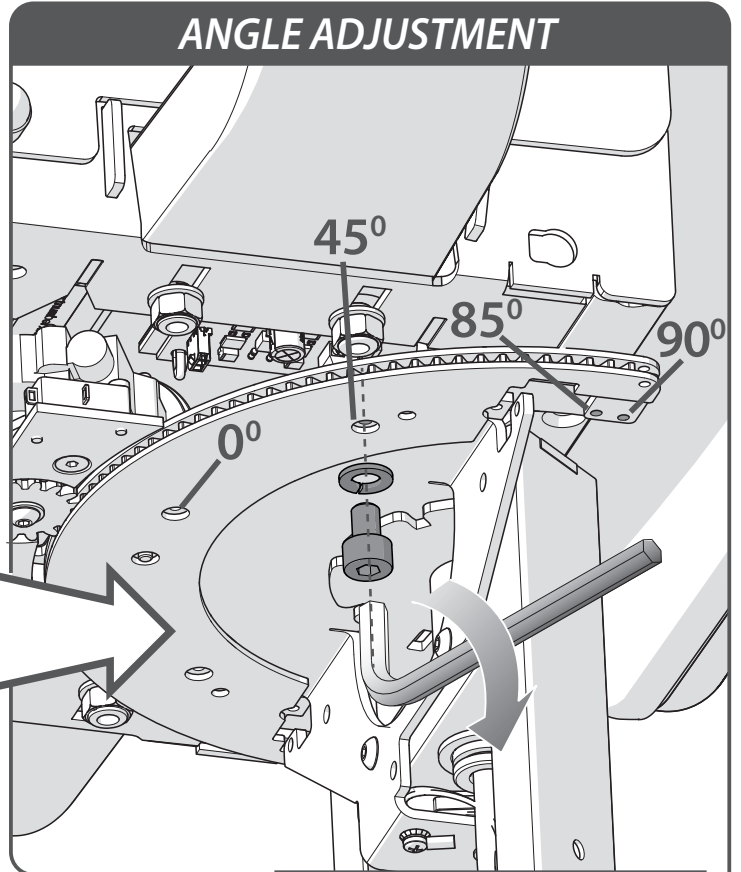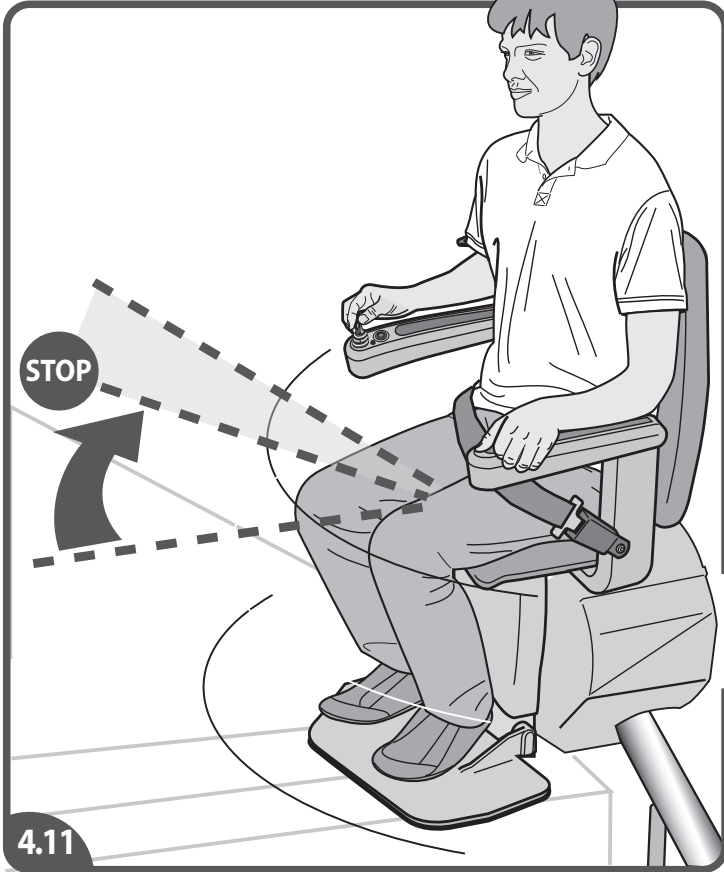5

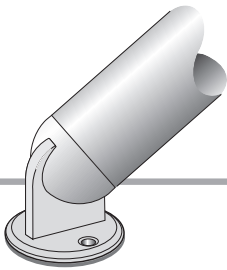11J00084

< 7 m < 276 inch	20 gr.	1 x
7-13 m 276-512 inch	20 gr.	2 x
13-19 m 512-748 inch	20 gr.	3 x
> 19 m > 748 inch	125 gr.	1 x

5.3

TIP

5.4

TIP
6,3 x 50 (3x)
AA20942

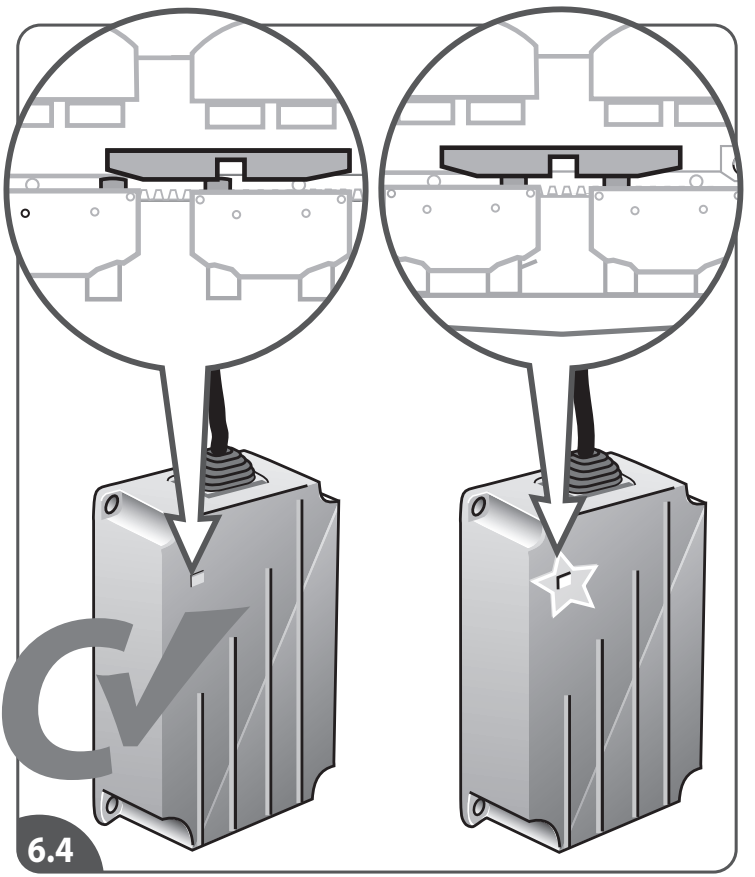
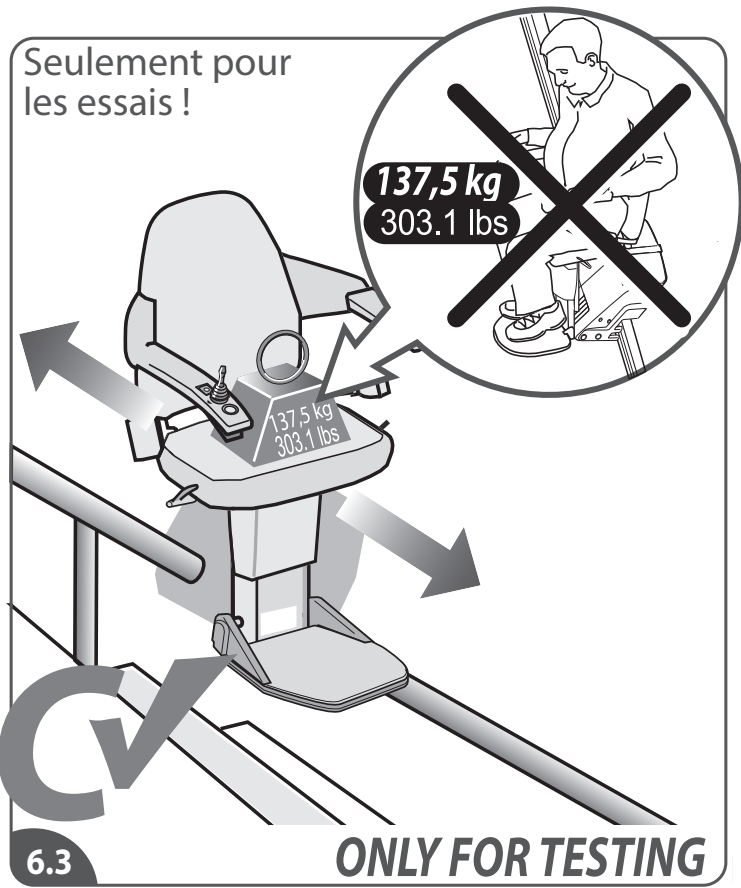
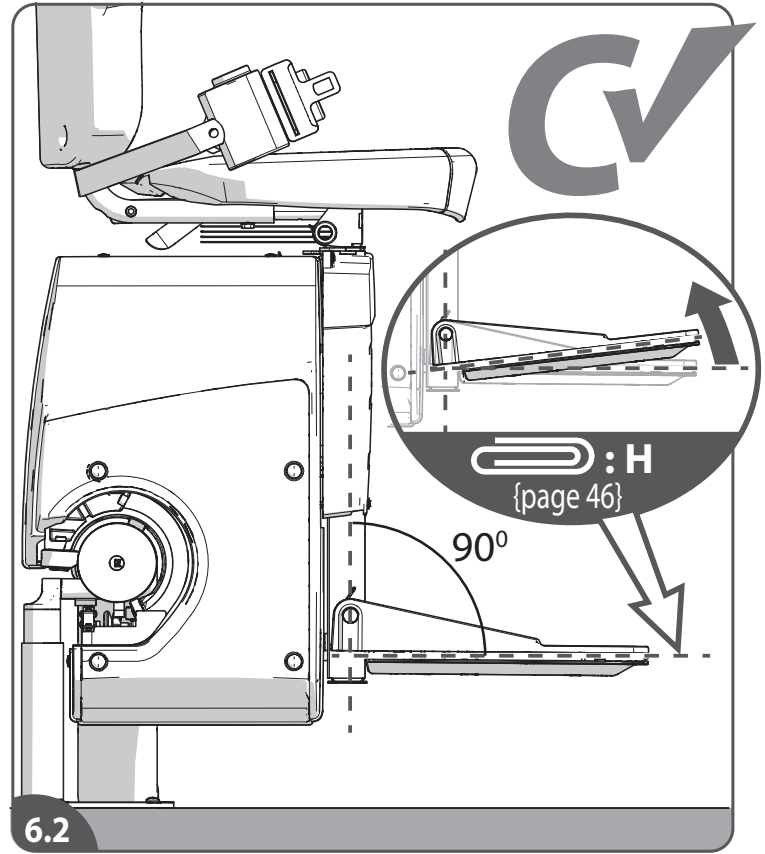
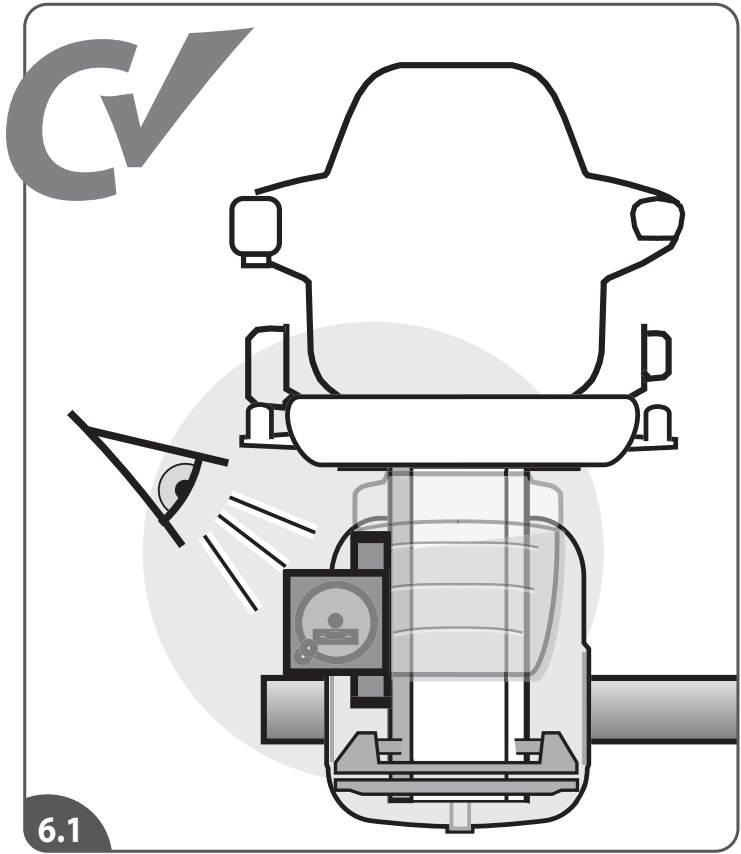
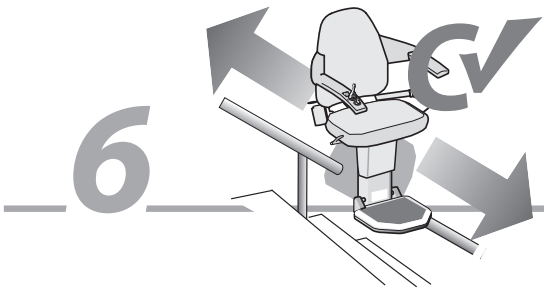
6.3 x 38 (3x)
8 mm (3x)
8 mm

5.5




6.3 x 38 (2x)
8 mm (2x)
8 mm


{page 42}

5.6

6

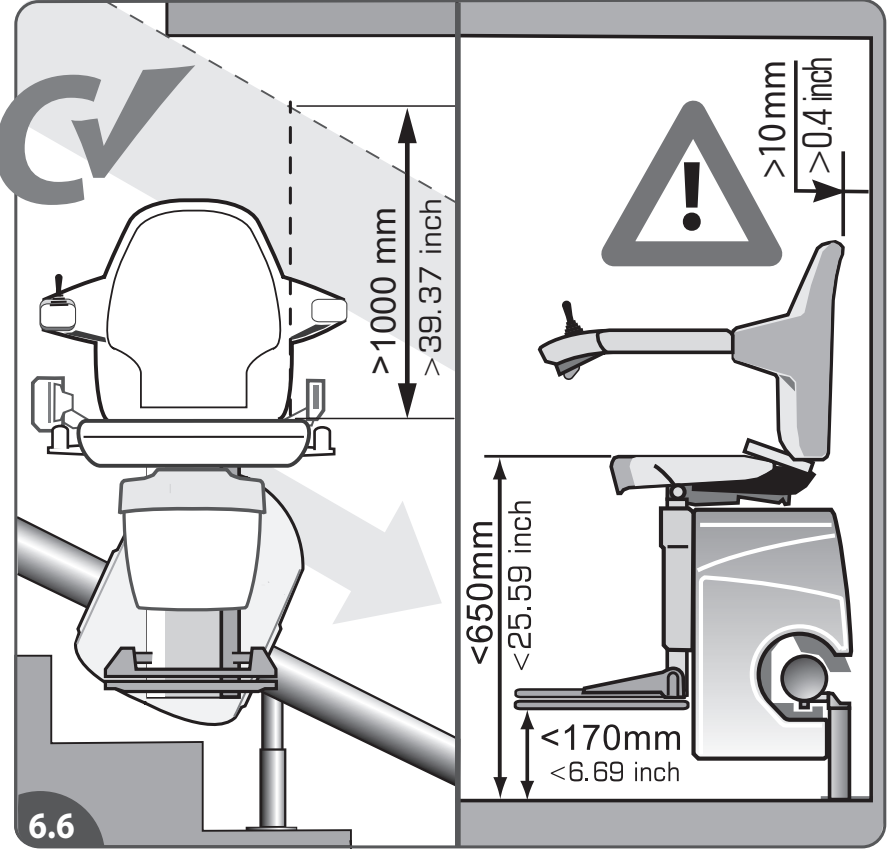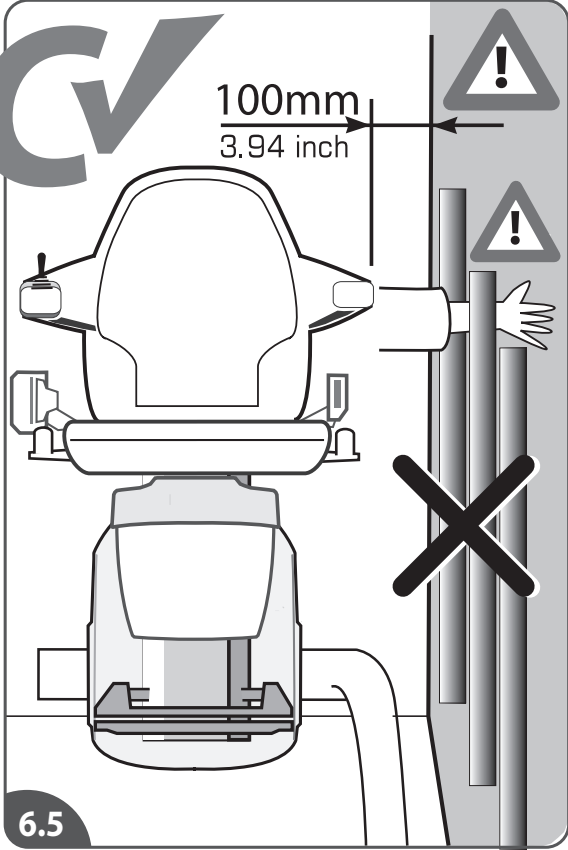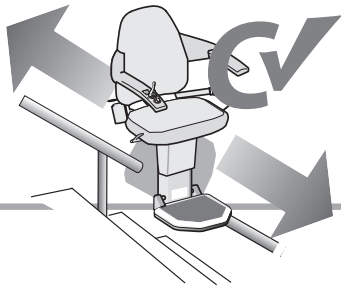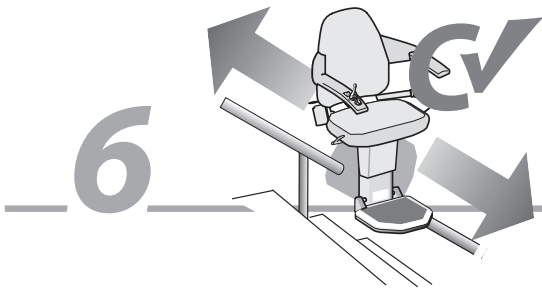FINAL CHECK

SAFETY

The declaration of conformity is legal after the installation checklist and the declaration instructed person have been signed off by the engineer and the user. The installation checklist and the declaration instructed person shall be saved by the dealer and made available to Handicare on request.

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INSTALLATION ▶ TEST/EXAMINATION

1 Description of installation

Location:	Manufacturer:	Yes
Rated load: kg (lbs/USA: lbs)	Stairlift serial no.:	
Rated speed in ft/min (ft/min)	Type:	
Contract electricity supply:	V Phase:	
Fuse rating:		
Confirm that the data of installation is filled in on the EC-declaration form	N/A	Yes
Confirm that the remote-control and hand on the wall operate correctly	N/A	Yes
Confirm that the safety belt has been fixed and functions correctly.	N/A	Yes

2 Rail

1) Confirm the railjoint fixations	N/A	Yes
2) Confirm the support fixations	N/A	Yes

3 Electrical drive unit tests

1) Confirm that the batteries do not leak:	N/A	Yes
2) Confirm that the batteries are securely fitted in the carriage structure.	N/A	Yes
3) Confirm that it is effectively indicated to the user when the stairlift is brought to rest out of reach of the charge contacts:	N/A	Yes
4) Confirm that the carriage chassis is effectively grounded:	N/A	Yes
5) Confirm that the polarity of the battery connection is correct	N/A	Yes

4 Clearance dimensions

1) Confirm in the correct clearance dimensions of chapter 6 (Int.Manual)	N/A	Yes
--	-----	-----

5 Mechanical tests

1) Confirm that the final limit switches operates correctly	N/A	Yes
Upper switch:	N/A	Yes
Lower switch:	N/A	Yes
2) Confirm that the joystick, the keyboard, emergency stop and locking switch on the armrest operate correctly.	N/A	Yes
3) Confirm that the sensitive edges and surfaces operate correctly.	N/A	Yes
4) Confirm that the swivel seat interlock and switch operate correctly.	N/A	Yes
5) State the stopping distance (max. 2 cm / 0.78 inch) of the carriage (with rated load) upon application of brake (for distance USB, use inches)		
up	mm (inch)	
down	mm (inch)	
6) Confirm that the emergency operation mechanism operates satisfactorily	N/A	Yes
7) Undergo without failure a dynamic test (with a load of 137.5 kg/305.1 lbs) to check the forces.	N/A	Yes
8) Confirm all stickers have been applied to the product imposed by the maximum working load	N/A	Yes
9) Confirm the function of the VoiceConnect (emergency alarm) if installed.	N/A	Yes

6.9

6 Electrical tests

a) Confirm that the polarity of the mains supply connection is correct:	N/A	Yes
b) State insulation resistance to earth	Ω	Moim
c) State motor reversal relay delay	s	

7 Declaration engineer

I/We certify that onI/We thoroughly examined this lift and that the foregoing is a correct report of the result.
I/We declare that the erected stairlift is compound with approved components and installed according to the instructions in the installation manual. I/We also declare that I/we have instructed the user by telling him/her all users practice.

Name Engineer: _____ Name Engineer: _____
 Qualifications: _____ Qualifications: _____
 Signing date: _____ Signing date: _____
 Signature: _____ Signature: _____
 Address: _____ Address: _____
(if employed by a company/association/nationality)

8 Declaration instructed person

Herewith confirms the user of the stairlift that the above mentioned engineer(s) has declared all users practice and that he/she is known with the complete operation as instructed person.

Name: _____ Address: _____
 City: _____ Phone number: _____
 Signing date: _____ Signature: _____

QUALITY CHECKLIST

	PACKAGING	COMPLETENESS	FUNCTIONING
RAIL	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
CHAIR	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
UNIT	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
REMARKS/IDEAS			

Fax (+31(0)72 5768877) or mail (data@handicare.com) a filled in and signed form to Handicare.

6.10

6 Electrical tests

a) Confirm that the polarity of the mains supply connection is correct:	Yes
b) State insulation resistance to earth	
c) State motor reversal relay delay	

7 Declaration engineer

I/We certify that onI/We thoroughly examined this lift and that the foregoing is a correct report of the result.
I/We declare that the erected stairlift is compound with approved components and installed according to the instructions in the installation manual. I/We also declare that I/we have instructed the user by telling him/her all users practice.

Name Engineer: _____ Name Engineer: _____
 Qualifications: _____ Qualifications: _____
 Signing date: _____ Signing date: _____
 Signature: _____ Signature: _____
 Address: _____ Address: _____
(if employed by a company/association/nationality)

8 Declaration instructed person

Herewith confirms the user of the stairlift that the above mentioned engineer(s) has declared all users practice and that he/she is known with the complete operation as instructed person.

Name: _____ Address: _____
 City: _____ Phone number: _____
 Signing date: _____ Signature: _____

QUALITY CHECKLIST

	PACKAGING	COMPLETENESS	FUNCTIONING
RAIL	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
CHAIR	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
UNIT	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
REMARKS/IDEAS			

Fax (+31(0)72 5768877) or mail (data@handicare.com) a filled in and signed form to Handicare.

6.11

IMPORTATION

EC DECLARATION OF CONFORMITY FOR MACHINERY
(Directive 2006/42/EC)

Manufacturer: Handicare
 Address: Newtonstraat 35
 1704 SB Heerhugowaard
 Postbus 39
 1700 AA Heerhugowaard
 Holland
 Tel.nr. +31 (0)72-5768888

Herewith declares that the

Freecurve

is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2006/95/EC and EMC Directive 2014/53/EU.

The stairlifts are 24V DC Powered Stairlifts moving in an inclined plane, used with an approved battery charger product.

The stairlift has been designed and manufactured in accordance with the following standards:

EN 81-40, ISO 9386-2, EN ISO 12100-1,
 EN ISO 12100-2, EN ISO 14121,
 EN60204-1, (including the EMC standards).

Liftnestuur B.V., Buikslotermeerplein 361, 1025 XE, Amsterdam, The Netherlands.
 Identification: D400 has issued an EC-certificate in compliance with above mentioned directives and standards with reference NL02-400-1001-015-006.

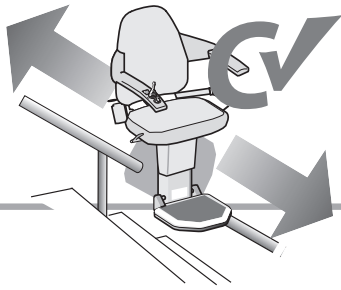
The Technical Construction File for the above Stairlift is retained at the manufacturers address. A copy is retained at the Liftnestuur B.V. address.

Heerhugowaard, The Netherlands, 15 may 2017

K.J. Ruder
 Managing director Handicare stairlifts HHW



Project: _____
 Date: _____

6

FRANCE

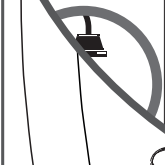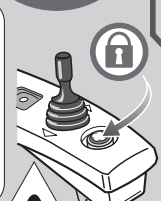REST OF WORLD

€

max. 125 kg

Freecurve

1 person

Year: _____

YEAR

USER MANUAL

Handicare stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T. +3172 576 8888, www.handicare.com

€

max. 125 kg

Monorail

1 personne

Année: _____

YEAR

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Handicare stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T. +3172 576 8888, www.handicare.com

2020 2021 2022

A

USA

handicare Model

Handicare Stairlifts B.V.
Newtonstraat 35, 1704SB Heerhugowaard
The Netherlands
T: +31 72 576 8888, www.handicare.com

DRIVE: RACK & PINION

RATED LOAD: 276lbs (125kg)

RATED SPEED:
21.7 ft/min (0.11m/s)

CONFORMS TO ASME A18.1 & ASME A17.5
CERTIFIED TO CSA B441.1 & CSA B355
DESIGNED AND TESTED TO IEC 60601-1-2
Indoor use only. Utilisation intérieure seulement.
One person only. Une seule personne.
Stairlift for persons with physical disabilities.
Monte-escalier pour personnes handicapées.

ASME A18.1 TO BE USED FOR INSPECTIONS AND TESTS

RATED INPUT FROM CHARGER: 1.5A/24V DC
OUTPUT TO MOTORS: 30A/24V DC

DUTY CYCLE: 30%

IP RATING: IP12

DATE OF INSTALLATION: _____

NAME OF INSTALLER: _____

LABEL ON POWERPACK

handicare

11P03021.5

C

B

CE

max. 125 kg

Freecurve

1 person

USER MANUAL

Handicare stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T. +3172 576 8888, www.handicare.com

2020 2021 2022

CE

max. 125 kg

Monorail

1 personne

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Handicare stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T. +3172 576 8888, www.handicare.com

6.12

ATTACHMENTS

ANNEXE

ANEXO

ANHANG

ANNEXE

APPENDICE

BILAG

BIJLAGE

ZALACNIK

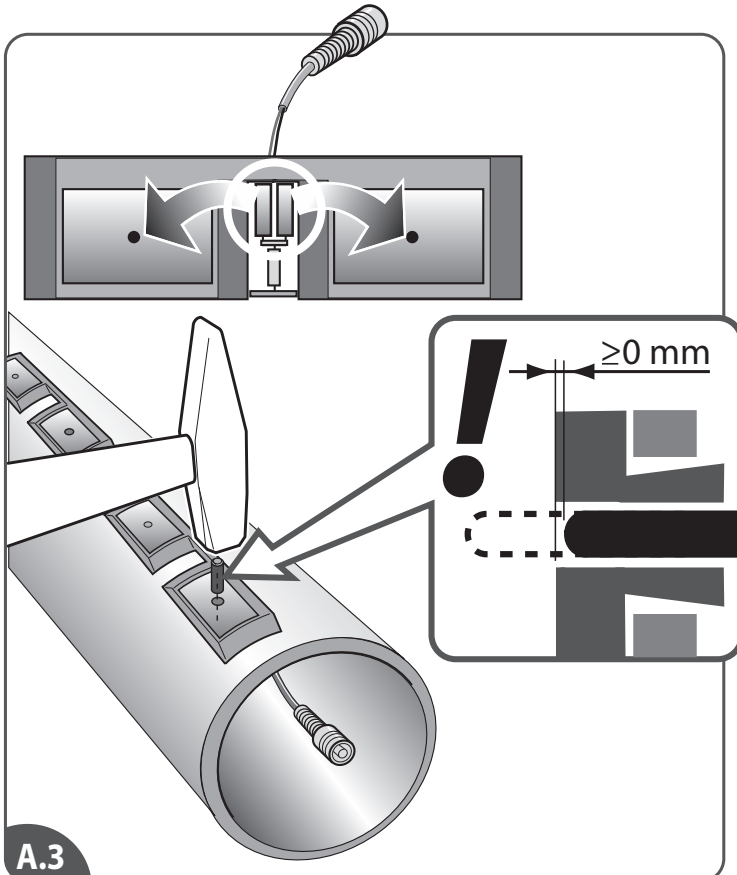
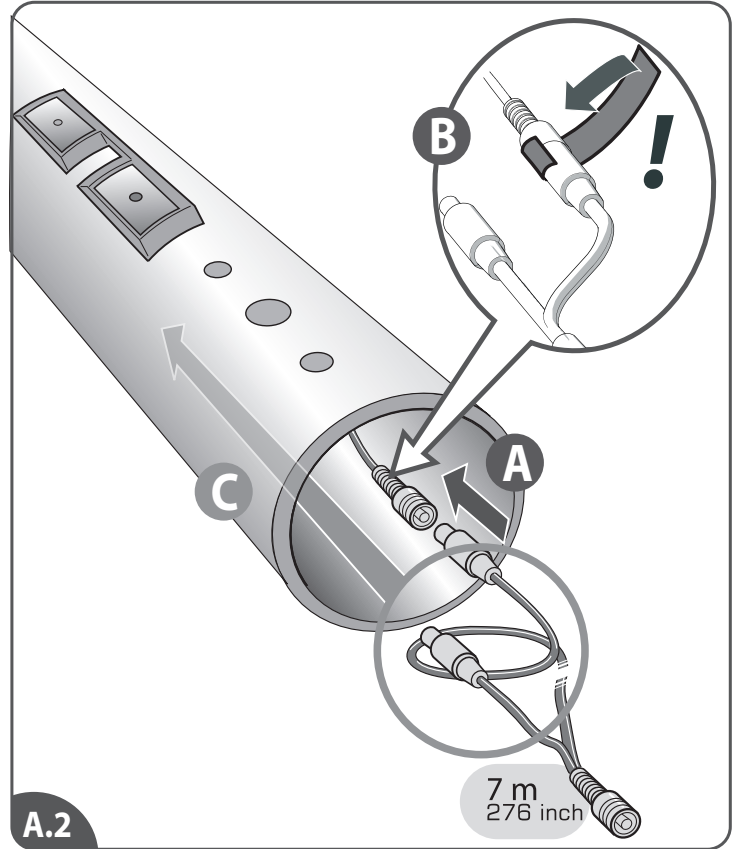
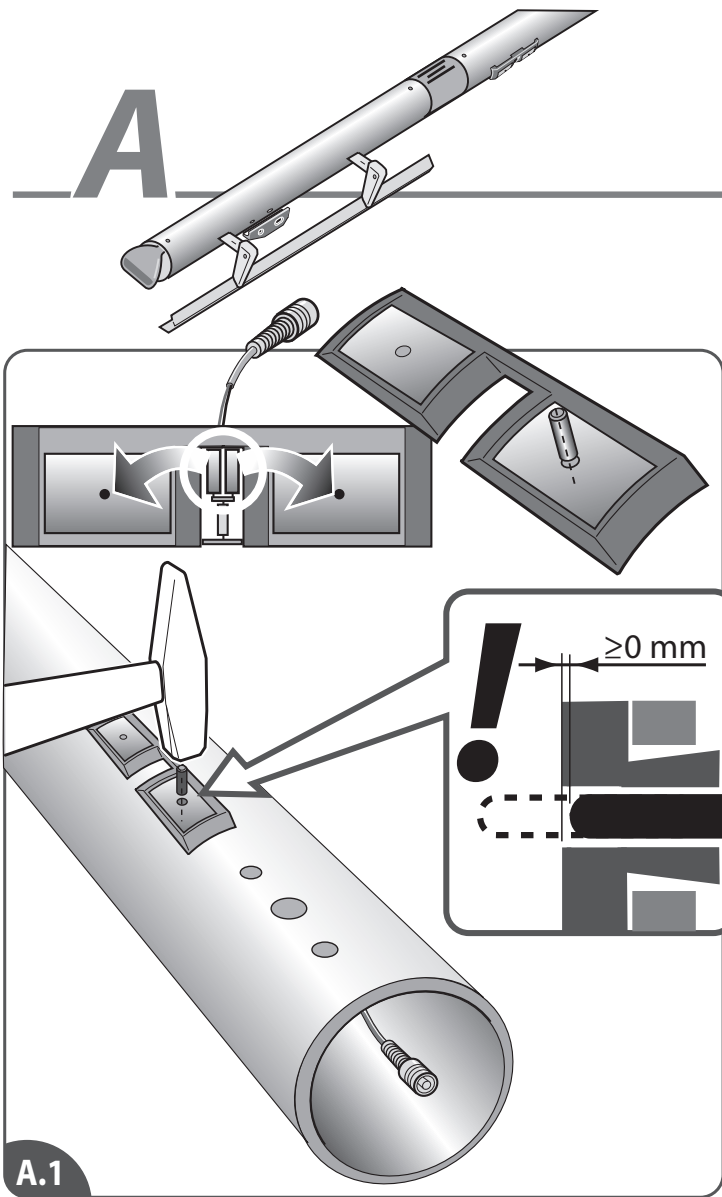
ANEXO

BILAGO


TILBEHØR






A




HINGE RAIL ESCAMOTABLE

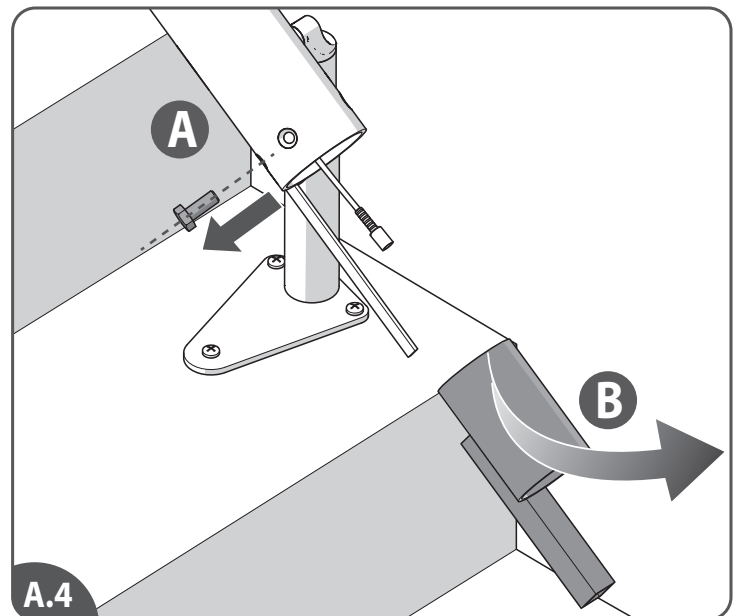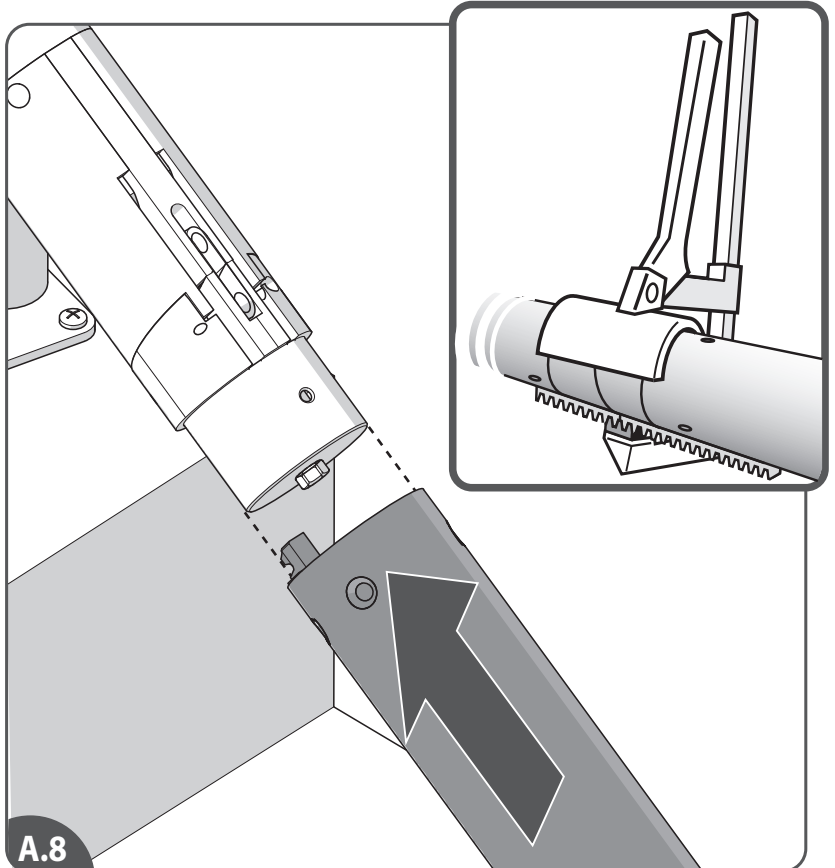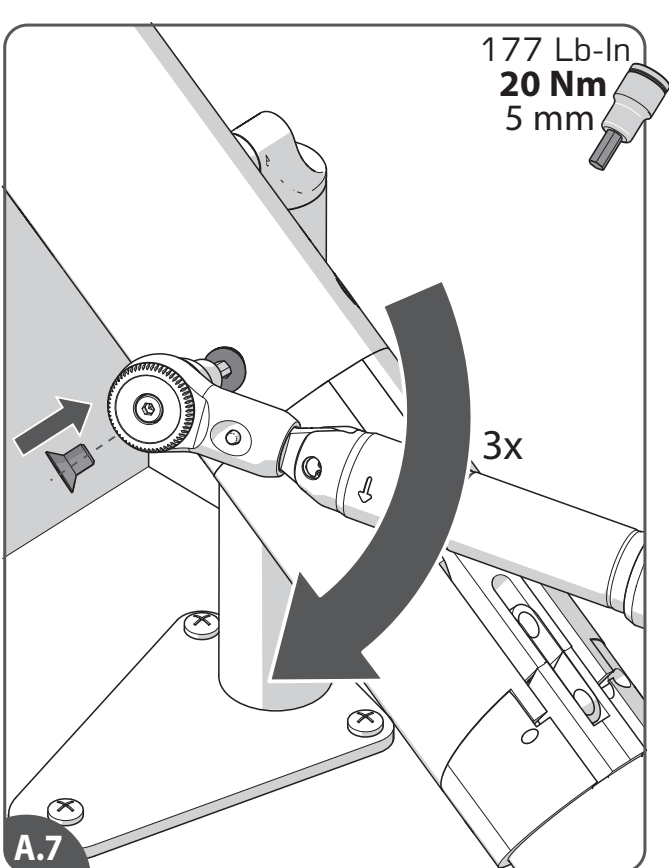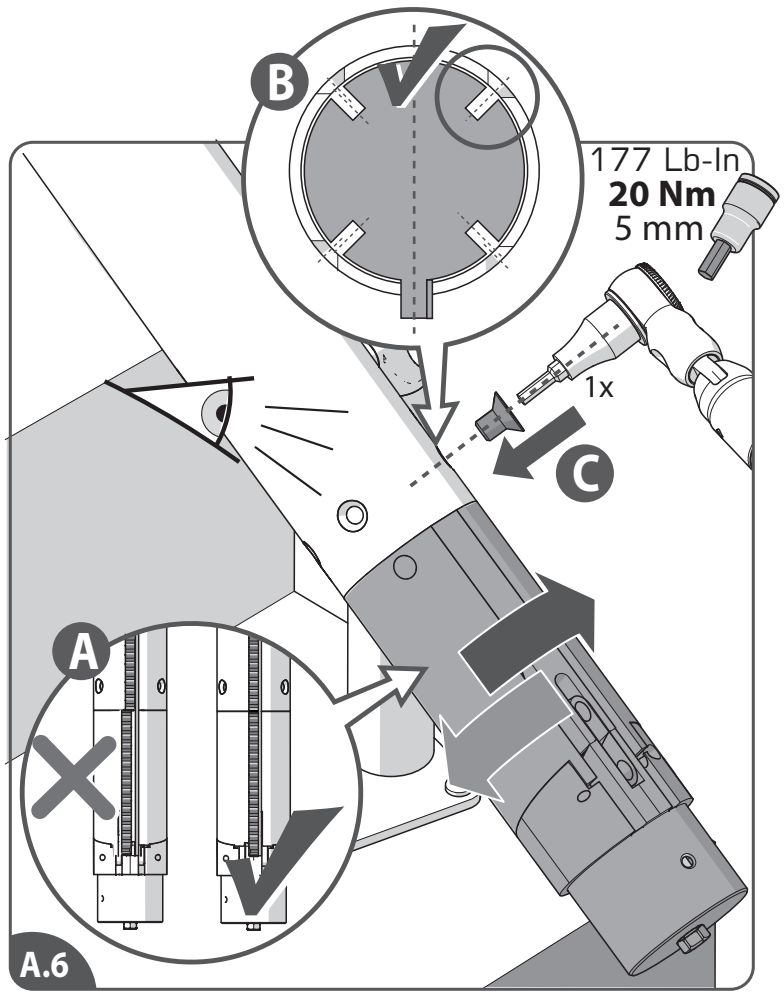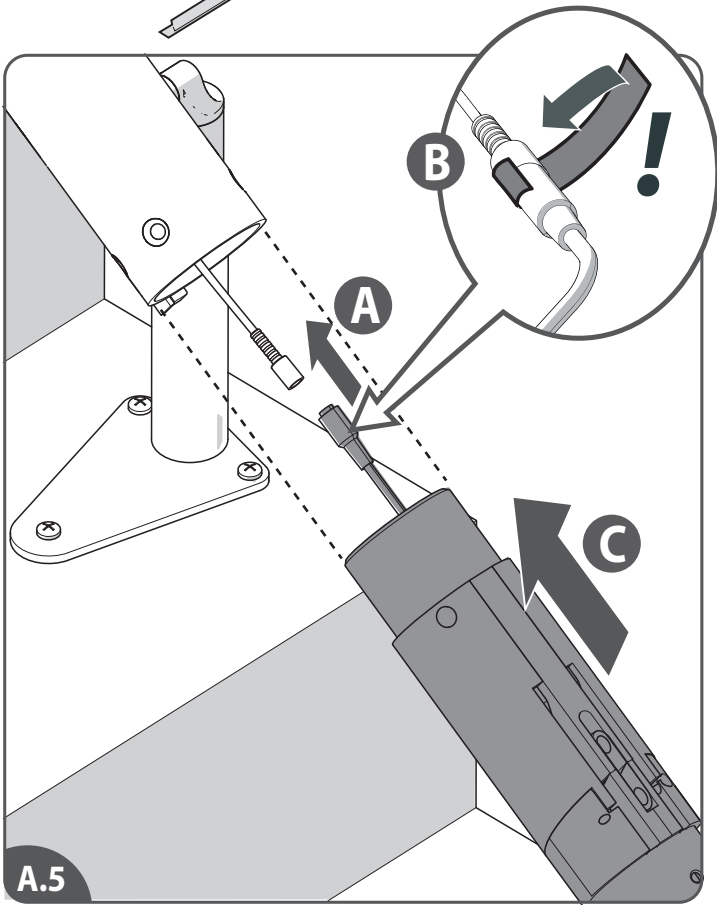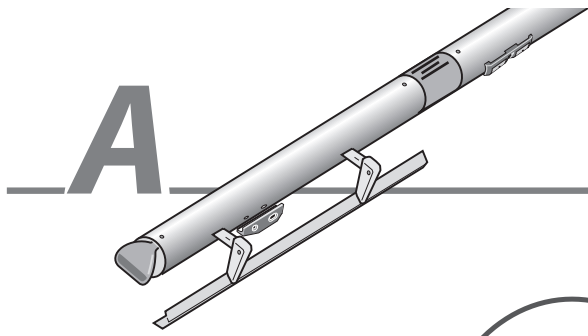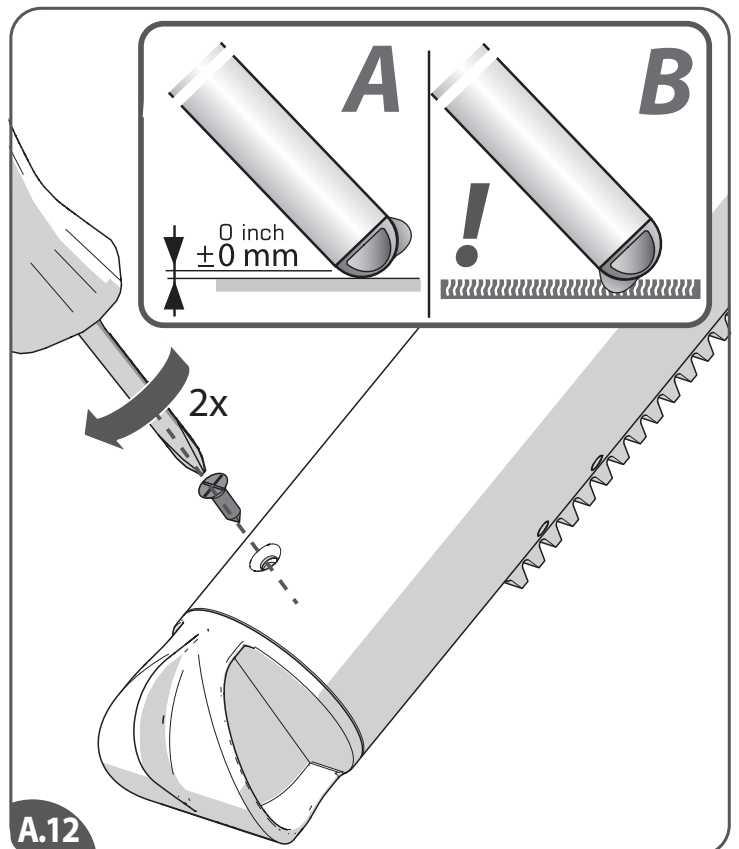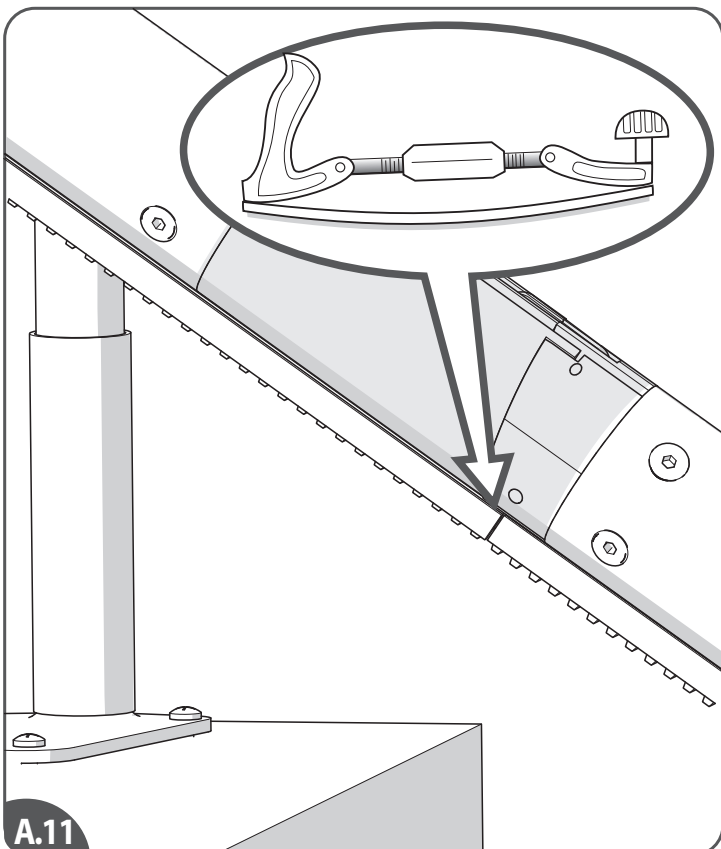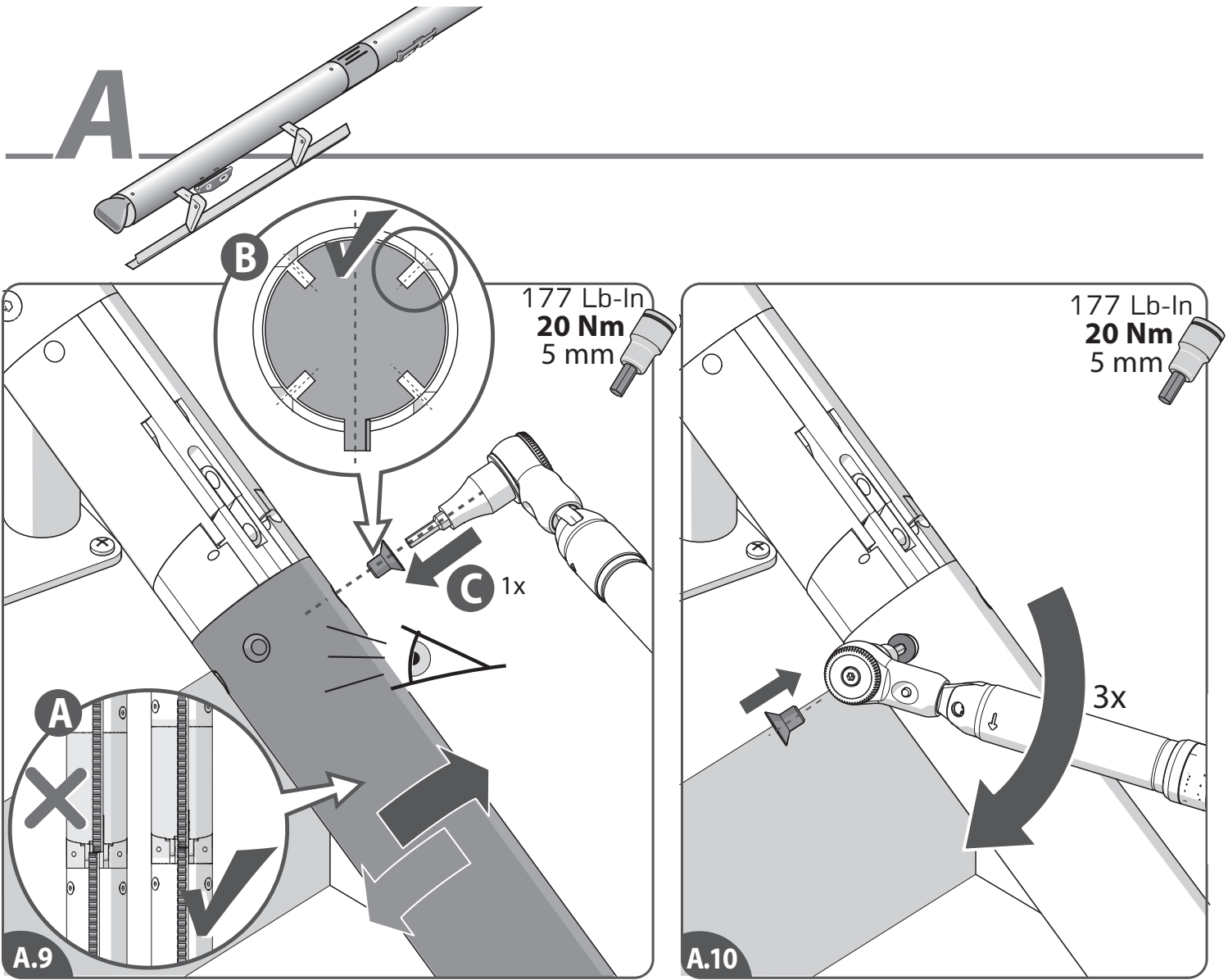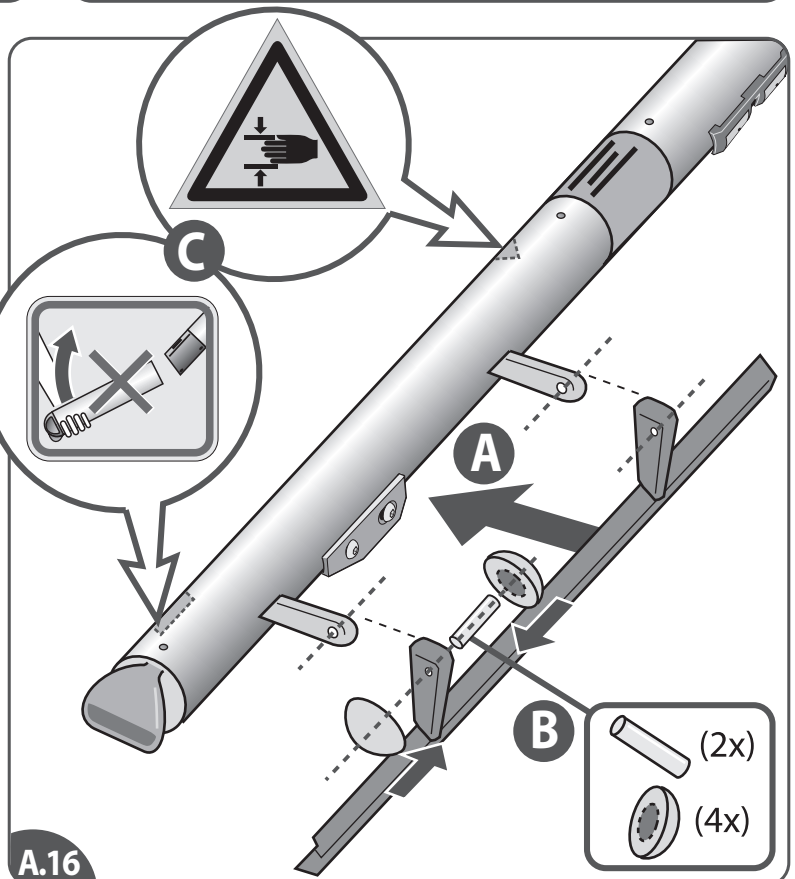
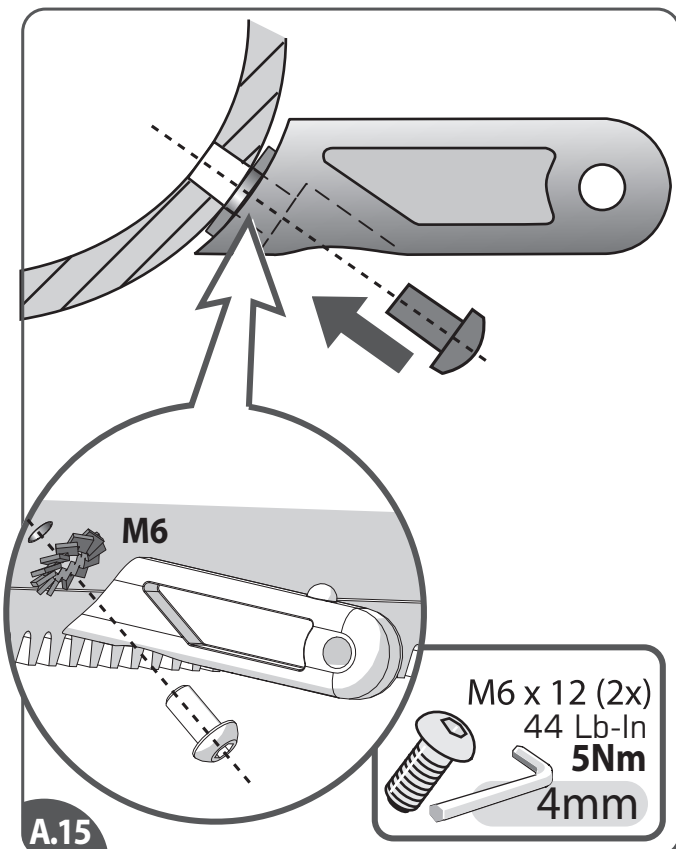
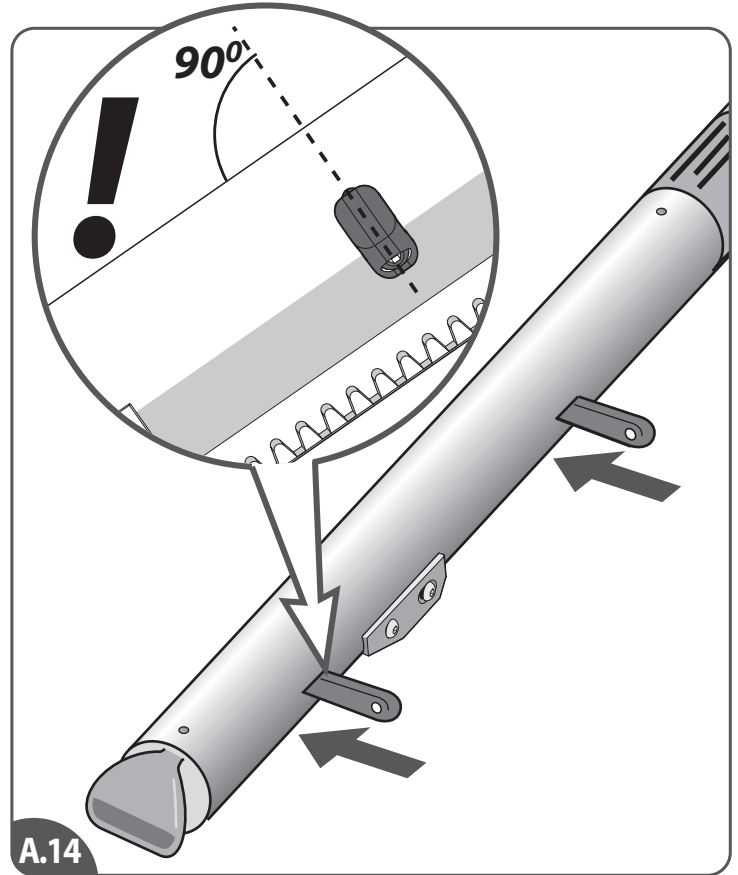
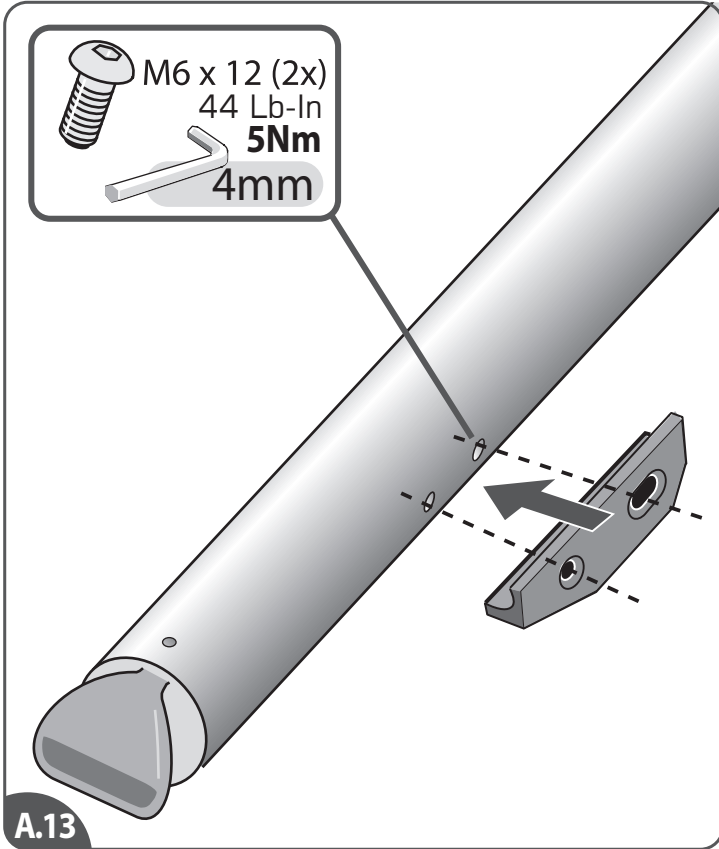
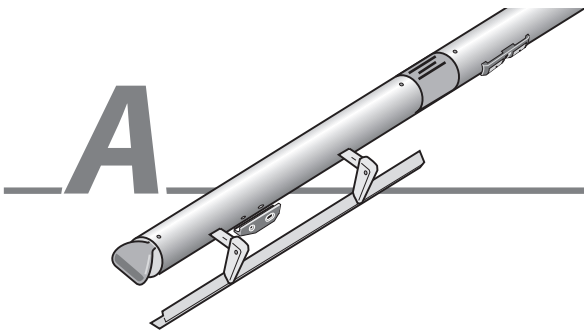
INSTALL THE HINGED RAIL PART IN THE SAME WAY AS AN ORDINARY RAIL PART. SEE CHAPTER 2 FOR A DETAILED INSTALLATION.

















B

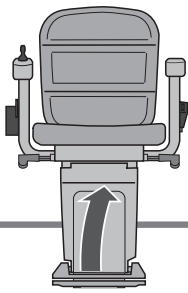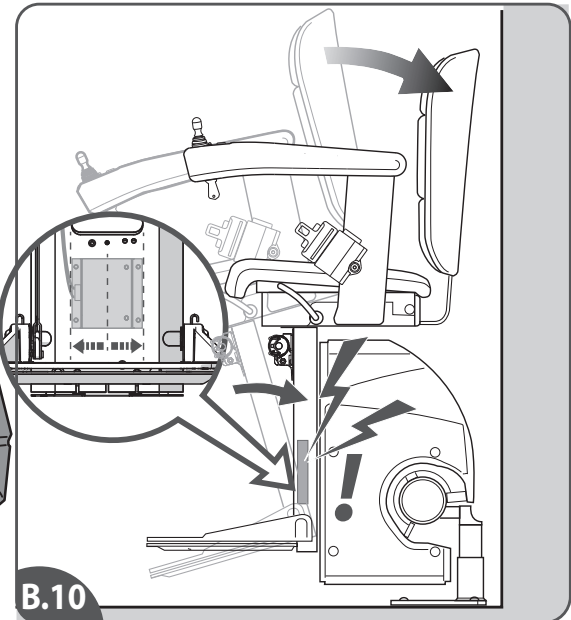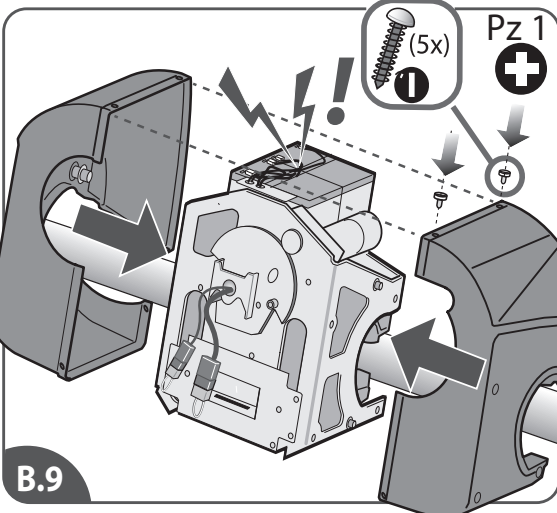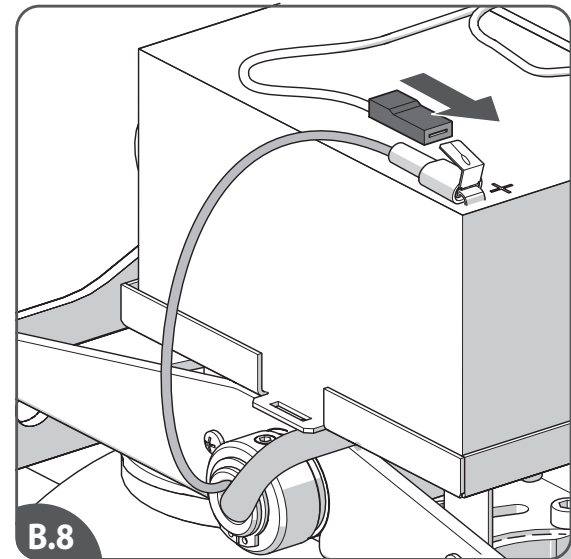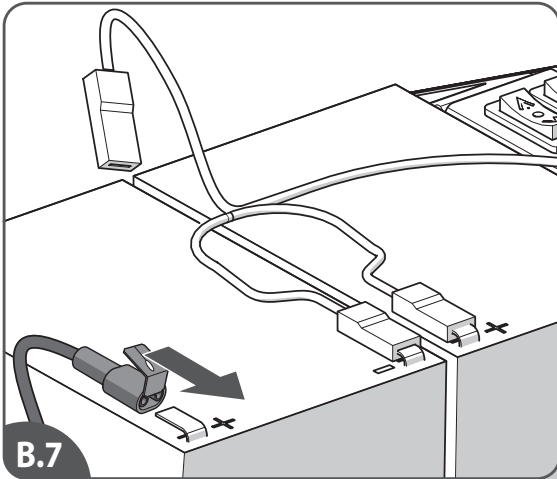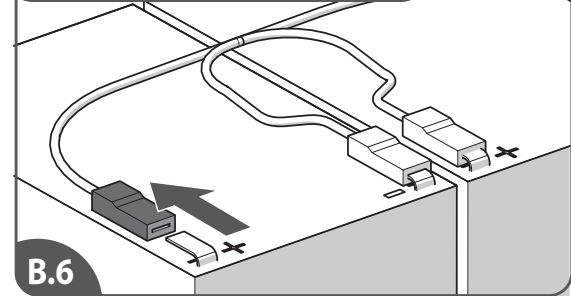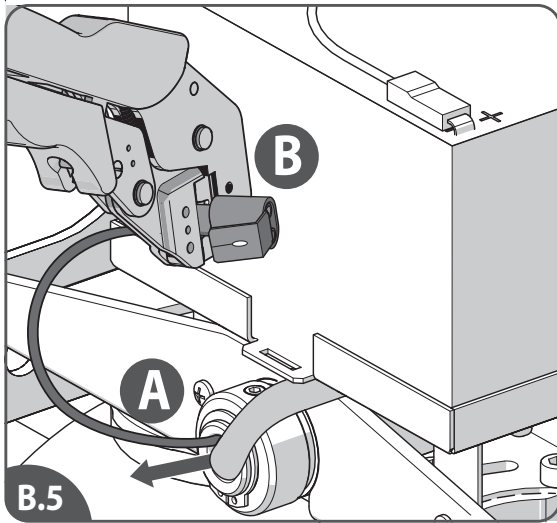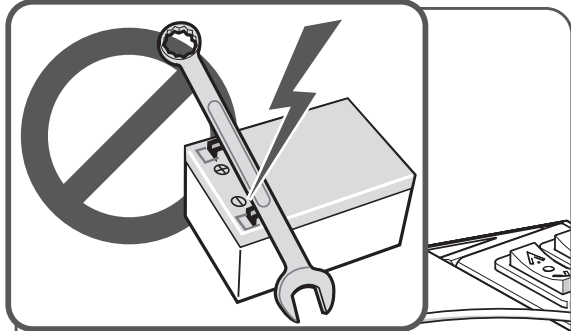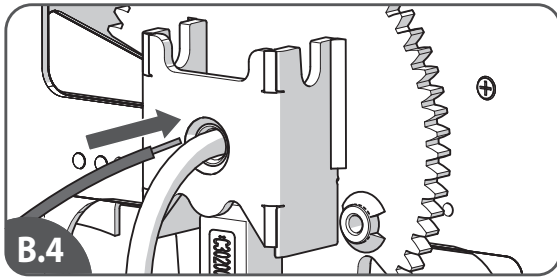POWERED FOOTREST REPOSE-PIEDS AUTOMATIQUE

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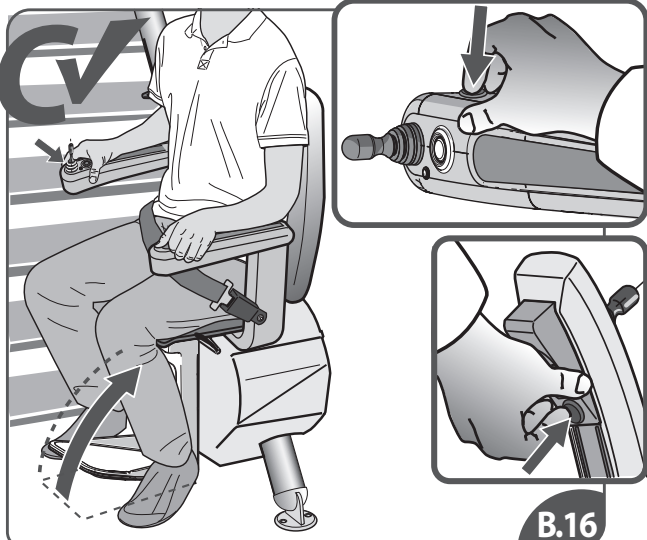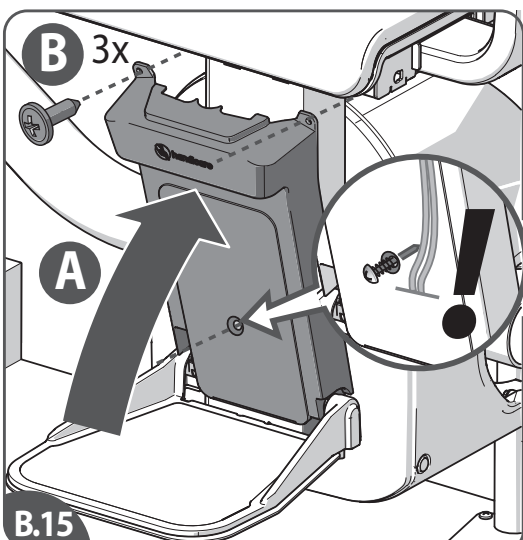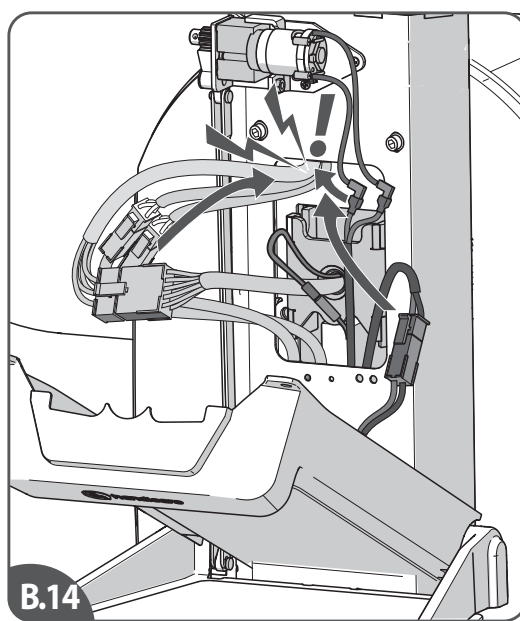
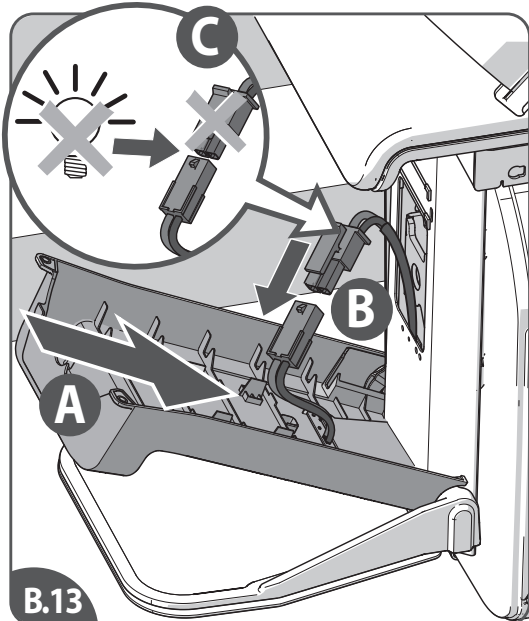
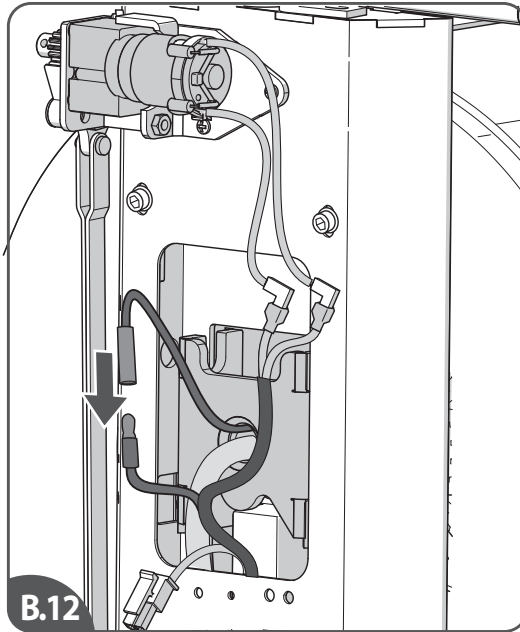
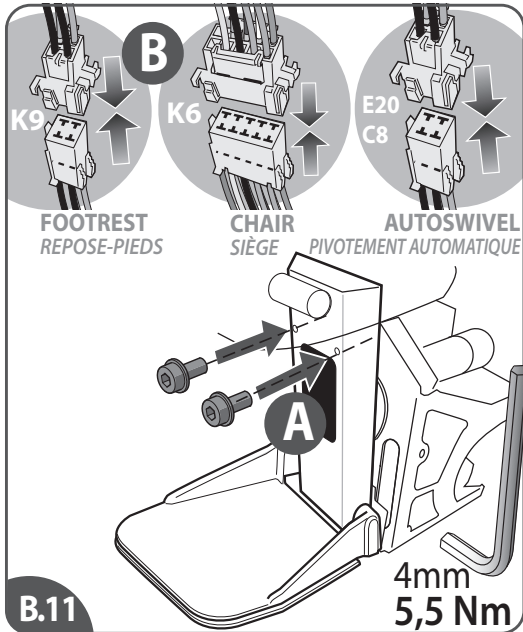
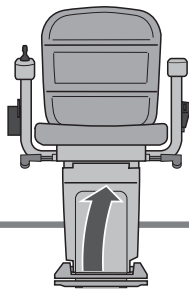
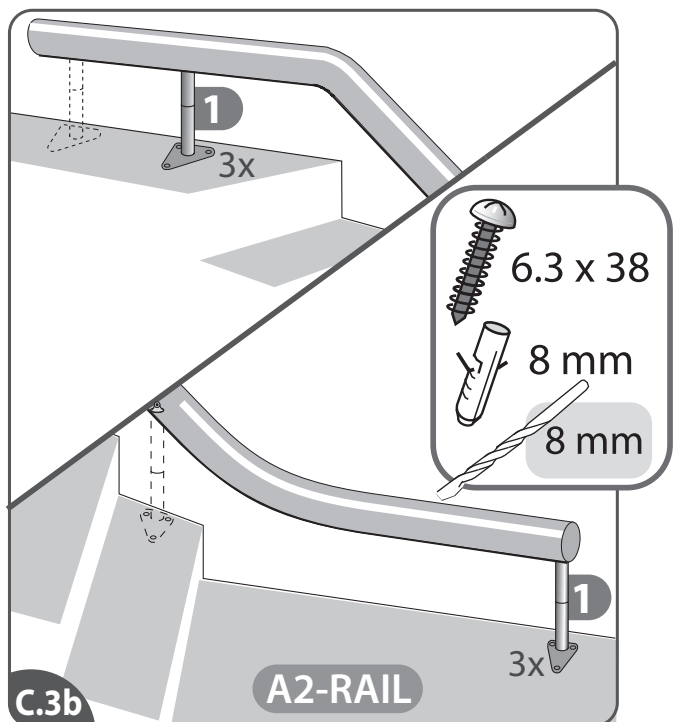
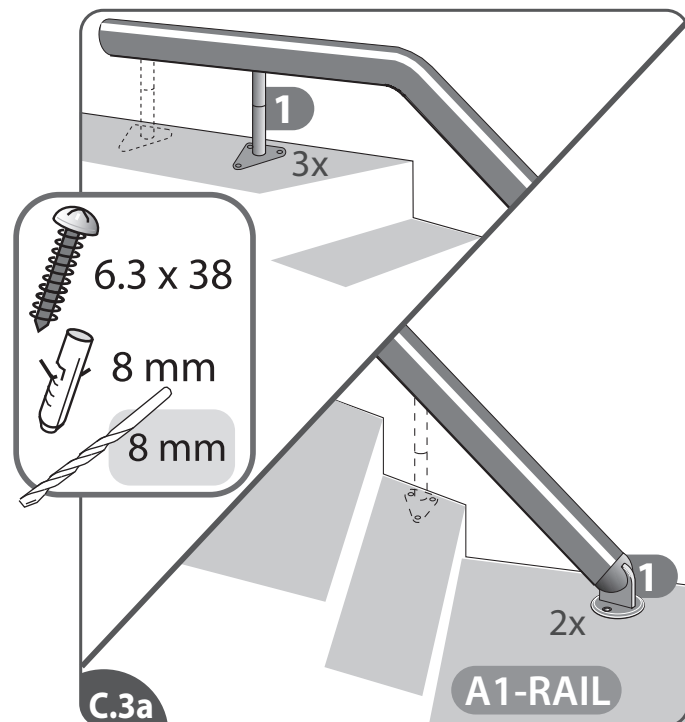
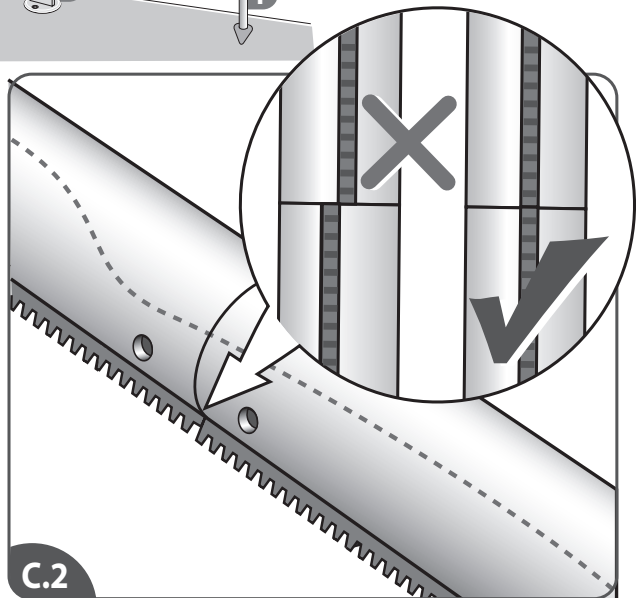
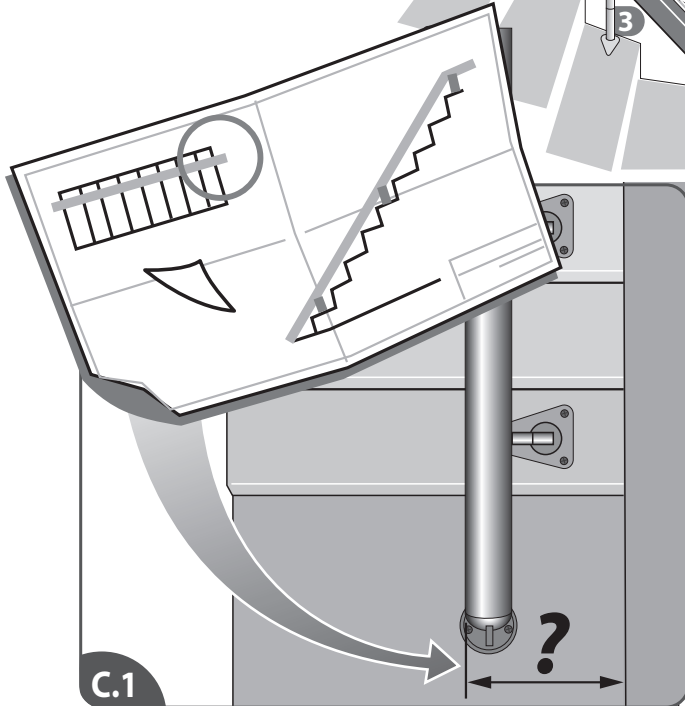
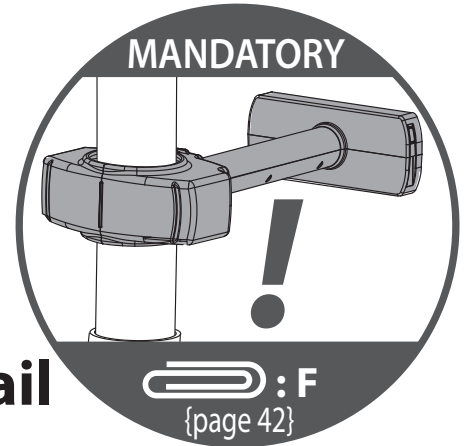
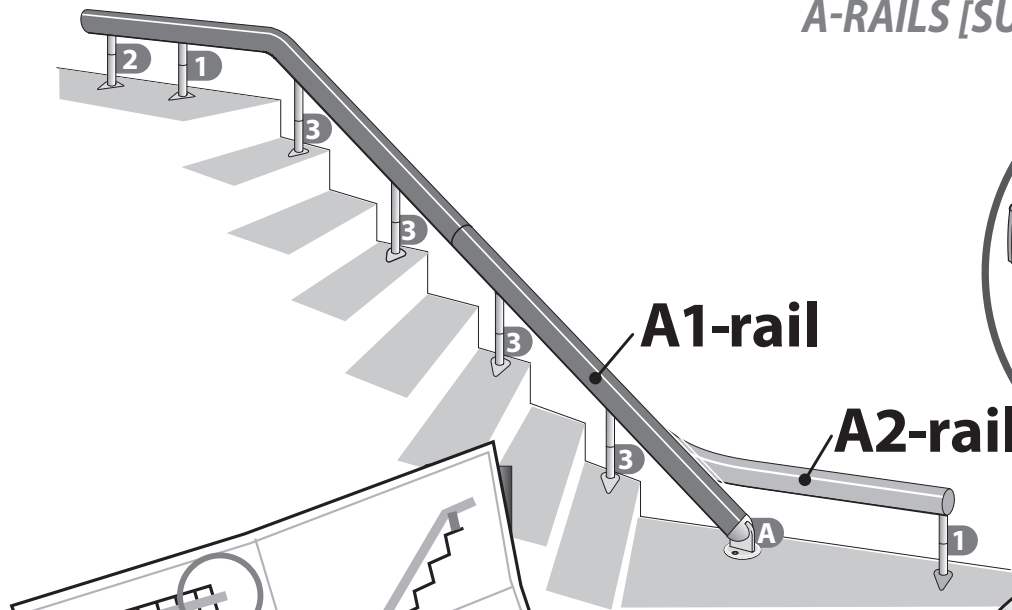
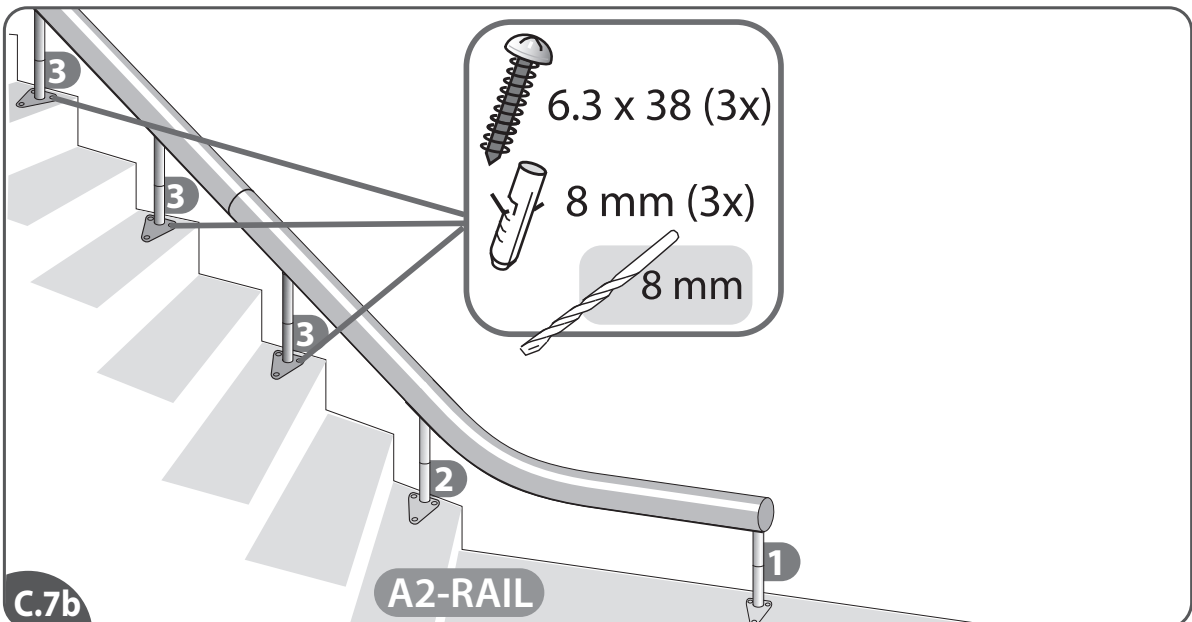
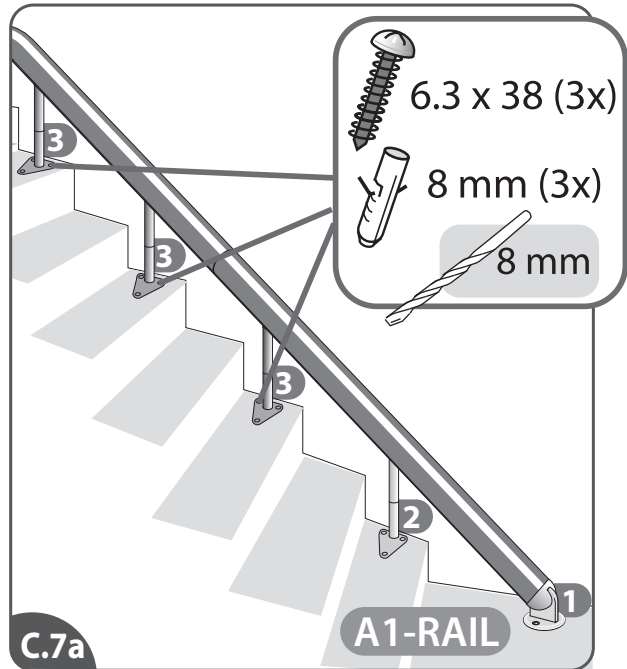
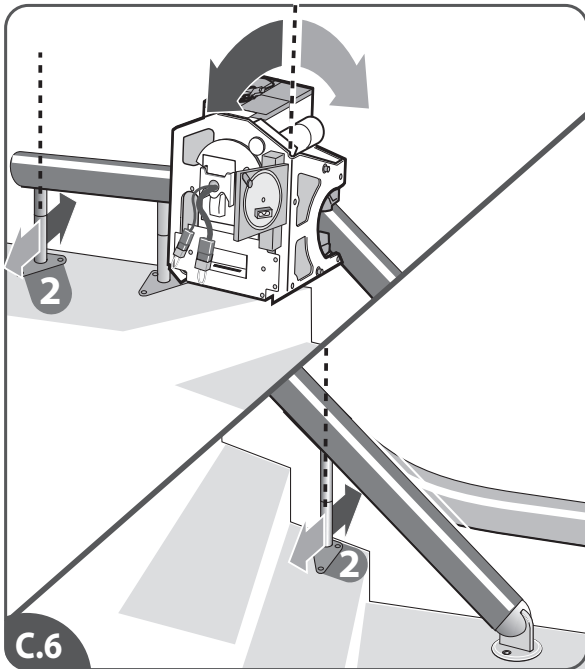
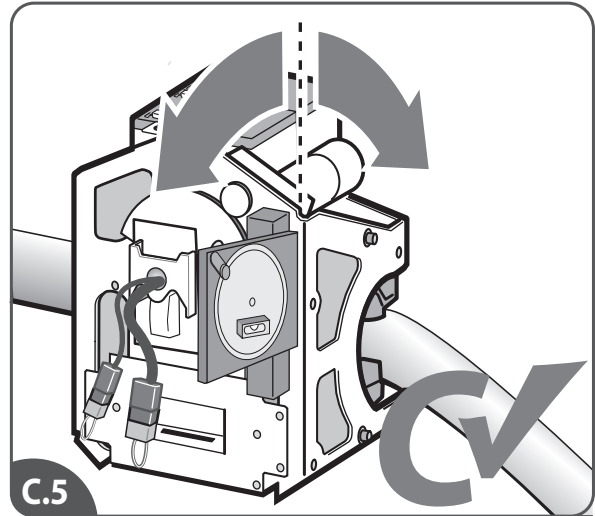
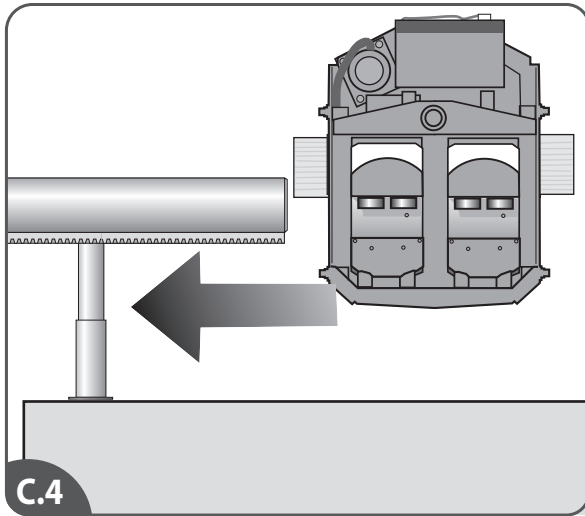
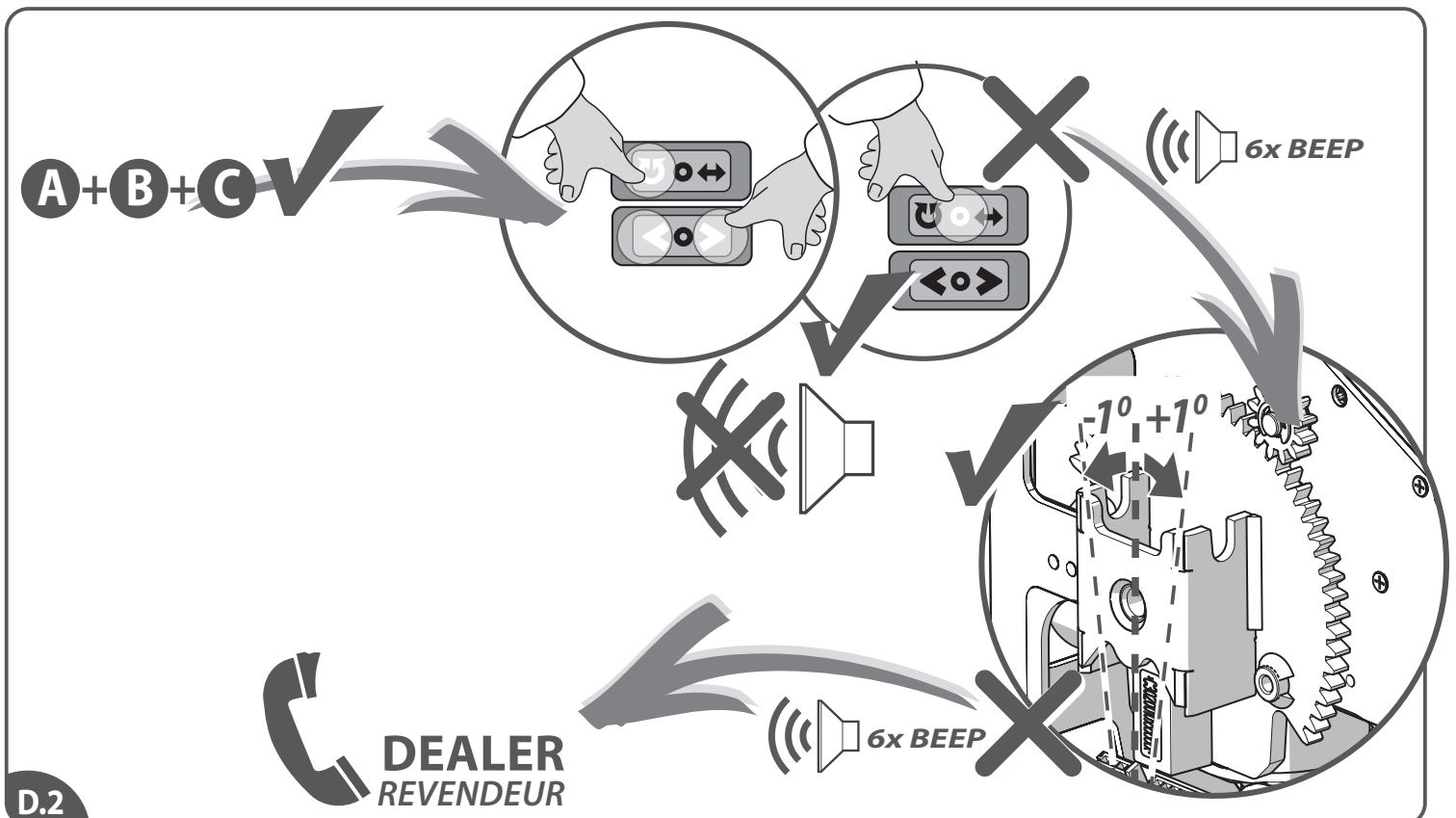
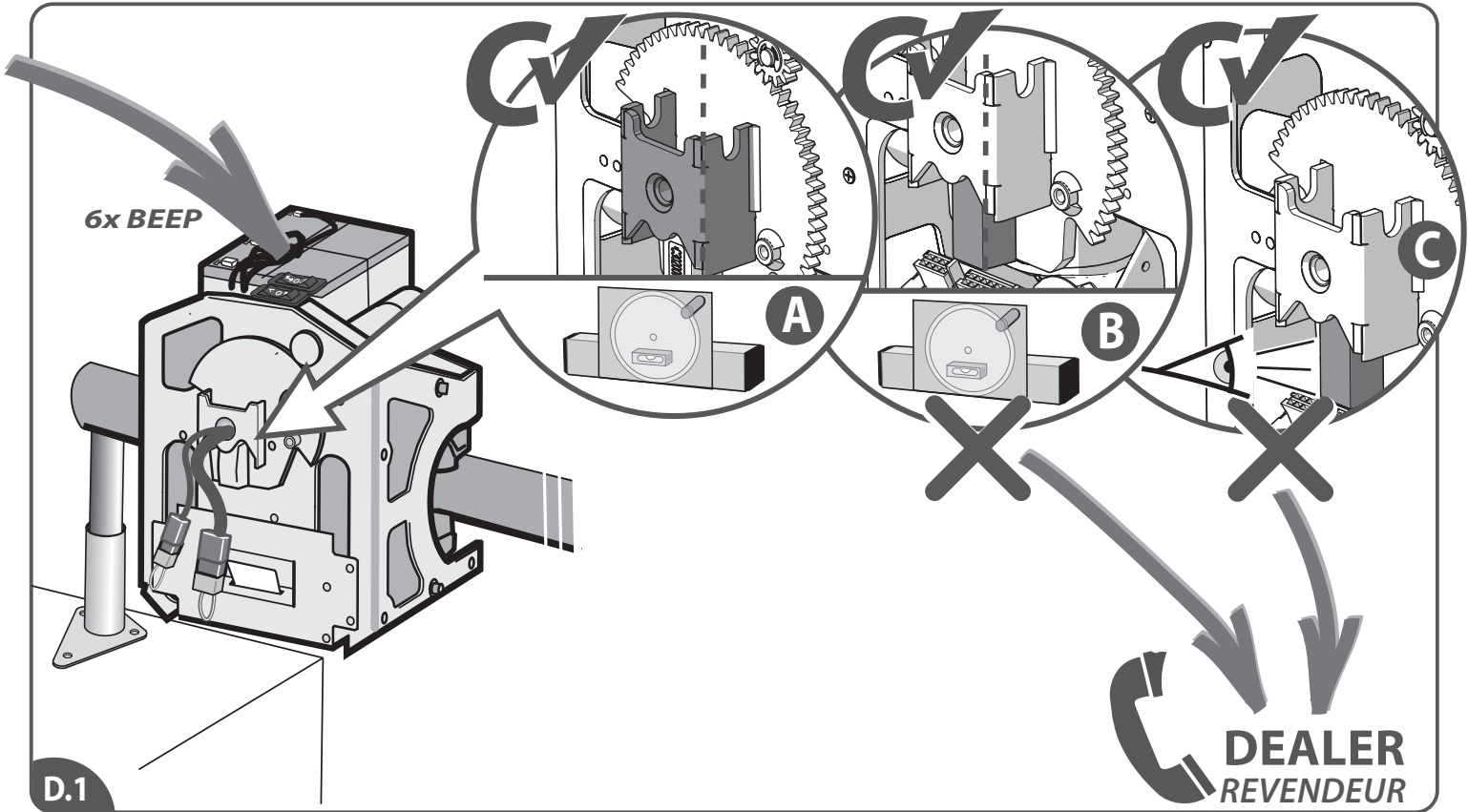
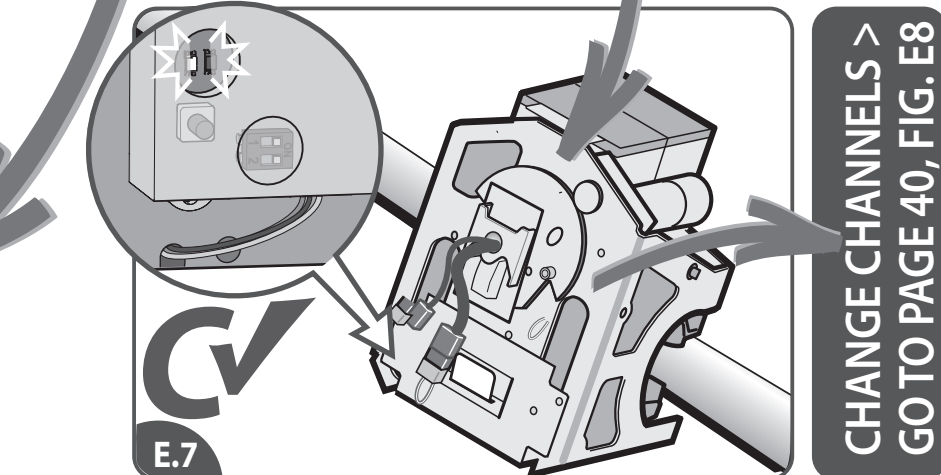
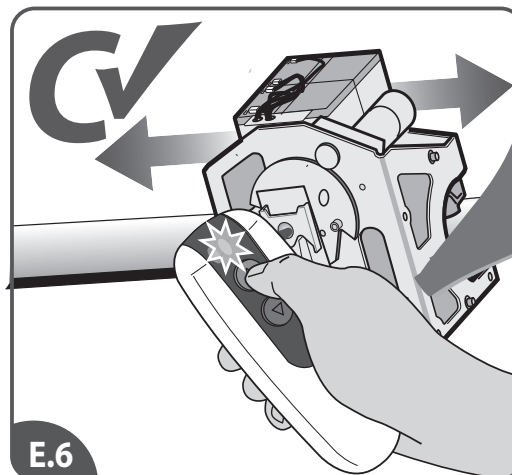
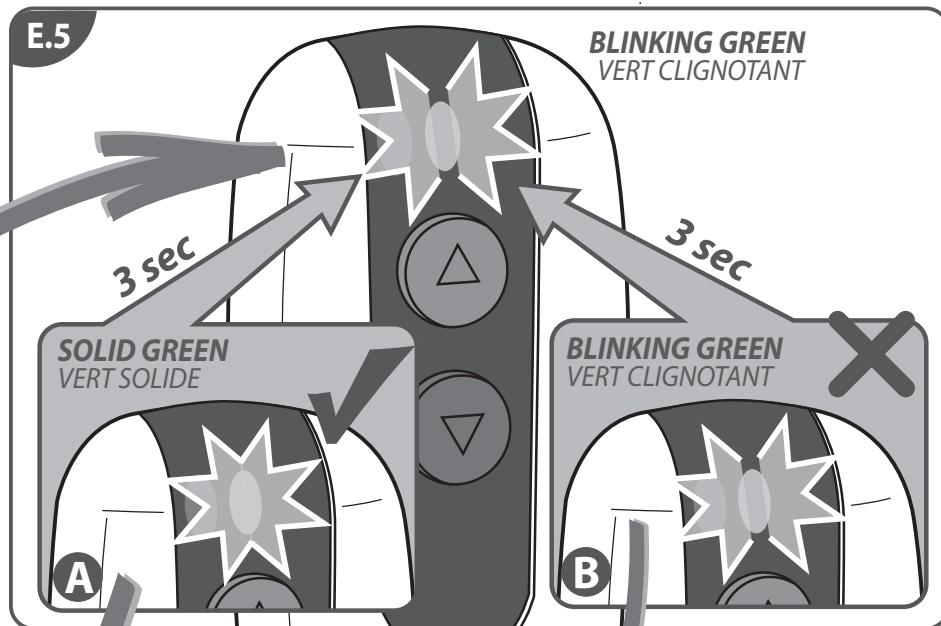
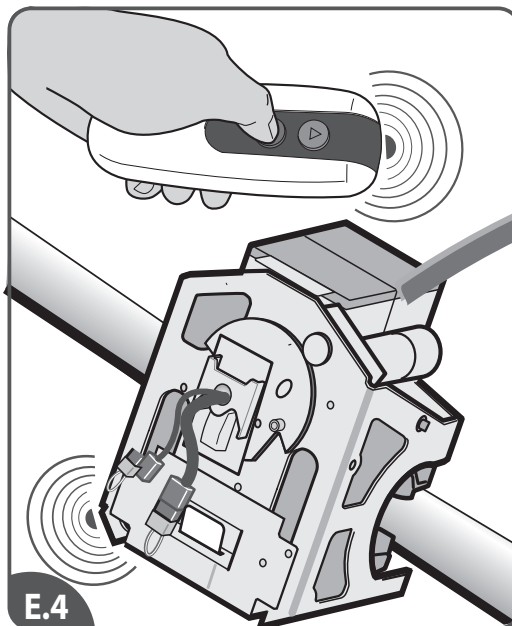
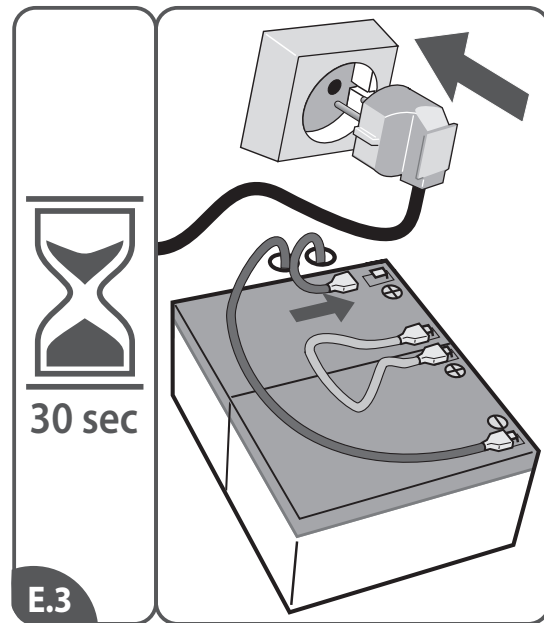
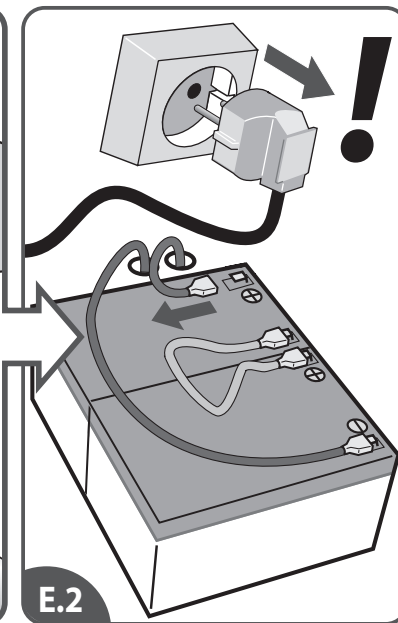
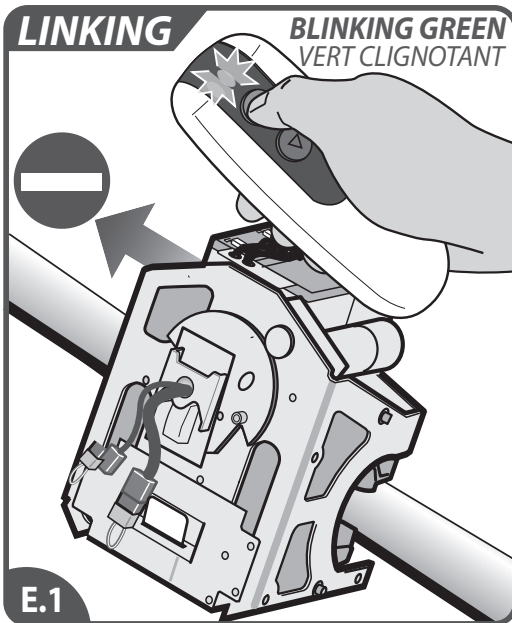
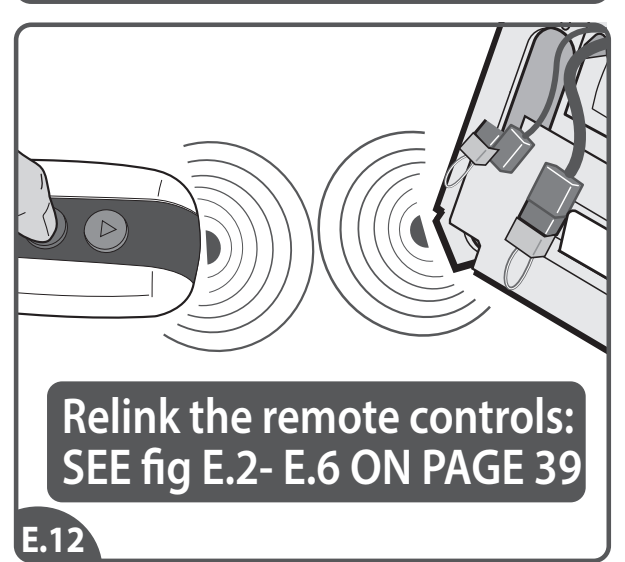
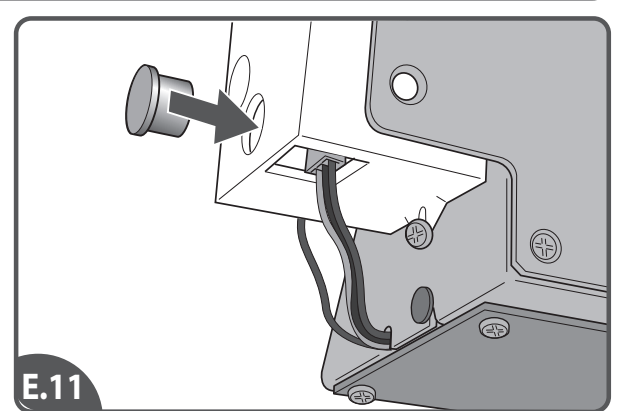
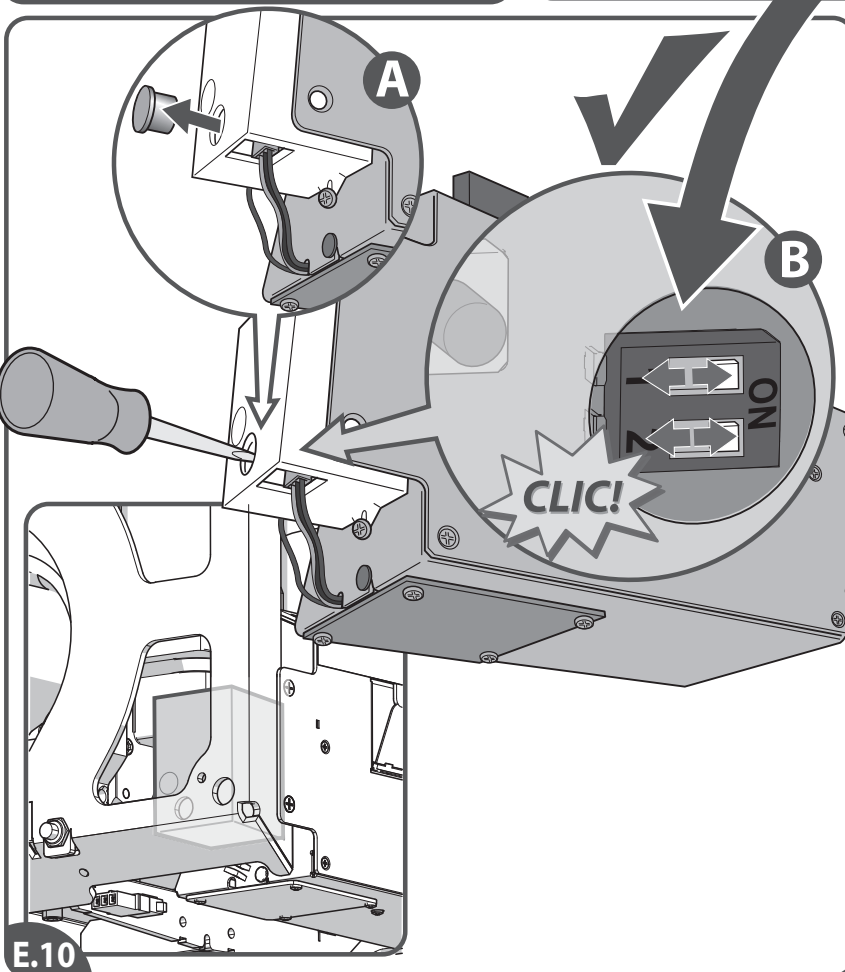
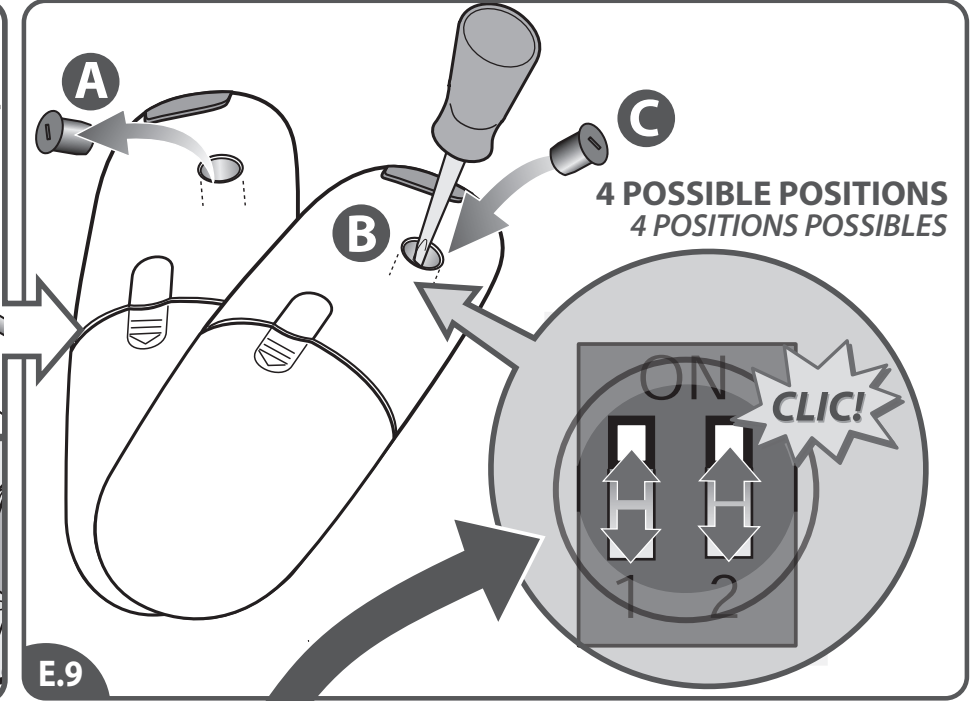
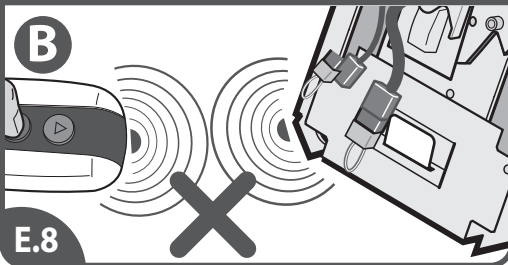
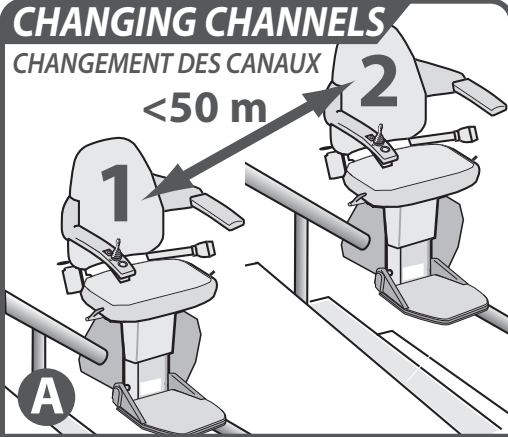
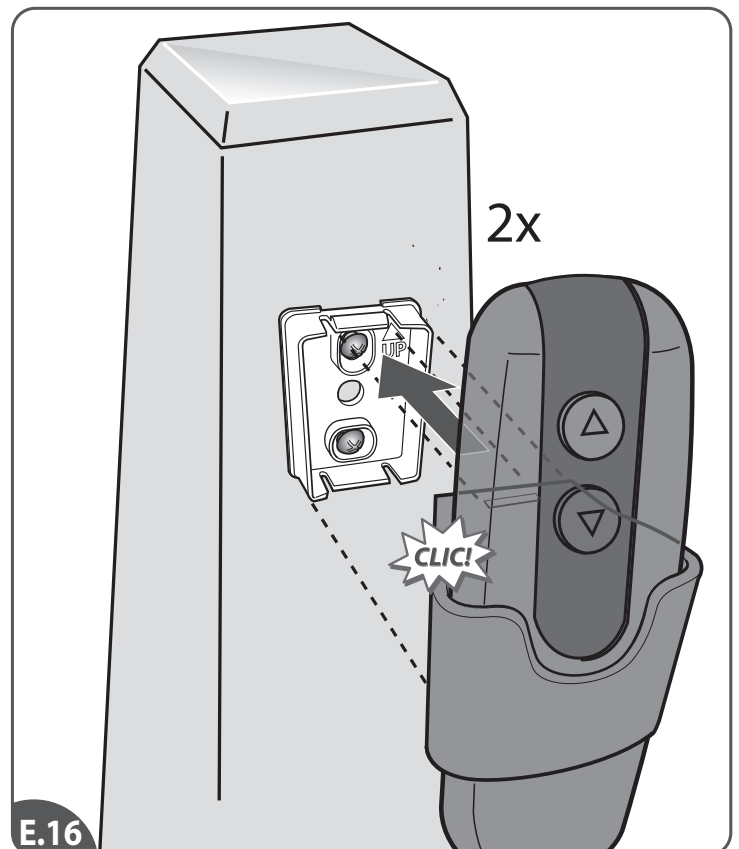
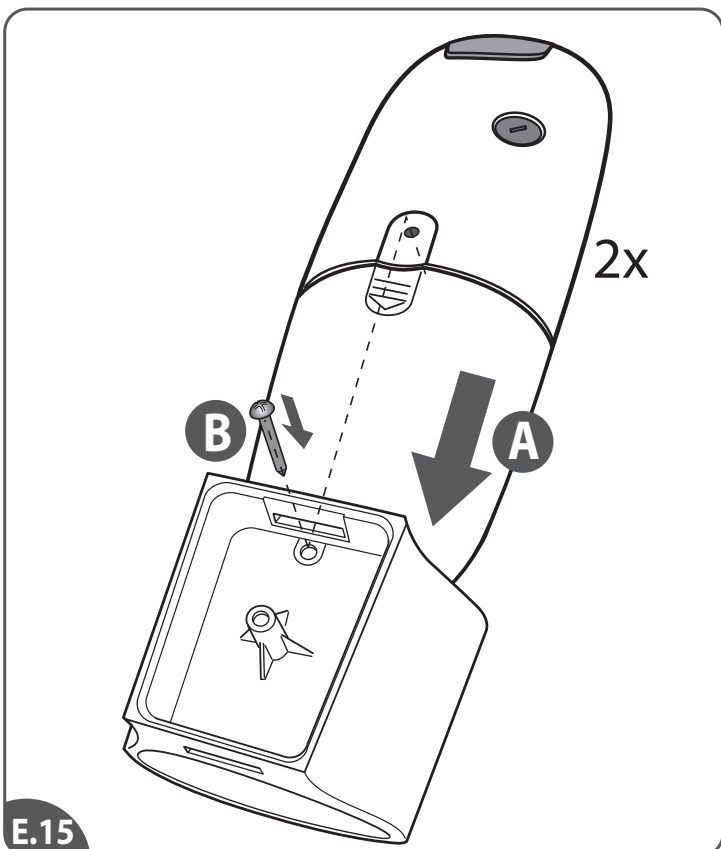
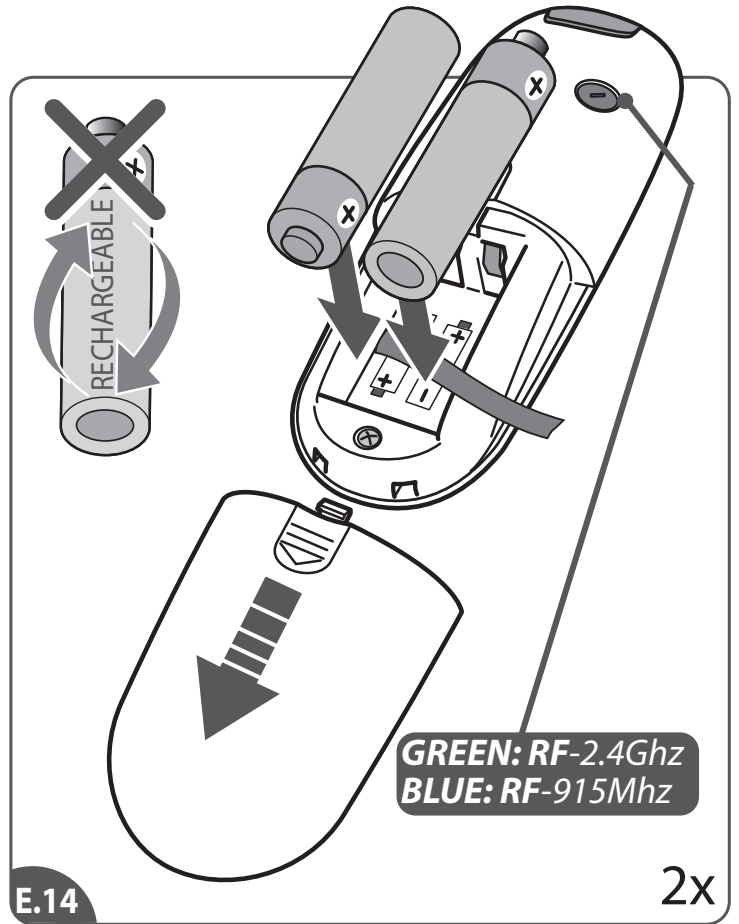
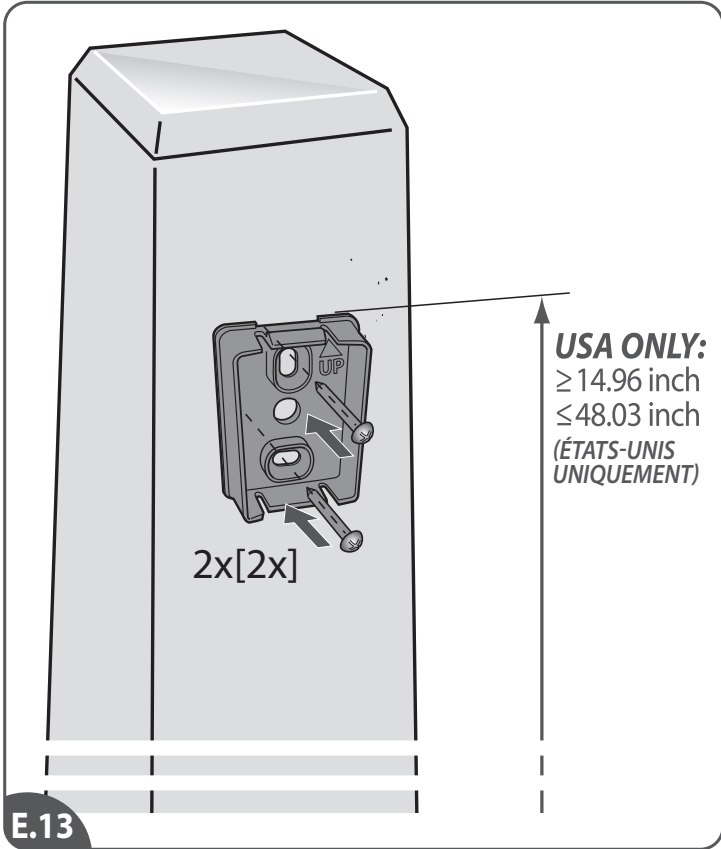
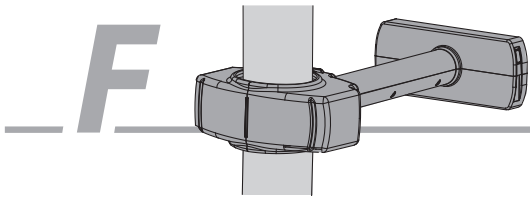
WALL SUPPORT

SUPPORT MURAL

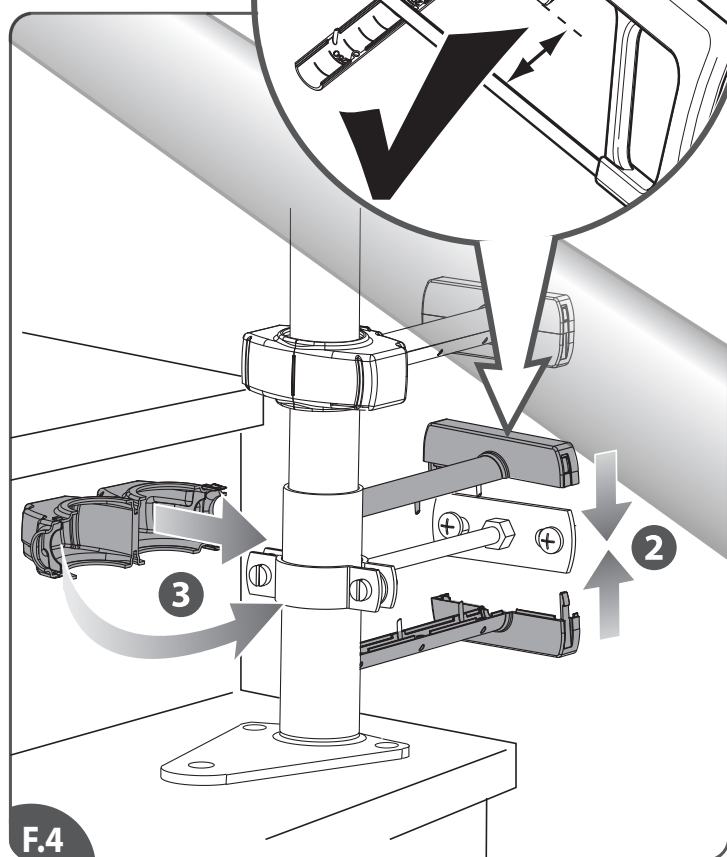
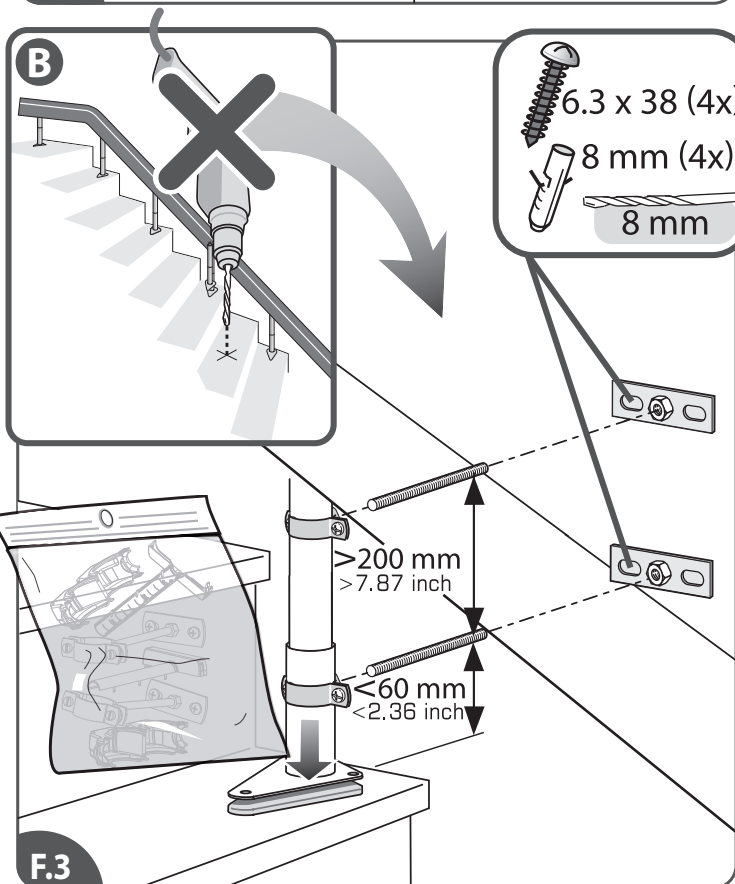
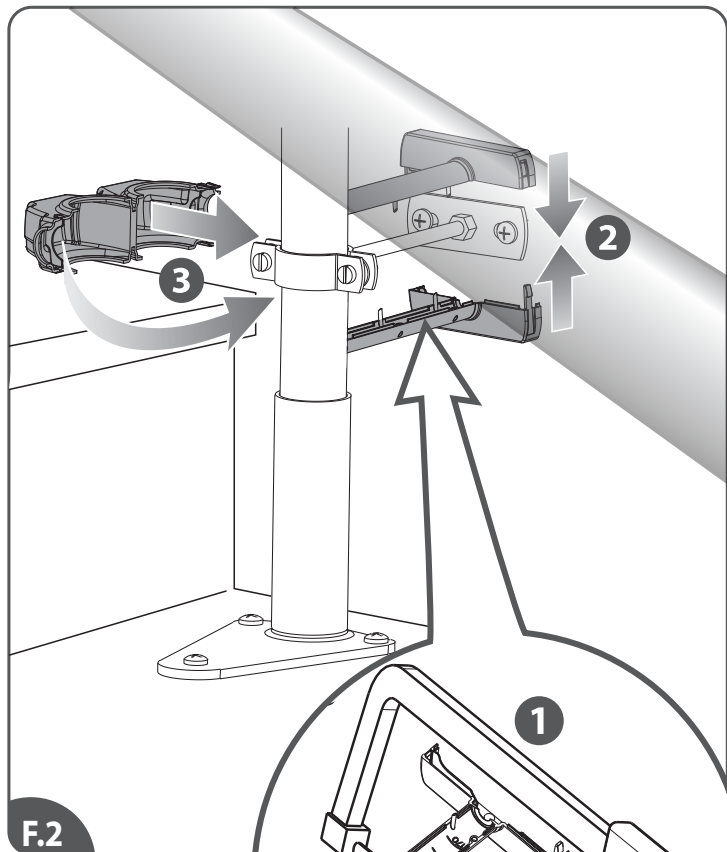
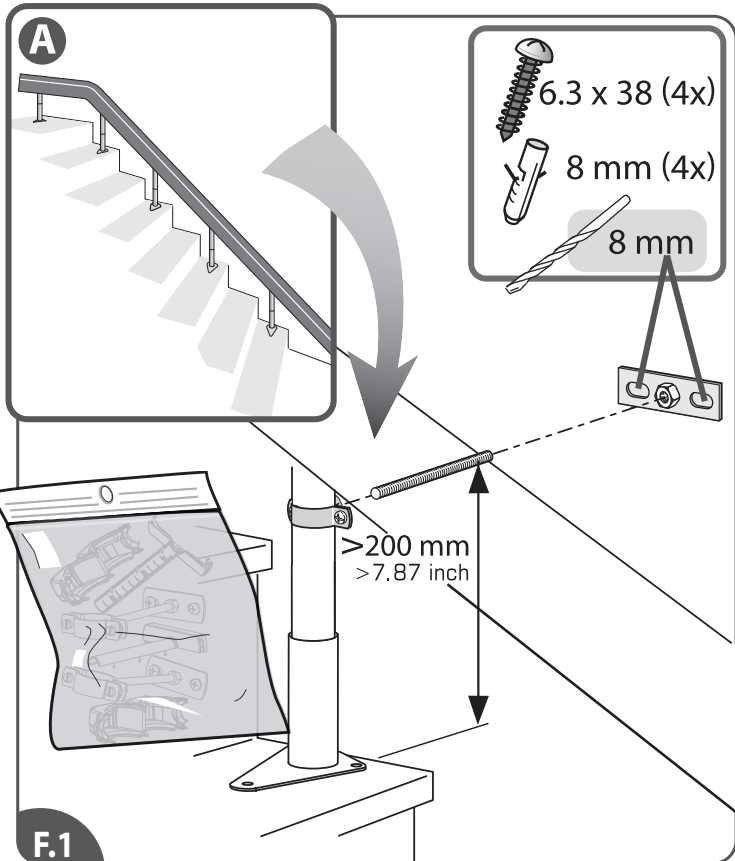
SEE SALESMANUAL (11P00442) IN WHICH SITUATIONS WALL SUPPORTS ARE MANDATORY

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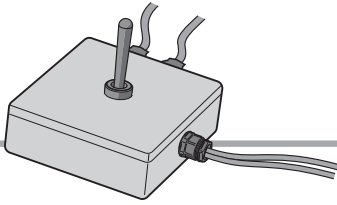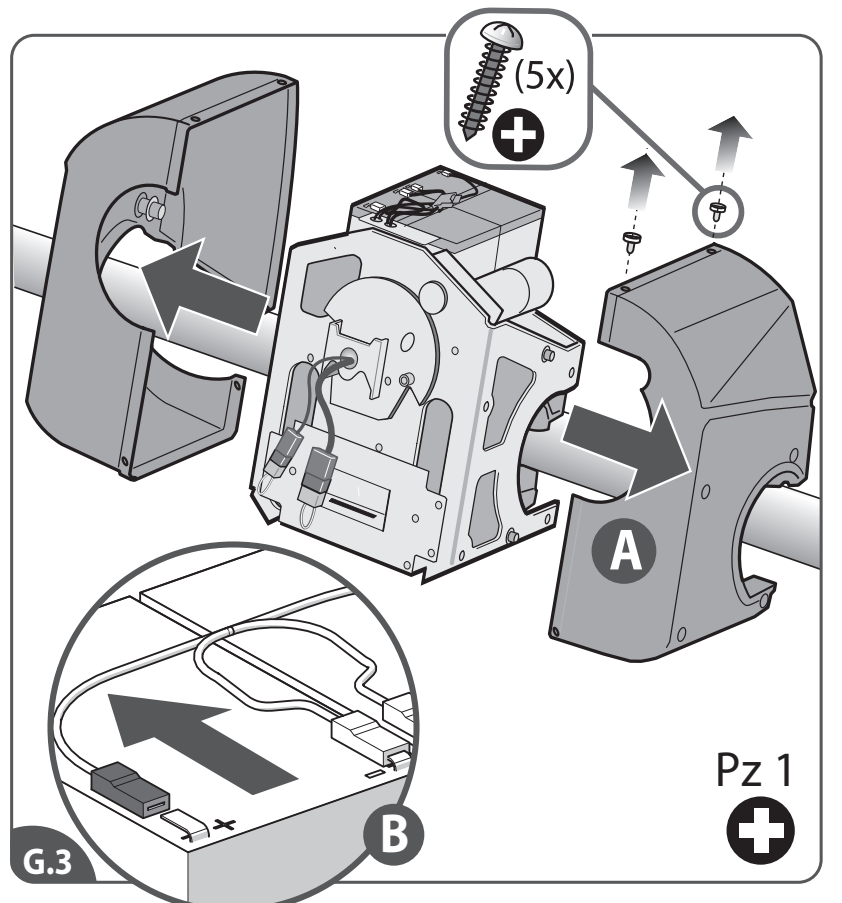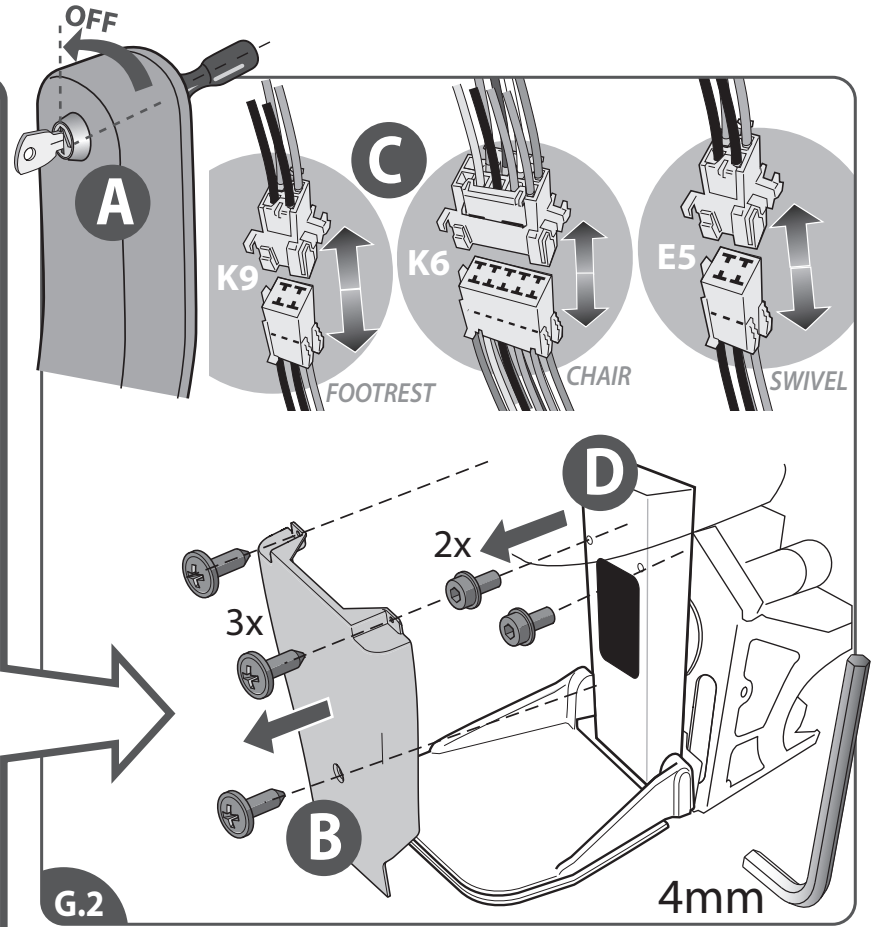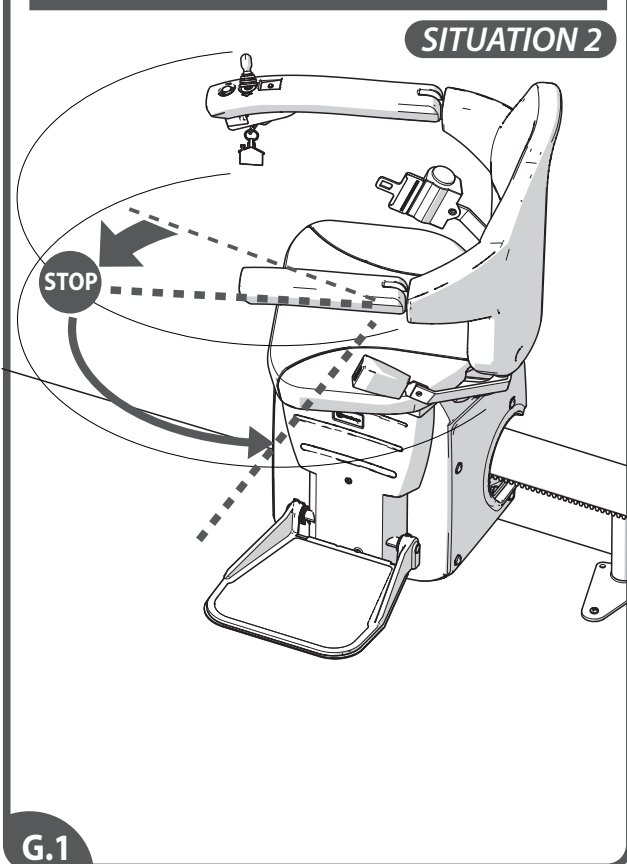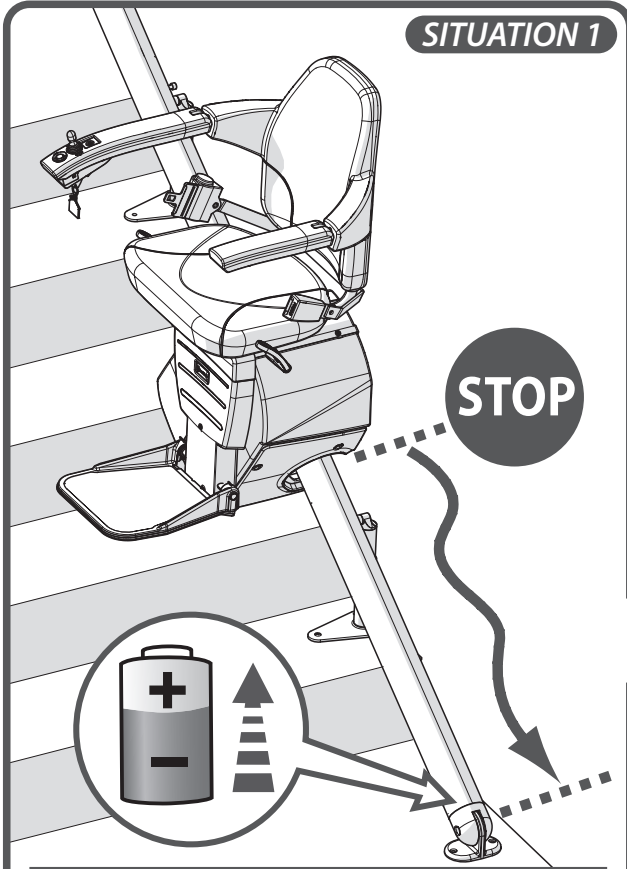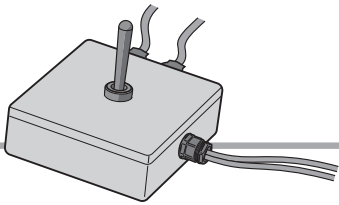
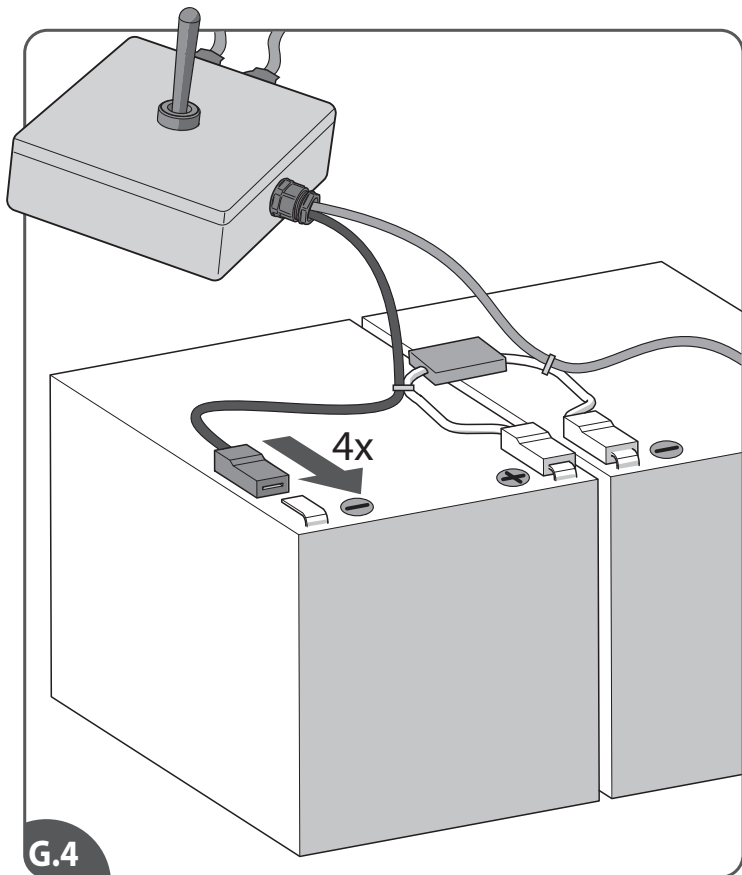
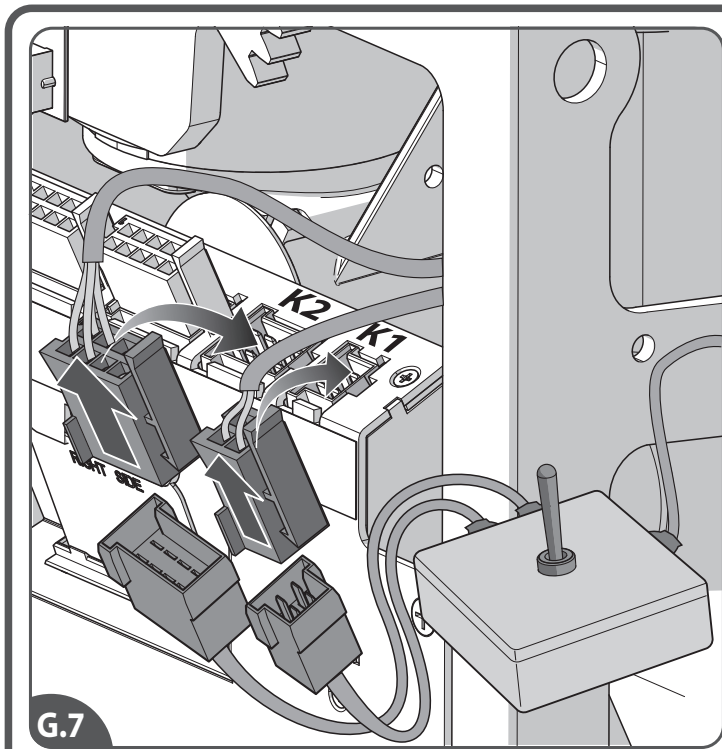
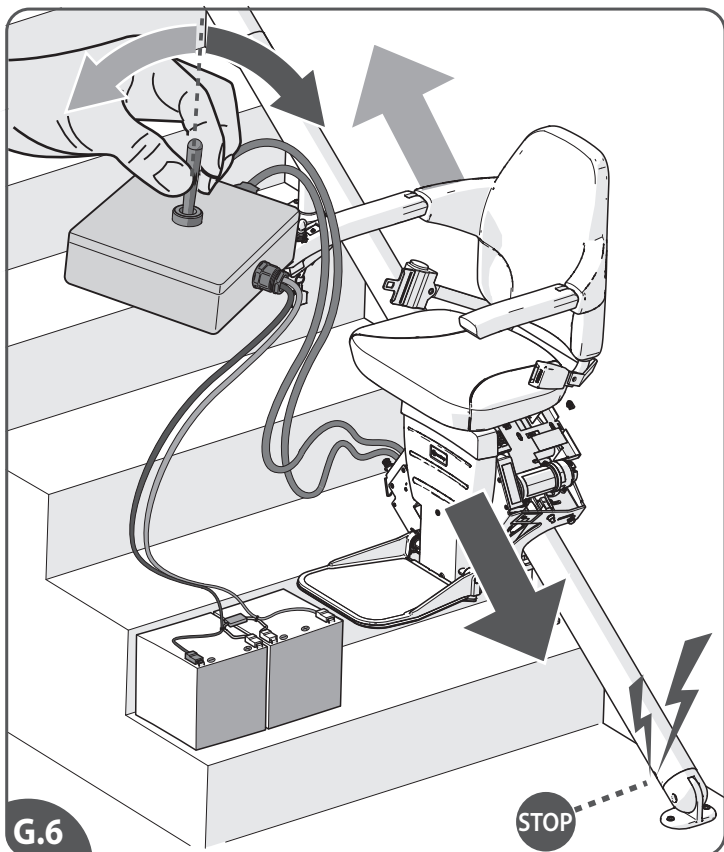
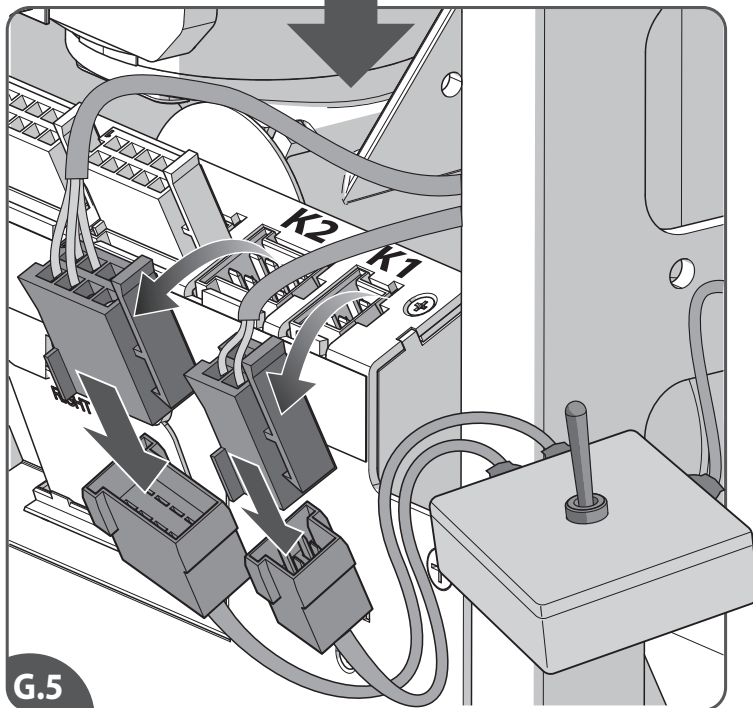
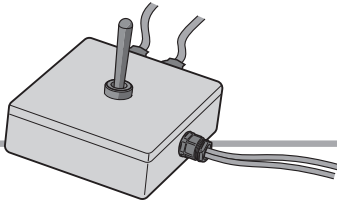
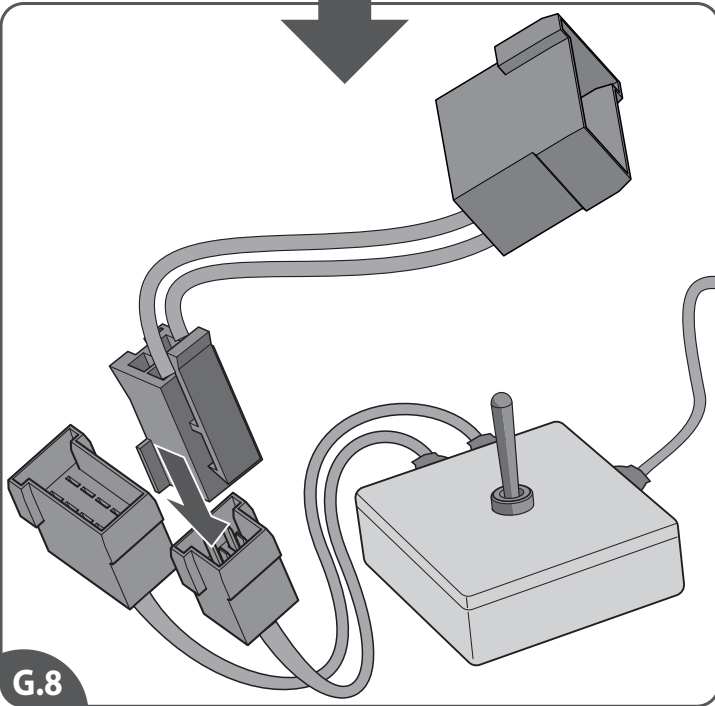
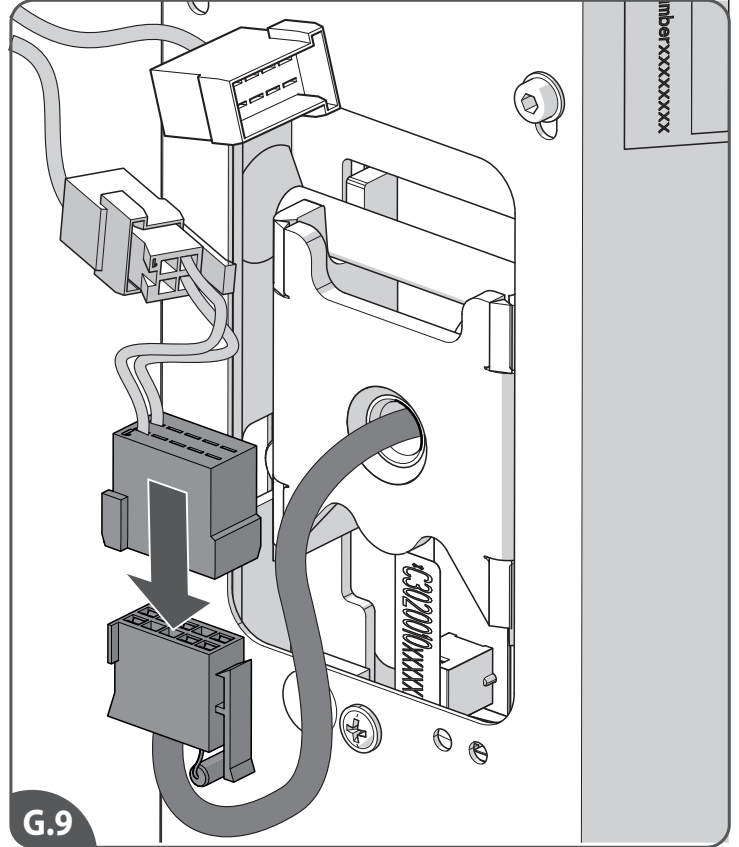
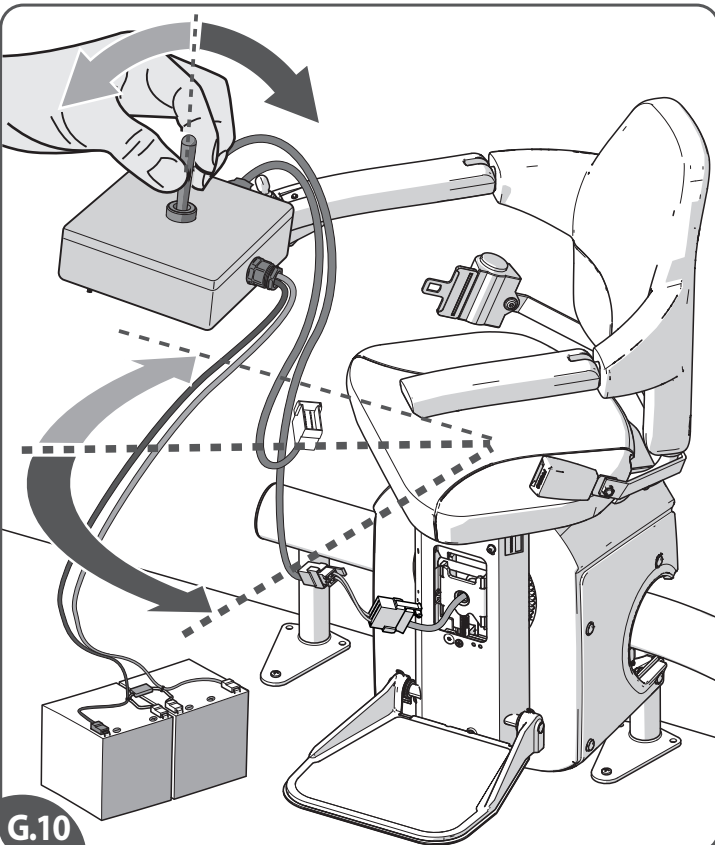
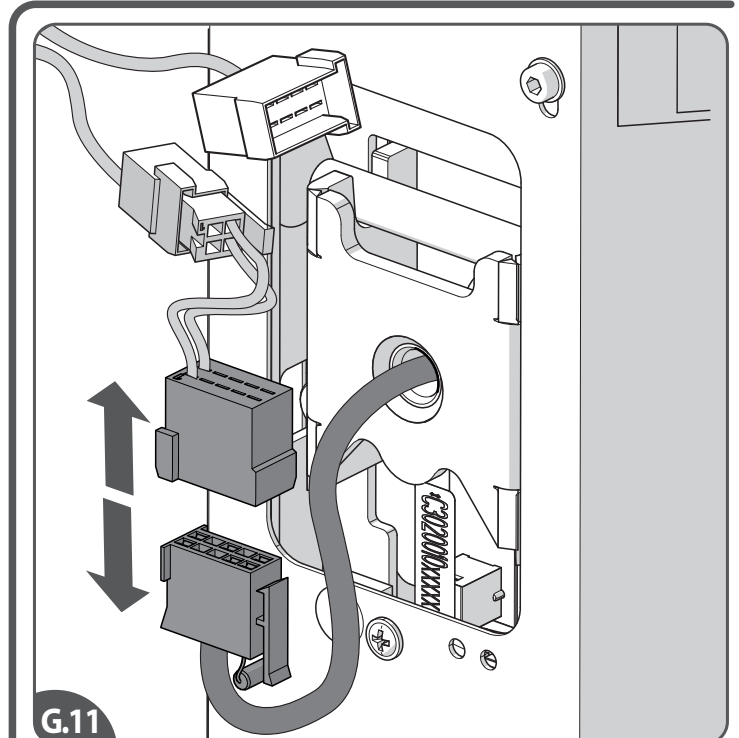
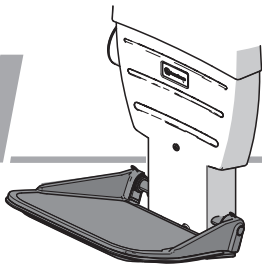
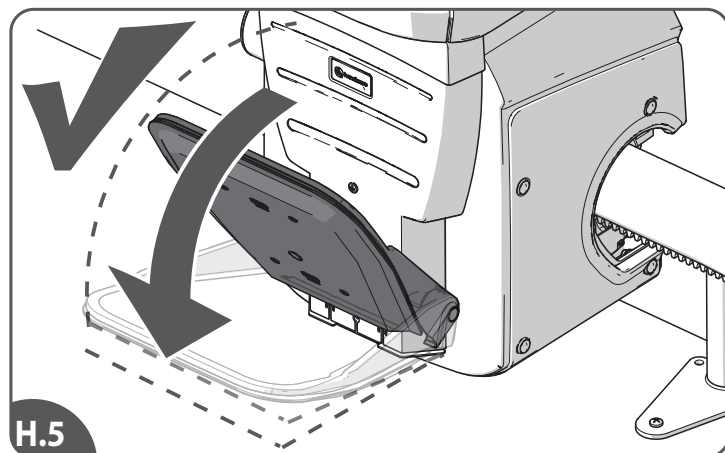
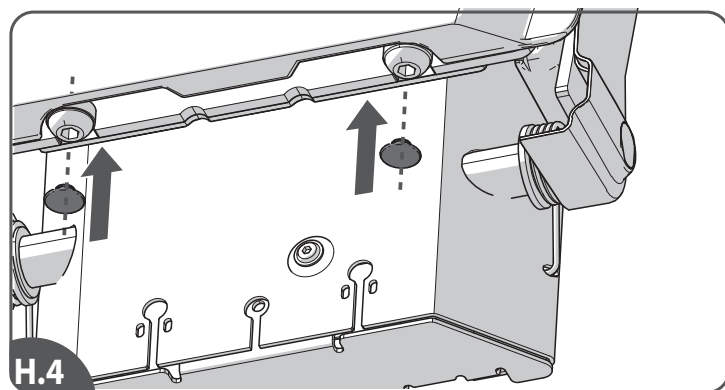
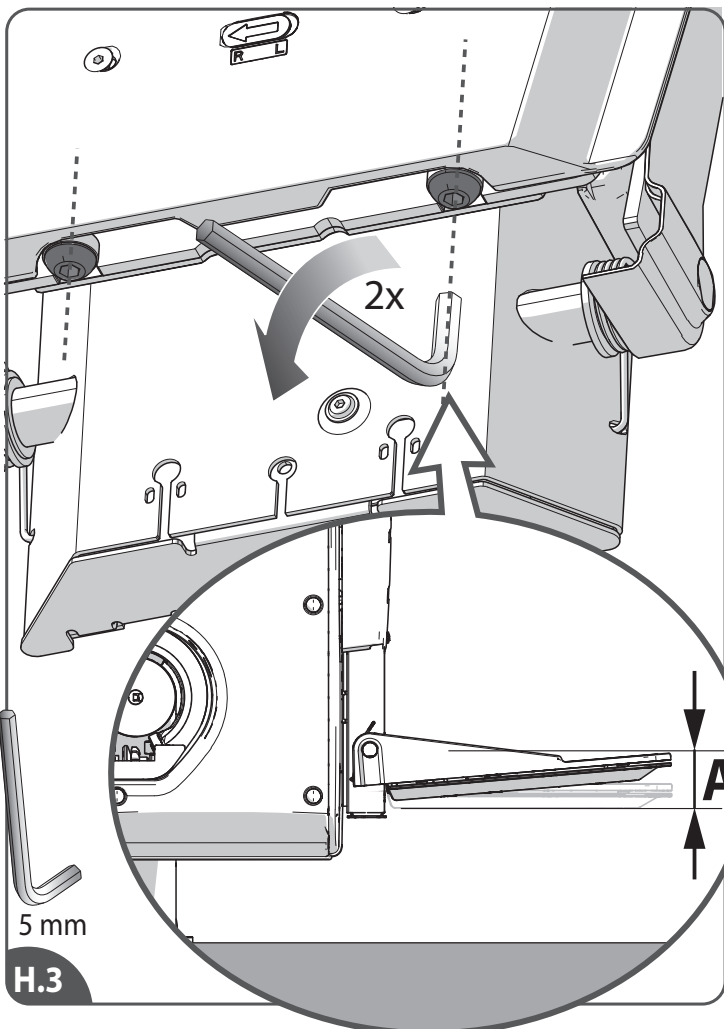
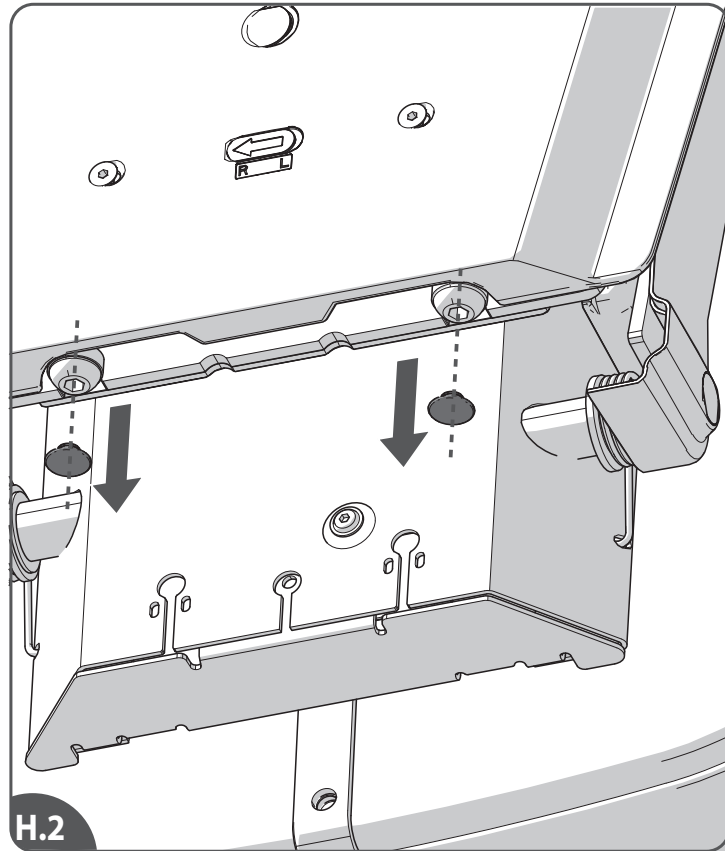
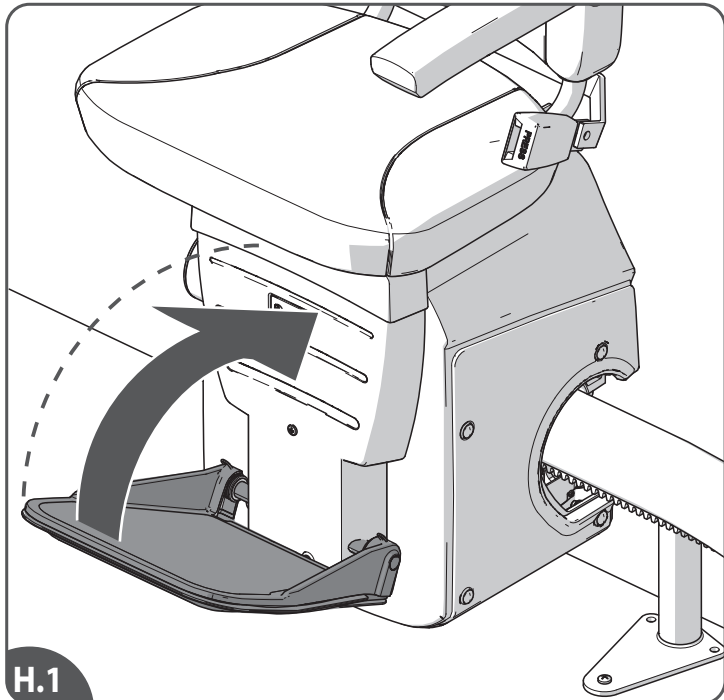
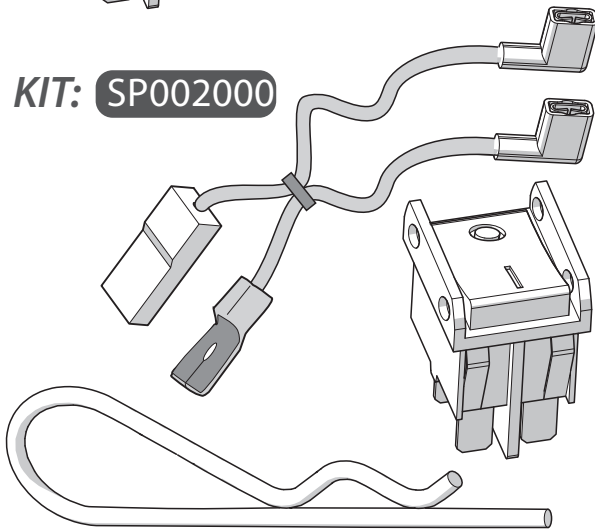
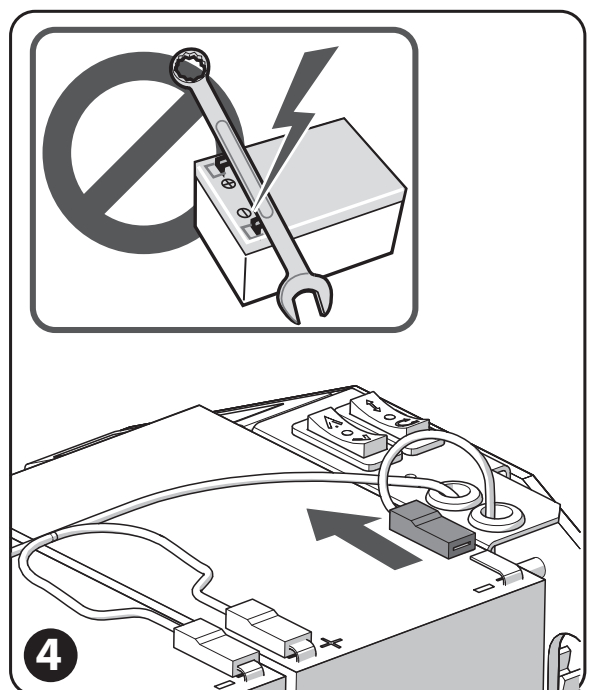
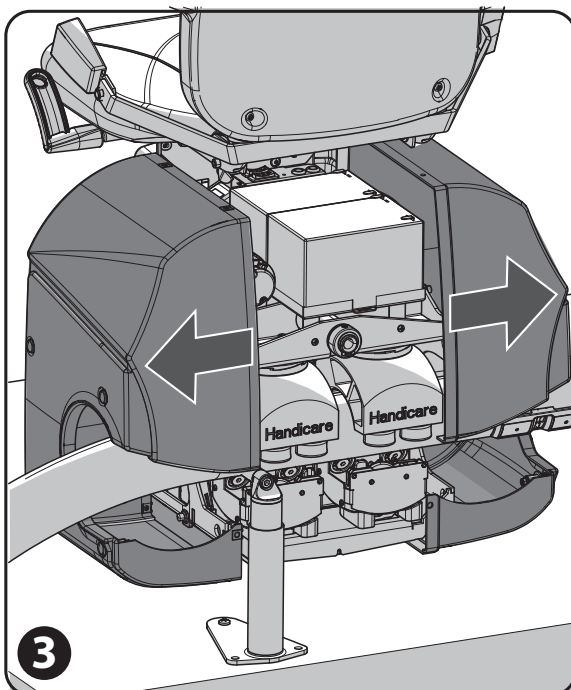
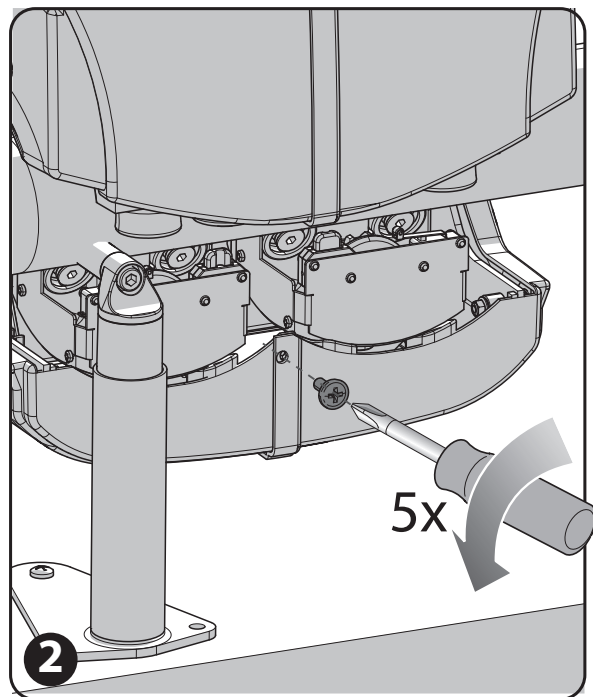
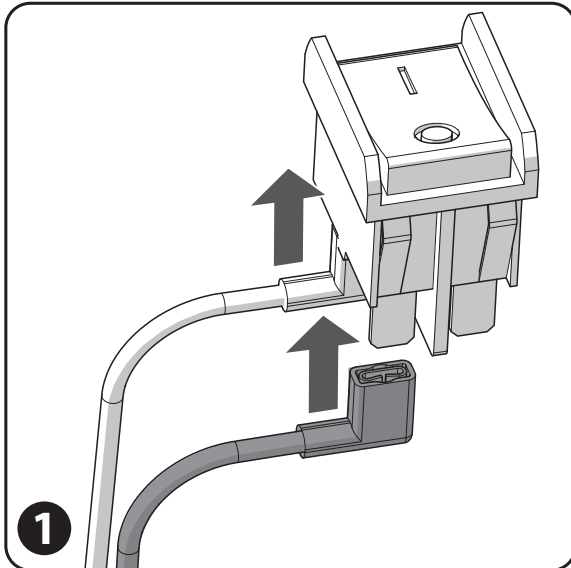
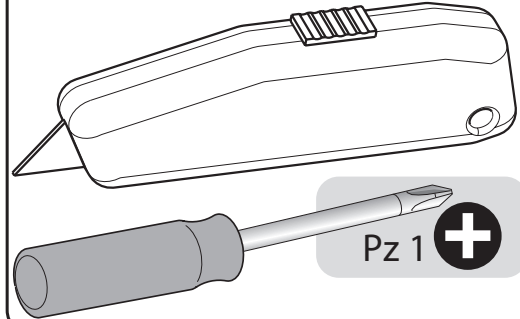
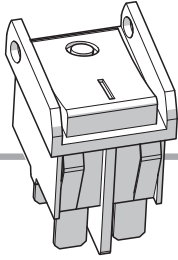
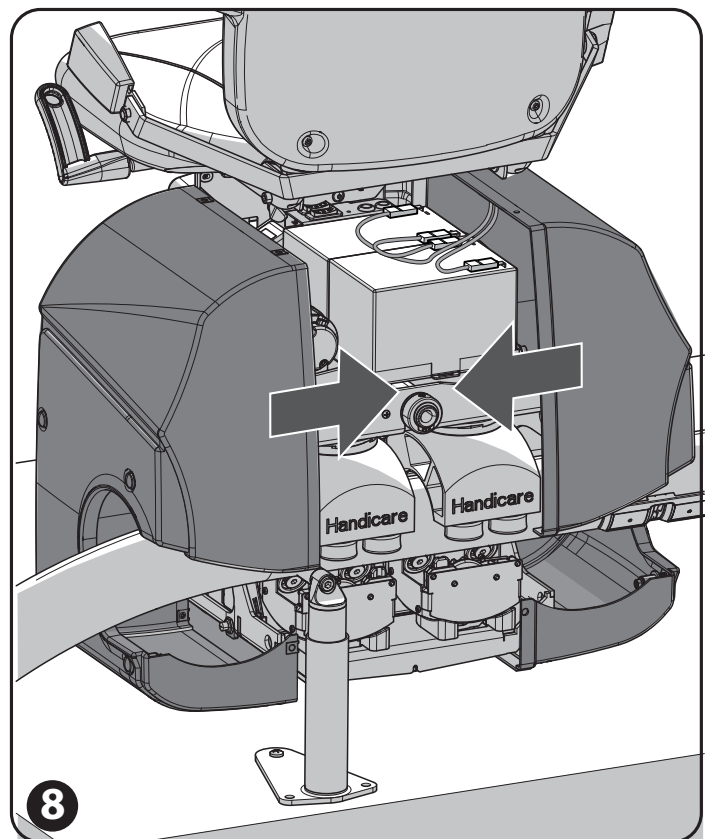
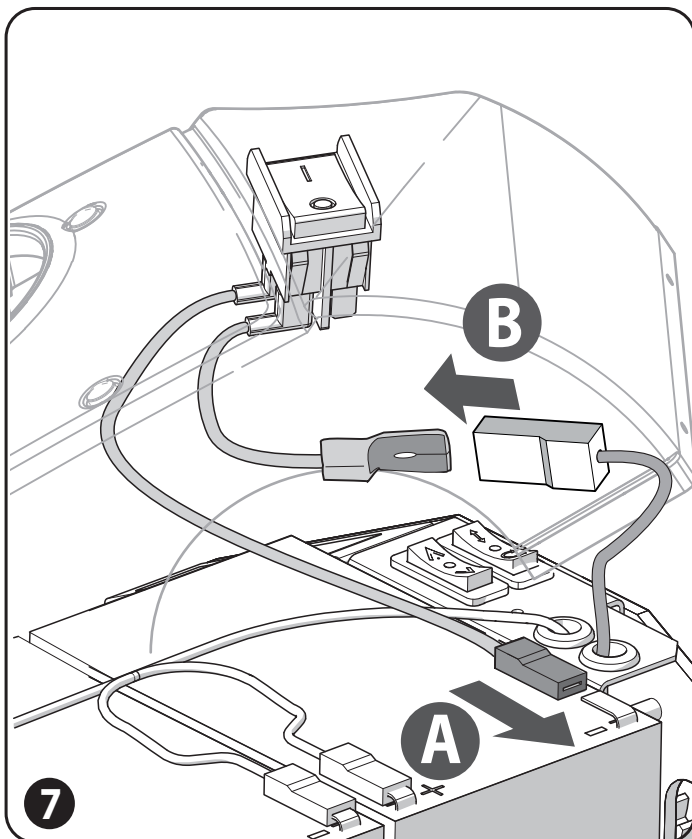
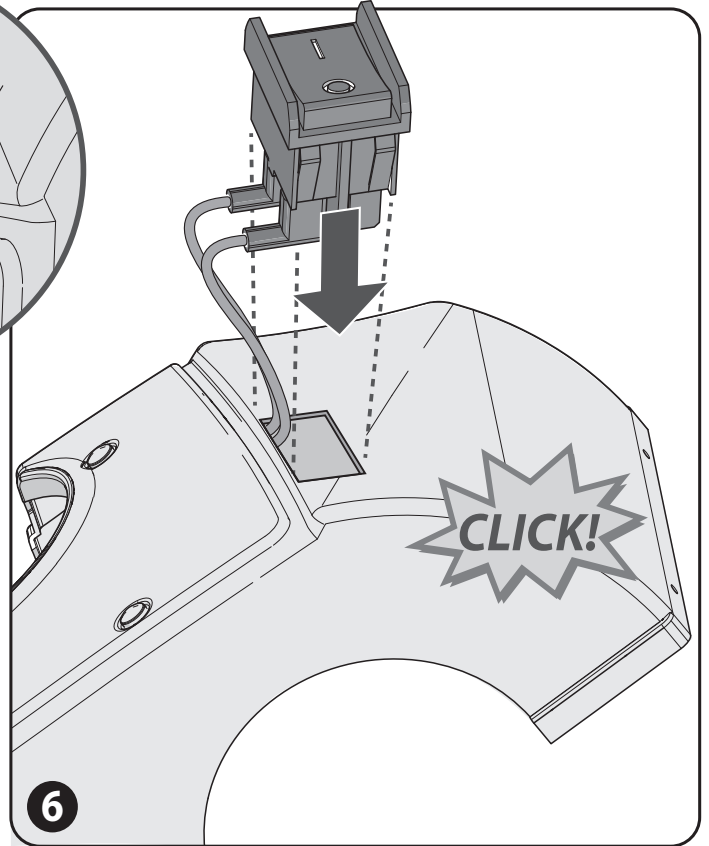
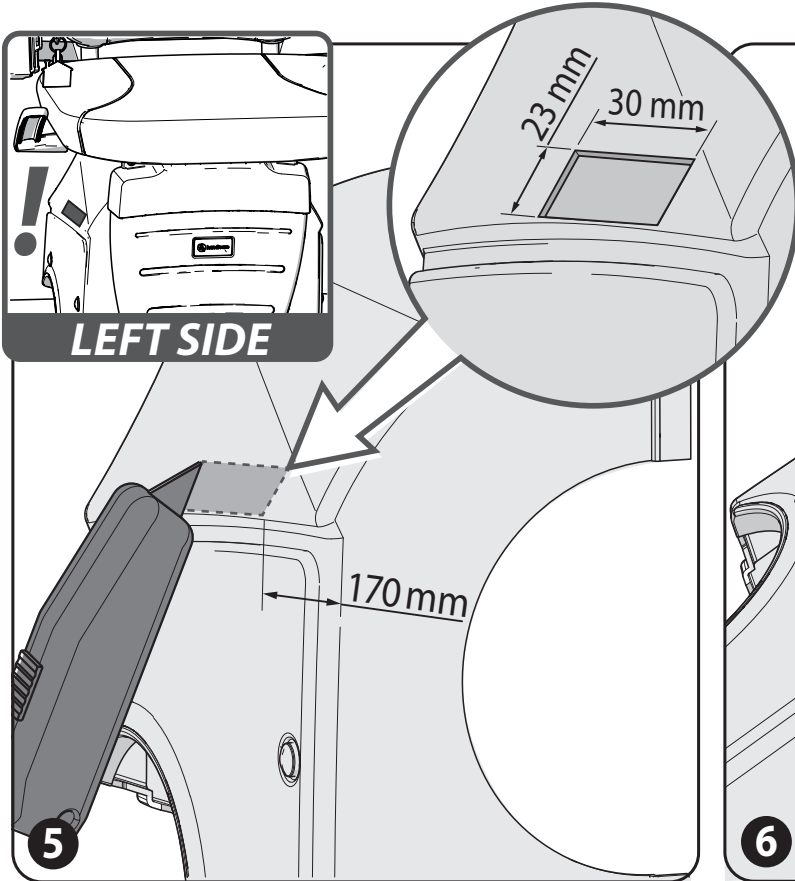
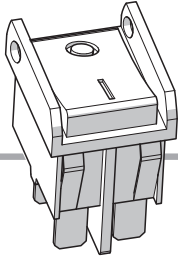
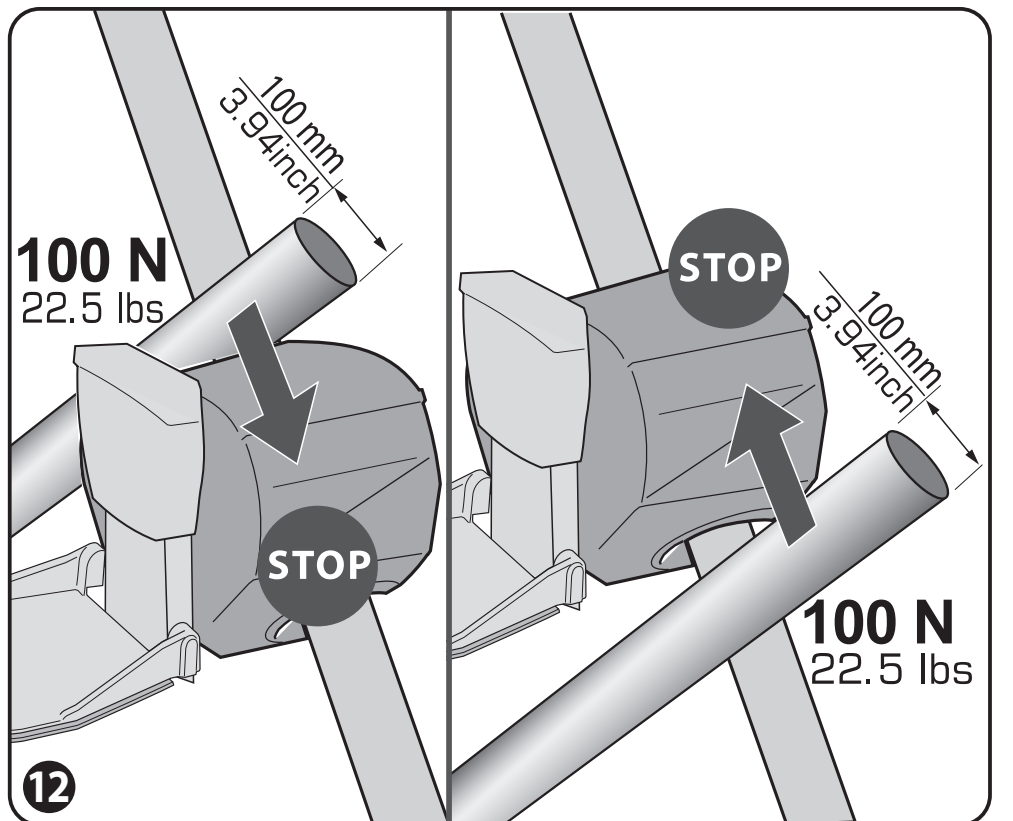
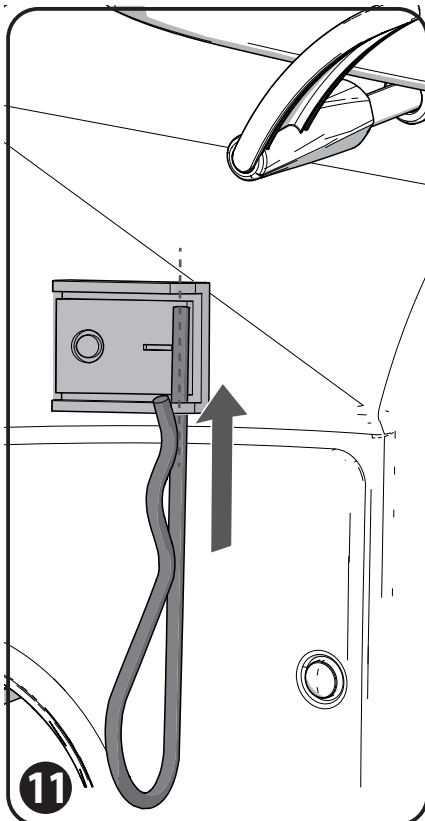
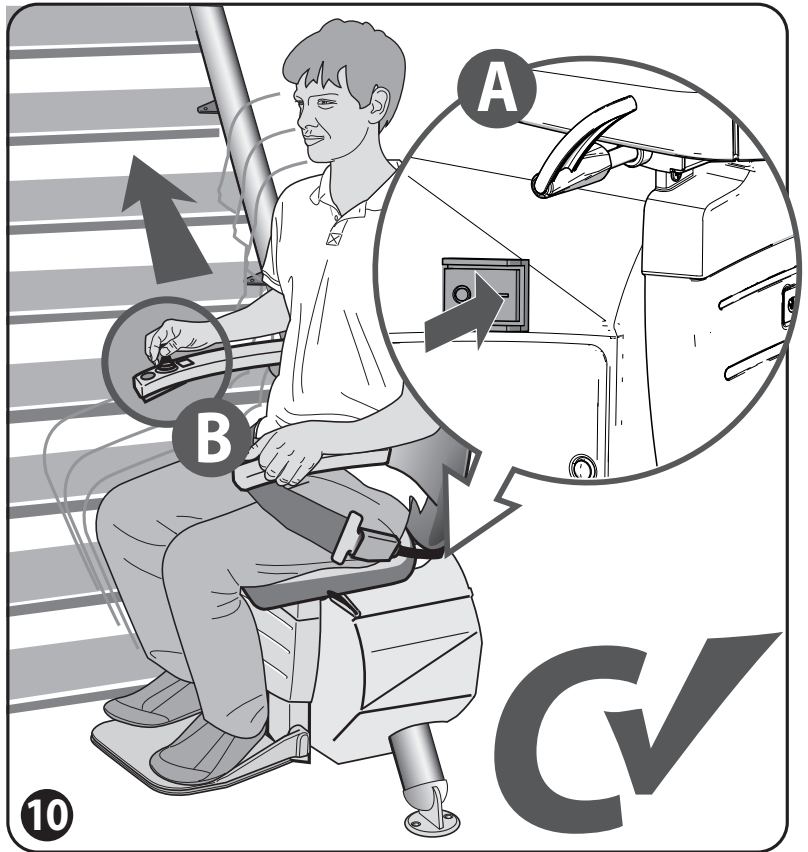
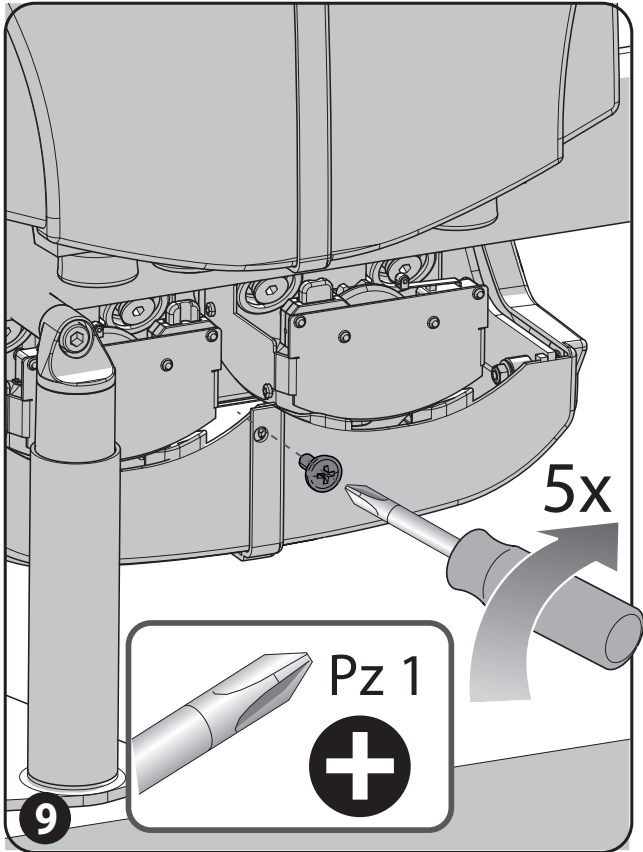
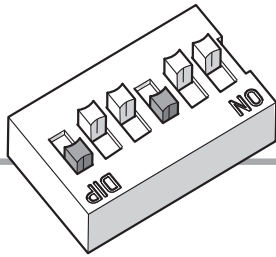
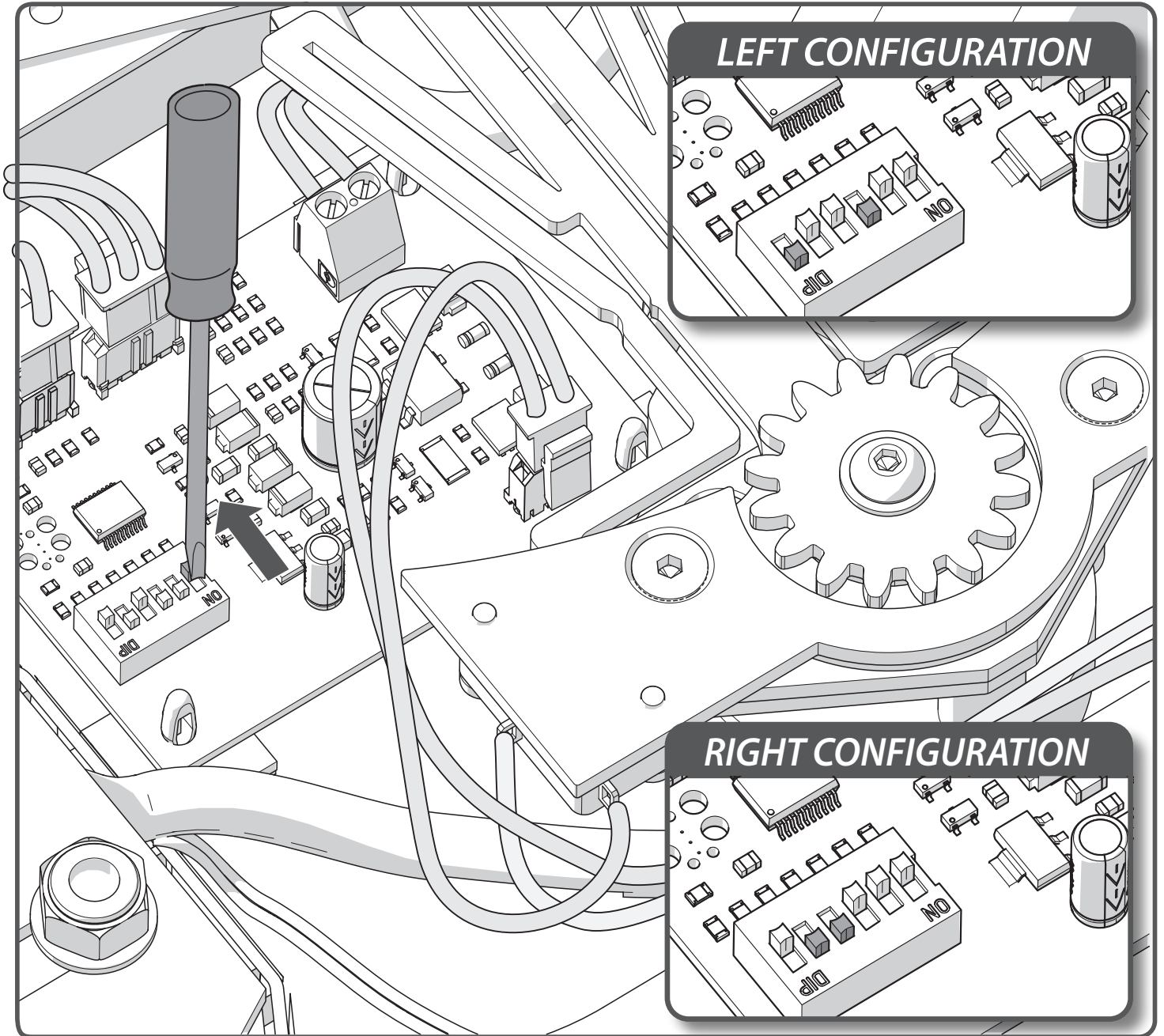
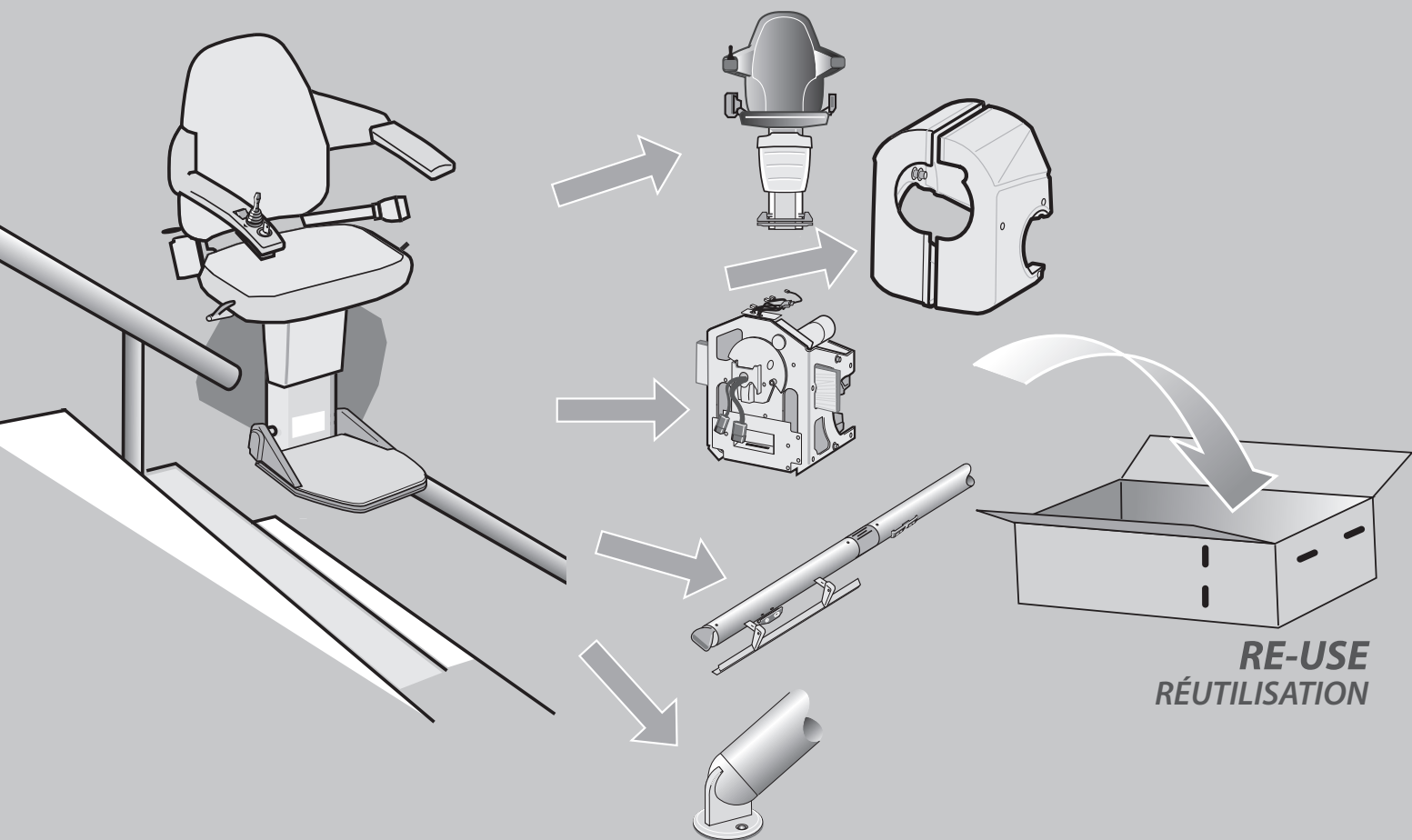
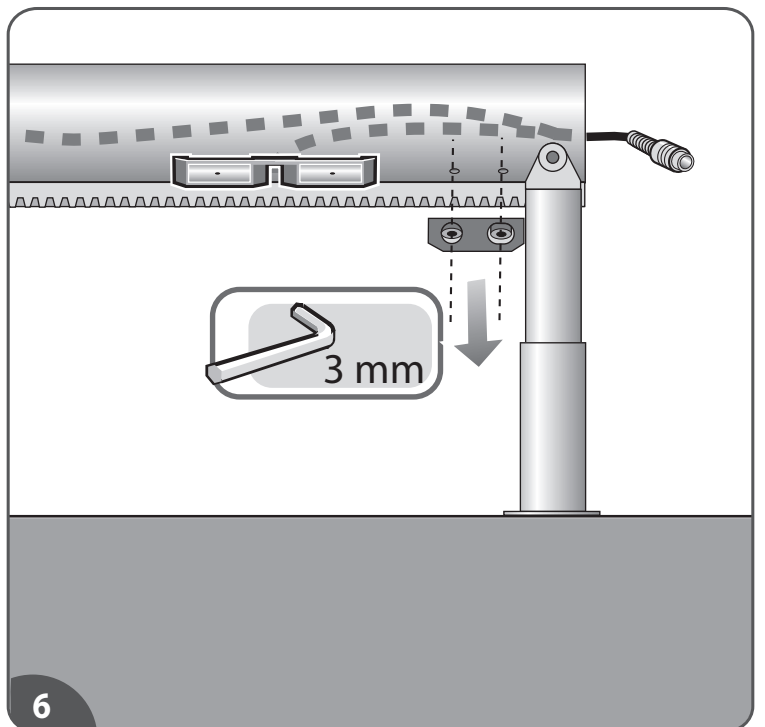
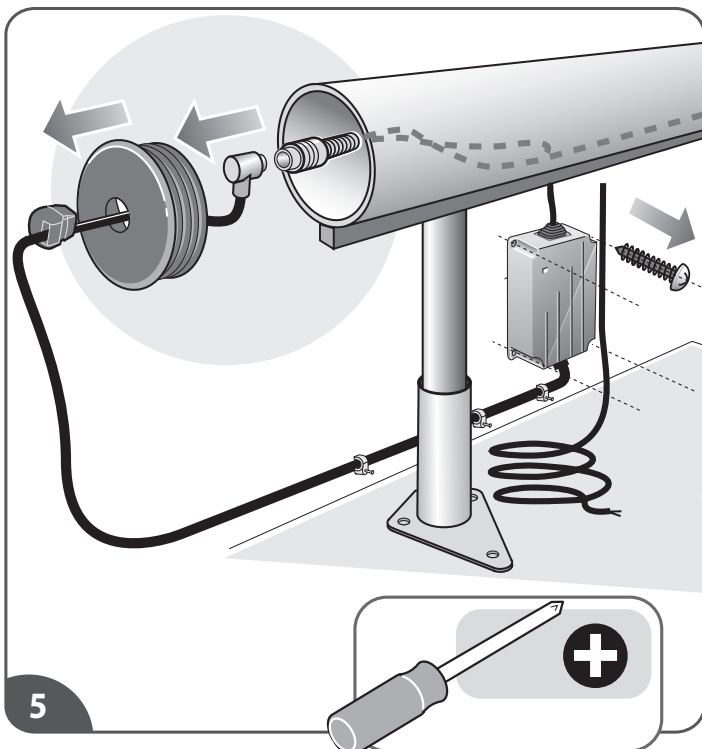
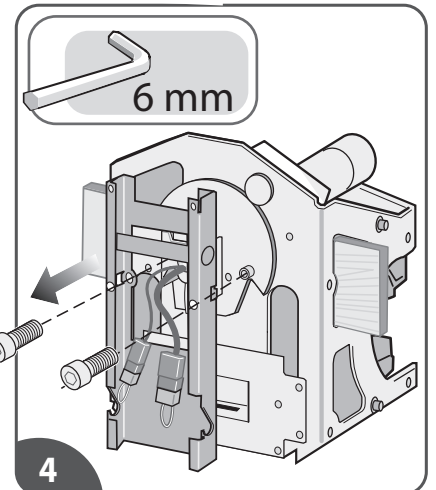
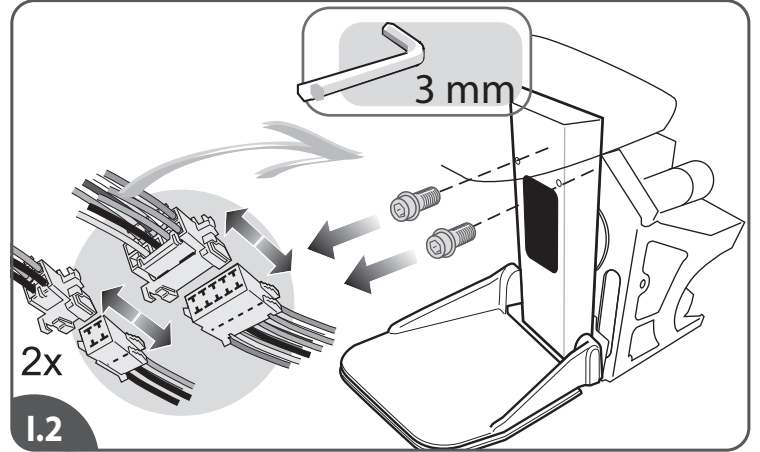
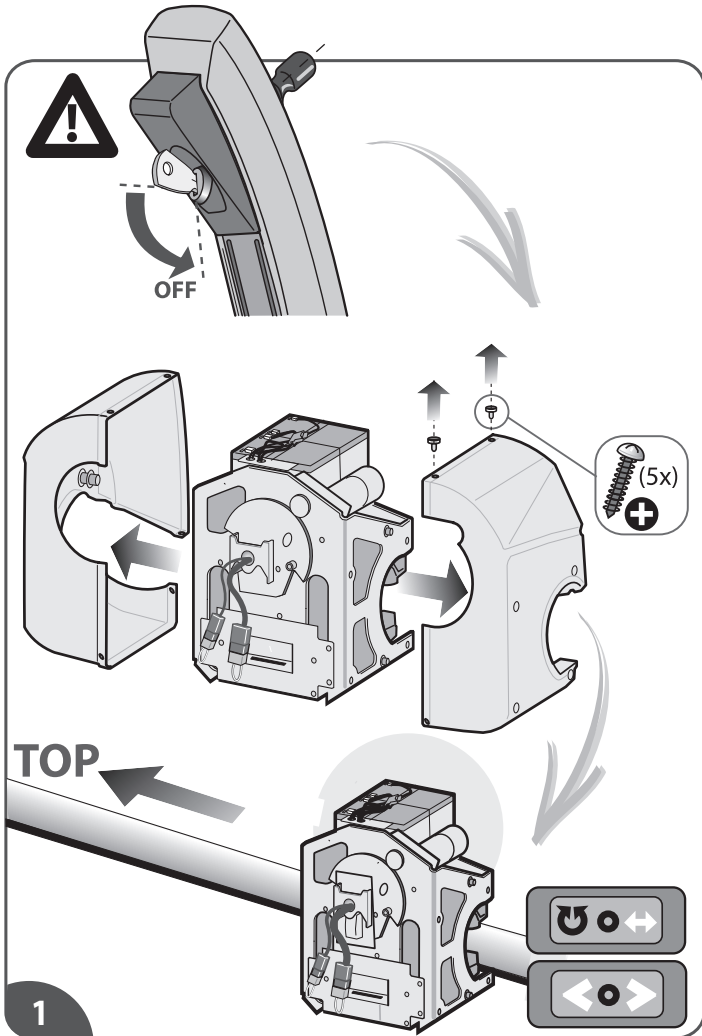
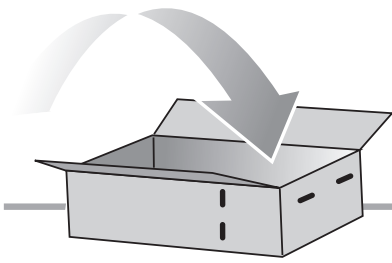
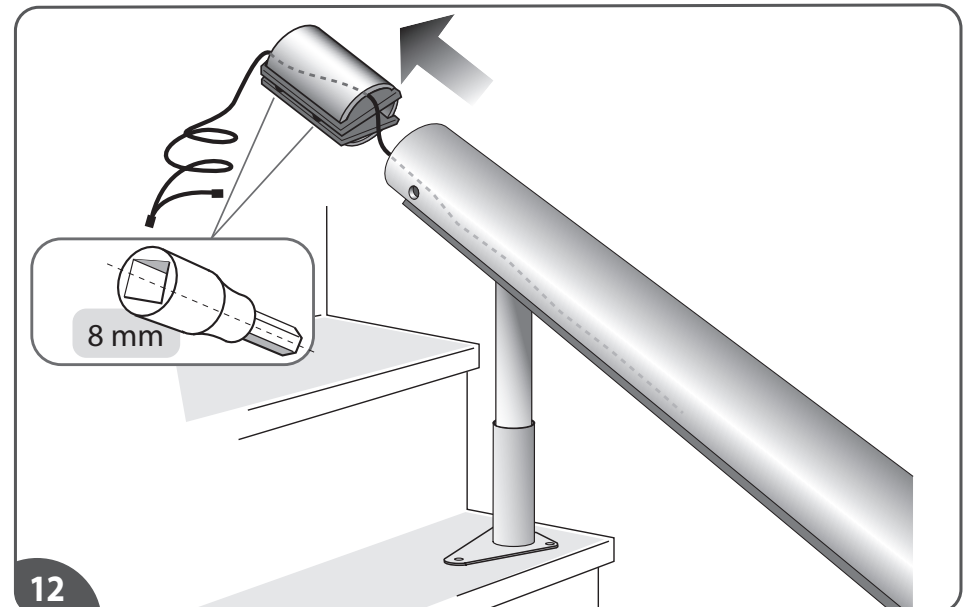
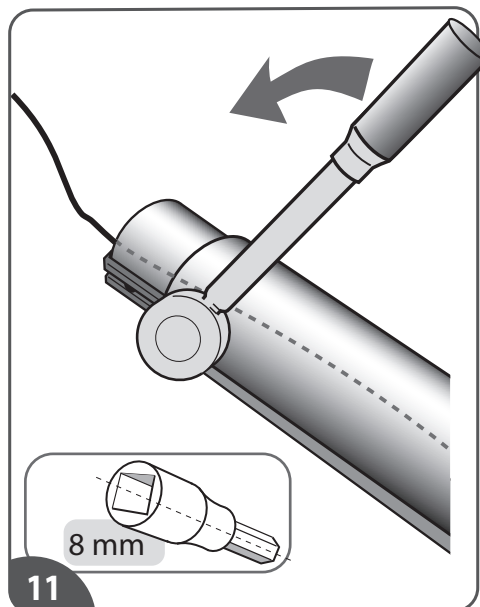
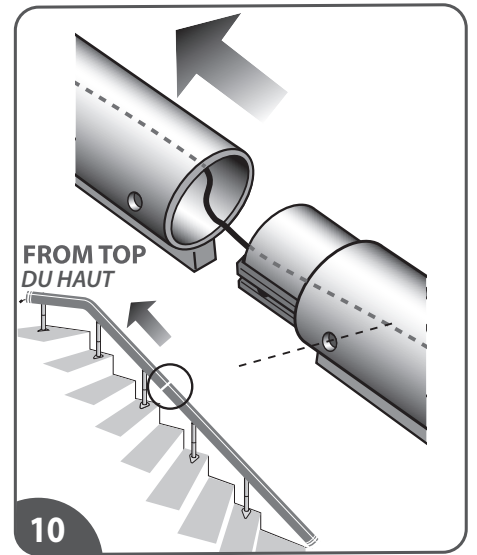
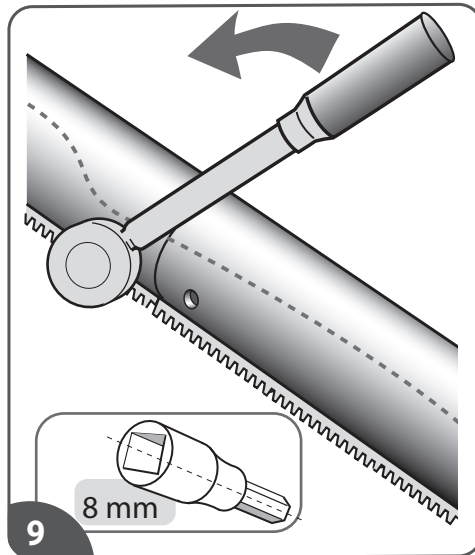
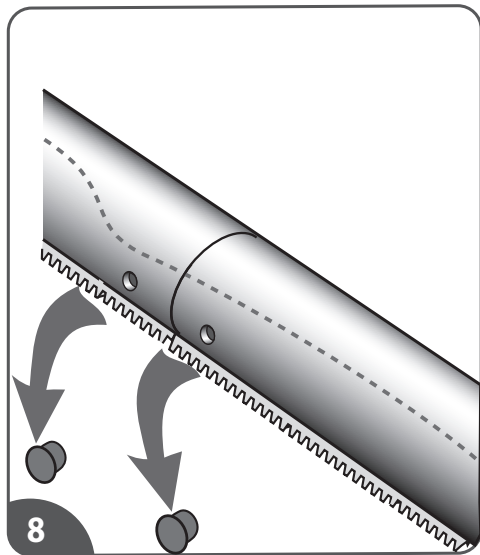
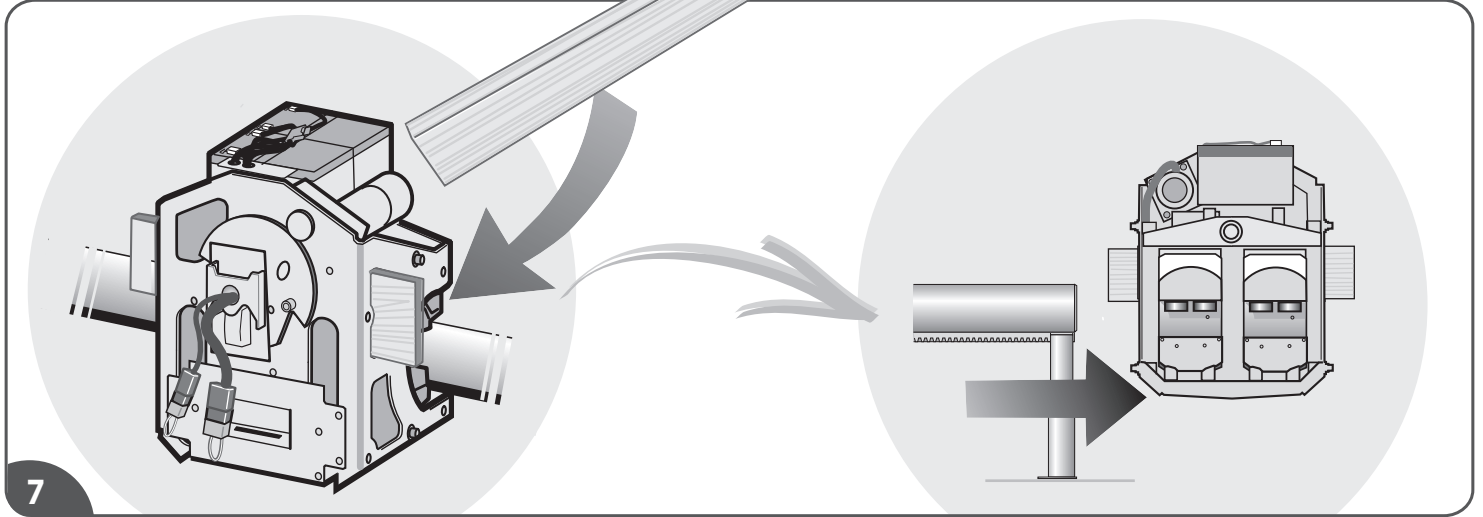
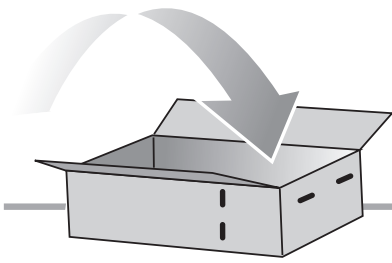
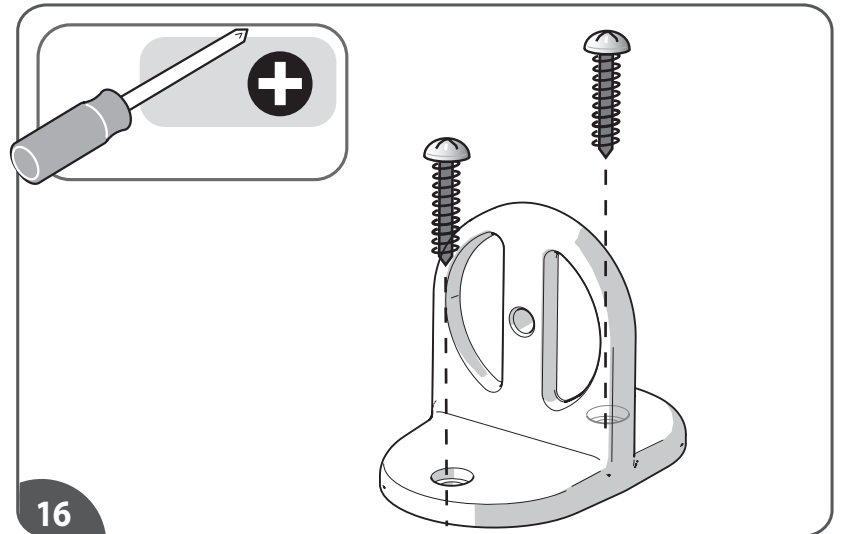
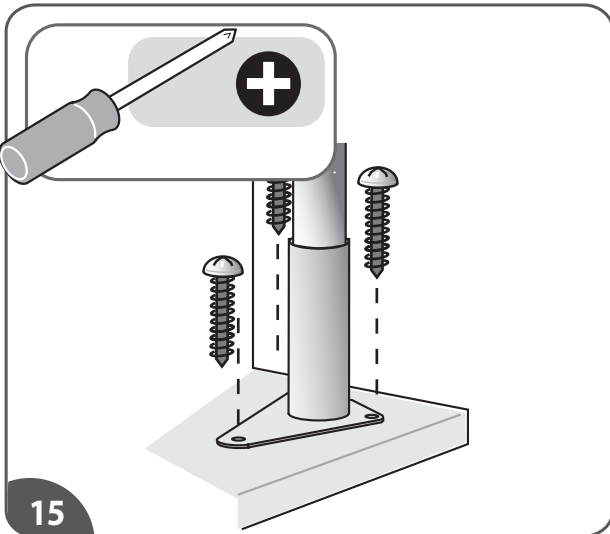
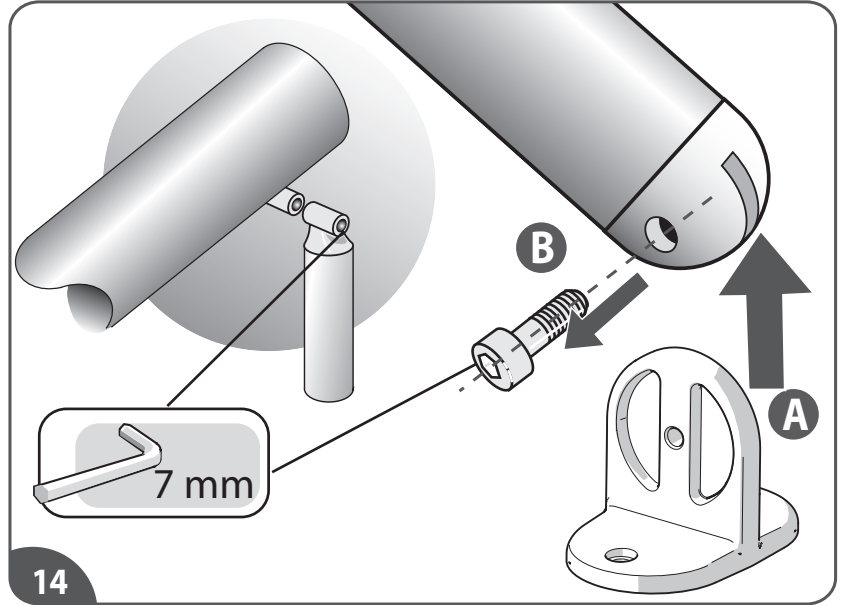
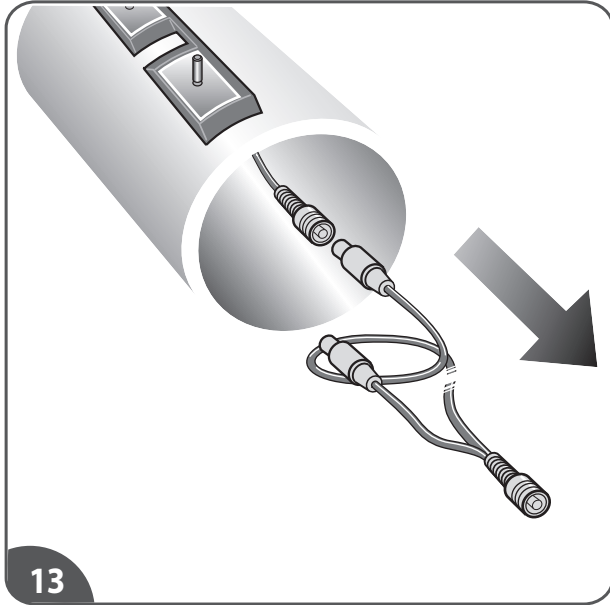
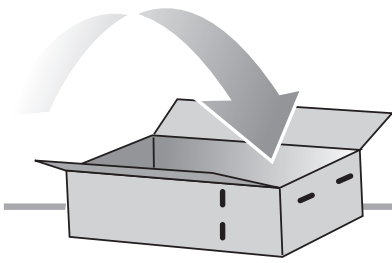
Handicare Accessibility Ltd.

82 First Avenue,
Pensnett Estate Kingswinford,
West Midlands DY6 7FJ,
England T +44 (0)1384 408700
enquiries@handicare.co.uk

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Savaria Canada

2 Walker Drive Brampton,
ON L6T 5E1 Canada
T 1.905.791.5555
T 1.855.728.2742
info@savaria.com

[www.handicare .com](http://www.handicare.com)

INSTALLATION MANUAL

MANUAL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONANLEITUNG
MANUALE PER L'INSTALLAZIONE
MANUAL DE INSTALACION
MANUAL DE INSTALACAO
INSTALLASJONSMANUALEN
INSTALLATIONSMANUAL
INSTRUKCJA

11P00045_P_151123

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Inc. 10888 Metro Court