Freecurve User Manual

Classic
Elegance
Alliance

CONTENT

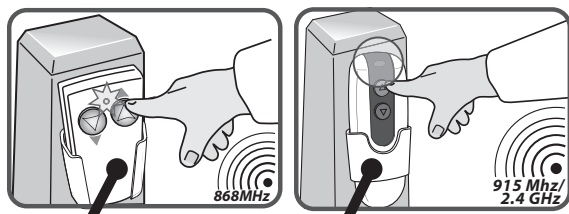
USER MANUAL	2
OPTIONS	11
IMPORTANT INFORMATION	15
FAULT FINDING	20
ATTACHMENTS	24
A. Reminders	24
B. Remote control	25
C. Tap 2 floor	26
D. Emergency operation	28

FOREWORD

Thank you for choosing a Handicare stairlift. This manual is for the Classic and Elegance/Alliance seat types, and includes instructions for the standard features and optional upgrades.

Make everyday life easier....

CLASSIC

8

1

2

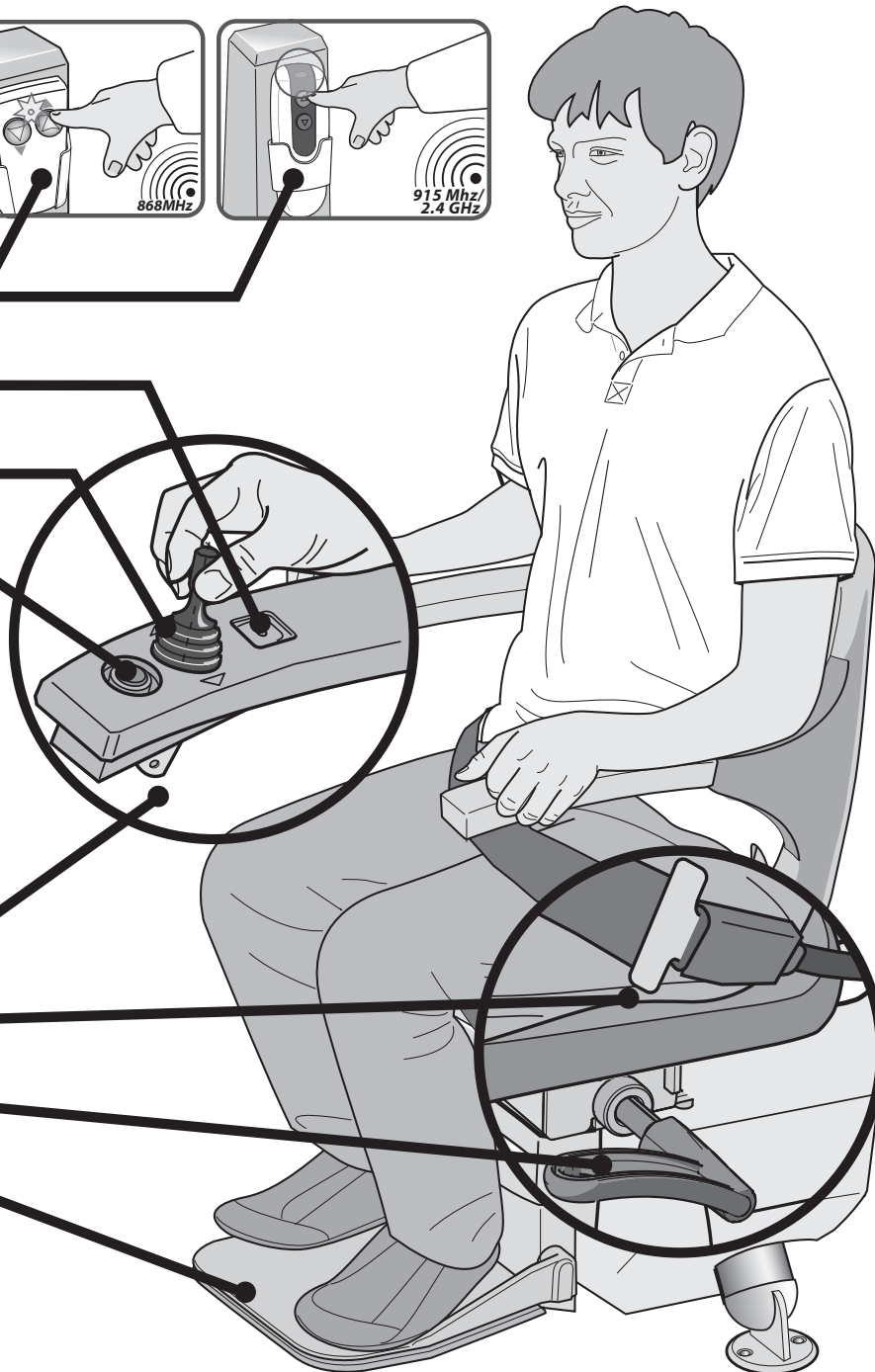
3

4

5

6

7

OPTIONS

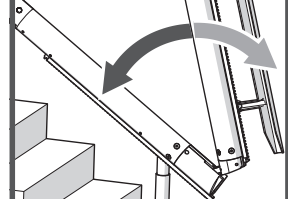
One-way
powered swivel


Pow. footrest

Folding hinge

Two-way
powered swivel

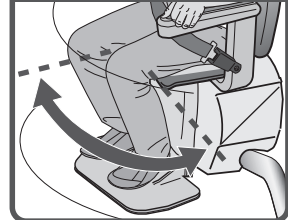
The illustration opposite shows the features used to operate the Classic seat. The Classic can be upgraded with the one-way powered swivel seat, powered footrest, hinged rail and two-way powered swivel options to create even more seat comfort and user-friendliness.


- [1] LED
- [2] Joystick
- [3] Stop button
- [4] Key switch
- [5] Seat belt

- [6] Seat swivel lever
- [7] Footplate
- [8] Remote control

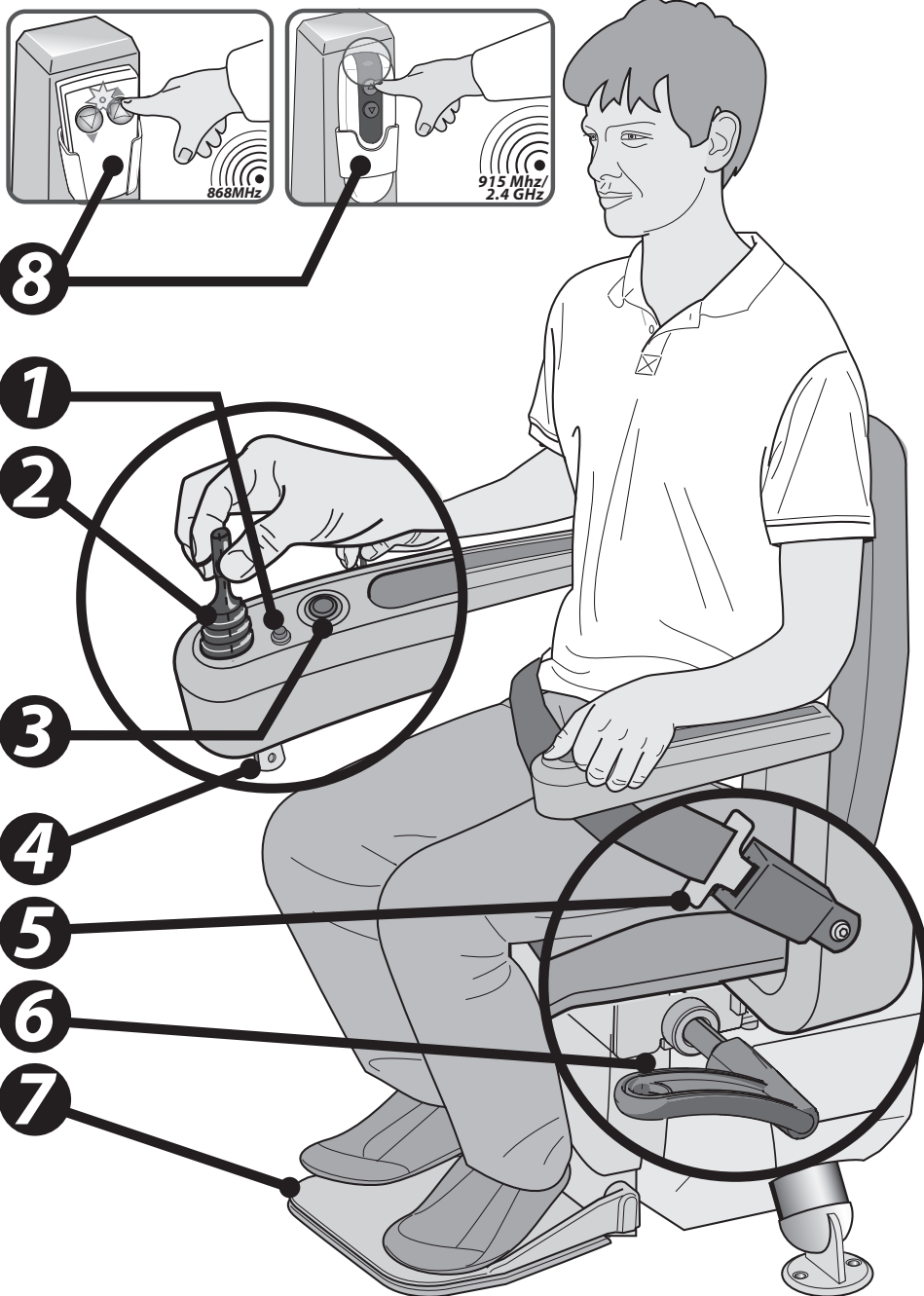ELEGANCE / ALLIANCE


OPTIONS

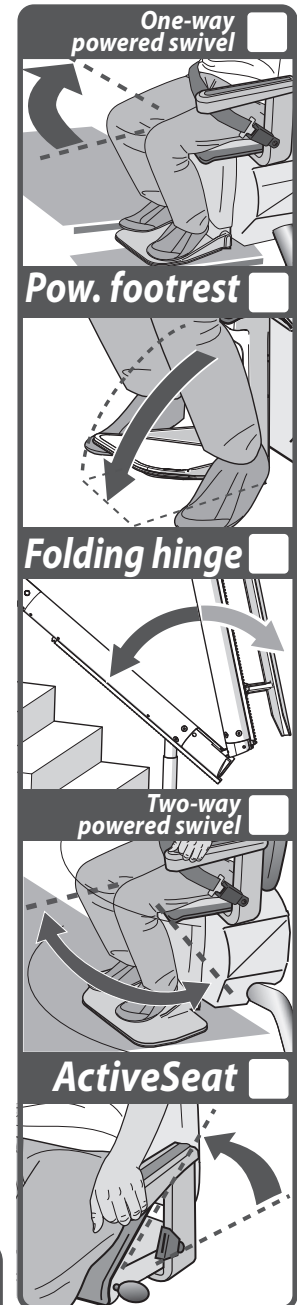The illustration opposite shows the features used to operate the Elegance/Alliance seat. The Elegance/Alliance can be upgraded with the one-way powered swivel seat, powered footrest, hinged rail, two-way powered swivel and Active Seat options to create even more seat comfort and user-friendliness.

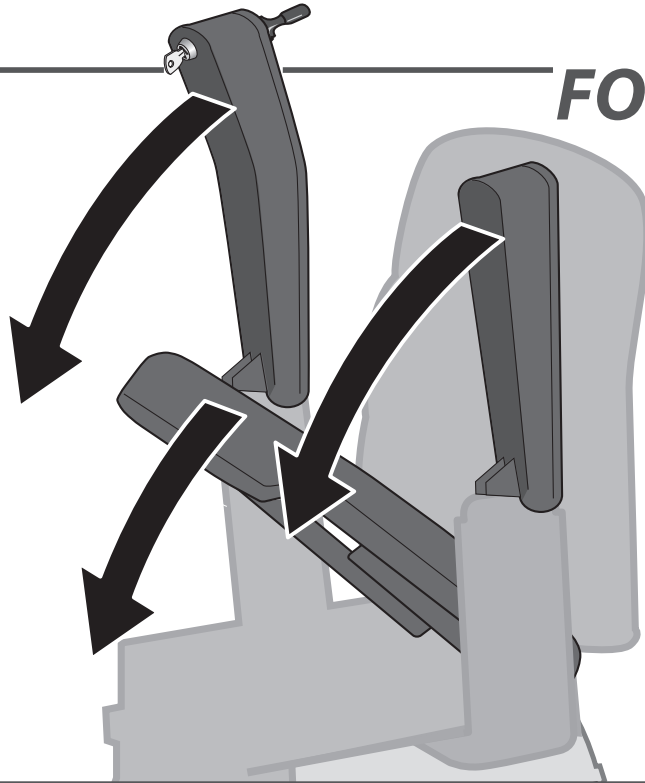- [1] LED
- [2] Joystick
- [3] Stop button
- [4] Key switch
- [5] Seat belt
- [6] Seat swivel lever
- [7] Footplate
- [8] Remote control

FOLD DOWN

Fold down the armrests and seat.

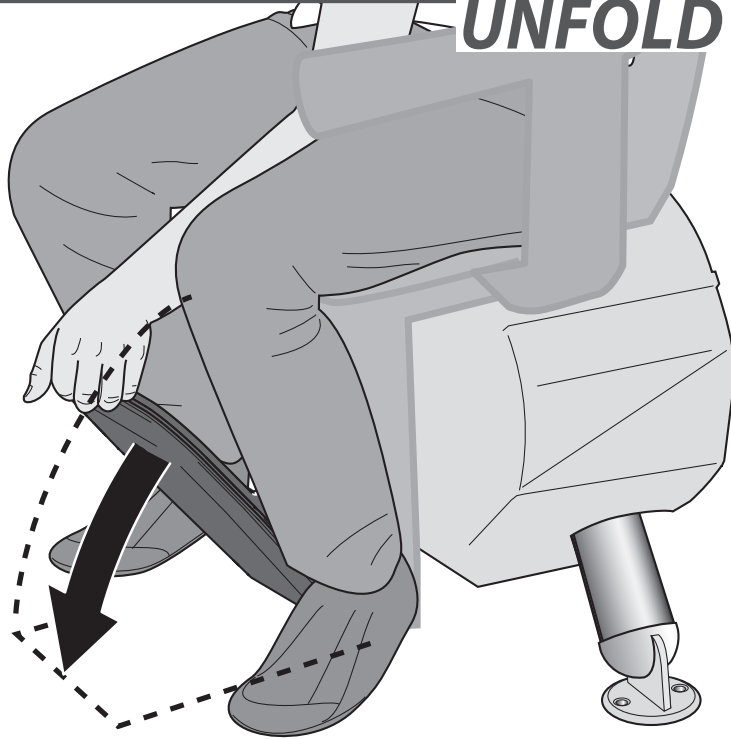
SIT DOWN

Sit down on the seat.

page 14 > OPTIONAL ACTIVE SEAT


UNFOLD FOOTREST


Fold down the footplate.

 [page 12 > OPTIONAL POWERED FOOTREST](#)

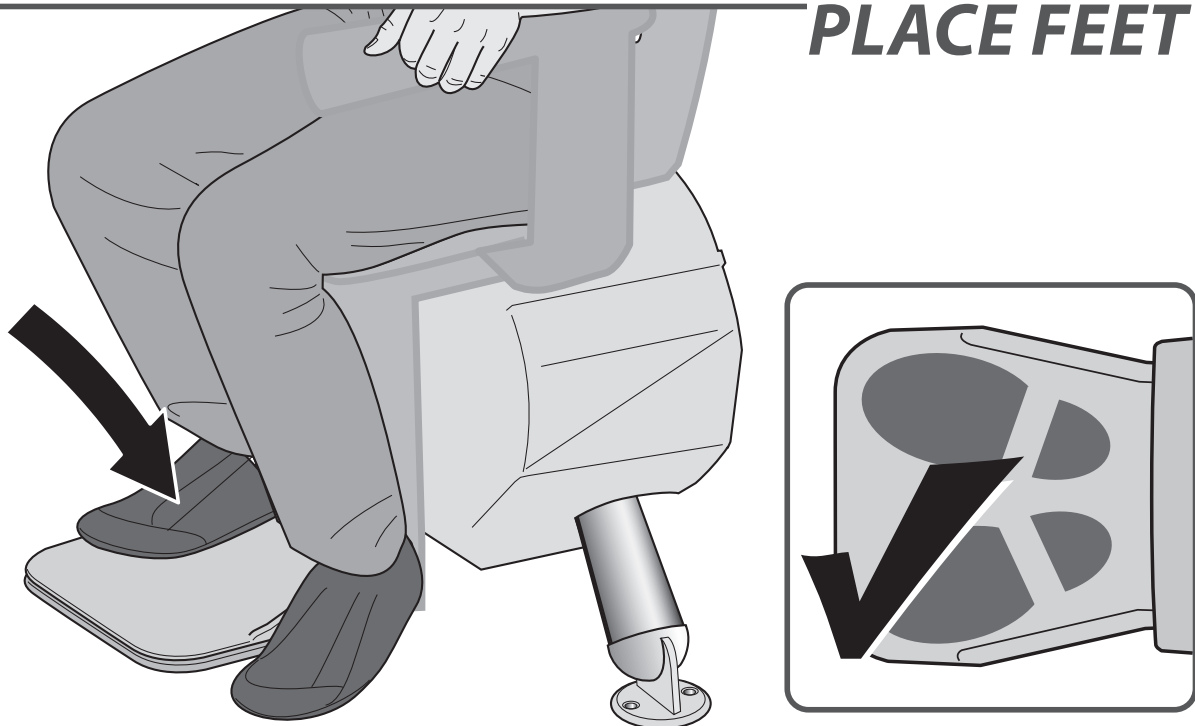
PLACE FEET

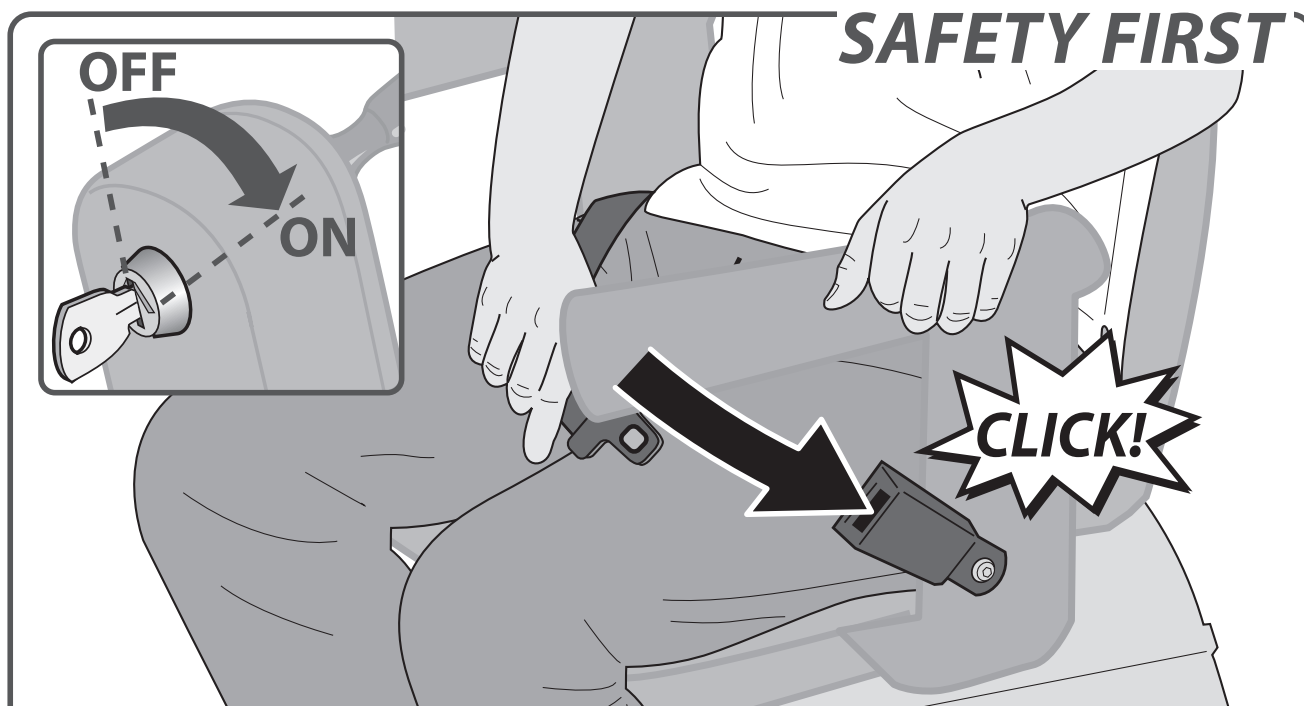Place your feet on the footplate. Ensure that both feet are completely on the plate.

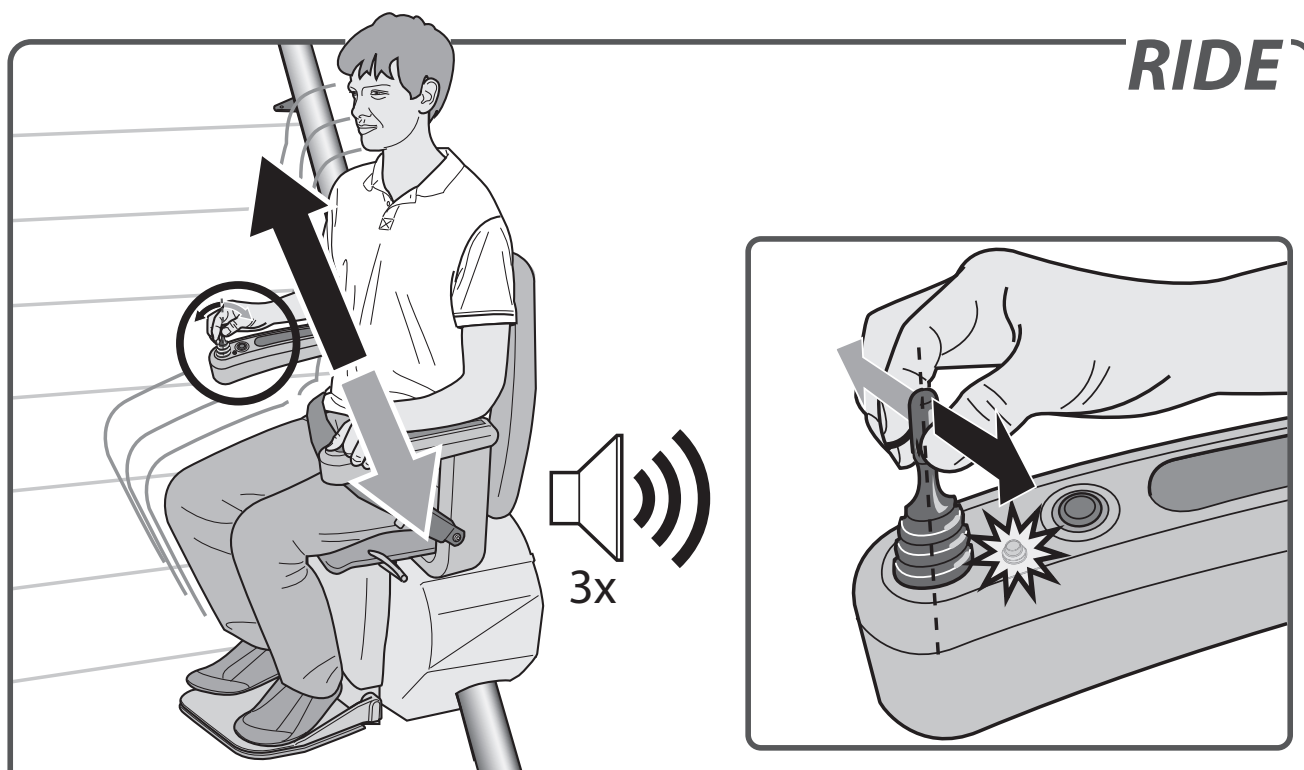Fasten the seat belt securely, ensuring that it clicks into the holder. Put the key in the switch and turn it ninety degrees clockwise. The LED on the armrest lights up faintly.

Move the joystick in the desired direction of travel. The LED lights up. The lift beeps three times, then gradually starts to move.

STOP BUTTON

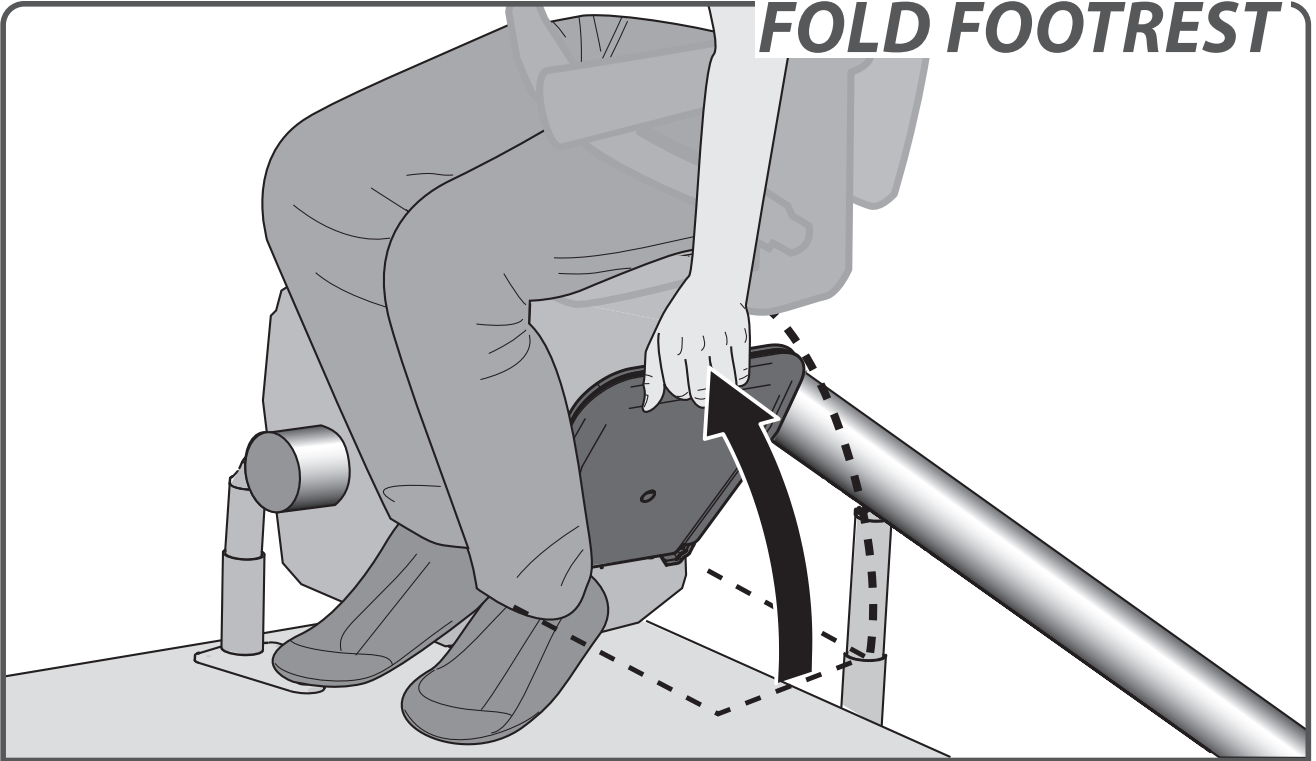
In an emergency, press the stop button once. The lift stops straight away. To release the lift, press the stop button again.
Optional: the emergency stop can be released by turning the emergency button.

SWIVEL


The lift stops automatically at the end of the rail. You can now swivel the seat manually, making it easier to get on and off. To do this, push down the lever{A}. Swivel the seat until you hear a 'click'. The seat is now in the correct position{B}.

page 11 > OPTIONAL AUTOSWIVEL

FOLD FOOTREST

Fold up the footplate.

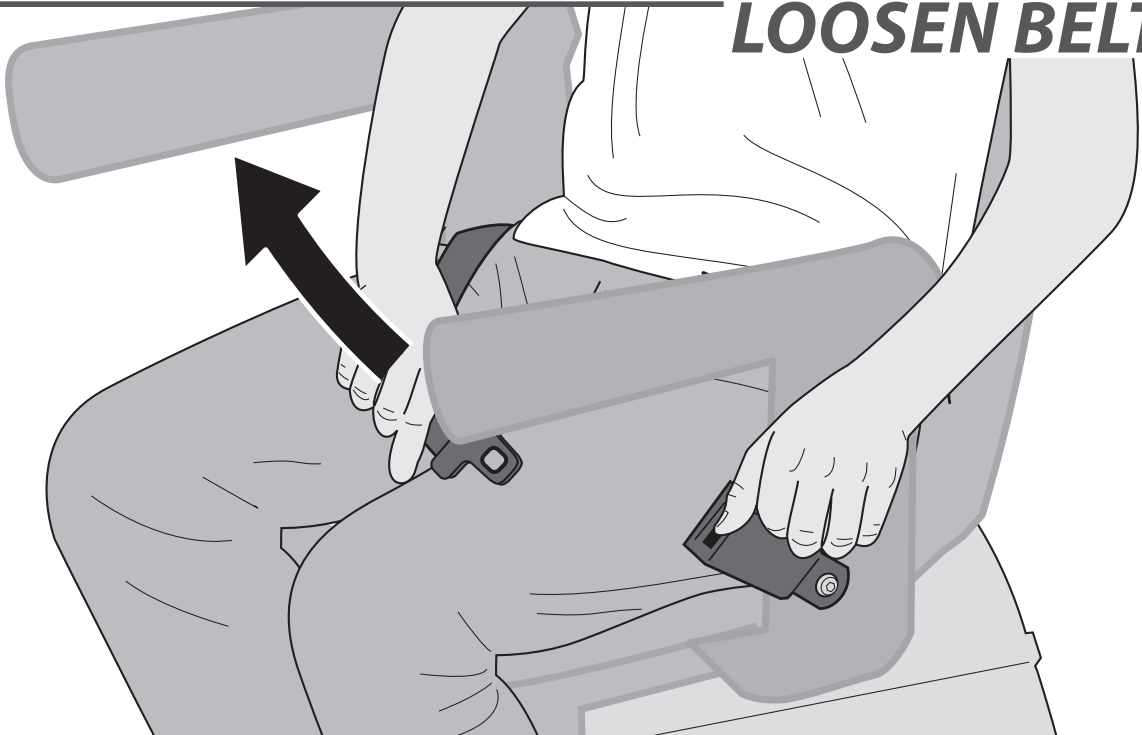 [page 12 > OPTIONAL POWERED FOOTREST](#)


LOOSEN BELT

Unfasten the seat belt.



STAND UP

Stand up from the seat.

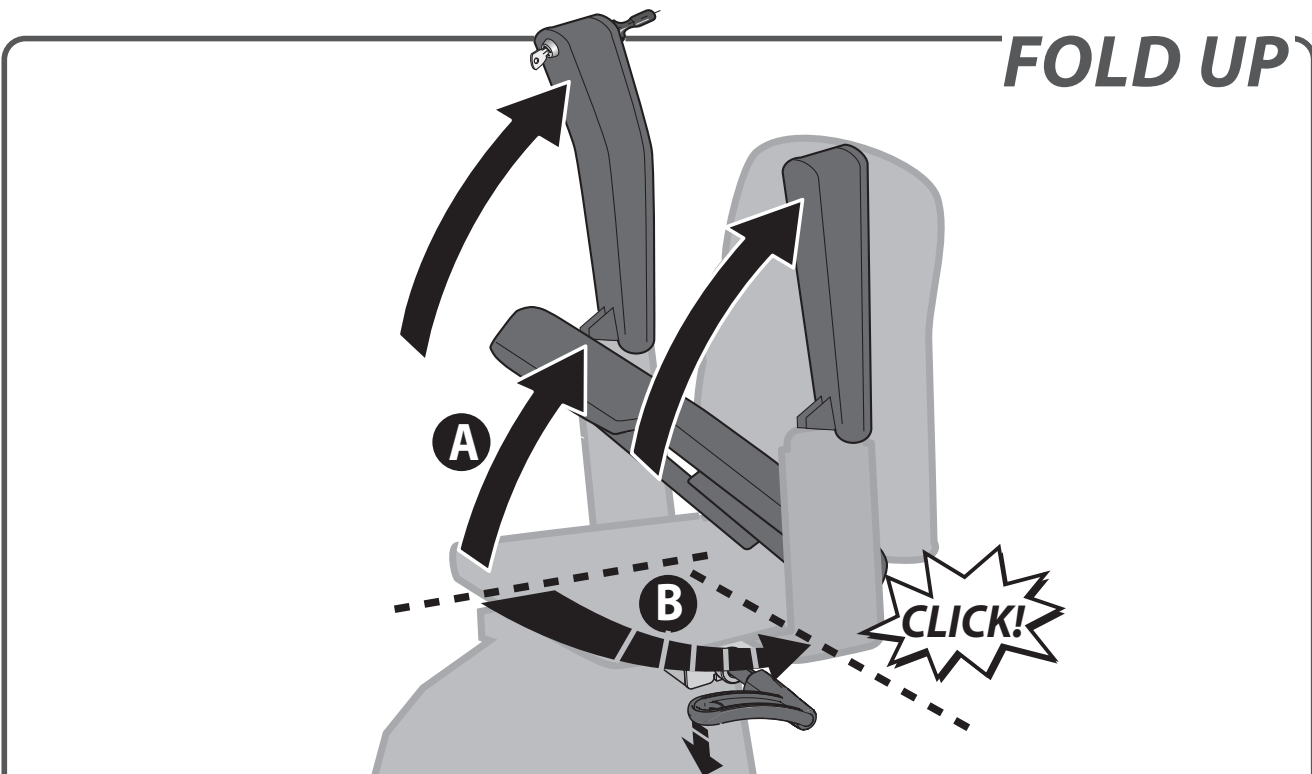 page 14 > OPTIONAL ACTIVE SEAT


FOLD UP

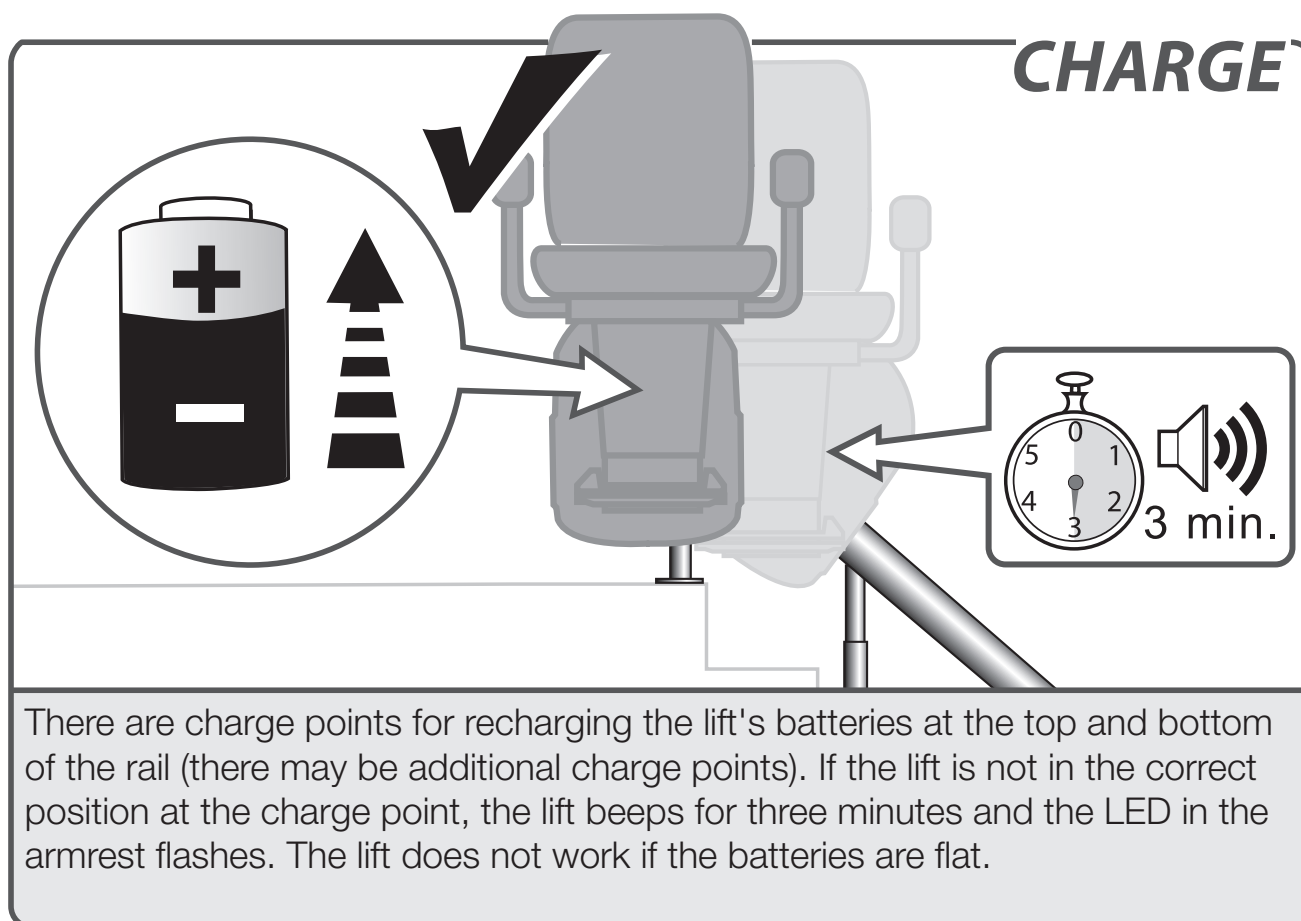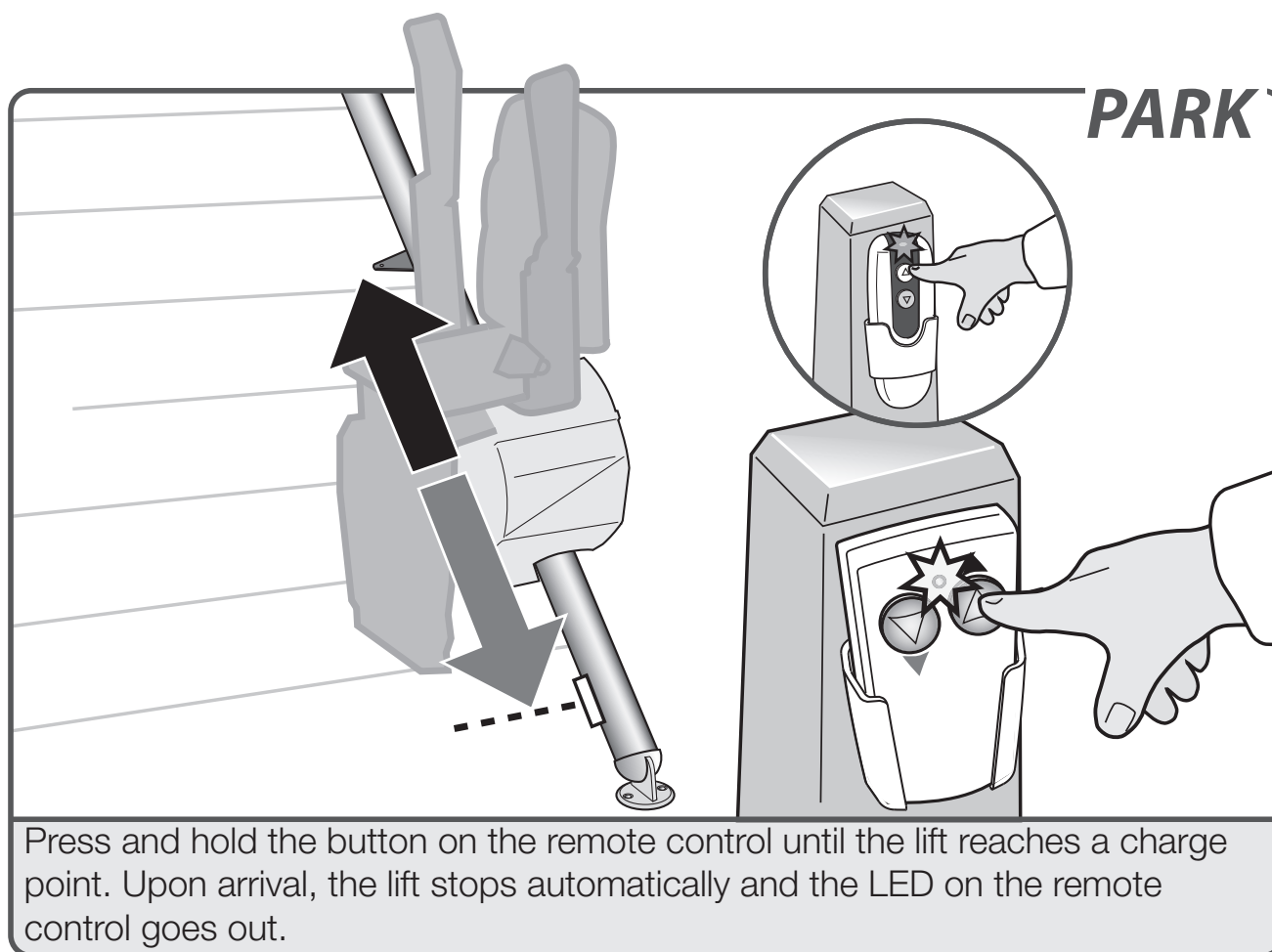
Fold up the armrests and seat {A}. Push the handle, located to the left or right under the seat, downwards. Swivel the seat until you hear a 'click'. The seat is now in the locked position {B}.


OPTIONS & FAULTFINDING

ONE-WAY POWERED SWIVEL	_____	<i>page 11</i>
POWERED FOOTREST	_____	<i>page 12</i>
FOLDING HINGE	_____	<i>page 12</i>
ACTIVE SEAT	_____	<i>page 14</i>
TWO-WAY POWERED SWIVEL	_____	<i>page 14</i>
OBSTACLE	_____	<i>page 20</i>
CHAIRPOSITION	_____	<i>page 20</i>
STOP BUTTON	_____	<i>page 21</i>
ARMREST	_____	<i>page 22</i>
KEY	_____	<i>page 22</i>
BATTERIES RC	_____	<i>page 23</i>
POWER	_____	<i>page 23</i>

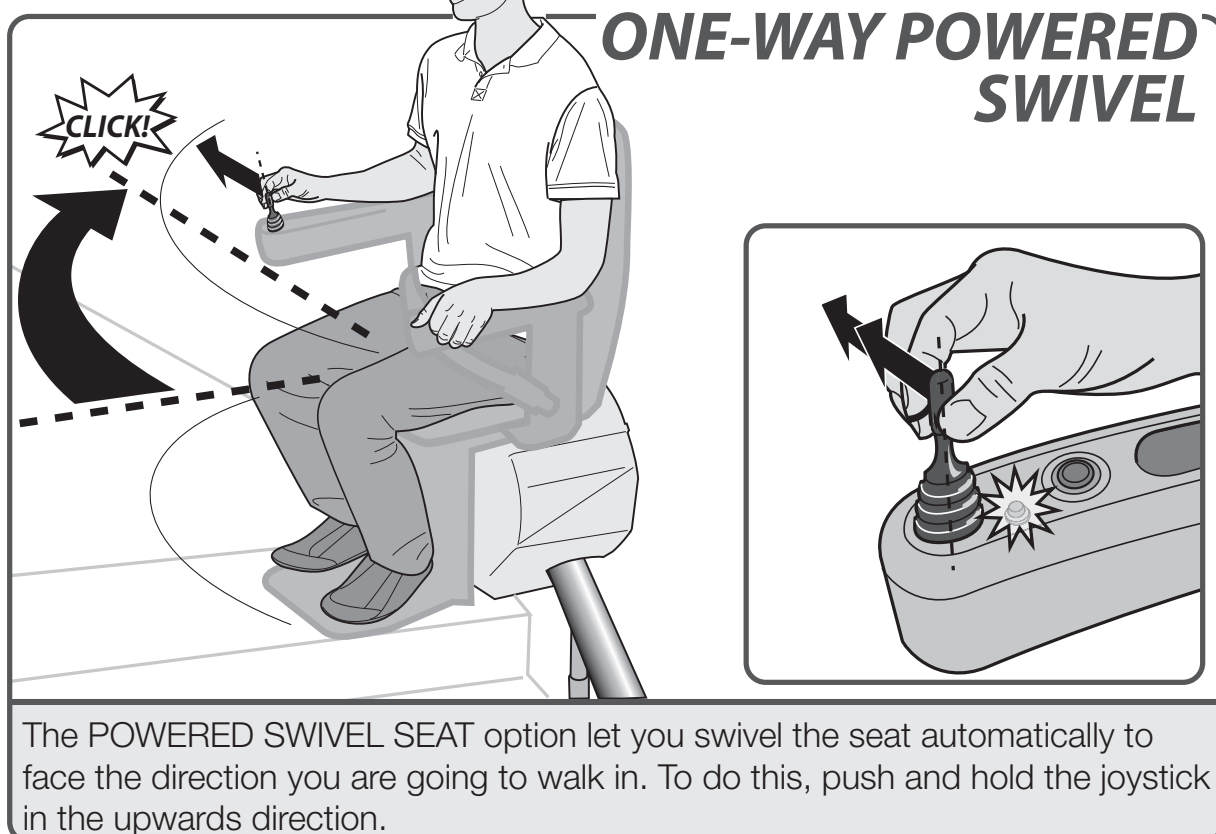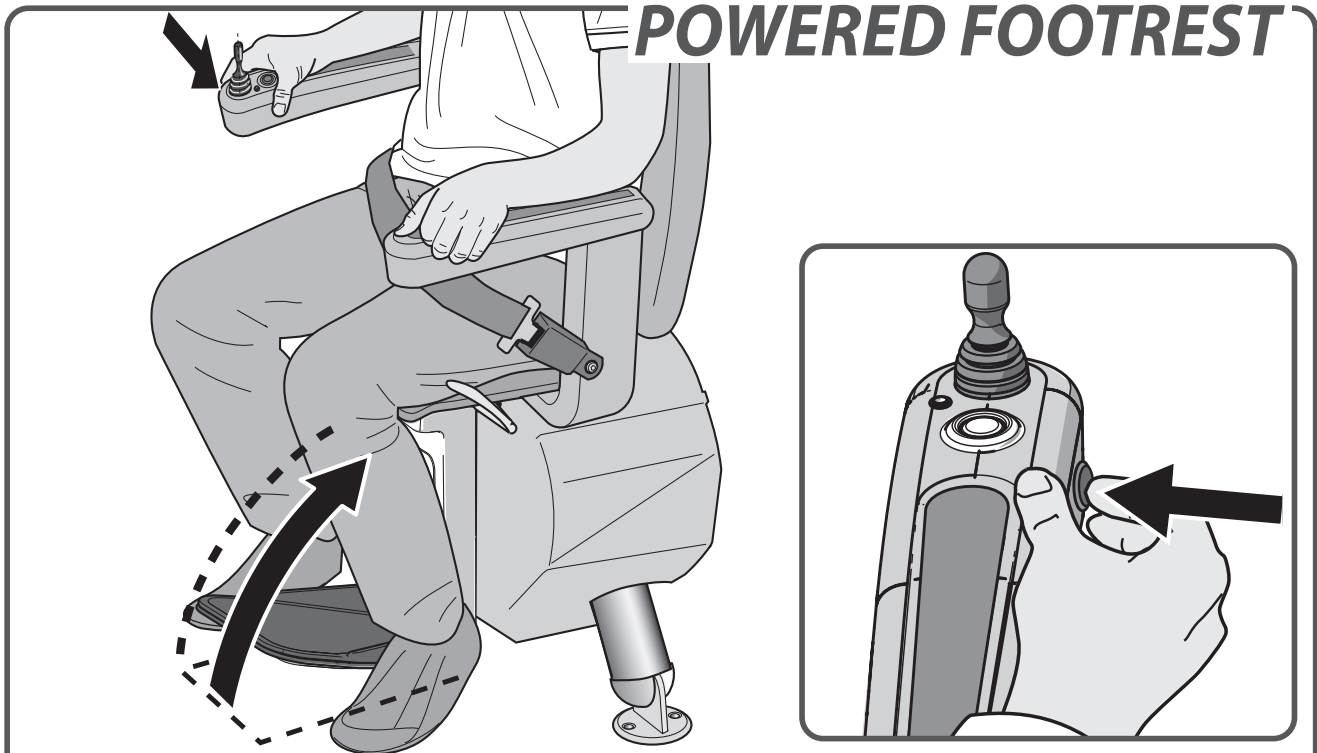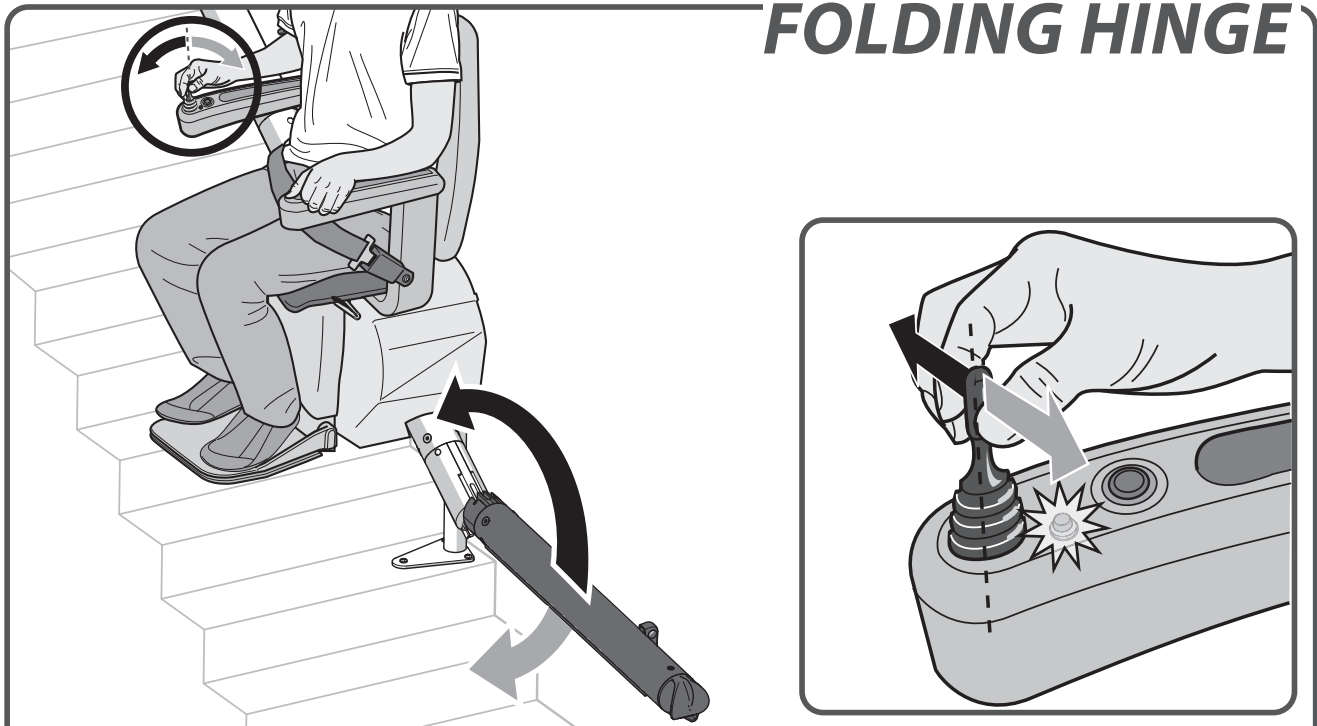
The POWERED FOOTPLATE option lets you fold the footplate up and down automatically using the switch on the side of the armrest. The footplate lights up when folded down.

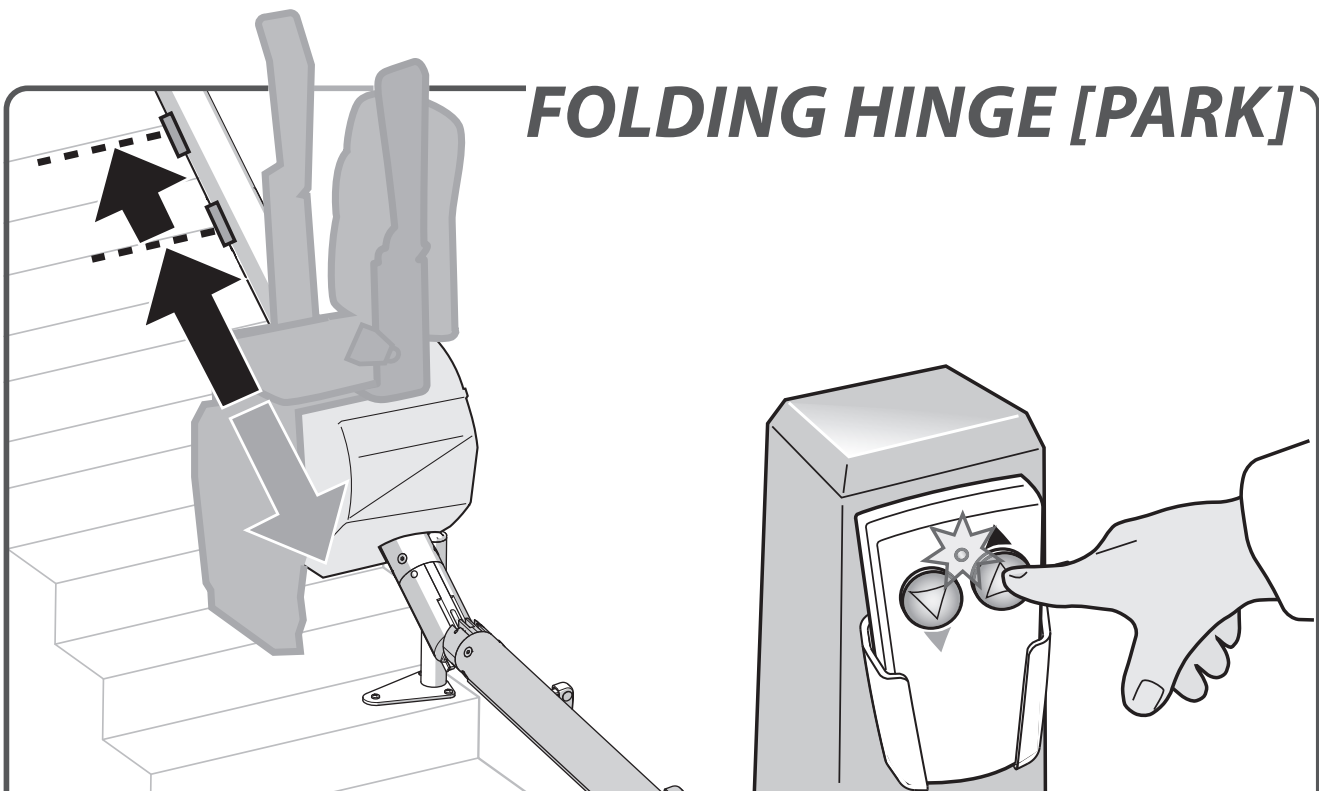
With the FOLDING HINGE option, the seat stops just above the hinged rail. Move the joystick up or down to fold or unfold the rail. When the hinge is not functioning, the lift can still ride in the upwards direction.


FOLDING HINGE [PARK]

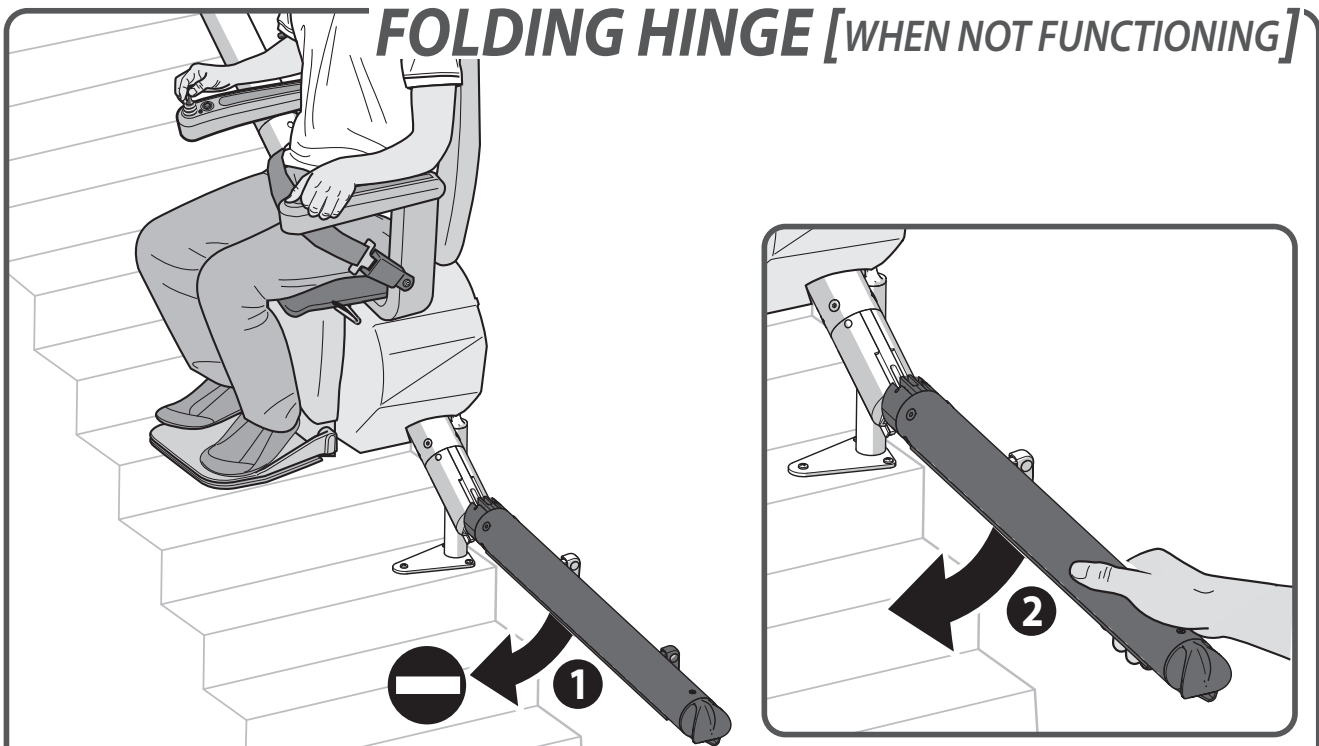Do not leave the lift on the hinged rail after use. Use the remote control to move the lift to a charge point and recharge the batteries. The first stopping point is at the hinged rail. Keep moving the lift until it reaches a charge point further up.


FOLDING HINGE [WHEN NOT FUNCTIONING]

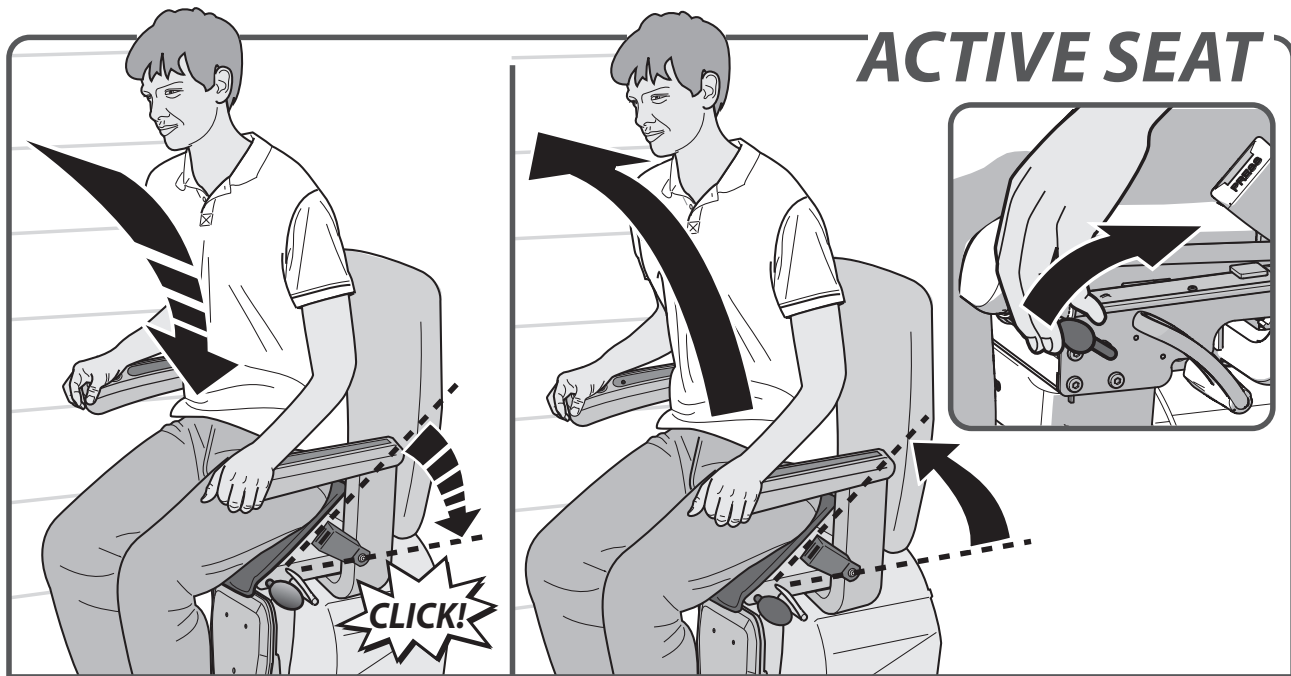When the FOLDING HINGE is not functioning [1], the hinged rail can (only in case of an emergency) fold down by hand [2].

With the ACTIVE SEAT option, the seat assists you when sitting down and standing up. To stand up, pull the handle, located to the left or right under the seat, upwards. The seat pushes you up. When you sit back down, the seat slowly returns to its normal position and clicks into place.

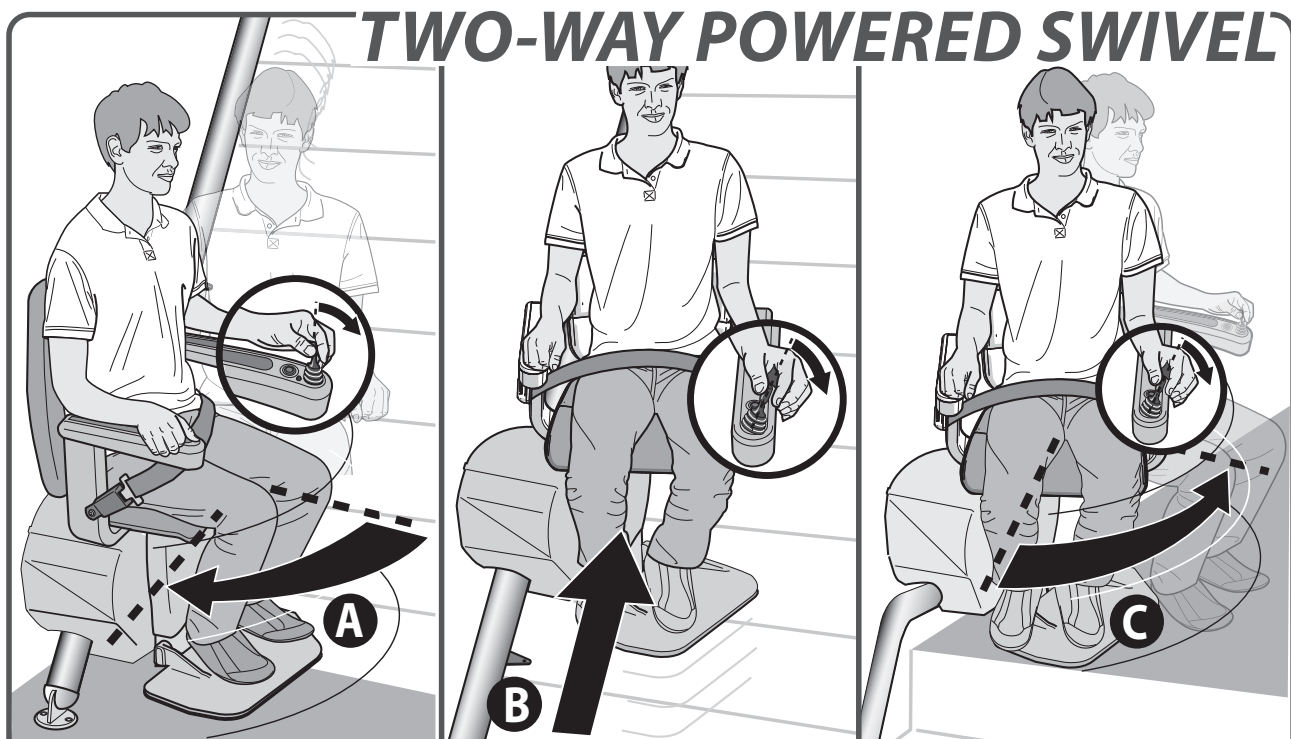The lift will not move until the seat is securely clicked in place.

The TWO-WAY POWERED SWIVEL option swivels the seat so that you travel up and down with your back to the stairs {B}. Hold the joy stick in the desired direction of travel, at the start and end of the rail {A and C}.

IMPORTANT INFORMATION

EC DECLARATION OF CONFORMITY FOR MACHINERY (Directive 2006/42/EC)

Manufacturer: Handicare
 Address : Newtonstraat 35
 1704 SB Heerhugowaard
 Postbus 39
 1700 AA Heerhugowaard
 Holland
 Tel.nr. +31 (0)72-5768888

Herewith declares that the

Freecurve

is in conformity with the provisions of the Machinery Directive 2006/42/EC,
 Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.

The stairlifts are 24V DC Powered Stairlifts moving in an inclined plane, used
 with an approved battery charger product.

The stairlift has been designed and manufactured in accordance with the
 following standards:

EN 81-40, ISO 9386-2, EN ISO 12100-1, EN ISO 12100-2, EN ISO 14121,
 EN60204-1, (including the EMC standards)
 ASME A18.1, CSA B-355

Liftinstituut B.V., Buikslotermeerplein 361, 1025 XE, Amsterdam, The Netherlands.
 Identificationnr 0400 has issued an EC-certificate in compliance with above
 mentioned directives and standards with reference NL02-400-1001-015-006.

The Technical Construction File for the above Stairlift is retained at the
 manufacturers address. A copy is retained at the Liftinstituut B.V.office.

Heerhugowaard, The Netherlands, 28 March 2019

J. Hopman
 Managing director Handicare stairlifts HHW

Project:

Date:

FOREWORD

Your stairlift is a sophisticated stairlift, specifically developed to carry one person sitting on the chair up to a maximum weight of 125 kg (275 lbs). The information provided in properties and working methods known to us at the time of publication; alterations and any improvements are therefore excepted. Before using the stairlift read this manual thoroughly and familiarise yourself with the working and operation of the stairlift. Users who can not see and hear should receive assistance when using the stairlift. Your stairlift has been constructed in accordance with state-of-the-art technology and satisfies recognized safety regulations. Nevertheless during the wrong use of the stairlift risk to the user or third parties, or damage to your stairlift or other goods, can arise. So use your stairlift only when it is in good working order, in accordance with the intended use described above and with due attention to this manual. Always keep this manual in the vicinity of the stairlift.

The strength and the suitability in general of wall, staircases, terraces, upper floors, banisters, plinths etc. which are to support or surround the installation have been verified and ensured by the customer, who therefore accepts all relative responsibilities. Neither Handicare nor its authorized dealers may be held responsible for the suitability of the structures which surround and support the installation. All tests, except for the operating test, shall be performed by the customer.

TECHNICAL SUPPORT

Should problems occur that are not dealt with in this document, contact your supplier. The Service department staff will always be pleased to help you. For quick handling it is important that you keep the following items/particulars to hand:

- this manual;
- your name, address, postcode, town and telephone number;
- lift number.

GUARANTEE AND LIABILITY

As the user of this stairlift, you are always fully responsible for the observance of locally prevailing safety regulations and guidelines.

Term of warranty – this warranty is valid for a period of 24 months from the date that the product ships from our factory. The purchase price for the product must be paid in full in order for the manufacturer to release parts under this warranty.

Coverage – this warranty applies to the repair or replacement, at the manufacturer's discretion, of parts that fail due to defective material or workmanship. The manufacturer may, at its discretion, provide factory reconditioned parts. This warranty is provided to the Authorised Handicare Dealer on behalf of the final purchaser of the product and is not transferable. The manufacturer's warranty does not cover labour charges for the removal, repair or replacement of warranty parts.

Excluded from guarantee: damage caused by normal wear and tear, overloading or misuse, the replacement of batteries in the remote control. Under no circumstances can the manufacturer accept responsibility for death, personal injury, damage to property, or incidental, unforeseen consequences and/or costs or damage resulting from the use of your stairlift. The carrying out of alterations to the product is not permitted. The manufacturer is not liable for damage resulting from such alterations. The manufacturer accepts no liability for damage or injury caused by the non-observance, or inadequate observance, of the safety regulations and safety instructions in this manual, or by negligence during the installation, use, maintenance and repair of this stairlift. Installation and servicing may

only be carried out under the responsibility of dealers or installers/service engineers authorised by the manufacturer, in the course of which locally prevailing regulations must be observed.

The wallbuttons of this stairlift work radiographic. Handicare does not accept any responsibility for the non-functioning of these wallbuttons due to interference by a source outside this stairlift.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

SAFETY INSTRUCTIONS

When using the stairlift always observe locally prevailing safety regulations regarding risk of electric shock and physical injury.

- 1.** Provide an unobstructed environment. The entire track must be free of obstacles.
- 2.** Take environmental influences into account. Do not expose the lift to damp or extreme sunlight. Make sure that the environment is well lit. The entire track must be visible from the stopping places.
- 3.** Keep uninstructed persons and domestic pets away from the stairlift.
Do not use the stairlift to carry animals or goods. Never use the stairlift with more than one person sitting on it. The stairlift may only be used by adults who have signed the instructed person's declaration. Keep children away from the stairlift. Remove the key after use.
- 4.** Keep limbs and clothing away from moving parts. Make sure that wide or loose clothing cannot become trapped between the chair and, say, the stairs or rail.
- 5.** Ensure a safe position. Always ensure a correct, stable position. Sit right up against the back support, use the armrests and place the feet squarely on the foot rest. Always use the safety belt. Never stand on the stairlift.
- 6.** Take care at all times. Concentrate on the operation of the stairlift.
- 7.** Check for damage. Before use check the lift for external damage. Report unusual noises, such as squeaking and creaking, to your supplier. The lift must comply with all prevailing safety regulations. Do not use the lift if a part is defective.

To prevent accidents, repairs and preventive maintenance must only be carried out by engineers qualified to do so.

8. The stairlift must not be used in the event of a fire or as a means of escape during a fire.
9. If the stairlift is not fitted with a communication device, we recommend that you always have one to hand.
10. Only leave the stairlift at the designated points at the top and bottom of the rail.

(only for the installer)

Open stair rails (e.g. banisters) should be closed off so that the lift is not accessible from behind.

The electrical connection must comply with section 4.3 of EN60204-1

11. Check for damage

Before the installation inspect the parts carefully for damage in order to make sure that the lift will function properly. The lift must comply with all prevailing safety regulations. Do not use defective parts; only use original parts.

12. Safety devices

All safety devices must be fitted correctly and may only be removed for maintenance and repair work by qualified service engineers trained to do so.

The product must never be used if the safety devices are incomplete or absent, or have been put out of action or are defective.

The safety devices must be checked regularly to see that they are working correctly and, if necessary, must be repaired immediately.

13. Use and installation in accordance with intended use.

Where necessary use safety equipment such as face and hearing protection, safety shoes and hard hat during assembly and servicing. Only use approved tools.

Before you start the assembly, please check whether the stairs and the environment meet the safety requirements.

14. Please note that the Freecurve stairlift has an electronically assisted (or controlled) emergency manual travel device. This can only be used by a qualified technician. In the event of equipment failure, the user must have the necessary skills to get out of the Chair lift on their own or to call for the help of a caregiver or a member of the emergency personnel.

MAINTENANCE

Cleaning

Clean the stairlift lift regularly.

Avoid excessive use of water. Clean the seat, back support, armrests and footrest only with a slightly damp cloth. Clean the track regularly with a dry cloth. Do not clean the metal rack. If dirty spots appear on the rail, these can easily be removed with a damp cloth using a mild detergent. In time the track wheels can leave a mark on the rail.

During the first 50 rides of the stairlift scales of paint might fall on the stairs due to running in of the stairlift.

Preventive maintenance

In addition to cleaning your stairlift, periodic preventive maintenance is necessary for long-term correct functioning.

The stairlift must be serviced at least once a year. This service must be carried out by a qualified person. Contact your supplier to conclude a service agreement. If your stair lift runs over more than 2 floors and/or has more than 2 users, it needs at least 2 maintenances a year.

Technical specifications

Supply voltage	230 VAC
Frequency	50/60 Hz
Current	1 A
Power	360 watt
Fuses	30 A
Load (max.)	125 kg / 275 lbs
Speed	0.15 m/s (max.) / 0.49 f/s (max.)
Ambient temperature	0 - 40 °C / 32 - 104 °F
Relative humidity	20 - 80 %
Users pattern	15 rides per 24 hours 10 rides continuous
Noise	56 dB
Remote control	915 Mhz / 2,4 Ghz

An electrical output socket shall be available to the stairlift for local lighting during inspection and servicing. Lighting to a minimum value of 50 lux shall be provided at the boarding points whilst the stairlift is in use.

The configuration of the Handicare stairlift, including in particular (but not limited to) its parts such as the rail system, the construction of the stairlift, the fixation parts, all swivelling and movable parts, and the motor, rely on Handicare's know-how. Work on this configuration or on one or more of its parts may only be performed by dealers (and their employees) who have been certified by Handicare to perform such work in accordance with the instructions given by Handicare. Only these employees have received the required training; only these qualified employees can provide the level of safety that Handicare stairlifts have when delivered ex works. Suppliers of used Handicare stairlifts other than dealers who have been certified by Handicare are not qualified to perform such work, e.g. installing a rail for a user other than the original Handicare rail.

This is important, because when work is performed on a Handicare stairlift or a used Handicare stairlift is re-installed with other parts, such as a different rail, by anyone other than the aforementioned qualified employees, Handicare's warranty and the CE mark for the stairlift in question will become invalid with immediate effect.

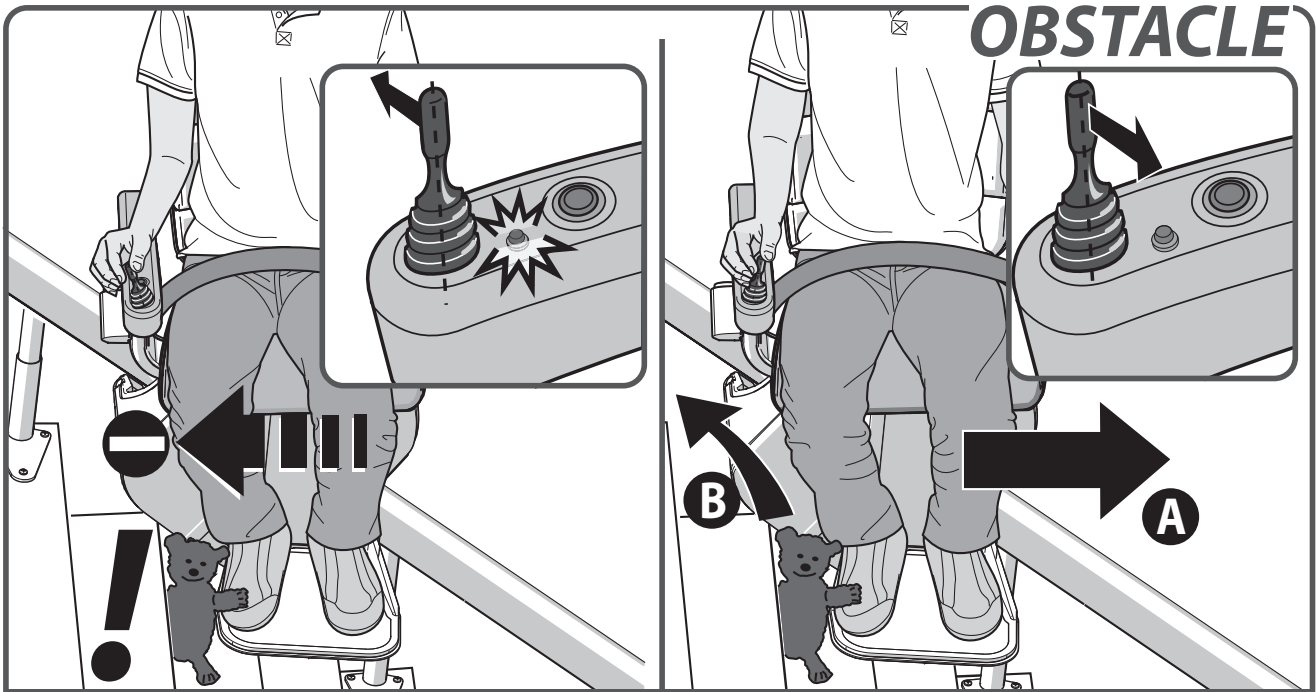
Once the CE mark has become invalid, this means that the stairlift in question no longer complies with European regulations. As a result, Handicare will no longer be responsible or liable for any consequences arising from the failure of the stairlift in question. Furthermore, maintenance on the Handicare stairlift may only be performed using original spare parts that have been approved by Handicare.

This is important, because if parts other than original spare parts approved by Handicare, such as a different rail, are used, the Handicare warranty and the CE mark for the stairlift in question will become invalid with immediate effect.

All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual can also be changed without prior warning. This manual is intended for the standard model of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications which differ from those of the standard model of the lift supplied to you. For information concerning adjustment, maintenance or repairs not provided in this manual, you are asked to contact the technical department of your supplier.

This manual has been compiled with every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof.

The Dutch manual is the original language


CAUSE

SOLUTION

Problem: the lift travels in one direction only and the LED flashes when the joystick is operated.

Cause: there is an obstacle between the lift and the stairs.

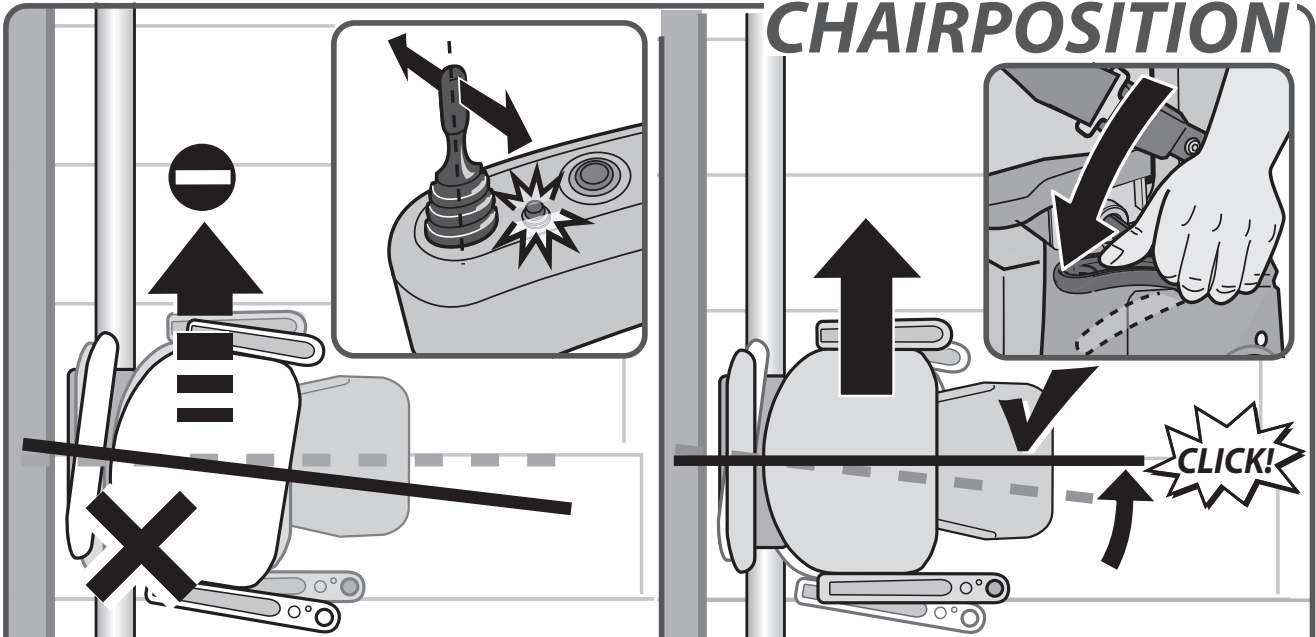Solution: move the lift in the opposite direction {A} and remove the obstacle {B}.

CAUSE

SOLUTION

Problem: the lift does not travel in either direction and the LED flashes when the joystick is operated.

Cause: the seat is not locked and is in a swivelled position.

Solution: swivel the seat using the handle located to the left or right under the seat.

STOP BUTTON

CAUSE

SOLUTION

Problem: the lift does not travel in either direction, the LED[1] flashes when the joystick is operated and the stop button [2] is illuminated.

Cause: the stop button has been pressed.

Solution: press the stop button once.

OPTIONAL STOP BUTTON

CAUSE

SOLUTION

Problem: the lift does not travel in either direction, the LED[1] flashes when the joystick is operated and the emergency stop [2] is pressed.

Cause: the stop button has been pressed.

Solution: the emergency stop can be released by turning the emergency button.

ARMREST

CAUSE **SOLUTION**

Problem: the lift does not travel in either direction and the LED on the armrest lights up when the joystick is operated. **Cause:** an armrest has not been folded down. **Solution:** fold down the armrest fully.

KEY

CAUSE **SOLUTION**

Problem: the lift does not travel in either direction and the LED on the armrest does not light up when the joystick is operated.
Cause: the key-operated switch is in the off position.
Solution: turn the key-operated switch ninety degrees clockwise.

BATTERIES RC

CAUSE

SOLUTION

Problem: the lift does not travel in either direction and the LED on the remote control does not light up when the remote control is operated.

Cause: the batteries in the remote control are flat.

Solution: replace the batteries in the remote control.

POWER

CAUSE

SOLUTION

Problem: the lift batteries are not recharging, the LED is flashing and the lift beeps for three minutes.

Cause: the lift is not in the correct position at a charge point or the plug is not in the socket.

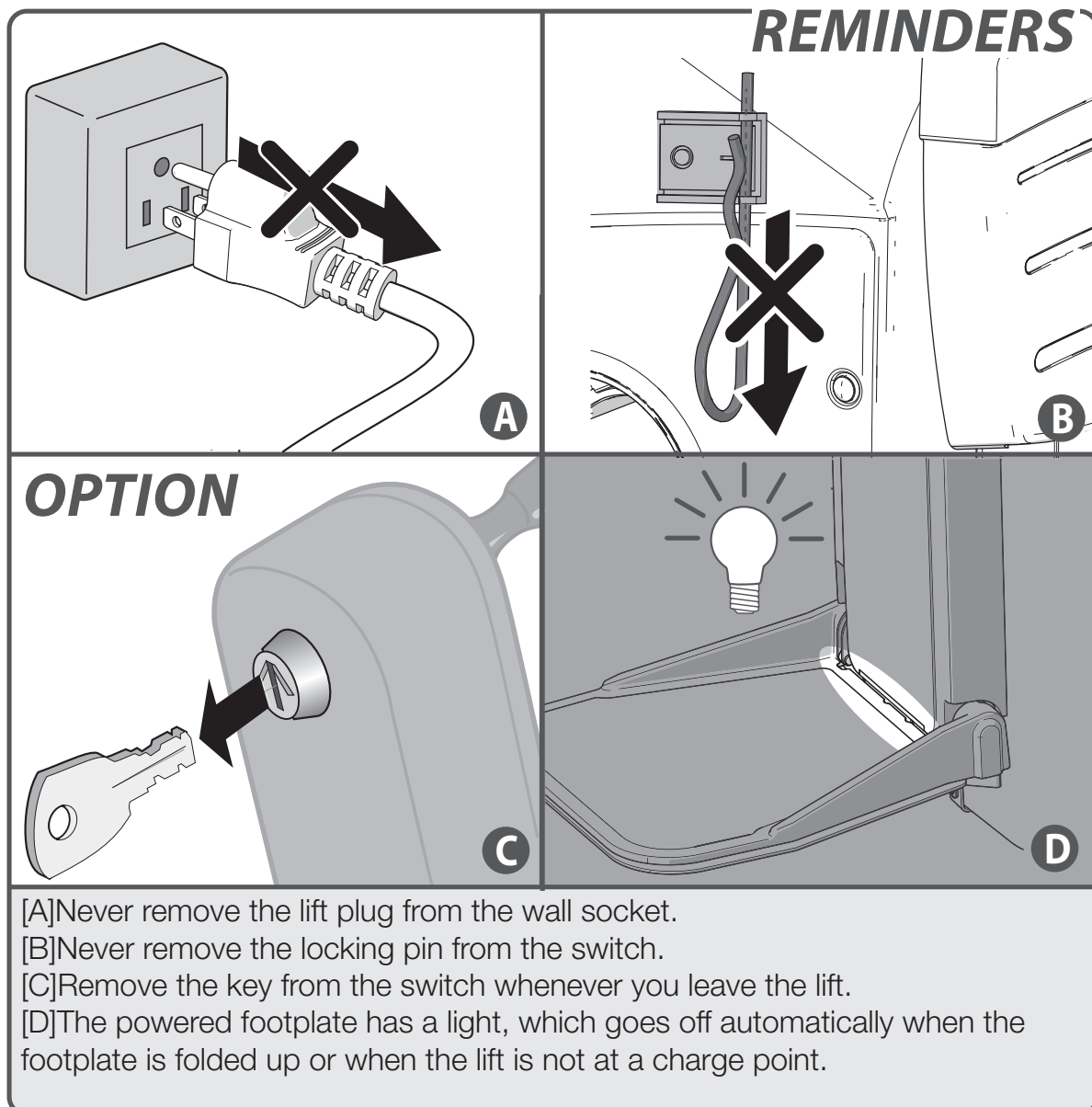
Solution: move the lift to a charge point or put the plug in the socket.

ATTACHMENTS

REMINDERS *page 24*


REMOTE CONTROL *page 25*

TAP 2 FLOOR *page 26*


Rembrandt / Van Gogh

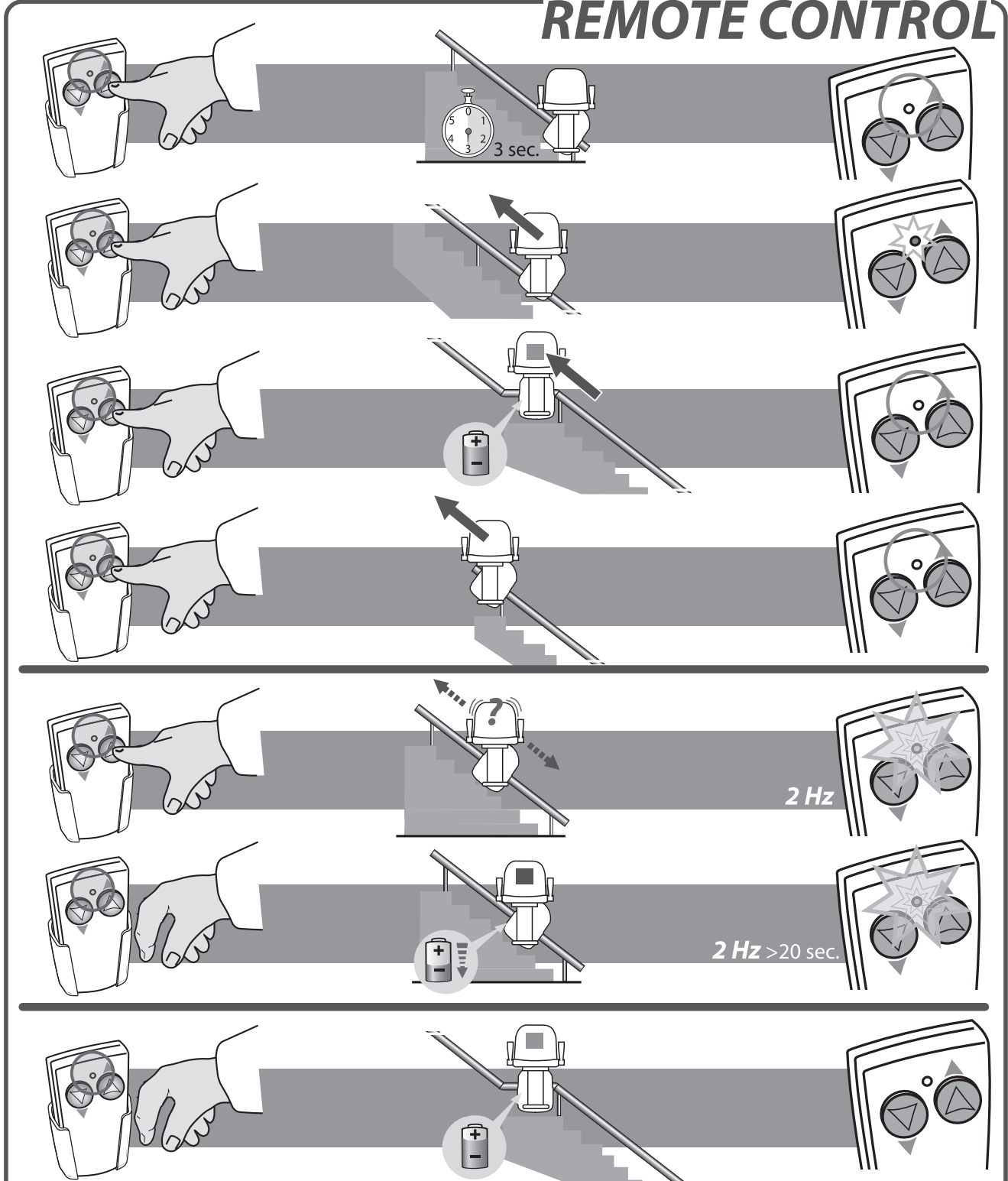REMOTE CONTROL


Operation	Situation	LED
when using remote control	stairlift in start-up phase (3 sec)	LED remains off
when using remote control	stairlift travelling	LED turns on
when using remote control	stairlift stopped at an active charging station	LED turns off
when using remote control	stairlift is stopped at the end stop and will not depart until it is called in the correct direction	LED remains off
when using remote control	stairlift is out-of-order and can no longer depart	LED flashes twice a second
when not using remote control	stairlift is stationary en route, i.e. not at a charging station	LED flashes twice a second for 20 seconds after operation has stopped. LED then turns off
when not using remote control	stairlift stopped at an active charging station	LED off

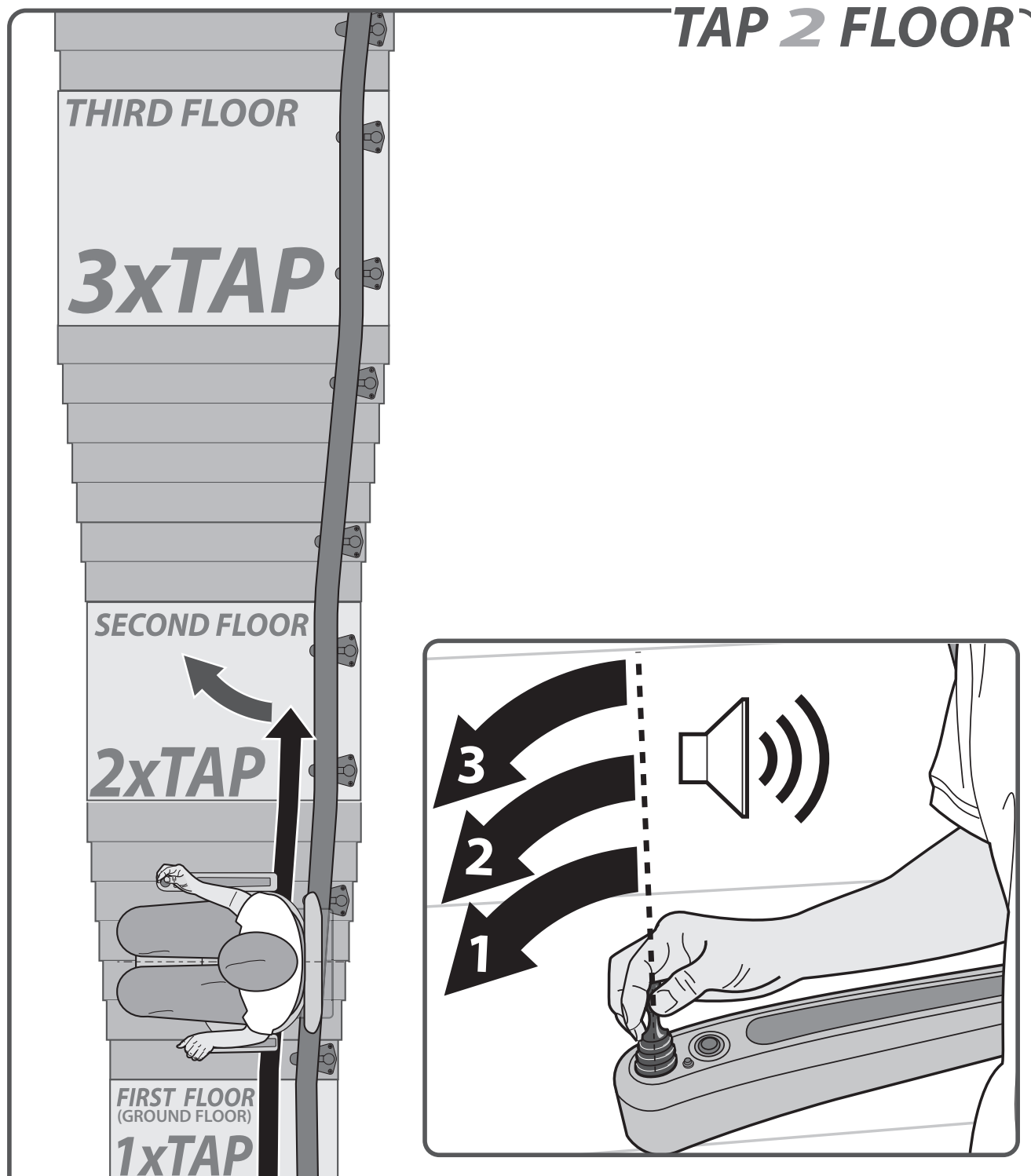TAP 2 FLOOR


Stairlift with powered swivel can swivel automatic on a preferent intermediate landing after tapping.

Tapping

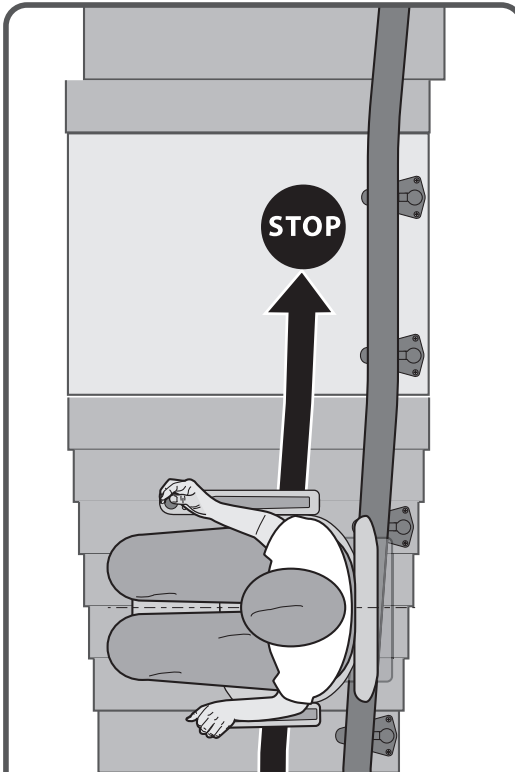Before the start of the lift, the joystick (not the remote control) is operated in the direction of travel (Tap). The number of operations (taps) indicates the desired floor. Tapping must be done rather quickly. After each tap you hear a beeping sound. LAST press (when holding the joystick to start driving) is NOT counted as a tap !!

**Riding**

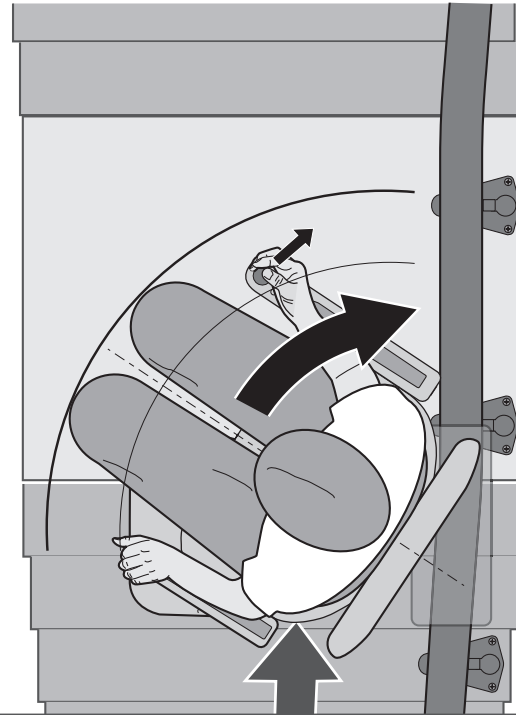After the lift has started to drive the lift will automatically stop at the desired floor.

TAP 2 FLOOR**Swivel**

After arrival you keep the joystick pushed and the chair will swivel automatically.

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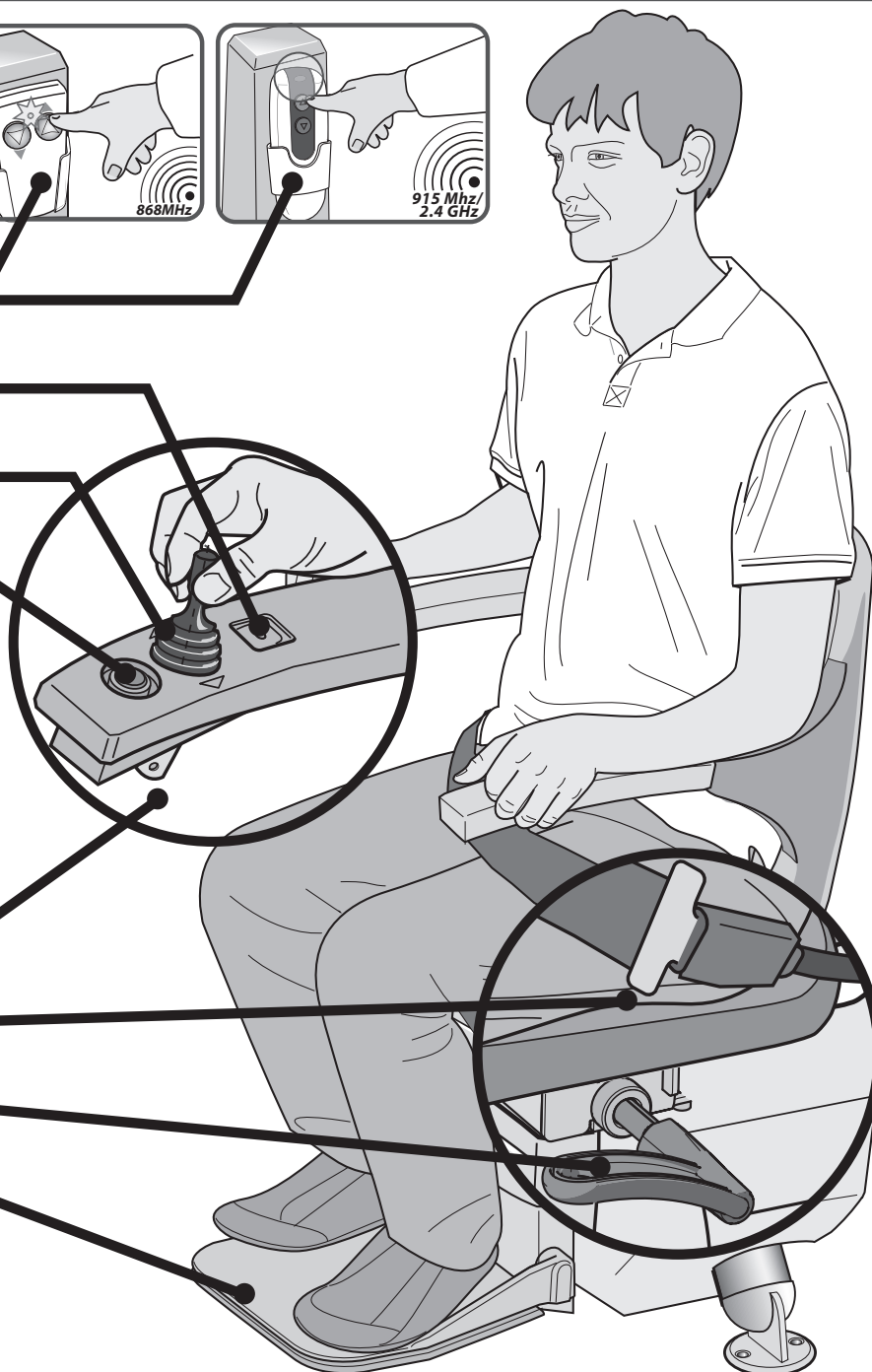
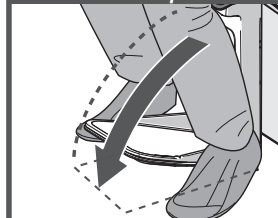
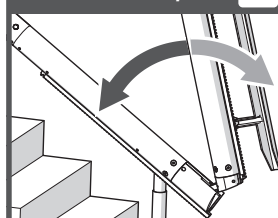
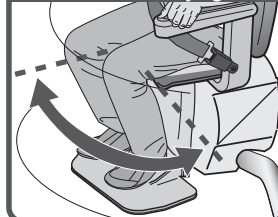
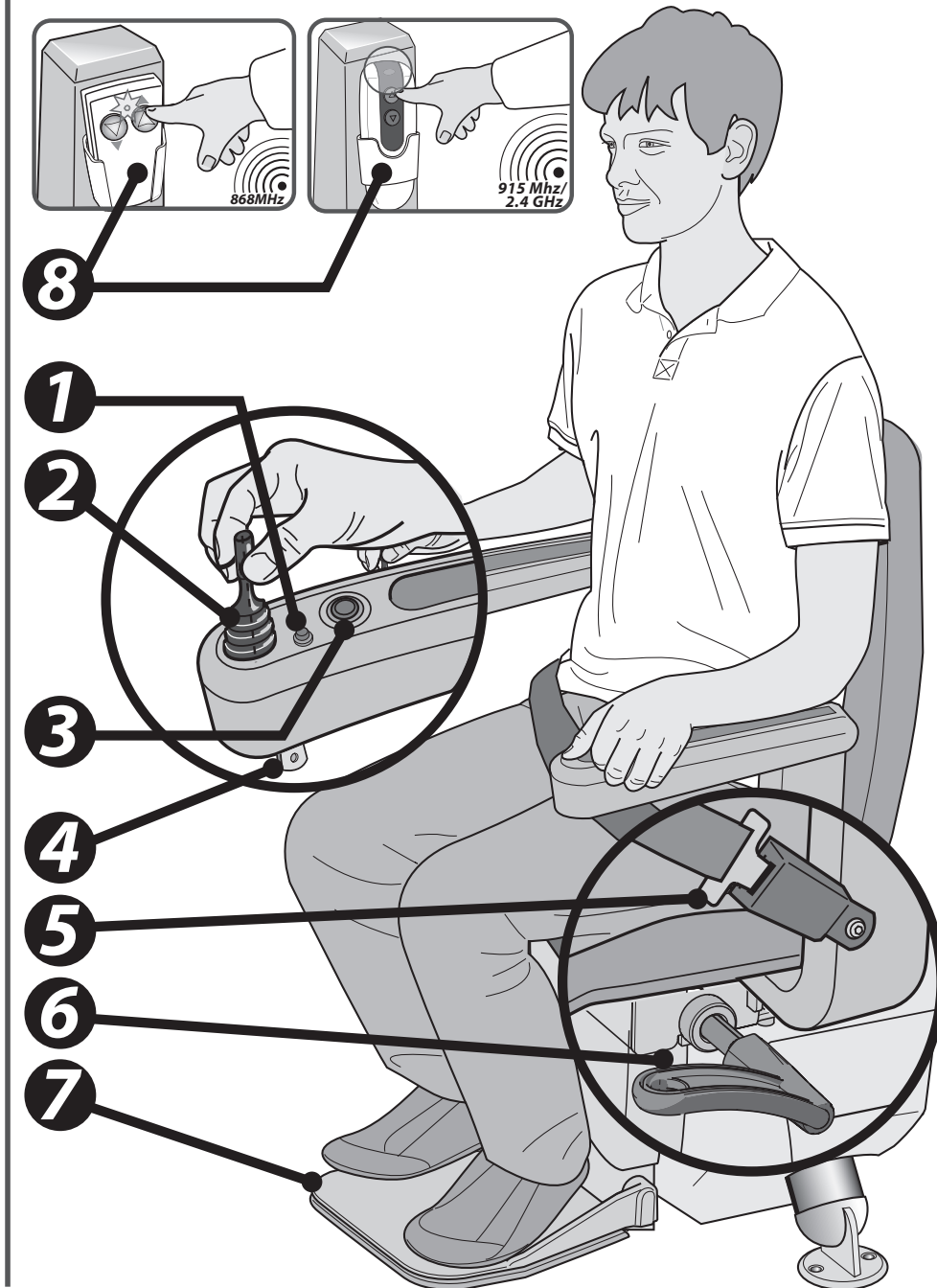
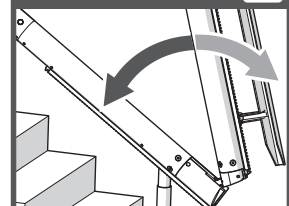
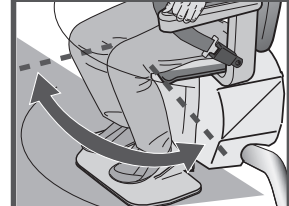
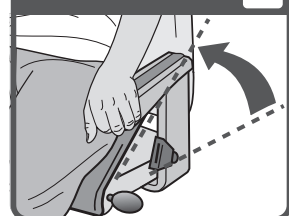
[1] LED
[2] Joystick
[3] Stopknop
[4] Sleutel-schakelaar
[5] Veiligheids-gordel

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Handicare GmbH
Handicare USA, Inc.