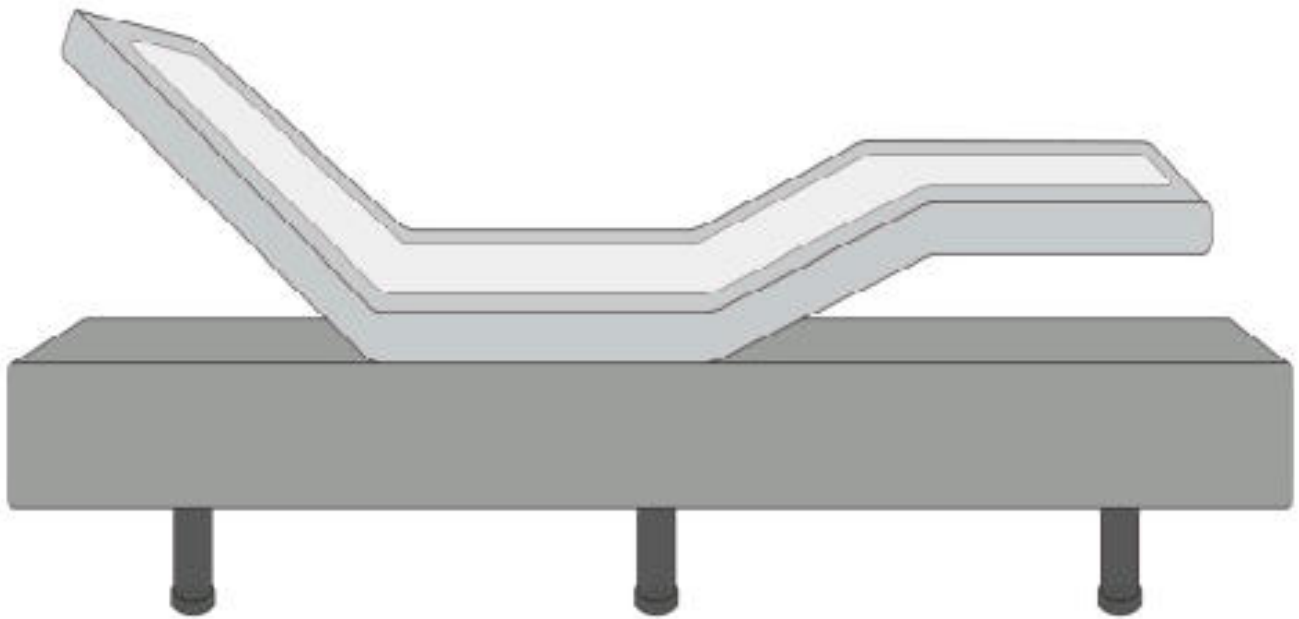
# Harmony™

## **Folding Adjustable Base**

# **Assembly and Operation Manual**

# Specifications

Weight Capacity	880 lb	Wall Hugger	X
Incline Degree	Backrest: 0°~65° Legrest: 0°~35°	Power Transformer Output	DC 29V 1.8A
Power Input	AC 100-240 50/60 HZ	Massage Vibrator	X
USB Charger	X	Bluetooth Control	X
Floor Light	X	Flash Light on Remote	X
Preset Position: ZG and QUIET SLEEP	X	Memory Position	X

## Table of Contents

<b>Safety Precaution and Usage Guide.....</b>	<b>2</b>
<b>To decrease the risk of electronic shock.....</b>	<b>2</b>
<b>Adjustable base performance.....</b>	<b>2</b>
<b>Restrictions of your adjustable base.....</b>	<b>3</b>
<b>Electronic system information.....</b>	<b>3</b>
<b>Parts List.....</b>	<b>4</b>
<b>Electronics Overview.....</b>	<b>5</b>
<b>Assembly Guide.....</b>	<b>6</b>
<b>Operating Your Adjustable Base.....</b>	<b>9</b>
<b>Wireless pairing operation.....</b>	<b>10</b>
<b>Synchronization of Two Bases (Option).....</b>	<b>11</b>
<b>Power outage or failure of remote control.....</b>	<b>12</b>
<b>Troubleshooting.....</b>	<b>13</b>

## 20 YEAR LIMITED WARRANTY

# Safety Precaution and Usage Guide

## Attention: SAVE THESE PRECAUTIONS AND USAGE GUIDES

Your new adjustable base has been strictly tested and inspected prior to dispatch. It is highly recommended to read these instructions thoroughly before you try your new adjustable base, in order to keep your high-quality adjustable base performing well in its lifetime but also to reduce the risk of fire, electric shock, burns or any injury to users.

### To decrease the risk of electronic shock, burns, or injury to users, children or pets, please follow these instructions.

1. After unpacking your adjustable, be aware of your **small children and pets**. Its packing material may put your **small children or pets** in **smothering dangers**.
2. **Unplug** the power cord from the power outlet **before installing** parts or **cleaning** your adjustable base.
3. **Keep** the power cord **away** from **water** and **heated areas**. If your adjustable base is **wet** or has a **frayed power cord**, please do not operate it.
4. **Do not** operate your adjustable base near **explosive gases** or **aerosol spray products** are being used.
5. Before you **disconnect** the power cable, please return your adjustable base to the **flat position** and **stop all operations**.
6. **Unplug** your electronic gadgets from the USB port (option) of your adjustable base if they are **fully charged**.
7. **Do not** use your adjustable base **in outdoors**. Your adjustable base is only designed for **in-home use**.
8. Use your adjustable base in compliance with the **restrictions**.

### To keep your high-quality adjustable base performing well in its lifetime, please follow these instructions.

1. **Do not** unscrew or open the **control box, lift motors, massage vibrators** (option), **SMPS, wired handset or remote control** (option), and **other electronic components** on your adjustable base, except for opening the battery compartment on your remote control or SMPS. The warranty will be void if there is any component damaged by users. Only **authorized service personnel** are allowed to repair or replace parts.
2. **Do not** mix **different** types of **batteries** or old batteries with a fresh one for your handset or SMPS.
3. **Remove** the **batteries** from your **remote control** (option) and **SMPS** if your adjustable base will not be used for a long time period to prevent damage caused by **battery leakage**.
4. **Do not** sit on the **head** or **leg section** while in the raised position. It can damage your adjustable base and lead to shorter life expectancy.
5. **Do not** lift or move the base **during operation**. It might cause damage to your system.
6. **Do not** lift or move your adjustable base by **grabbing** its **mattress platform** or its **wood lower frame (option) furnished with fabric**. If it is necessary to lift or move your adjustable base, please do it by **grabbing** the **cross bars** of its metal frame.
7. Your adjustable base must be on **flat ground** to avoid damage to your product.
8. Use a **compatible mattress** for your adjustable base. A **memory foam mattress** or **latex mattress** is recommended. Some spring mattresses or pocket spring mattresses, for example, may not suit your adjustable base. Consult your mattress specialist to verify your mattress is compatible.

## Restrictions of your adjustable base

1. Do not operate lift motors above the **10% duty cycle**. For example, after continuous 2-minute operation 18-minute rest is required.
2. Do not continuously or intensively operate the massage vibrators (option) above maximum 2 hours within 6 hours.
3. **Weight Restrictions:** the maximum weight limits on your adjustable base for Twin, Twin XL, Full and Queen are at a maximum of **400kg/880lbs** (evenly allocated across the mattress platform). This capacity includes the mattress and occupants. The warranty of your adjustable base will be void if excessive weight of 400kg/880lbs is loaded on your adjustable base.

## Electronic system information

### Radio Frequency Compliance:

Radio Frequency: 2.4 MHZ.

FCC ID: **O3YRF075A**

Your Device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

### Attention:

In compliance with FCC RF exposure requirements, there must be no change to the antenna or the device without permission. Users may be deprived of his/her right to the device if the device exceeds the RF exposure requirements caused by any change the antenna.

### Power Ratings:

Input: AC100-240V 50/60Hz 1.5A

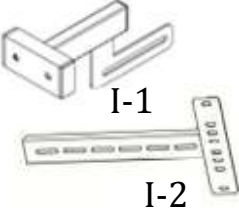
Output: DC29V 1.8A

Protection Class: Class II



# Parts List

**IMPORTANT NOTE:**

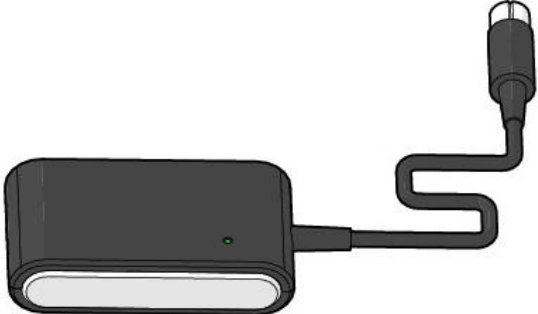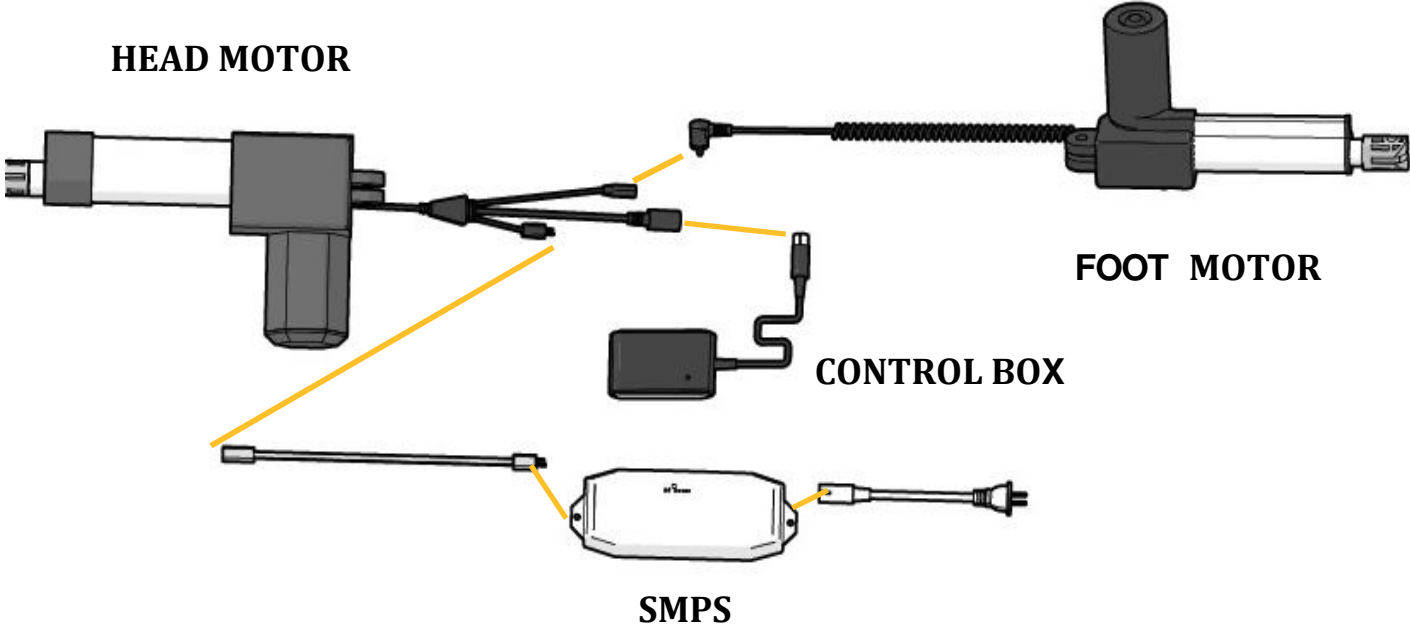Check and confirm the following items are included in the carton, BEFORE disposing of any packing materials.

	<p>A. Remote Control * 1</p>		<p>B. SMPS * 1 (Switched-Mode Power Supply)</p>
	<p>C. AC Main Cable * 1</p>		<p>D. AAA Battery * 2</p>
	<p>E. Support Leg * 6 sets</p>		<p>F. Mattress Retainer * 1</p>
	<p>G. Clevis Pin * 2</p>		<p>H. Cotter Pin * 2</p>
	<p>I. (Option) Headboard Brackets * 2</p>		<p>J (Option) Wrench * 1</p>
	<p>K. (Option) Hex Key * 1</p>		<p>L. (Option) Bolt for Headboard * 8</p>
	<p>M. (Option) Washer for Headboard * 16</p>		<p>N. (Option) Nut for Headboard * 8</p>
	<p>O. Side Retainer * 2</p>		<p>(Option) Sync Box* 1 (TXL only)</p>

# Electronics Overview

# Assembly Guide

## IMPORTANT NOTE:

Two people are required to set up your adjustable base.

### STEP 1

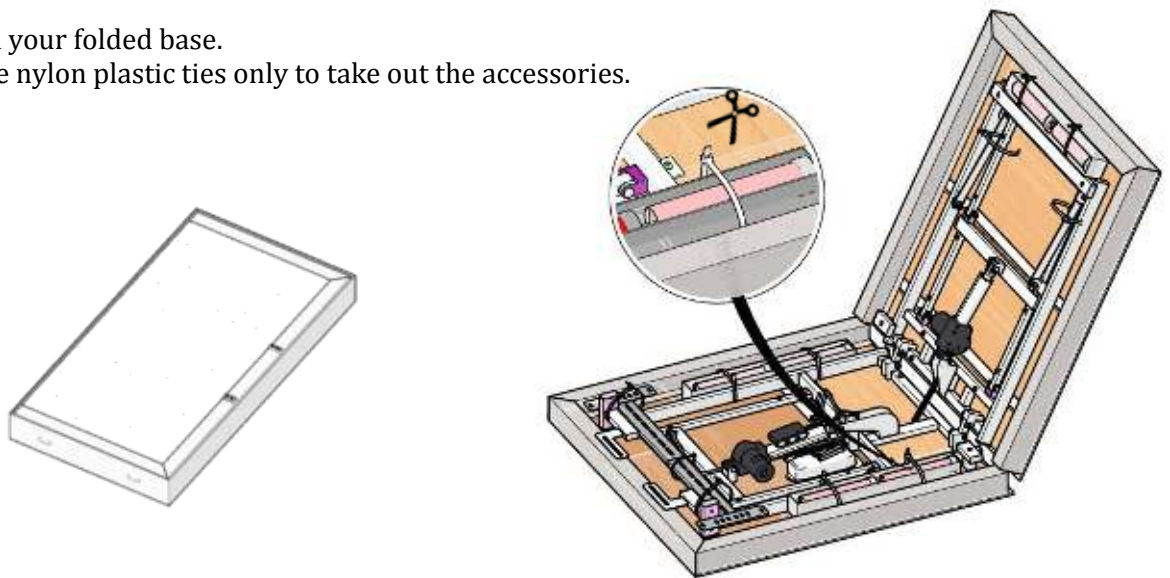
Leave the carton in the flat position. After opening this carton, carefully take your Adjustable Base to the desired location and leave it topside down.

### STEP 2


Firstly open your folded base.

Cut off white nylon plastic ties only to take out the accessories.

### STEP 3

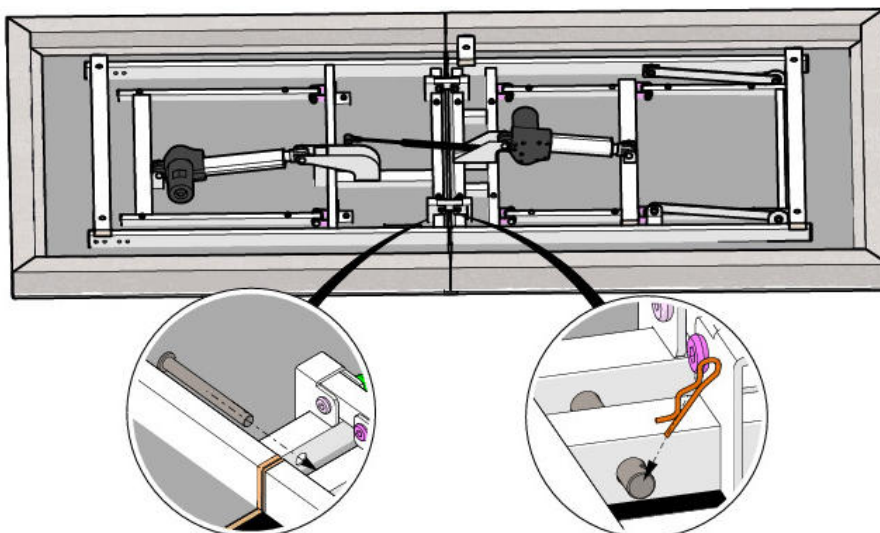Insert Clevis Pin (G) through the Cross bars and insert Cotter Pin (H) through the hole of Clevis Pin, as shown in the picture. Make sure it is securely locked. Repeat it on the other side.


## STEP 4

Take out Support Legs(E) \*6 and securely tighten onto your Adjustable Base and leave no gap between them.

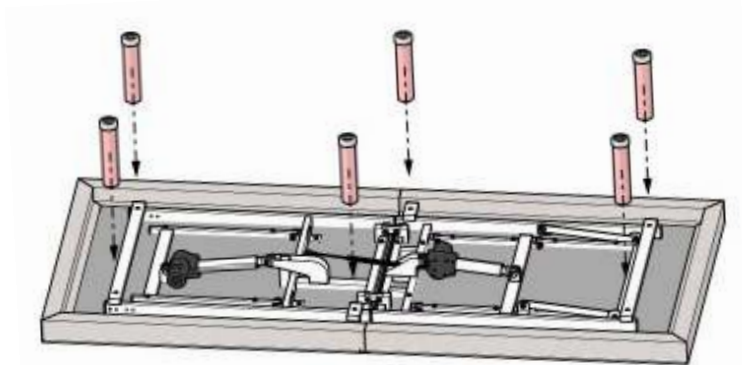\*\*If you have two or three-sectional legs, you may allow to choose the suitable height for your base.


## STEP 5

Insert two AAA Batteries (D) into the compartment of the Remote Control (A).

## STEP 6

Find Extension Cable from the Head Section of your Adjustable Base and plug it into SMPS (B).

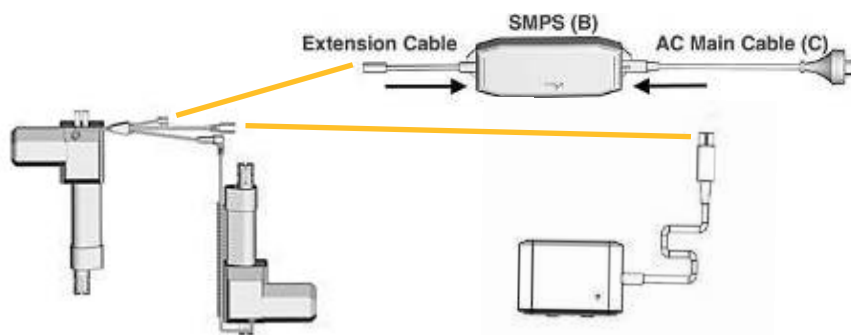Take out AC Main Cable (C) from accessories box and plug it into SMPS (B).

## STEP 7

Carefully flip over your topside-down Adjustable Base. Lift up the Base to desired location.

### IMPORTANT NOTE:

Be Careful when flipping your Base:

The installed legs could get damaged or bent by excessive pressure, if you lean your Adjustable Base on the legs.

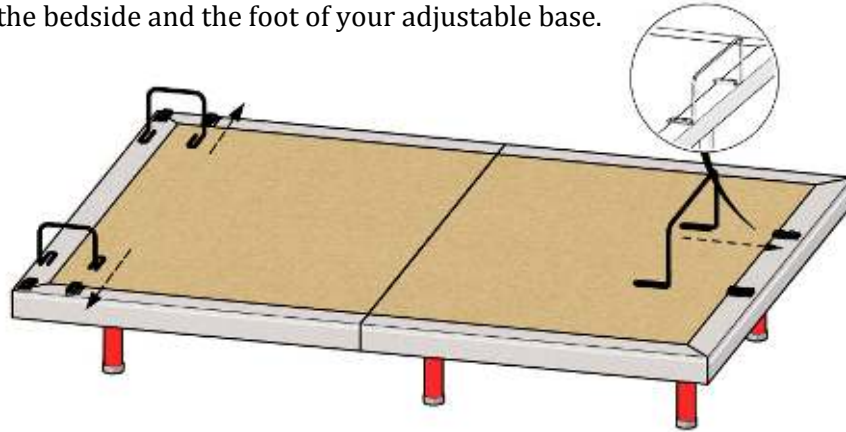
1. Do not recline the Base against the installed legs to flip it over.
2. Do not drag the Base across the floor.

## STEP 8

Plug the AC Main Cable (C) in an electrical outlet with 100-240V, 50/60 Hz. A surge protector (not included) is recommended for longer life of your Adjustable Base.

## STEP 9

Take out Retainer\*1 (F) and Side Retainers\*2 (O). Insert the retainers into the corresponding holes on the Retainer Bar Fixation at the bedside and the foot of your adjustable base.

## STEP 10 (Installing Headboard-Option)

Raise the Backrest by using Remote Control (A).

Take out the Hex Key (K), Wrench (J), 8x Bolts (L), 8x Nuts (N) and 16x Washers (M) from the accessories box.

Refer to the following picture, place the Headboard Bracket (I-1) on the tube of the right hand side of your Adjustable Base.

Insert the Bolt (L) with the Washer (M) through the pre-drilled holes on the Bracket (I-1) and the vertical tube on your Adjustable Base.

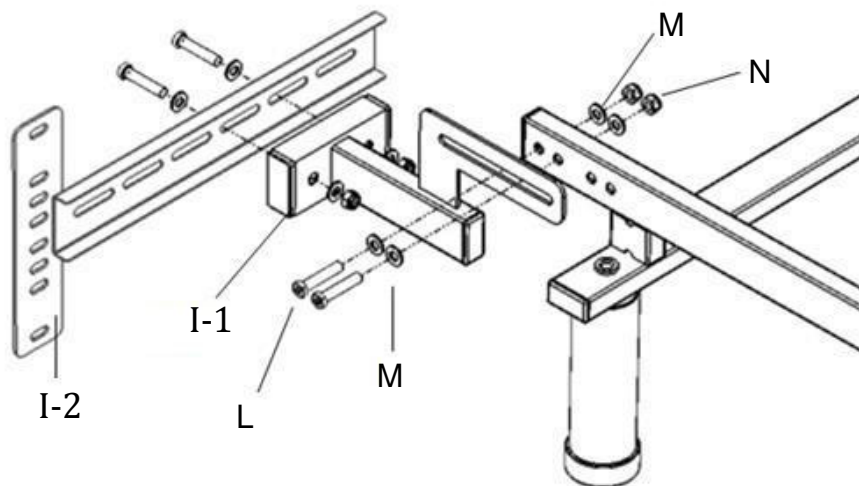
Fasten the Bolt (L) with the Washer (M) and the Nut (N) by using the Wrench (J) and Hex Key (K).

The Bracket (I-2) allows adjustment to meet the dimension of user's Headboard.

After choosing desired pre-drilled holes on the Bracket (I-2), pass the Bolt (L) with the Washer (M) through the holes on H-1 and H-2 Brackets.

Fasten these Bolts (L), Washers (M) and Nuts (N) by using the Wrench (J) and Hex Key (K).

Redo these steps on the other side of your Adjustable Base.

## Step 11

Now place your mattress on your Adjustable Base. All the installation procedures have been completed.

# Operating Your Adjustable Base

\* Press and hold the button to RAISE BACK REST. Stops when released.

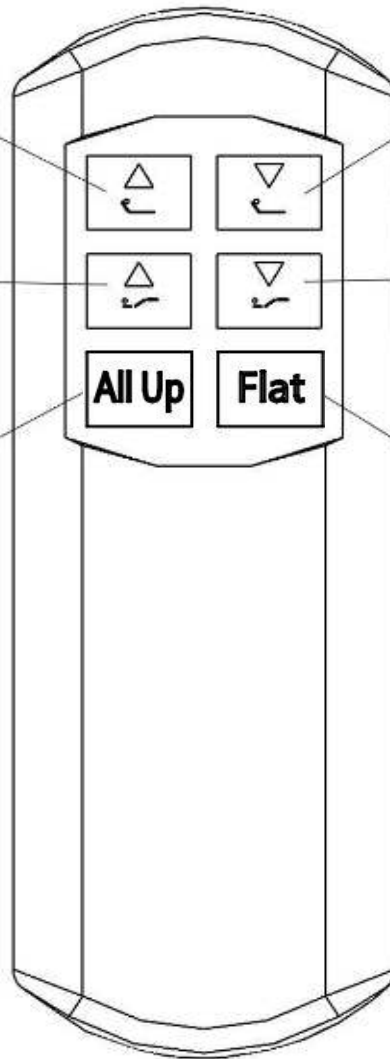
\* Press and hold the button to LOWER BACK REST. Stops when released.

\* Press and hold the button to RAISE LEG REST. Stops when released.

\* Press and hold the button to LOWER LEG REST. Stops when released.

\* Press and hold the button to RAISE BACK REST and LEG REST SIMULTANEOUSLY. Stops when released.

\* Press and hold the button to LOWER BACK REST and LEG REST SIMULTANEOUSLY. Stops when released.


## Wireless pairing operation

### Step1:

Make sure the RF receiver is properly connected to the Motor.

### Step2:

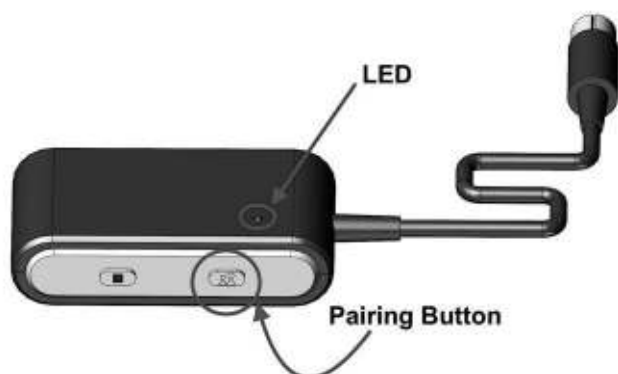Press the pairing button on the RF receiver until the green LED turns on. Now you have 10 seconds for pairing process.

### Step3:

Press and hold simultaneously Heads-Up and Heads-Down buttons on the Remote (A) until the LED on the RF receiver starts blinking. The blinking confirms successful pairing.

Please repeat all the above steps if the pairing operation failed.

#### Important Notes:

For pairing the distance between the Remote (A) and the RF receiver should be less than 2 meters.

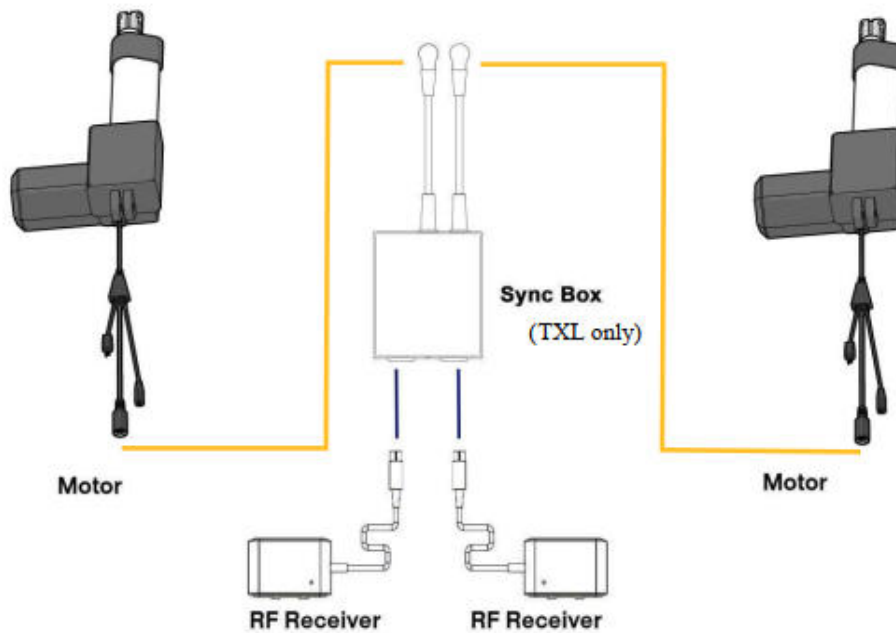
## Synchronization (OPTION)

Synchronization may be required for setting up two TXL Adjustable Bases.

Insert one cable of the Sync Box into the female end of the Head Motor and insert the RF Receiver into the Sync Box as shown in the picture. Repeat the same action to your second base.

Now, you can operate both of your Adjustable Bases simultaneously with either one of the Remotes.

**NOTE :** Your bases will freeze in place if both Remotes are used at the same time until one of the Remotes are released.


## Power outage or failure of remote control

In the case of failure of the Remote Control, press and hold the Emergency Button on the RF receiver to flatten your Adjustable Base. When releasing the Emergency Button, all the movements will stop immediately.

In the case of the power outage, insert two 9V Batteries (not provided) to the Battery Compartment on the backside of SMPS. It can be used as the backup power.

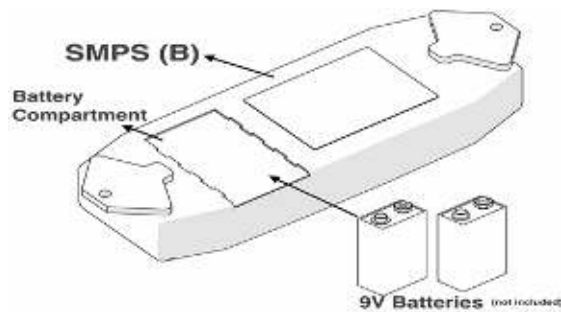NOTE : It's only for flattening the Base without load. Do not use the Massage Vibrator when using the backup power.

## Troubleshooting

TROUBLE	SOLUTION
There is no feature of the base that can be activated.	<p>Remote control illuminates</p> <ul style="list-style-type: none"> <li>• Reset the electronic components by unplugging your base for 30 seconds. Re-plug it into the power outlet.</li> <li>• Ensure the surge protection or the power outlet is working. Test it by plugging in another working electronic appliance.</li> <li>• The unworked power outlet may be led by the tripped electrical circuit breaker. Check the box of electrical service breaker.</li> <li>• Re-pair the remote control (see Wireless pairing operation).</li> </ul> <p>Remote control does not illuminate</p> <ul style="list-style-type: none"> <li>• Replace batteries in the remote control.</li> <li>• Install 3 AAA batteries correctly.</li> </ul>
The base cannot return to the flat position.	<ul style="list-style-type: none"> <li>• Ensure the base is not obstructed. Remove the obstruction.</li> <li>• Ensure there is a sufficient space between the wall or headboard and the edge of the mattress. The insufficient space may obstruct the movement of the base.</li> </ul>
Excessive noise of massage function.	<ul style="list-style-type: none"> <li>• Place carpet pieces or soft material beneath each leg of the base.</li> <li>• If headboard is equipped ensure the hardware of the headboard brackets are tightened firmly.</li> <li>• Ensure the base stands clear of walls, nightstands or other objects.</li> </ul>
Minor interference with lifts functions.	<ul style="list-style-type: none"> <li>• Replace batteries in remote control if necessary.</li> <li>• Ensure you're operating within 10% duty cycle (see Restrictions of your adjustable base).</li> <li>• Press the adjust buttons squarely and accurately.</li> <li>• Control box may be subject to the common radio frequency from other devices. Reprogram the remote control and the control box on a different radio frequency if necessary (see Wireless pairing operation).</li> </ul>
USB ports do not charge electronic gadgets.	<ul style="list-style-type: none"> <li>• Try another well-working USB cable.</li> <li>• Unplug all USB cables for 10 minutes to reset and try it again.</li> </ul>

# 20 YEAR LIMITED WARRANTY

Your New Adjustable Base has a 20 Year Limited Warranty, subject to the terms and conditions set fourth herein.

The Purchaser must show original proof of purchase for Warranty to be valid.

The decision for repair or replacement of defective parts under this limited warranty is the option and sole discretion of the manufacturer.

## **Year-1: Full Coverage of Parts and Labor**

This adjustable base is warranted by the manufacturer against defects and workmanship for a period of one year from the date of purchase.

During this time, the manufacturer will repair or replace, at no cost to the original purchaser, any warranted parts found to be defective by the manufacturer.

## **Year 2 – 3: Full Coverage of Parts Only**

During 2 through 3 years from the original purchase date, the manufacturer will offer replacement parts for any defective adjustable base parts determined by the manufacturer to be defective.

The manufacturer's liability is limited to the replacement of the defective electrical and mechanical parts only.

The purchaser shall bear all transportation costs for parts and installation labor related to the delivery and replacement of the defective parts.

## **Year 4 – 20: Full Coverage of Mechanical Parts Replacement**

During 4 through 20 years, the manufacturer will offer mechanical replacement parts for any mechanical base part to be found defective at 1/16 of the current replacement parts multiplied by the number of years from the original purchase date.

The manufacturer's liability is limited to the replacement of the defective mechanical parts only. This does not apply to the fabrics, electrical components, lift motors, massage motors or wood components.

The purchaser shall bear all transportation costs for parts and installation labor related to the delivery and replacement of the defective part.

## **Additional Terms and Conditions:**

This warranty is void and does not apply:

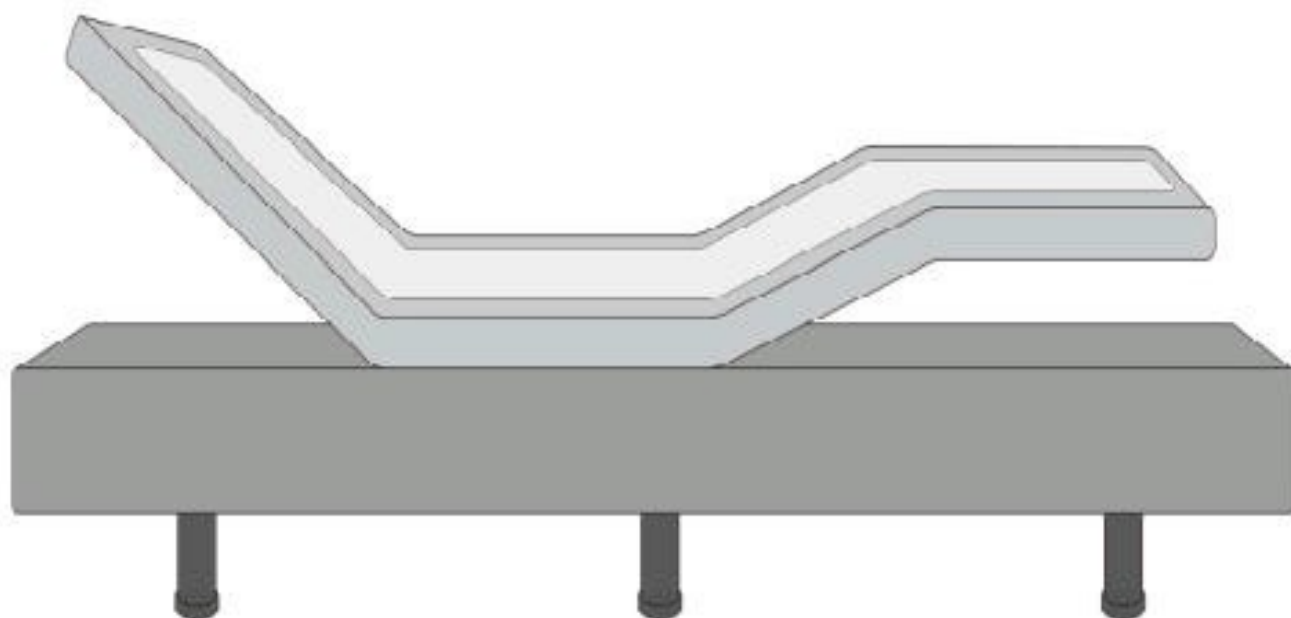
- To any damaged caused by user
- If damaged occurred during transit or mishandled
- Subjected to physical or electrical abuse or misuse
- Exceeded the Load Capacity recommended by the manufacturer in your Owner's Manual

Repair or replacement of adjustable base parts will be the sole remedy of the purchaser. There shall be no liability on the part of the manufacturer and their respective parents, subsidiaries, divisions or affiliates for any special, indirect, incidental or consequential damages or other damages, claim or loss not expressly covered by terms of this Limited Warranty.

For further assistance, please contact the dealer you purchased your Adjustable Base from.

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