


102662 Nitro Sprint

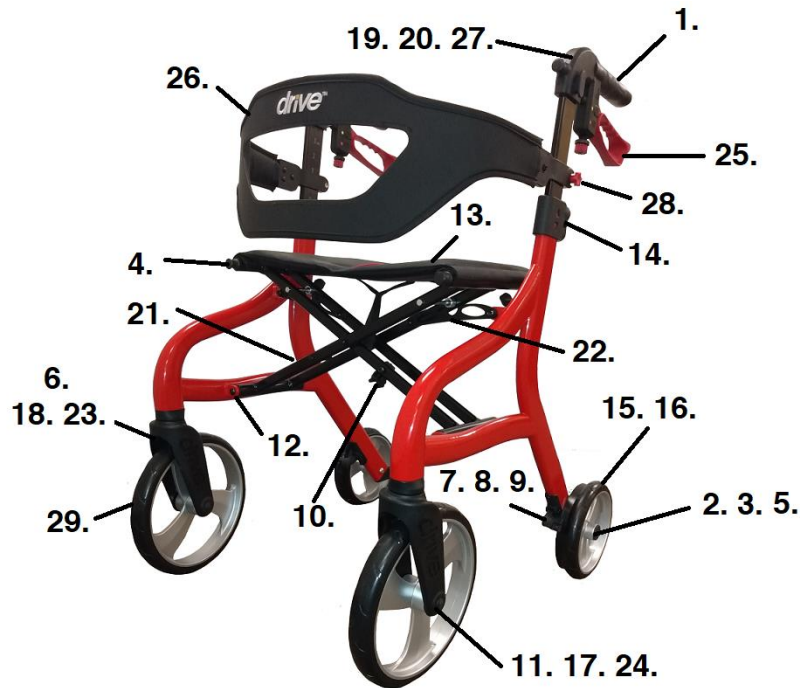

- | | |
|--|--|
| 1. Hand Grip- 102662-01 | 17. Front Wheel Bearings (pr)- 102662-17 |
| 2. Rear Wheel Bearings (pr)- 102662-02 | 18. Fork- 102662-18 |
| 3. Rear Wheel Hub Cap- 102662-03 | 19. Handlebar Assy Left- 102662-19 |
| 4. Cross Frame End Caps (pr)- 102662-04 | 20. Handlebar Assy Right- 102662-20 |
| 5. Rear Axle & Spacer- 102662-05 | 21. Cross Frame- 102662-22 |
| 6. Fork Stem Bearing- STDS1060 | 22. Cross Frame Link- 102662-23 |
| 7. Brake Cable Wire- 102662-07 | 23. Fork Stem Bolt & Spacer- 102662-24 |
| 8. Wheel Brake Assy. Left- 102662-08 | 24. Hub Cap (pr)- 102662-25 |
| 9. Wheel Brake Assy. Right- 102662-09 | 25. Hand Brake Handle- 102662-26 |
| 10. Folding Lock Mechanism- 102662-10 | 26. Backrest- 102662-28 |
| 11. Front Axle & Spacer- 102662-11 | 27. Brake Housing Assy- 102662-29 |
| 12. Cross Frame Mtg Screws (4 ea)- 102662-12 | 28. Backrest Adj. Knob (pr)- 102662-31 |
| 13. Seat Upholstery w/Handle- 102662-13 | 29. Front Wheel- 102662-32 |
| 14. Back Height Adj Latch - 102662-14 | N/S Tote Bag- 102662-27 |
| 15. Rear Wheel Left- 102662-15 | N/S Step Tube- 102662-06 |
| 16. Rear Wheel Right- 102662-16 | |