



INVACARE® CANADA  
**Patient Sling**  
Reference Guide



Yes, you can.®

- *Full Body and Divided Leg Slings*
- *Toileting, Standing and Transport Slings*
- *Premier Slings*
- *Sizing Guide*



# Invacare® Patient Sling Reference Guide


Application: Use with Invacare® 9805 and Reliant™ 450 and 600 series power or manual lifts.  
 Models: 9805, RHL450-1, RPA450-1, RPL450-2, RPL600-1, RPL600-2

## FULL BODY SLING

The Full Body Sling is an easy sling to use while transferring the resident from the bed to a wheelchair, toilet or shower-chair or from the floor to a bed. It offers total head and neck support and the padded lower section provides additional under-thigh comfort and support. Use the full body sling for residents who are totally dependent, partially-dependent, non-weight bearing, very heavy or have limited head control. Use this sling for amputee residents (in a reclined position). Choose from solid polyester fabric or bathing mesh, with or without commode opening. The mesh slings may be used for bathing or when the sling will be left under the resident. The solid polyester slings should not be left under the resident for extended periods because the padding retains body heat. Residents may be seated on "pressure-reduction" cushions. However, the efficacy of those cushions is diminished when solid padded slings are left under the resident.


Model	Fabric & Style	Size/Trim Colour
R110	Mesh, Full Body	Medium/Purple
R111	Mesh, Full Body	Large/Green
R112	Solid Fabric, Full Body	Medium/Purple
R113	Solid Fabric, Full Body	Large/Green
R114	Mesh, Full Body w/Opening	Medium/Purple
R115	Mesh, Full Body w/Opening	Large/Green
R116	Mesh w/Opening	Extra Large/Blue
R117	Solid Fabric	Extra Large/Blue
R140*	Heavy Duty Mesh	XXL/Black
R141*	Heavy Duty Mesh w/Opening	XXL/Black

\*Use Reliant 600 Lift

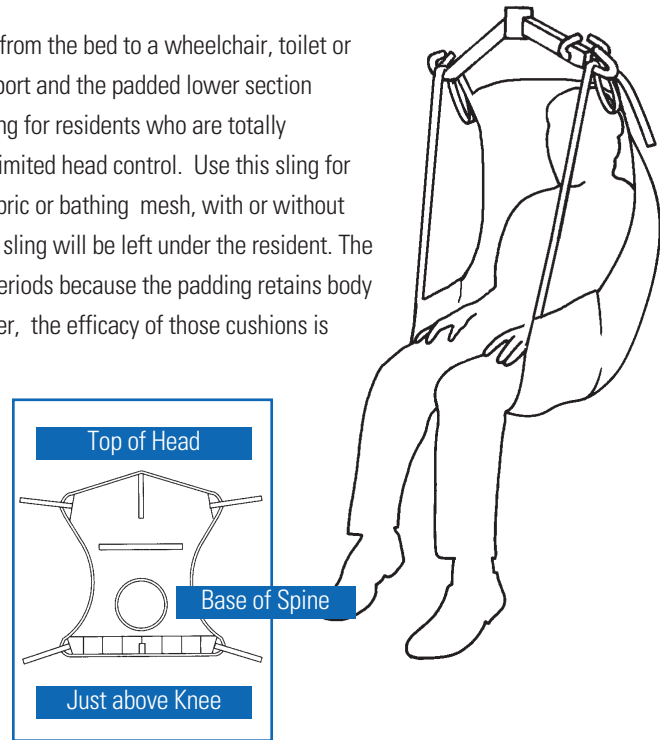

Model	Fabric & Style
9042	Solid Fabric, Full Body
9043	Solid Fabric, Full Body w/Opening
9046	Mesh, Full Body
9047	Mesh, Full Body w/Opening

## DIVIDED LEG WITH HEADREST

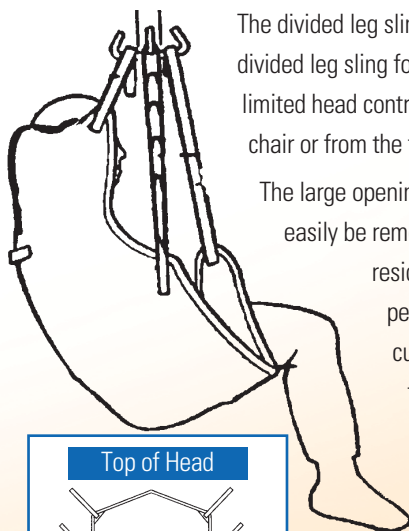


The divided leg sling is a secure, easy-to fit general purpose sling, designed for the majority of sling users. Use the divided leg sling for residents who are totally dependent, partially dependent, non-weight bearing, very heavy or have limited head control. Use this sling when transferring the resident from the bed to a wheelchair, recliner or shower-chair or from the floor to a bed.

The large opening also accommodates toileting. This versatile sling provides full head and neck support and can easily be removed or applied to a resident in a seated or supine position. The solid fabric has extra padding for residents with delicate skin. The solid polyester slings should not be left under the resident for extended periods because the padding retains body heat. Residents may be seated on "pressure-reduction" cushions. However, the efficacy of those cushions is diminished when solid padded slings are left under the resident.


Model	Fabric & Style	Size/Trim Colour
R100	Solid Fabric, Divided Leg w/Headrest	Medium/Purple
R101	Solid Fabric, Divided Leg w/Headrest	Large/Green
R102	Solid Fabric, Divided Leg w/Headrest	Extra Large/Blue
R100P	Solid Fabric, Divided Leg w/Headrest	Petite/Navy

## TOILETING SLING

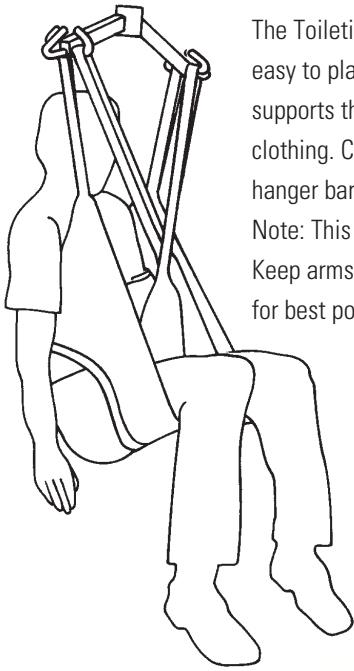

The Toiletting Sling offers superb access for quick toileting and hygiene. It is easy to place and remove in a bed or chair situation. The fully padded design supports the back, offers under thigh comfort and allows for removal of clothing. Colour coded positioning straps help align the sling properly on the hanger bar.

Note: This sling requires the patient to have FULL head and neck control. Keep arms on outside of toiletting sling. Place leg sections 3-4" above knee for best positioning.

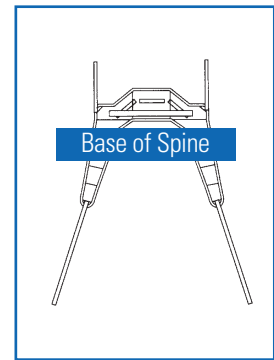

Model	Fabric & Style	Size/Trim Colour
R121	Solid Fabric, Toileting	Large/Green

## SLING POSITIONING (Reclined)

Place fan-folded or rolled sling under patient with grab strap away from skin. Roll patient from side to side, smoothing each half of the sling. Bed rail on opposite side should be raised. Refer to Invacare Sling Owner's Manual for detailed instructions.





Full Body



Divided Leg

## SLING POSITIONING (Seated)

For Divided Leg or Toiletting Sling, slide behind seated patient until belt is at waist level or bottom of back section is at the base of the spine. Place leg sections under thighs, smoothing fabric. Attach to hanger bar in one of the three methods: crossed between legs, under both legs or single loop under each leg.

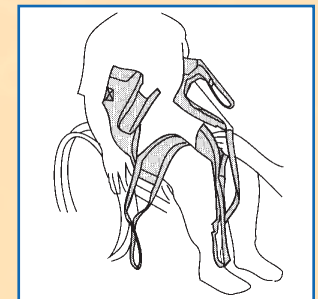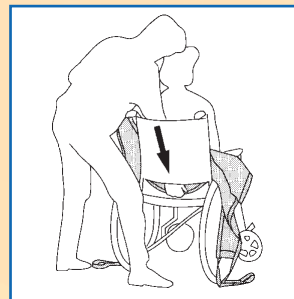

### Sling Specifications

	P	Divided-leg				Full-body			Full-body w/Commode			Toileting
		M	L	XL	M	L	XL	M	L	XL	L	
Dimensions												
Width	33"	37.5"	41"	44.5"	41.5"	45.5"	45.5"	41.5"	45.5"	45.5"	36"	
Commode opening	n/a	n/a	n/a	n/a	n/a	n/a	n/a	8"	8"	9"	n/a	
Length	59.8"	62.8"	67.8"	72.3"	54.7"	60.5"	65.3"	54.7"	60.5"	65.3"	37"	
Commode opening	n/a	n/a	n/a	n/a	n/a	n/a	n/a	11"	11"	13"	n/a	
Back	29.2"	35.7"	41.7"	43.2"	n/a	n/a	n/a	n/a	n/a	n/a	n/a	


Complete instructions for lift usage and sling procedures are found in the Invacare Operating Manual. Only Invacare slings and accessories should be used on Invacare lifts.



Application: Use only with Reliant Stand Up Lift.  
Models: RPS350-1, RPS350-2

## STANDING SLING R130

The R130 and R131 standing slings are for residents who are partially dependent, have at least 60% weight bearing capacity, have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance), and are able to bend at the hip, knees and ankles. Use the R130 or R131 standing slings for standing assistance, quick toileting, weight bearing practice and for transferring resident from bed to chair, or chair to bed.

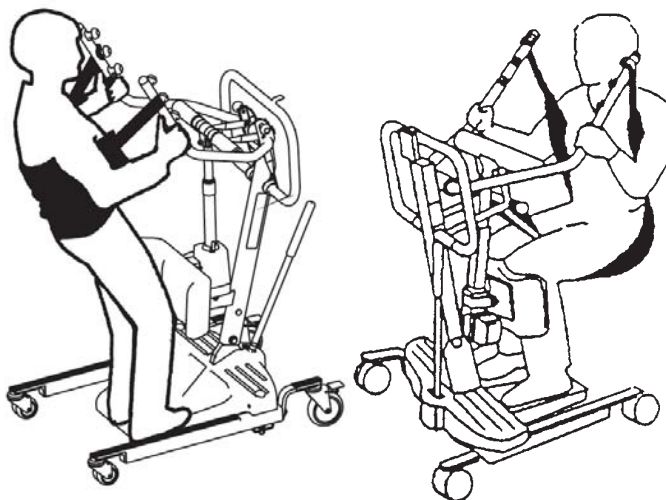

## TRANSPORT SLING R134

The R134 and R136 transport slings are for transferring residents with minimal weight-bearing capacity, who have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance), and are able to bend at the hips, knees and ankles. Use the R134 and R136 transport sling for transferring resident from bed to chair, chair to car or general in-room transport. Do not raise resident to a full standing position while using the transport sling.

## Sling Binding Colors




Note: Sling size and fit can vary significantly depending on patient weight and girth. These are general guidelines. Consult physician before sling selection.

Washing instructions are on each sling label. Washing temperature should not exceed 180°F. Air-dry or dry at low temperatures.

All slings are constructed of polyester/-nylon fabrics which can be used in wet or dry environments.

### Sling Sizes

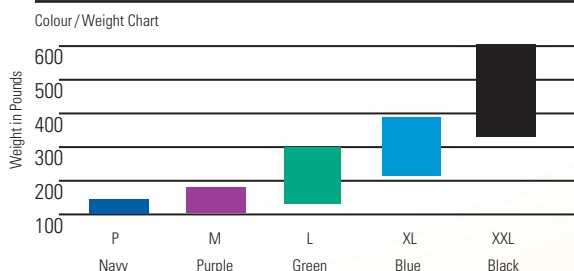

Invacare® Reliant Full-Body Slings with Commode Opening



Invacare® Reliant Toileting Sling with Belt



Invacare® Reliant Divided-Leg Slings with Headrest



Invacare® Reliant Transport Sling


The new Invacare® Premier Series slings balances solutions for any transfer. We say that with confidence because these slings were developed in collaboration with Occupational and Physical therapists. Invacare Premier slings are available in a wide variety of styles and can be adjusted to ensure a comfortable and secure fit for the resident. Clearly labeled slings and easy attachment points make it simple for the caregiver to quickly attach the sling and lift the resident.

**Application:** For use with Invacare® Reliant 350, 450 and 600 series power lifts, the Invacare® Premier Series Jasmine™ and Roze™ lifts.

**Models:** ROZE, JASMINE, RHL450-1, RPS350-1, RPS350-2, RPL450-1, RPL450-2, RPL600-1, RPL600-2



## UNIVERSAL HIGH SLING PLUS

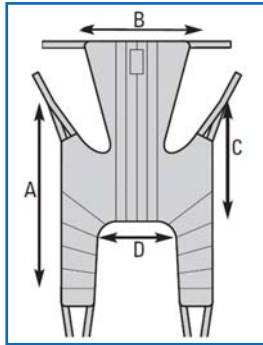


The Universal High Sling supports the whole body and head in transfers to/from seated or lying position. It offers a comfortable recline for those with limited head, trunk or hip control and allows arms to hang outside the sling.

*Compatible with the following lifts*  
 • Jasmine™ • Reliant™ 450, 600

### PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	D	POLYESTER FABRIC
S	54.4"	33.1"	34.7"	13.4"	2485960
M	59.5"	37.4"	36.6"	17.7"	2485961
L	62.3"	43.3"	38.2"	22.1"	2485962
XL	62.6"	46.9"	38.2"	24.8"	2485963

## STAND ASSIST SLING

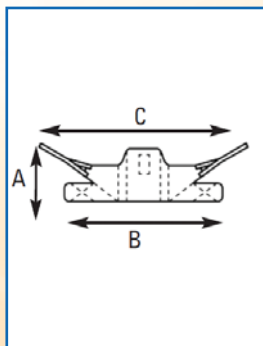


The Stand Assist Sling offers a comfortable and safe transfer for those who have some weight bearing ability, but cannot stand independently. The sling is well padded and features non-slip material on the side that touches the resident to assure a comfortable transfer position.

*Compatible with the following lifts*  
 • ROZE™ • Reliant™ 350

### PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	POLYESTER FABRIC
S	10.2"	33.1"	34.7"	2484685
M	13.0"	35.3"	46.1"	2484686
L	14.2"	37.4"	50.0"	2484687
XL	14.2"	40.2"	71.3"	2484688

## TRANSFER STAND ASSIST SLING

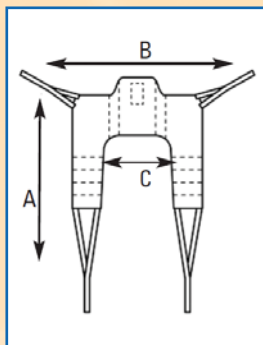


Also designed for optimal use with our Roze and Reliant 350 lifts, the Transfer Stand Assist Sling is a high-quality sling which provides simple seated transfer for residents with minimal weight bearing ability.

*Compatible with the following lifts*  
 • ROZE™ • Reliant™ 350

### PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	POLYESTER FABRIC
S	33.9"	33.1"	9.5"	2484689
M	40.2"	35.3"	10.2"	2484690
L	46.5"	37.4"	13.0"	2484691
XL	48.5"	40.2"	13.8"	2485117

## CRADLE SLING

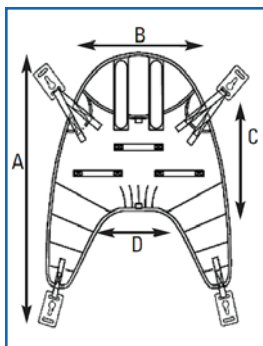

Designed for use with our new Jasmine lift. Comfort and support of the whole body and head are crucial for those with head, trunk and hip concerns. 4-point attachment clips onto the cradle bar for best positioning in horizontal-to-seated or seated-to-seated transfers. Head support is optional by removing rigidizers.


Compatible with the following lifts

- Jasmine™ with cradle accessory

PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	D	POLYESTER FABRIC
S	42.6"	33.1"	28.8"	15.0"	3483777
M	55.9"	39.4"	32.7"	15.8"	3483778
L	61.1"	43.3"	36.2"	20.1"	3483779
XL	66.2"	45.3"	36.2"	21.7"	3485949

## UNIVERSAL HIGH SLING

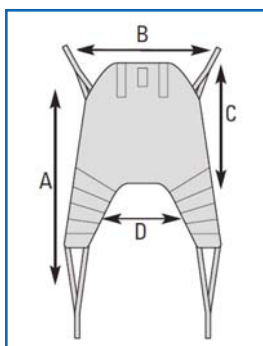

Supports the whole body and head in transfers to/from seated or lying position. Offers a comfortable recline for those with limited head, trunk or hip control. Universal High Sling allows arms to hang outside the sling.


Compatible with the following lifts

- Jasmine™
- Reliant 450, 600

PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	D	POLYESTER FABRIC	SPACER FABRIC
S	42.6"	33.1"	28.8"	15.0"	2483777	2485956
M	55.9"	39.4"	32.7"	15.8"	2483778	2485957
L	61.6"	43.3"	36.2"	20.1"	2483779	2485958
XL	66.2"	45.3"	36.2"	21.7"	2485949	2485959

## COMFORT SLING and COMFORT SLING with COMMODE OPENING

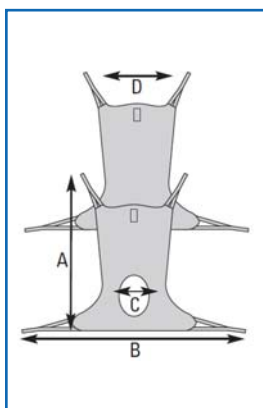

Amputees and residents with chronic or postural pain have delicate needs that may require a constant seated position. Our Comfort Slings transfer them from a horizontal to seated position, with an additional model if toileting is also required.


PRODUCT NUMBER/MATERIAL  
COMFORT SLING COMFORT SLING w/TOILETING OPENING

SIZE	A	B	C	D	NET FABRIC	SPACER FABRIC	POLYESTER FABRIC	SPACER FABRIC
S	42.6"	33.1"	28.8"	15.0"	2485968	2485554	2451092	2451097
M	55.9"	39.4"	32.7"	15.8"	2485969	2485777	2451093	2451098
L	61.6"	43.3"	36.2"	20.1"	2485970	2485778	2451094	2451099
XL	66.2"	45.3"	36.2"	21.7"	2485947	2451090	2451095	2451100

Compatible with the following lifts

- Jasmine™
- Reliant™ 450, 600

## EASY-FIT SLING

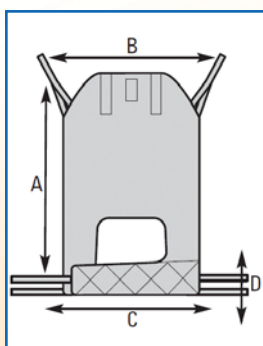

For those with more challenging physical disabilities, including amputees, this sling supports the whole body and head in an upright position with separate leg supports and optimal hip position.

Compatible with the following lifts


- Jasmine™
- Reliant™ 450, 600

PRODUCT NUMBER/MATERIAL

SIZE	A	B	C	D	POLYESTER FABRIC
S	44.5"	29.9"	25.2"	7.1"	2451102
M	47.3"	31.5"	26.6"	7.5"	2451103
L	50.0"	33.9"	28.4"	8.3"	2451104
XL	54.8"	37.0"	30.7"	8.7"	2451105



## DRESS TOILETING HIGH SLING

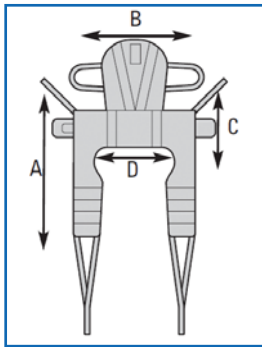


Featured belt offers secure support and positioning. Easy-to-apply Dress Toileting High Sling is suitable for seat-to-seat transfers and dressing or undressing during transfers to toilet. Dress Toileting High Sling is specially designed for users with reduced head control.

SIZE	A	B	C	D	PRODUCT NUMBER/MATERIAL
					POLYESTER FABRIC
S	34.3"	28.0"	7.9"	16.9"	2478444
M	37.4"	31.5"	9.1"	19.3"	2478445
L	40.2"	35.5"	9.1"	23.6"	2478446
XL	43.3"	42.6"	10.6"	29.9"	2484780

Compatible with the following lifts

- Jasmine™
- Reliant™ 450, 600

## POSITIONING THE SLING

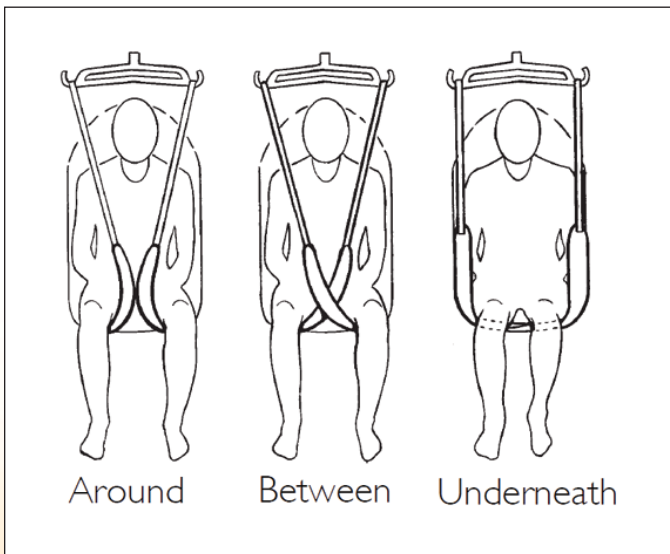

**Application: Use with Divided Leg, Universal, Easy-Fit, Toileting, Dress Toileting High Slings**

Divided Leg, Universal High, Easy Fit, Toileting, and Dress Toileting High styles may be used for patients on which sling removal is more difficult. Support ranges from upper back to beneath the thighs. These slings can be positioned with the patient seated or lying flat.

## PREMIER SLING FABRICS



Polyester Solid Sling Fabric - Offers low shear and enables easy transfers with minimal friction. The lightweight woven design is breathable and can be left in place for short periods of time.



Spacer Hybrid Poly Net - An optimal solution for long term care applications where a sling cannot be removed for short periods of time. Ideal for use with fully dependant users.



Premier Net Fabric - A woven poly mesh that is ideal for commode and bathing applications. Premier Net Fabric is also suitable for daily transfer applications and is an ideal single solution for partially dependant users.



**Yes, you can.®**



**Inspect sling before each use for wear, tears and loose stitching. Bleached, torn, cut, frayed or broken slings are unsafe and could result in injury. Discard immediately. Do not alter slings. Use only on Invacare lifts.**

### **Warning!**

BEFORE using the Patient Lift, READ and UNDERSTAND the Owner's Manual for proper operation and safety procedures.

Use the sling that is recommended by the individual's doctor, nurse or medical attendant. Before lifting, check all sling straps for secure points of attachment on lift device. Do not exceed weight limitation posted on lift. Use only with genuine Invacare Patient Lifts.

Bleached, torn, cut, frayed or broken slings are unsafe and could result in injury. Discard immediately. Do not alter slings. Use only on Invacare lifts.

**For additional information on training manuals or videos,  
contact Invacare Canada at (800)668-5324.**

**Invacare Corporation**  
www.invacare.ca

**Canada**  
570 Matheson Blvd. E.  
Unit 8  
Mississauga, Ontario  
L4Z 4G4 Canada  
(800) 668-5324

©2010 Invacare Corporation.  
All rights reserved.  
Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.

© 2010  
Invacare Corporation  
Form No. 10-765C  
Rev. 11/10

Use only Invacare lifts, slings and lift accessories to maintain patient safety. Use of other manufacturers products is prohibited and will void the warranty.

